
FSC

Philips Consumer Lifestyle BV
Tussendiepen 4, 9206 AD Drachten,
The Netherlands / les Pays-Bas
www.philips.com/avent

Trademarks are the property of Koninklijke Philips N.V.
© 2020 Koninklijke Philips N.V.
All rights reserved. / Tous droits réservés.
3000.052.3045.1 (2020-05-20)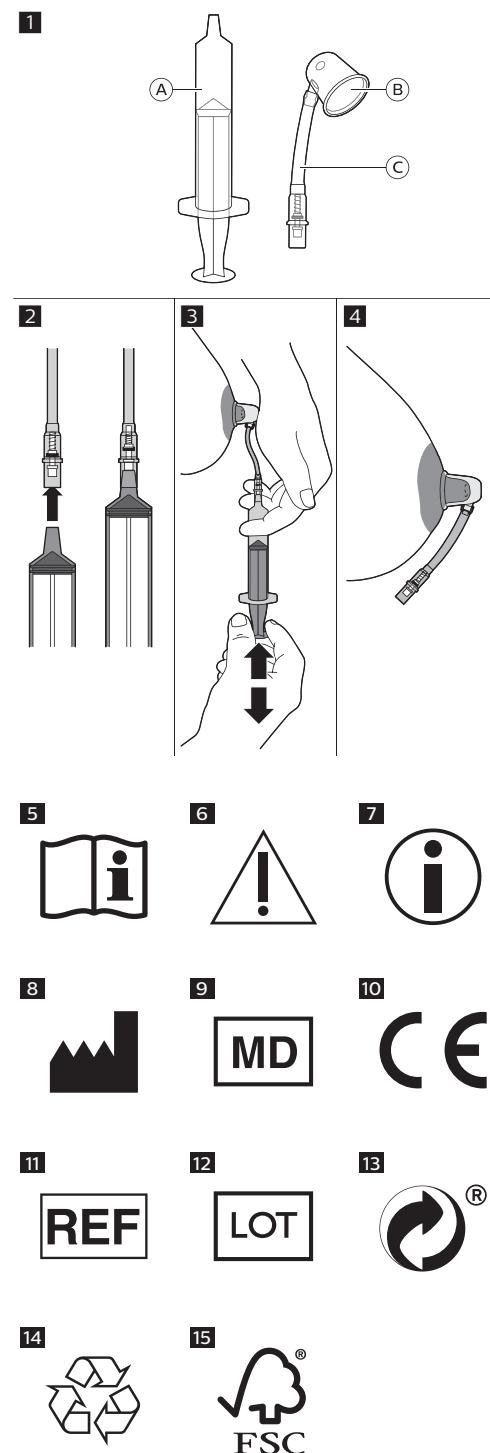

ENGLISH

Introduction

Flat and inverted nipples can be the result of short lactiferous (milk-bearing) ducts. The Niplette is designed to lengthen these ducts by the application of gentle suction. The Niplette can correct flat and inverted nipples and it has proven to be a long-lasting solution for both cosmetic purposes and to allow for breastfeeding.

The Niplette set includes:

- Syringe (Fig. 1A) – 1 or 2 pcs*
- Tube (Fig. 1C) with a thimble cup (Fig. 1B) – 1 or 2 pcs*
- Disposable breast pads – 2 pcs

* See on the packaging for your specific set.

Intended use

The Philips Avent Niplette is intended to correct flat and inverted nipples. The Niplette is intended for a single user.

Indications for use

The Philips Avent Niplette is indicated for use to correct flat and inverted nipples which have developed as a result of hereditary factors or during puberty. It is also indicated for users having had unsuccessful surgery to obtain a cosmetic correction.

Important safety information

Read this important information carefully before you use the product and save it for future reference.

⚠ Contraindications

- Do not use the Niplette during the last trimester of pregnancy.
- The Niplette is not suitable for lactating women.
- Do not use the Niplette on flat and inverted nipples developing later in life without medical advice. If in doubt, consult your healthcare professional.

Undesirable side effects

When using the product, undesirable side effects that may occur are pain, blisters, swelling, bleeding or discharge of fluid. If you experience any of these symptoms, first decrease the amount of suction. If the symptoms persist, contact a healthcare professional.

⚠ Warnings to avoid swallowing of small parts and choking:

- Always keep this product under adult supervision.
- Keep all parts not in use out of the reach of children.
- Do not allow children to play with small parts or packaging materials.

Other warnings:

- Inspect carefully before each use. If the Niplette appears to be damaged or becomes damaged, stop using it and throw it away.
- Only use accessories or parts specifically recommended by Philips to avoid unsafe situations.
- Do not modify the product to avoid unsafe situations.
- Do not use the on damaged skin.
- Any serious incident that has occurred in relation to this product should be reported to Philips via www.philips.com/support and to the competent authority of the Member State in which the user is established.

Cleaning

⚠ Warning: Clean the Niplette before first use and after each use.

⚠ Note: The Niplette is not suitable for any disinfection method, for example boiling in water, sterilization in microwave or in electric steam sterilizer, using alcohol-based wipes. Do not clean in the dishwasher.

- 1 Detach the tube from the syringe. Do not remove the thimble cup from the tube.
- 2 Clean the two parts in warm, soapy water and rinse under the tap.

⚠ Warning: Do not use aggressive cleaning agents when you clean the Niplette.

- 3 Assemble the parts.
- 4 Remove residual water from the tube by pushing the syringe down.
- 5 Disassemble the parts and let them dry on a clean surface.

Using the Niplette

- 1 Push the syringe firmly into the open end of the valve (Fig. 2).
- 2 Hold the thimble cup over the nipple with one hand and apply suction by pulling the syringe with the other hand (Fig. 3). Pull as firmly as is comfortable. Do not pull too hard. A small smear of lotion or cream at the base of the Niplette will improve the seal and suction.

⚠ Warning: Start with minimal suction and gradually increase the suction over time. Reduce the vacuum level if you experience pain. Excessive suction force applied to the nipple could cause pain, blisters, swelling, bleeding or discharge of fluid.

- 3 When the nipple has been pulled out, let go of the Niplette by holding the valve and by carefully separating the syringe from the valve (Fig. 4).

4 You are now free to continue your daily activities wearing the thimble cup and tube. It is easily concealed under loose clothing. You can also wear the Nipplette overnight.

⚠ Warning: Start wearing the product for a few hours and gradually increase the time you are wearing it. Depending on the duration of use, results can be expected within 3–12 weeks.

5 To remove the Nipplette, push the syringe into the valve to release the vacuum.

⚠ Warning: Release the vacuum on the nipple before re-applying the vacuum.

Storage – Store in a cool, dry place.

Disposal – According to local regulations.

Support

Visit www.philips.com/support for a list of frequently asked questions or contact the Philips Consumer Care Center in your country.

Technical information

- Materials
 - Thimble cup: Polypropylene (PP)
 - Tube: Polyvinylchloride (PVC)
 - Syringe: Polycarbonate (PC), Polypropylene (PP)
- Operating temperature: 5 °C to 40 °C (41 °F to 104 °F)
- Service life: 3 months

Explanation of symbols

The warning signs and symbols are essential to ensure that you use this product safely and correctly and to protect you and others from injury. Below you find the meaning of the warning signs and symbols on the label and in the user manual. This symbol indicates:

- to follow the instructions for use (Fig. 5).
- important information such as warnings and cautions (Fig. 6).
- usage tips, additional information or a note (Fig. 7).
- the manufacturer (Fig. 8).
- that the product is a medical device (Fig. 9).
- that the manufacturer has taken all necessary measures to ensure that the product complies with the applicable safety legislation for distribution within the European Community (Fig. 10).
- the catalog number (Fig. 11).
- the batch number of the medical product (Fig. 12).
- that the packaging should be recycled and packaging fees have been paid to the Green Dot recycling organizations (Fig. 13).
- that an object is capable of being recycled – not that the object has been recycled or will be accepted in all recycling collection systems (Fig. 14).
- Forest Stewardship Council. The FSC trademarks enable consumers to choose products that support forest conservation, offer social benefits, and enable the market to provide an incentive for better forest management (Fig. 15).

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