HC8900

ENGLISH 6

DEUTSCH 14

NEDERLANDS 22

Introduction

Congratulations on purchasing your new Philips Microlux. Due to its patented construction it is the most invisible hearing enhancer on the market. The Microlux is not only completely invisible when you wear it, but it also produces a much better sound. This is thanks to your own auricle which, after all, works as a reflector. Half of the amplifying effect of your ear can be attributed to the auricle. You owe the other half to the cavity just in front of the entrance to the auditory canal. With the Philips Microlux, this cavity is not filled up, as is still the case with today's in-the-ear hearing aids. Consequently the (directional) quality of the sound is retained and you hear your own voice without resonance.

The irksome squeaking and whistling you hear when, for example, you make a telephone call or move your lower jaw (talking, chewing) are things of the past; you have far less trouble with wind in your ears and you can participate in sports. Greater amplification is possible as the Microlux is positioned deeper in the ear and earwax is no longer a problem because of the characteristic construction of the Microlux. The Microlux does not cause interference to mobile telephones or vice versa. The Philips Microlux will give you many years of uninterrupted pleasure!

General description (Fig. 1)

Hearing capsule

- A** Pull tab
- B** Microphone opening
- C** Programming output
- D** On/off switch
- E** Battery

Customized sleeve

- F** Pull tab
- G** Sleeve edge
- H** Left/right identification dot
- I** Sound output

Accessories

- J** Battery insertion aid
- K** Storage box
- L** Battery insertion aid of storage box

Important

Read these instructions for use carefully before using the Microlux and save them for future reference.

- ⓘ If you have two Microlux hearing capsules, make sure you do not switch them because the programming for the left ear is often different from that for the right ear.
- ⓘ Never leave the hearing capsule, customized sleeve and battery unattended. If these items are swallowed by children or pets, they may cause suffocation.
- ⓘ Regularly clean the hearing capsule and the customised sleeve, to avoid skin irritation or inflammation (see chapter 'Cleaning').
- ⓘ Regularly check if the sound output is not blocked, because this will deaden the sound.
- ⓘ Never let anyone else use your Microlux because this can cause injury or damage to the other person's hearing and ear.
- ⓘ Keep the Microlux dry.
- ⓘ Do not wear your Microlux during swimming, showering and when blow-drying your hair or in combination with other heat sources close to the ear.
- ⓘ This appliance meets the requirements of MDD 93/42/EEC for medical appliances.

Preparing for use

Putting the hearing capsule in the customised sleeve

Always place the hearing capsule and customised sleeve on a clean surface.

- 1** Slide the hearing capsule (battery first) into the customised sleeve and press it as far as it will go. (Fig. 2)

8 ENGLISH

- ▶ Make sure that the pull tab of the hearing capsule is positioned on top of the pull tab of the customised sleeve.
- 2** Switch on the Microlux by setting the on/off switch to the 'on' position. (Fig. 3)

You can check whether the Microlux is working by holding it in the closed palm of your hand. You will hear a whistling sound if the Microlux is functioning. If it does not whistle, check the sound output and the position of the battery.

Using the Microlux

Inserting the Microlux in your ear

If you have two Microlux hearing capsules, make sure you do not switch them, because the programming for the left ear is often different from that for the right ear.

- 1** Grip the assembled Microlux by both pull tabs and insert it loosely into the auditory canal. (Fig. 4)
- ▶ The Microlux with the red identification dot is for your right ear, the Microlux with the blue identification dot is for your left ear.
 - ▶ The identification dot must point upwards when the Microlux is in your ear.
- 2** Place your forefinger or thumb behind both pull tabs at the location of the programming contact and press the Microlux further into your ear.
- ▶ You may have to guide the Microlux upwards and backwards when doing so, depending on the shape of the auditory canal.
 - ▶ If necessary, stretch the auditory canal with your other hand by pulling your auricle upwards and backwards.
 - ▶ It may help to open your mouth wide when inserting the Microlux.
 - ▶ If necessary, massage the Microlux with the point of your forefinger to ease it further into the ear.

Note: It is not possible to insert the Microlux too deeply into the auditory canal.

Removing the Microlux from your ear

- 1 Slowly pull at both pull tabs to remove the Microlux from the auditory canal.
- 2 Switch off the Microlux and check if it is off by holding it in the closed palm of your hand.

The Microlux does not whistle if it has been switched off.

Adjusting the sound

The hearing capsule has a high sound quality and many possibilities for adjusting the amplification. The sound is adjusted by your audiologist using a computer program.

You cannot adjust the sound of the Microlux yourself. Consult your audiologist if you have complaints about the sound.

Earwax

Earwax protects your ears from detrimental effects from outside, including dirt and infections. When there is something (e.g. a hearing aid) in your ear, more wax is produced than normally as a reaction to the foreign body. This is not a problem with the Philips Microlux hearing enhancer, because it takes some earwax with it every time you remove it from your ear. This slows down the buildup of an earwax plug.

- ▶ An earwax plug can lead to squeaking in your Microlux hearing enhancer or you may suddenly hear less well, even though there is still enough energy in the battery. Have your ears examined by your audiologist to determine if you have an earwax plug. If so, have it removed by a medical professional.

Do not try to clean your ears yourself with, for example cotton swabs.

Cleaning

If you have two Microlux hearing capsules, always clean them one at a time to prevent switching them.

Tip: Clean the Microlux hearing capsule daily to prevent dirt or moisture from collecting between the hearing capsule and the customised sleeve.

- 1** Firmly grip the pull tab of the hearing capsule (not the pull tab of the customised sleeve).
- 2** Place the thumbnail of your other hand against the edge of the customised sleeve and pull the hearing capsule out of the sleeve.
- 3** Clean the hearing capsule by rubbing it with a dry cloth or tissue.

Keep the hearing capsule dry.

- 4** Check if the sound output and microphone opening are still clean and open. If they are not, ask your audiologist to clean them.

Never clean the capsule housing with cleaning agents yourself.

- 5** Thoroughly clean and disinfect the customised sleeve by using the cleaning agent provided by your audiologist.

Do not use any other cleaning agent than the one provided by your audiologist.

- 6** Let the customised sleeve dry before you reinsert the hearing capsule.

Replacing the battery

Tip: If you have two Microlux hearing capsules, do not take both of them apart at the same time to prevent switching the hearing capsules and sleeves.

Tip: Always work above a table or similar flat surface.

Note: The correct position is very important when placing the battery. The + of the battery should be on the same side as the pull tab.

Note: Always use the battery insertion aid provided to insert the battery. If, for any reason, you have to insert the battery with your fingers, place the battery horizontally between the battery contacts to prevent short-circuiting.

- 1** Remove the hearing capsule from the customised sleeve (see chapter 'Cleaning').
- 2** Remove the battery by placing your fingernail between the hearing capsule and the round part of the battery. (Fig. 5)
- 3** Remove the sticker from the new battery.
Only use type 10 batteries (yellow sticker).
Do not remove the sticker before you are going to use the battery.

Replacing the battery with the battery insertion aid in the storage box

- 1** Slide the lid of the storage box completely open and place the new battery in the battery cavity. (Fig. 6)
- 2** Press the hearing capsule into the storage box over the battery. (Fig. 7)
Make sure the pull tab is at the same side as the + side of the battery.

Replacing the battery with the loose battery insertion aid

- 1** Place the battery in the battery cavity of the battery insertion aid. (Fig. 8)
- 2** Press the hearing capsule into the battery insertion aid. (Fig. 9)
Make sure the pull tab is at the same side as the + side of the battery.

Storage

- ▶ Store the Microlux in the storage box provided.

Environment

- ▶ Batteries contain substances that may pollute the environment. Do not throw empty batteries away with the normal household waste, but hand them in at an official collection point. Always remove the battery before discarding the Microlux and handing it in at an official collection point.

12 ENGLISH

- Do not throw the Microlux away with the normal household waste at the end of its life, but hand it in at an official collection point for recycling. By doing this you will help to preserve the environment. (Fig. 10)

Guarantee and service

If you need information or if you have a problem, please visit or contact your audiologist.

Frequently asked questions

This chapter lists the questions most frequently asked about the Philips Microlux hearing enhancer. If you can't find the answer to your question, please contact your audiologist.

Question	Answer
Why doesn't the Microlux hearing enhancer work?	The battery may be empty. Replace the battery (see chapter 'Replacing the battery').
	The Microlux may not be on. Switch on the Microlux by setting the on/off switch to the 'on' position (see chapter 'Preparing for use').
	The sound output and/or microphone opening may be blocked. Check if they are and, if so, ask your audiologist to clean them.
	Dirt and/or moisture may have collected between the customised sleeve and the hearing capsule. Remove the hearing capsule from the customised sleeve and clean both parts (see chapter 'Cleaning').
How can I adjust the sound of the Microlux hearing enhancer?	You cannot adjust the sound yourself. Consult your audiologist if you have any complaints about the sound.

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PHILIPS DAP B.V.