

## User Manual


Clean the product before using it for the first time.

### Size instruction:


One size fits most mothers, with an upper bust between 82.5 and 105 cm (32–41 inches), and under bust between 70 and 92.5 cm (28–36 inches).

### Instructions:


It is recommended to use this pumping bra together with Philips Avent breast pump.

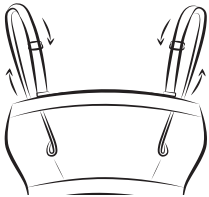

Adjust the bra and shoulder strap to fit your size.

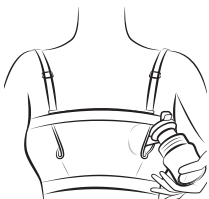

Insert the pump funnel into the circular cutout in front of the bra, install the accessories and main body in place according to the user manual of your breast pump.

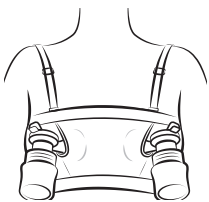

Keeping the pump funnel in contact with your breast, switch on the breast pump, select a comfortable level, you can then begin hands-free pumping.

### Washing instructions:

- Wash only with similar colors.
- Wash gently by hand.
- Do not bleach.
- Hang in the shade to dry.
- Do not iron.
- Do not dry clean.


## 产品使用说明书

初次使用前，请先清洗本产品。


### 尺寸说明：

一款尺寸可满足大多数妈妈的需求的胸衣。上胸围 82.5-105cm，下胸围 70-92.5cm 均可使用。


**使用说明：建议配合飞利浦新安怡电动吸乳器使用免手扶吸乳文胸。**

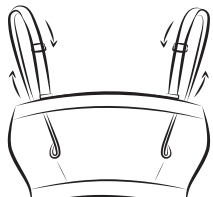

将胸衣和肩带调整至适合您的尺寸。

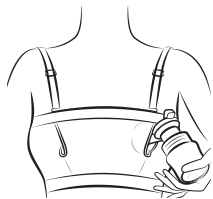

将吸乳器喇叭口装入文胸前部的圆形切口，根据您所使用的电动吸乳器说明书，将配件和主机安装到位。

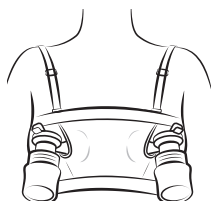

请保持吸乳器的喇叭口紧贴在乳房上，开启电动吸乳器电源开关，调至使您觉得舒适的档位，即可开始免手扶吸乳。

### 洗涤说明：

- 与同色衣物一起洗涤
- 轻柔手洗
- 不可漂白
- 阴凉处悬挂晾干
- 不可熨烫
- 禁止干洗

## PHILIPS AVENT

飞利浦新安怡

产品	： 飞利浦新安怡免手扶吸乳文胸
成分	： 面料: 77% 锦纶, 23% 氨纶 内里: 95% 棉, 5% 氨纶
执行标准	： FZ/T 73012-2017
安全技术类别	： GB 18401-2010(A类) 直接接触皮肤类
检验	： 合格
质量等级	： 一等品
产地	： 中国 江西 吉安
维护方法	：

飞利浦（中国）投资有限公司  
上海市静安区灵石路718号A1幢  
全国顾客服务热线：4008 800 008



出版日期：2021/06/30  
3000.045.6668.3