
# Always here to help you

Register your product and get support at  
[www.philips.com/welcome](http://www.philips.com/welcome)

PR3840

# PHILIPS

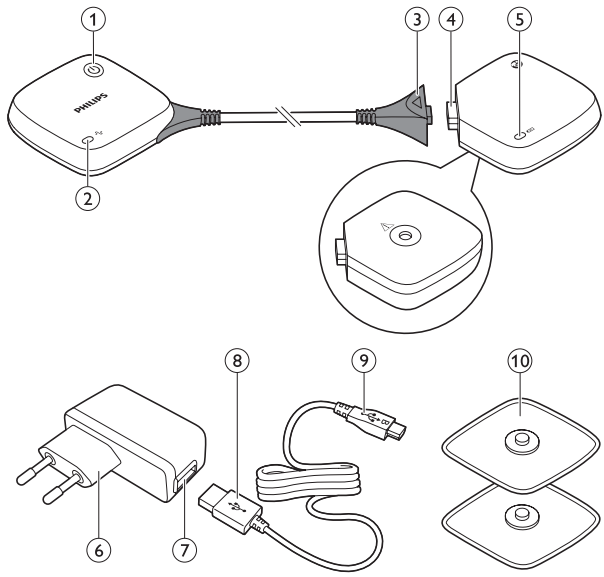**ENGLISH 4**

**DANSK 52**

**ESPAÑOL 101**

**SUOMI 153**

**NORSK 199**

**PORTUGUÊS 247**

**SVENSKA 298**

## Table of contents

Introduction	4
Contra-indications, reactions and side effects	6
Important safety information	8
Pre-set programs	16
Product overview	22
Preparing for use	22
Using the device	24
After use	32
Troubleshooting	34
Signals on the device	37
Replacement	38
Ordering accessories	38
Guarantee and support	39
Recycling	40
Specifications	41
Explanation of symbols	43
Electromagnetic emissions and immunity	49
Electrode placement guide	50

## Introduction

Congratulations on your purchase and welcome to Philips! To benefit fully from the support that Philips offers, register your product at **[www.philips.com/pain-relief/register](http://www.philips.com/pain-relief/register)**.

Read this user manual carefully before you use the PulseRelief device. The PulseRelief device is available without prescription. To achieve optimal treatment success with your PulseRelief device, we recommend that you use it according to the treatment instructions in this user manual. If you need further information or have questions, please visit **[www.philips.com/pain-relief](http://www.philips.com/pain-relief)** or contact the Philips Consumer Care Centre in your country.

---

## Intended use

---

The PulseRelief is intended to be used by adult consumers who experience mild to moderate chronic pain or acute post-surgical pain. The purpose of this device is to provide pain relief through transcutaneous electrical nerve stimulation (TENS). You can also use it for electrical muscle stimulation (EMS). The device is suitable for home use.

---

## TENS and pain management

---

Transcutaneous electrical nerve stimulation (TENS) is a non-invasive pain relief method for which clinical proof exists that it can help relieve pain. Physiotherapists and other healthcare professionals use TENS daily as part of their treatment. TENS treatment passes electrical pulses across the intact surface of the skin to activate the underlying nerves. Clinical evidence suggests that TENS produces pain relief in a similar way to the nervous system's own pain-relief mechanisms, for instance similar to massaging a painful area.

The device uses a rechargeable battery to generate pulses. These pulses are applied to the skin through self-adhesive hydrogel electrodes. You can choose one of the 15 pre-set TENS programs with different pulse settings and you can adjust the intensity of the pulse according to your own preference.

---

## Electrical muscle stimulation

---

Electrical muscle stimulation (EMS), also known as neuromuscular electrical stimulation (NMES) or electromyostimulation, uses electric pulses to evoke muscle contraction. A stimulation device generates pulses and delivers them through electrodes on the skin close to the muscles you want to stimulate.

The pulses mimic the way the central nervous system makes the muscles contract. You can choose one of the 5 pre-set EMS programs.

### **Pain management**

---

We all want to enjoy doing our daily activities. When you experience pain, however, the instinctive reaction is to avoid pain by reducing your activity level. As a result, you become less fit and less able to enjoy doing the things you like. Staying active, without overdoing it, is important to avoid being trapped in a persistent pain cycle. The key to self-managing your pain is pacing: distributing periods of activity and periods of rest over the day in a way that feels most comfortable for you. The key to pacing is spreading out your activities and 'taking a break before you need it'. To get more tips on pacing and how to self-manage your pain, tap the info icon in the Philips Treatment App and select 'Pain Toolkit'.

### **Contra-indications, reactions and side effects**

#### **Contra-indications**

---

Do not use the device if:

- you have a cardiac pacemaker, implanted defibrillator or other implanted metallic or electronic device (e.g. a drug delivery system), as use in these cases could cause electric shock, burns, electrical interference, or even death.
- you have a heart disease.
- you have epilepsy.

- you are in the first trimester of your pregnancy, as the effects of TENS on the development of the foetus are as yet unknown. Avoid use over the uterus during pregnancy to reduce the risk of inducing labour. If you are pregnant, always consult your doctor or midwife before use.
- you have a cognitive impairment.
- you have venous or arterial thrombosis or thrombophlebitis. If you use the device for EMS, there is a risk of mobilising the blood clot.
- you have undiagnosed pain, unless your doctor recommends use of this device.

---

## Reactions

Stop using the device and consult your doctor if you experience adverse reactions from the device.

Possible adverse reactions may include the following:

- skin irritation beneath the electrodes, although the gel used on the electrodes is not known to cause allergic reactions
- burns beneath the electrodes
- headaches or other painful sensations
- nausea or feeling faint

---

## Side effects

- After EMS, some muscle soreness may occur some time after the treatment.
- Some reddening of the skin under or around the electrodes may occur during and shortly after EMS treatment. Skin reddening usually disappears within two hours after treatment. If the skin continues to be red for more than a day, please consult your doctor.

## Important safety information

### Warnings

#### General warnings

- Read this user manual carefully and always adhere to the treatment instructions.
- This is a medical device. Keep the device out of the reach of children. Children may not be able to use it according to the instructions in this user manual.
- This appliance is not intended for use by adults with reduced sensory or mental capabilities. They may not be able to use it according to the instructions in this user manual and they may become agitated by the treatment.
- Do not use the device on children, as the device is not designed to use on children.
- Do not modify the device or the electrodes. This could cause improper functioning.
- Only use the adapter supplied to charge the device. Use of another adapter may cause a hazardous situation.

- This device is designed for use by and on a single adult person.

**Do not apply to or treat the following parts or areas**

- Stimulation on the sides of the neck over the carotid sinus may have severe adverse effects on your heart rhythm or blood pressure.
- Stimulation on the front of the neck may cause severe muscle spasms that could close your airway and result in breathing difficulties.
- Do not apply stimulation across your chest because the introduction of electrical current into the chest may cause rhythm disturbances to your heart, which could be lethal.
- Stimulation across the head or on opposite sides of the head must be avoided, as the effects on the brain are unknown.
- Do not place the electrodes on skin that is not intact, clean or healthy. Open, irritated or otherwise damaged skin may lead to the delivery of too much current to the area, which may potentially cause burns.

- Do not place the electrodes near cancerous lesions, as this may have a negative impact on the cancerous lesion.
- Do not place the electrodes on areas of skin that lack normal sensation. This may cause you to be unaware of the high intensity current administered, which may result in electrical skin burns.
- Do not place the electrodes over open wounds or rashes. Open wounds may lead to the delivery of too much current to the area, potentially leading to burns. It might also cause substances in the electrode to enter the skin.
- Do not place the electrodes over swollen, red, infected or inflamed areas or skin eruptions (e.g. phlebitis, thrombophlebitis and varicose veins). Stimulation must not be applied to areas with known thrombosis or thrombophlebitis, as stimulation may increase circulation, resulting in a greater risk of emboli.

- Do not place the electrodes inside body cavities, e.g. the mouth. This device is not designed for internal application.

### **Do not use the device in the following conditions**

- Do not use the device if you are in the care of a doctor and have not consulted your doctor first.
- Do not use the device if you have a tendency to bleed internally due to any impact or injury.
- Do not use EMS stimulation to contract a muscle if contraction of the muscle may disrupt healing. For instance, if the muscle or tendon is torn, muscle contraction may exacerbate the tear, just like a voluntary contraction. This may also be the case after a recent surgical procedure or following acute trauma or fracture. Muscle contraction in case of tendinitis may also worsen symptoms.
- Do not use the device while driving, operating machines or performing any other activity in which electrical stimulation can put you at risk of injury.

- Do not use if you are likely to fall asleep during the treatment, as this may cause you to sense injuries too late. If you use the device at bedtime, set the timer to make it switch off automatically.
- Do not use while in the bath, shower or swimming pool. This increases the risk of electric shocks and burns to the skin.
- Do not apply stimulation in the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), which may not operate properly when the electrical stimulation device is in use.
- Do not use the device when you are connected to high-frequency surgical equipment. This may result in burns on the skin under the electrodes and may damage the device.
- Do not use the device within less than 1 metre from shortwave or microwave medical equipment. Close proximity to this equipment may cause unstable device output.

- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

---

### **Cautions**

- If you had surgery recently, consult your doctor before you use the device. Using the device may disrupt the healing process.
- Use Burst TENS programs while resting, as these programs can result in muscle contraction in the treatment area.

- Make sure that you end the treatment in the Philips Treatment App or press the on/off button on the device before you remove the device or the electrodes. If you do not end the treatment, you may experience an unpleasant sensation in your fingers when you touch the magnets. This sensation is not harmful, but can be unpleasant.
- Only use this device with the adapter, cord and accessories recommended by the manufacturer.
- Always check the device and the electrodes for damage before use. Do not use the device or an electrode if it is damaged.
- The device is not waterproof. Do not use the device in wet surroundings and prevent it from getting wet.
- Although you can use the device indoors and outdoors, it does not withstand all weather conditions.
- The device does not withstand high and low temperatures. Please check the operating conditions in chapter 'Specifications'.

- The electrodes have a limited shelf life. Please check the packaging for the use-by date prior to use. Do not use electrodes whose use-by date has expired. For ordering replacement electrodes, see chapter 'Ordering accessories'.
- Do not use plaster or tape to attach the electrodes to the skin.
- Always use and store the electrodes according to the instructions in chapters 'Using the device' and 'After use'.
- If the device does not function as described here, stop using it and contact the Consumer Care Centre. For contact details, see chapter 'Guarantee and support'.

---

### **Compliance with standards**

- The device meets the relevant standards for this type of Class IIa electrical medical appliance and appliances using electrical stimulation for home use.
- This Philips appliance complies with all applicable standards and regulations regarding exposure to electromagnetic fields.

## Pre-set programs

The device comes standard with 15 pre-set programs for TENS and 5 pre-set programs for EMS. Please note that the order of the programs in the list does not signify a recommendation of specific programs for specific conditions. As the effect of a treatment program may strongly differ from person to person, we advise you to try out different treatment programs to find out which one works best for you.

## Program overview

---

### Conventional TENS programs

Conventional TENS is a common TENS program with high-frequency stimulation. It is based on the Gate Control Theory that states that this type of stimulation will block transmission of pain signals in the nervous system. It is comparable to 'rubbing the pain away'. Most people use Conventional TENS for 30 minutes per session, but you can use it as often and as long as it feels comfortable. The effect is usually most perceptible during stimulation and generally wears off rather quickly after stimulation. You have to adjust the intensity of the program so that you clearly feel the stimulation. Stimulations must not feel unpleasant or painful.

Name of Conventional TENS program	Program description	Suggested treatment time
Conventional	May give you a continuous tingling stimulation	No limit
Thorough stimulus	May give you a more thorough stimulating sensation	No limit
Pre-activity	May give you a more local and soft stimulating sensation	No limit
Soft stimulus	May give you a softer stimulating sensation, e.g. for sensitive body parts	No limit
Deep stimulus	May give you a deeper stimulating sensation	No limit
Radiating pulse	May give you a more radiating stimulation sensation	No limit

### **Burst TENS programs**

Burst TENS offers a high-frequency stimulation that comes in trains of pulses. This type of stimulation is known to trigger endorphin release. Endorphins are painkillers produced by the body. You can use Burst TENS around 30 minutes multiple times a day, for example 3 times. We advise you to adjust the intensity of the Burst program so that you feel the stimulation strongly. Burst programs generally give a sharper sensation, which can feel somewhat unpleasant. It is also advised to adjust the intensity up to a level that produces visible muscle contractions in the treatment area. These contractions are not harmful. If you find these contractions too uncomfortable, you can decrease the intensity, choose a different program or move the electrodes to a different place. Apply Burst TENS programs while resting, as they can result in muscle contraction in the treatment area.

Name of Burst TENS program	Program description	Suggested treatment time
Continuous burst	May give you a pulsing burst stimulation to release endorphins in your body	30 min.*
Slow kneading	May give you a slower and massaging burst stimulation to release endorphins	30 min.*
Mild kneading	May give you a milder burst stimulation to release endorphins in your body	30 min.*
Deep kneading	May give you a more local and deeper burst stimulation to release endorphins in your body	30 min.*
Diffuse burst	May give you a more diffuse burst stimulation to release endorphins in your body	30 min.*

\* optionally multiple times a day, can be changed in the app

### **Frequency-modulated TENS programs**

Frequency-modulated TENS is a program in which the frequency of the pulse varies from high to low. This can be more comfortable or effective than the other programs in some cases. You can use frequency-modulated TENS for around 30 minutes multiple times a day, for example 3 times.

Name of Freq. Mod.TENS program	Program description	Suggested treatment time
Mixed frequency	May give you a flowing stimulation sensation	30 min.*
Pulsing massage	May give you a more pulsing stimulation sensation	30 min.*
Deep massage	May give you a slower and deeper flowing sensation	30 min.*
Gentle massage	May give you a mild flowing stimulation sensation	30 min.*

\* optionally multiple times a day, can be changed in the app

### **Electrical muscle stimulation (EMS) programs**

Electrical muscle stimulation (EMS), also known as neuromuscular electrical stimulation (NMES) or electromyostimulation, uses pulses to evoke muscle contraction. This can for example help to strengthen muscles.

Use EMS only on healthy, uninjured muscles. Apply EMS programs while resting (sitting, lying), as they result in muscle contraction. First-time users can experience electrical muscle stimulation as an unfamiliar and unusual sensation. We therefore advise you to start at a low intensity setting to get used to the sensation before you choose a higher intensity setting. In general, stimulation must always feel comfortable. The muscle contractions must not feel exhausting, strenuous or tiring. After strenuous exercise or exertion, we advise you to use a lower setting to avoid muscle fatigue.

Name of EMS program	Program description	Suggested treatment time
Muscle stimulation pulse	May give you a nice soothing sensation	30 min.*
Muscle stimulation flow	May give you a comfortable tingling sensation	20 min.*
Muscle stimulation wave	May give you a mild contracting and relaxing sensation	30 min.*
Muscle stimulation flux	May give you an energising muscle sensation	25 min.*
Muscle stimulation bounce	May give you a rhythmically relaxing sensation	20 min.*

\* optionally multiple times a day, can be changed in the app

### Program specifications

The table below gives information on the frequency, pulse width and pre-set time of each program. The current pulse in all programs has a symmetrical bi-phase waveform.

Program name	Program type	Frequency	Pulse width	Pre-set time
Conventional	Conventional	100 Hz	200 $\mu$ s	continuous
Thorough stimulus	Conventional	40 Hz	200 $\mu$ s	continuous
Pre-activity	Conventional	100 Hz	150 $\mu$ s	continuous
Soft stimulus	Conventional	80 Hz	60 $\mu$ s	continuous

Program name	Program type	Frequency	Pulse width	Pre-set time
Deep stimulus	Conventional	80 Hz	150 $\mu$ s	continuous
Radiating Pulse	Conventional	60 Hz	150 $\mu$ s	continuous
Continuous burst	Burst	100 Hz, 2 Hz	250 $\mu$ s	30 min.
Slow kneading	Burst	80 Hz, 1 Hz	200 $\mu$ s	30 min.
Mild kneading	Burst	80 Hz, 3 Hz	200 $\mu$ s	30 min.
Deep kneading	Burst	80 Hz, 2 Hz	150 $\mu$ s	30 min.
Diffuse burst	Burst	80 Hz, 2 Hz	180 $\mu$ s	30 min.
Mixed frequency	Freq. Mod.	8-100 Hz	250 $\mu$ s	30 min.
Pulsing massage	Freq. Mod.	2-8 Hz	250 $\mu$ s	30 min.
Deep massage	Freq. Mod.	2-80 Hz	180 $\mu$ s	30 min.
Gentle massage	Freq. Mod.	8-80 Hz	150 $\mu$ s	30 min.
Muscle stimulation pulse	Ampl. Mod.	40 Hz	200 $\mu$ s	30 min.
Muscle stimulation flow	Ampl. Mod.	50 HZ	150 $\mu$ s	20 min.
Muscle stimulation wave	Ampl. Mod.	65 Hz	200 $\mu$ s	30 min.
Muscle stimulation flux	Ampl. Mod.	40 Hz	350 $\mu$ s	25 min.
Muscle stimulation bounce	Ampl. Mod.	50 Hz	300 $\mu$ s	20 min.

## Product overview (Fig. 1)

- 1 On/off button
- 2 Status indicator
- 3 Detachable connector
- 4 Micro-USB socket of device
- 5 Battery indicator
- 6 Adapter
- 7 Socket for standard USB plug
- 8 Standard USB plug
- 9 Micro-USB plug
- 10 Self-adhesive hydrogel electrodes


## Preparing for use

### Charging the device


To be able to use the device, you first have to charge it. Charging the device takes approx. 5 hours at room temperature. When the rechargeable battery is fully charged, it contains sufficient energy for approx. 8 hours of continuous use under normal operating conditions.

*Note: If the device is not fully charged when you start a treatment, the battery may run out during the treatment. We advise you to always fully charge the device before you start a treatment.*


*Note: Treatment is not possible when the device is charging.*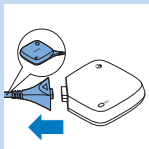

- 1** Unplug the cable that connects the two units from the micro-USB socket of the device unit with the battery indicator.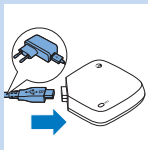


- 2** Insert the micro-USB plug of the adapter cable into the micro-USB socket of the device.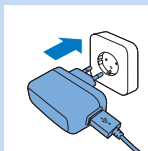

- 3** Insert the USB plug into the socket of the adapter and insert the adapter into a wall socket.


*Note: Insert the USB plug into a wall socket that is properly accessible so that the USB plug can be removed easily. The device is only connected to the mains during charging.*

- ▶ The battery indicator is solid orange during charging. When the battery is fully charged, the battery indicator turns solid green.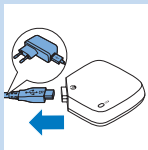

- 4** Remove the micro-USB plug from the micro-USB socket of the device.

*Note: The USB socket may only be used for charging with the adapter supplied or for connecting the cable that connects the two units. Do not use this socket for any other purpose.*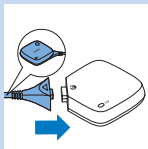

- 5** Reconnect the cable that connects the two units by inserting the device connector into the micro-USB socket of the device.

### **Installing and setting up the Treatment App**

You need the Philips Treatment app to operate the device. Please go to the Apple App Store or GooglePlay, search for the 'Philips Treatment' app and press install. Refer to the quick start guide for more details.

*Note: The Treatment App is an app that can be used for PulseRelief (PR3840) and BlueTouch (PR3733). The installation procedure is the same for both devices.*

## Using the device

### Attaching the electrodes and the device

- 1 Make sure that the skin you want to place the electrodes on looks healthy and is free from wounds or rashes.

For TENS: Do not place the electrodes on red or inflamed skin and skin with open wounds, cancerous lesions or rashes.

For EMS: Only place the electrodes on healthy, uninjured muscles.


- 2 Clean the skin with a moist cloth and dry it. The skin you want to place the electrodes on must be dry, clean and free from cream or lotion.

*Note: Dirt, grease or particles left on the skin may lodge themselves in the hydrogel of the electrodes. This has an adverse effect on the adhesive power of the electrodes.*


- 3 Take the electrodes from the plastic bag and check them for damage and signs of wear.

Caution: Do not use electrodes whose use-by date has expired. The use-by date is shown on the resealable plastic bag.

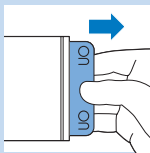
*Note: Replace the electrodes when they are damaged or dirty, when they have lost their adhesive power or when stimulation becomes uncomfortable, i.e. when you experience an unpleasant stinging or biting sensation.*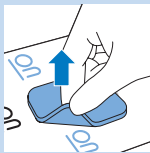

- 4** Pull the electrode off the liner and place it on the skin at the place you want to treat. See section 'Electrode placement' below and the placement guide.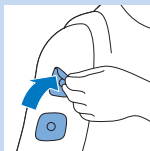

- 5** Place the second electrode on the body part you want to treat.


*Note: Always place the electrodes on the skin before you connect the device.*

*Note: Do not place the electrodes on top of each other or so close to each other that they touch each other.*

- 6** Attach the device to the electrodes. The device units have magnets in the centre to ensure easy connection to the metal connectors of the electrodes. When the device units lock onto the electrodes properly, you hear a click.

*Note: You can wear the electrodes with the device units attached to them under your clothes, also during treatment.*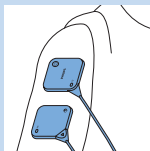

*Note: Consult the electrode placement guide for the recommended position for the electrodes during treatment. This is only a recommendation. You can perform all activities allowed as intended use and not warned against in chapter 'Important' when you wear the device on your body. You can also continue to wear the device on your body in between treatments.*

### **Electrode placement**


To get maximum results from stimulation, it is important to place the electrodes correctly. Always make sure that you have normal skin sensation in the treatment area before you attach the electrodes.

The optimal place may vary slightly from person to person, so try moving the electrodes around until you get the most effective results. In some cases, it can be helpful to mark the spot for future use. For suggestions regarding placement of the electrodes, see the 'Electrode Placement Guide'. Please note that you may need assistance to place the electrodes on certain spots on your body.


As a rule, place the electrodes on or near the treatment area. Stimulation may result in visible muscle contraction, especially in burst TENS programs (programs 7 to 11). This is not harmful, but if you find these contractions uncomfortable, try moving the electrodes to a different place. Please note that when you use EMS programs (programs 16 to 20), muscle contraction is a desired effect. If no muscle contraction occurs in an EMS program, you have not placed the electrodes correctly or you have chosen a too low intensity.

**Do not place electrodes over the carotid sinus on the sides of your neck or on the front of your neck, across your chest or on your head. Do not place electrodes on red or inflamed skin and skin with open wounds, on cancerous lesions or rashes (see chapter 'Contra-indications, reactions and side effects').**

### **Using the app to start and stop a treatment**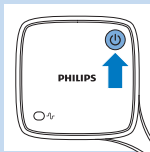

- 1** Switch on Bluetooth on your mobile device.
  - 2** Use the on/off button to switch the device to standby.
- ▶ The status indicator on the device lights up solid green to indicate that the device is in standby mode and ready for treatment.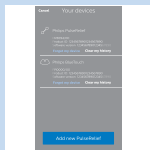


- ▶ If you do not start the treatment within 5 minutes, the device switches off automatically. If this happens, you first have to press the on/off button again to start a treatment.

**3** Open the Philips Treatment App.


**4** The app automatically selects PulseRelief if only a PulseRelief device is paired to the app. If both a PulseRelief and a BlueTouch device are paired to the app, tap 'PulseRelief' to select the device.

### Selecting a treatment area

At first-time use, you always have to select a treatment area. If you have used the device before, you can choose between 'My previous treatment' to repeat the previous treatment, or 'Select new treatment' to start a different treatment.

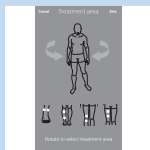
**1** Rotate the body animation on the screen.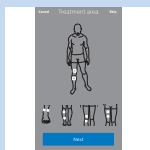


**2** Tap on one of the suggested locations for the electrodes to select the area you want to treat.

### Rating your pain intensity

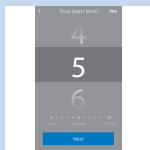
**1** Swipe vertically to rate the intensity of your pain on a scale from 0 to 10. If you experience no pain tap 0 and if you experience the worst pain ever tap 10. If your pain is somewhere in between, rate it by tapping the appropriate number.

*Note: The pain intensity you indicate is not used by the app to determine the intensity of the treatment. The rating is only stored in your treatment diary. You can switch off this feature in the settings tab of the app.*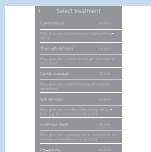


## Selecting your treatment

- 1 Scroll through the list of programs and tap a program to select it.
- 2 The start screen of the treatment appears.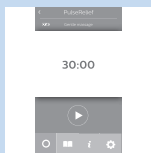


## Starting and stopping a treatment

- 1 Tap the play button on the screen.
  - ▶ The treatment screen opens and the time starts to count down.
  - ▶ The status indicator on the device starts to flash orange.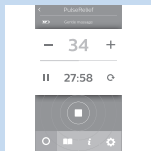

- 2 On the treatment screen, you can tap the + or the - to increase or decrease the intensity to a comfortable level.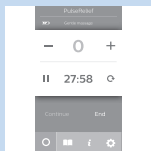

- 3 In programs with a specified maximum treatment time, treatment continues until the timer has counted down to 0. In programs that have no specified maximum treatment time, the timer counts up from 0. It continues to count up until you stop the treatment. If you want to stop or interrupt the treatment, tap the square in the centre of the pulsing treatment indicator to stop the program .
  - To resume the program, tap 'Continue' or the arrow next to the time.
  - To end the program, tap 'End treatment'.

## Switching to another program

- 1 If you want to switch to another treatment program while a program is running:
  - on a smartphone or iPod, tap the name of the program in the treatment screen.
  - on a tablet, tap 'Treatment list'
- 2 The list of programs opens. Tap the treatment program you want to use.
- 3 The start screen of the treatment appears. Tap the play button to start the treatment.
- 4 Tap the + to increase the intensity to a comfortable level.

*Note: When you switch to another program during treatment, the intensity is automatically reset to 0. This is a safety measure. The intensity level of the previous program may be too high for the new program.*

## Adapting the treatment time


- 1 Tap the time displayed in the treatment screen.
- 2 Change the remaining treatment time.

## Quick start


From the start screen, you can immediately start the program you used during your previous treatment.

- 1 From the start screen, tap the name of the treatment under 'My previous treatment'.
- 2 Swipe vertically to rate the intensity of your pain on a scale from 0 to 10.
- 3 Start the treatment.


## Other app features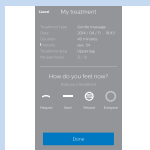


- 1 After the program finishes, you get to a screen that asks you to describe how you feel. You can indicate this by tapping one of the icons.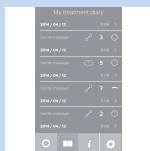

- 2 Tap the book symbol to open your diary. The diary shows when treatments took place, which program you used and how long the treatment lasted.
  - Tap the arrow to see more details.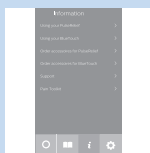


- 3 Tap the 'i' to open the information screen. This screen contains the following items:
  - Using your PulseRelief
  - Using your BlueTouch device
  - Contact us
  - Pain toolkit
  - Terms and conditions
  - Privacy policy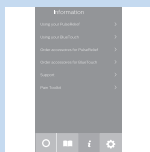

- 4 Tap the settings symbol to open the settings screen. This screen contains the following items:
  - Your devices
  - Pain level tracking
  - Sensation tracking
  - Treatment area tracking
  - Run treatment in background
  - About

## Using a second device

To treat a larger area of the body or to treat different body areas at the same time, you can pair a second PulseRelief device to the app.

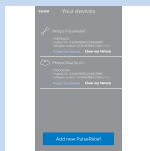
**1** Tap the settings symbol and then tap 'Your Devices'.


**2** At the bottom of the screen, tap the device you want to pair: PulseRelief.

**3** Switch on the device when the app tells you to.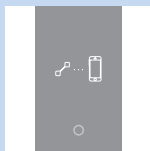

**4** The app enters pairing mode.

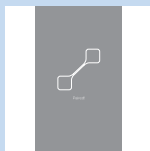
**5** When the device is paired to the app, the smart phone icon disappears and only the device is shown.

**6** Tap 'Back' to go back to the treatment screen.

## After use

Caution: Make sure that you end the treatment in the app or press the on/off button to switch off the device before you remove the device or the electrodes. If the treatment has not ended when you remove the device or the electrodes, you may experience an unpleasant sensation in your fingers when you touch the connectors. This sensation is not harmful, but can be unpleasant.

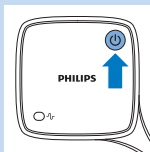- 1 Press the on/off button to switch off the device.

- 2 Detach the device from the electrodes by pulling it off.

*Note: If you forget to switch off the device, it will switch off automatically 5 minutes after you detach it.*

- 3 If you have used the device quite frequently during the day, it is advisable to recharge it before your next treatment.


- 4 Peel the electrodes off the skin by pulling at the edge.


*Note: You can leave the electrodes at the same spot as long as the treatment is effective and no skin irritation occurs underneath the electrodes. If treatment at a spot is no longer effective or skin irritation occurs, move the electrodes to a different spot.*


## Storage of the electrodes

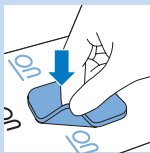- 1 Place the electrodes back on the liner.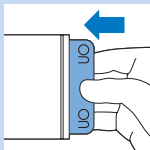

- 2 Slide the liner with the electrodes back into the bag.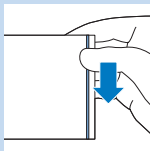

- 3 Reseal the bag for dust-free storage.

*Note: To keep the electrodes in good condition and to prevent the hydrogel layer from drying out, always store them in the resealable bag when they are not attached to your body for treatment. Seal the resealable bag to guarantee dust-free storage.*

*Note: If you want to store electrodes for more than a month, keep them at temperatures between  $+5^{\circ}\text{C}$  and  $27^{\circ}\text{C}$ . Do not store the electrodes in the freezer or the refrigerator. Avoid high temperatures and exposure to direct sunlight.*

## Cleaning

---

Do not immerse the device in water or rinse it under the tap.

Do not immerse the electrodes in water.

- 1** Clean the device with a damp cloth and a mild cleaning agent, e.g. dishwashing liquid.

*Note: You can also clean the device with 70% isopropyl alcohol (IPA).*

Caution: Do not use any other cleaning agents than the ones mentioned above, as other cleaning agents could damage the material, e.g. cause loss of gloss.

- 2** If the electrodes are dirty, put a drop of water on your finger and gently rub the dirt off the surface.

*Note: Do not use soap or alcohol to clean the electrodes.*

## Troubleshooting


This chapter summarises the most common problems you could encounter with the PulseRelief. If you are unable to solve the problem with the information below, visit **[www.philips.com/support](http://www.philips.com/support)** for a list of frequently asked questions or contact the Philips Consumer Care Centre in your country.

Problem	Possible cause	Solution
The battery indicator on the device was orange and then the device switched off.	The battery is empty.	Charge the device (see chapter 'Preparing for use').
The battery indicator on the device is solid orange.	The battery is low.	This means that the battery has 30% of its capacity left. Recharge the device as soon as possible (see chapter 'Preparing for use').
I experienced an unpleasant sensation when I touched the connectors of the device or the electrodes.	You did not end the treatment in the app or did not press the on/off button on the device before you removed the device.	Always end the treatment in the app or press the on/off button on the device before you remove the device from your skin. In this way, you avoid the unpleasant sensation when you touch the connectors of the device or touch the electrodes. This sensation is unpleasant but not harmful.
The treatment feels different or less pleasant than usual.	The intensity is too high or too low.	On the program settings screen in the app, tap + or - to increase or decrease the intensity. You can also choose a different treatment program.

Problem	Possible cause	Solution
	The electrodes are not in the right place.	Switch off the device and detach it from the electrodes. Remove the electrodes from the skin and move them to a slightly different place.
	The electrodes are not properly attached.	Switch off the device and detach it from the electrodes. Check if the electrodes are attached to the skin properly.
	The electrodes are too dirty.	You can try to clean the electrodes (see chapter 'Cleaning'). If this does not help, replace the electrodes.
	The electrodes are worn.	Replace the electrodes.
	You are using electrodes that are past their use-by date.	Remove the electrodes and use new ones that are not past their use-by date.

## Signals on the device

### Indicator signals on the device

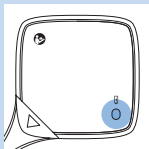#### Battery indicator

- The battery indicator shows the following signals:
  - 1 The battery indicator is solid orange when the rechargeable battery of the device is low and during charging.
  - 2 The battery indicator is solid green when the device is connected to the mains and when the rechargeable battery is fully charged.

#### Status indicator

- The status indicator on the device shows the following signals:
  - 1 The status indicator is solid green when the device is switched on and in standby mode.
  - 2 The status indicator flashes orange during treatment to indicate that the treatment is in progress.
  - 3 The status indicator flashes green when the device is pairing.

## Replacement

### Electrodes

Replace the electrodes if:

- they are damaged or torn.
- they are past the use-by date indicated on the resealable bag.
- they have lost their adhesive power. Never use plaster or tape to attach them to your skin.
- stimulation feels less strong.
- when stimulation is uncomfortable, i.e. when you experience an unpleasant stinging or biting sensation.

*Note: Always replace the electrodes with electrodes recommended for this device by the manufacturer. You can order new electrodes at [www.philips.com](http://www.philips.com).*

## Ordering accessories

To buy accessories or spare parts, visit [www.shop.philips.com/service](http://www.shop.philips.com/service) or go to your Philips dealer. You can also contact the Philips Consumer Care Centre in your country.

### Electrodes

You can order new electrodes on [www.philips.com](http://www.philips.com).

**Caution: Only use this device with the adapter, cord and electrodes recommended by the manufacturer.**

## Guarantee and support

Your device has been designed and developed with the greatest possible care to guarantee an expected service life of 5 years.

If you need service or information or if you have a problem, please visit our website at

**[www.philips.com/support](http://www.philips.com/support)**. You can also contact the Philips Consumer Care Centre in your country. Its telephone number is:

- Denmark: 35 44 41 30 (Monday to Friday 09:00 - 18:00)
- Finland: 09 88 62 50 41 (Monday - Friday 09:00 - 18:00)
- Norway: 22 97 19 50 (Monday to Friday 09:00 - 18:00)
- Portugal: 800 780 903 (free of charge, Monday - Friday: 09:00 - 18:00)
- Spain: 91 349 65 80 (Monday - Friday: 08:00 - 20:00; Saturday: 09:00 - 18:00)
- Sweden: 08-5792 9067 (Monday to Friday 09:00 - 18:00)
- United Kingdom: 0844 338 04 89 (5p/min. from a BT landline, other landline and mobile providers may charge more).

### Guarantee restrictions

The terms of the guarantee do not cover the electrodes, as the electrodes are subject to wear and have to be replaced regularly.

### Manufacturer's legal address

Philips Consumer Lifestyle B.V.

Tussendiepen 4, 9206 AD Drachten, Netherlands

## Recycling

- This symbol means that this product shall not be disposed of with normal household waste (2012/19/EU).

- This symbol means that this product contains a built-in rechargeable battery which shall not be disposed of with normal household waste (2006/66/EC). Follow the instructions in section 'Removing the rechargeable battery' to remove the battery.
- Follow your country's rules for the separate collection of electrical and electronic products and rechargeable batteries. Correct disposal helps prevent negative consequences for the environment and human health.
- All plastic parts are marked with recycle symbols.

### Removing the rechargeable batteries

*Note: Always remove the rechargeable batteries before you hand in the device at an official collection point. If you have trouble removing the batteries, you can also take the device to a Philips service centre, which will remove the battery pack for you and will dispose of it in an environmentally safe way.*

- 1** Make sure the rechargeable batteries are empty. Switch on the device as often as is necessary until you are no longer able to switch it on because the batteries have run out completely.

- 2 Insert the tip of a flat-head screwdriver between upper and lower shell at the micro-USB socket.

*Note: This action destroys the device and must only be executed if you intend to discard the device.*

- 3 Turn the screwdriver to separate the two shells.

*Note: You may have to insert and turn the screwdriver a few times to break the two shells loose from each other.*

- 4 When the shells have come apart, use the screwdriver to remove the batteries. Dispose of the plastic parts and the batteries separately according to local rules

## Specifications

Model	PR3840
Rated voltage(V) - adapter	100-240 ~ 150mA
Rated frequency (Hz) - adapter	50-60
Rated output - adapter	5V  500 mA
Class	Medical device Class IIa
Ingress of water - device	IP22
Battery type - device	Li-Ion
Adapter	The adapter is specified as part of the medical system.

### Pulse output parameters

Frequency range	1-100 Hz
Pulse duration	60-350 $\mu$ s for programs 1 to 20
Maximum current output	60mA at 500-1000 Ohm
Maximum output voltage	60V
Current pulse shape	Biphase symmetrical (net current 0 DC)

### Operating conditions

Temperature	from +5°C to +40°C
Relative humidity	from 15% to 93% (non-condensing)
Atmospheric pressure	from 700hPa to 1060hPa


### Storage and transport conditions

Temperature - electrodes	from 0°C to +40°C
Temperature - device	from -10°C to +50°C
Relative humidity	less than 93% (non-condensing)
Atmospheric pressure	from 700hPa to 1060hPa

**Note:** If you want to store the electrodes for more than a month, keep them at temperatures between +5°C and +27°C.

**Note:** Do not store the electrodes in the freezer or refrigerator. Do not expose them to high temperatures, nor immerse them in water, or leave them outside of the plastic bag.

## Explanation of symbols

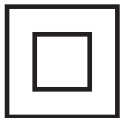### Symbols on the adapter and the device

- This symbol indicates that the adapter is double insulated (Class II), according to IEC 60601-1.

- This symbol means: Do not throw away with the normal household waste. For further instructions, see chapter 'Recycling'.

### Symbols on the device

- This is the symbol for serial number. This symbol is followed by the manufacturer's serial number.

- This is the symbol for reference number. This symbol is followed by the manufacturer's reference number, which is the type number of the device.

- This symbol on the device means: protected against access to hazardous parts with a finger and against vertically falling water drops when tilted up to 15 degrees.

- This symbol means that the part of the device that comes into physical contact with the user (also known as the applied part) is of type BF (Body Floating) according to IEC 60601-1. The applied parts are the electrodes.

- This symbol means that this device emits non-ionising radiation. All devices and systems that include RF transmitters or that intentionally apply RF electromagnetic energy must have a label with this symbol.

- This symbol means: caution TENS output. You find this symbol near all electrode connections.

- This symbol means 'Manufactured by' and appears next to the address of the legal manufacturer.

- This symbol means: Conforms to EC Directives. CE stands for 'Conformité Européenne'. 0344 is the number of the notified body.

- This symbol means: read the user manual before you start using the device.

- This symbol is the mark that identifies the standby button.

- This is the pulse symbol. It appears next to the status indicator.

- This is the battery symbol. It appears next to the battery indicator.

**Symbols on the adapter**

- This symbol means: alternating current.

- This symbol means: direct current.

- This symbol means: Conforms to EC Directives. CE stands for 'Conformité Européenne'.

- This symbol means: may only be used indoors.

- This symbol means: USB connector.

- This symbol means: micro-USB connector.

- This symbol is the certification logo that includes identification of the accredited compliance test house.

- This symbol is the certification mark for technical equipment. The GS mark is based on the German Equipment and Product Safety Act.


- This symbol shows that the no-load power CEC (California Energy Commission) Efficiency Level is V (five) in order to meet EU requirements.


### **Symbols on electrode bag**


- This symbol means: Do not use while driving, operating machines or performing other activities that may present a risk of injury.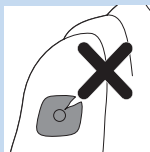


- This symbol means: Do not use damaged or worn electrodes or electrodes that have lost their adhesive power.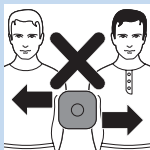


- This symbol means: The electrodes should be used by one person and not shared with others.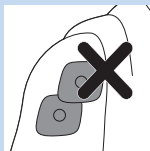

- This symbol means: Do not place the electrodes on top of each other or so close to each other that they touch each other.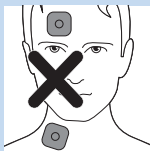

- This symbol means: Do not place electrodes on your head or on your neck.

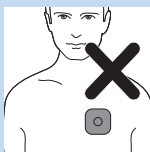
- This symbol means: Do not place the electrodes on your chest.

- This symbol means: Do not put the electrodes on open wounds, cancerous lesions, rashes and infected or inflamed skin.

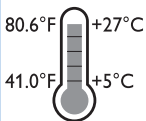- This symbol indicates the minimum and maximum storage temperature for electrodes if they are stored for more than a month.

- This is the symbol for 'use by'. It indicates after which date the electrodes must not be used anymore.


## Electromagnetic emissions and immunity

### Electromagnetic Compatibility (EMC)

The PulseRelief device is approved according to EMC safety standard EN 60601-1-2. It is designed to be used in typical domestic or clinical environments.


**Electrode placement guide****TENS**

For a description of the TENS programs (programs 1 - 15), refer to chapter 'Pre-set programs'.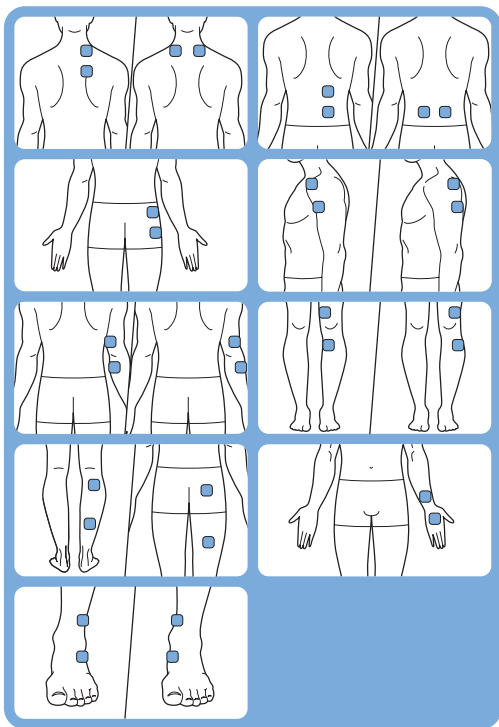

**EMS**

For a description of the EMS programs (programs 16 - 20), refer to chapter 'Pre-set programs'.

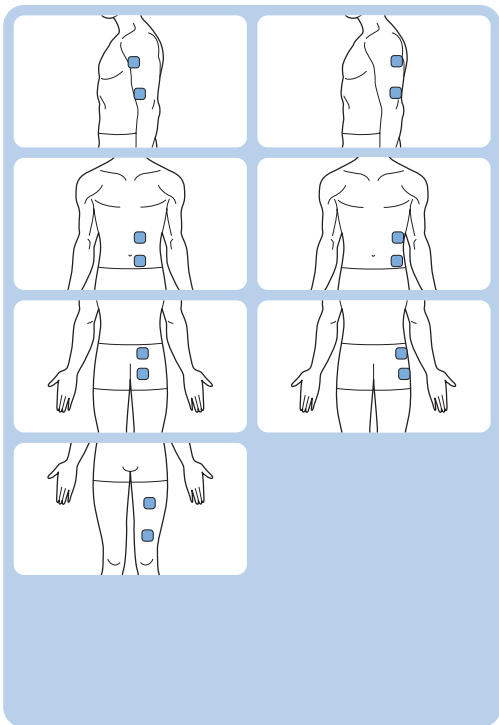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