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Register your product and get support at /
请登录以下网址并得到相应的帮助



www.philips.com/welcome

PR3092

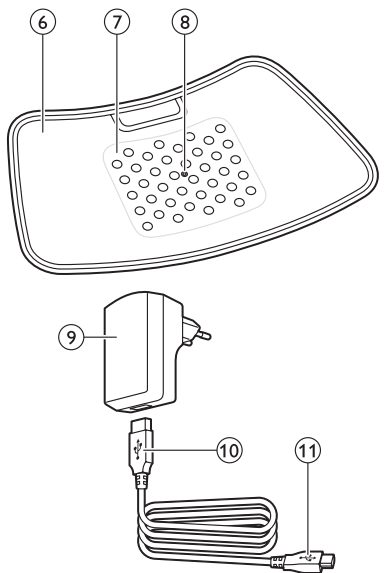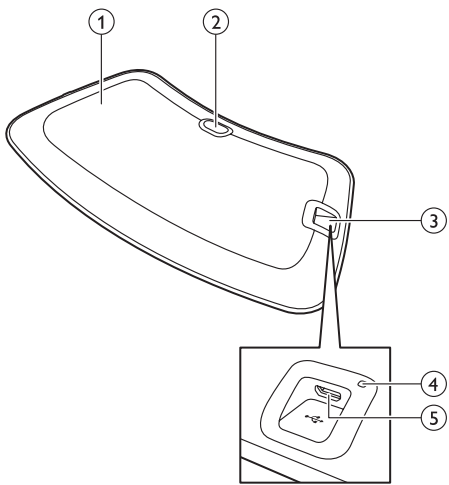PHILIPS

ENGLISH 6

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Table of contents

Introduction	6
Intended use	6
Important	7
Important safety information	7
Contra-indications	8
Compliance with standards	10
Electromagnetic emissions and immunity	10
Explanation of symbols	10
General description	14
Charging	15
Using your BlueTouch Pain Relief Patch	16
Treatment schedule	16
Treatment modes	16
Preparing the BlueTouch Pain Relief Patch for use	17
Treatment procedure	19
After treatment	20
Possible reactions and side-effects	21
Light and sound signals during use	21
Cleaning	22
Recycling	23
Guarantee and support	23
Troubleshooting	24
Specifications	25

Introduction

Read this user manual carefully before you use the BlueTouch Pain Relief Patch and save it for future reference. This medical device is available without prescription. To achieve the optimal treatment success, you have to use the device according to the treatment schedule and the instructions in this user manual. In case of any further questions, please ask your pharmacist or doctor. BlueTouch is a trademark owned by Koninklijke Philips N.V.

Intended use

The BlueTouch Pain Relief Patch is a wearable medical device that intends to treat back pain during normal daily activities by supplying light and heat to the body by means of optical power and thermal conduction. Special blue LED light relaxes and helps heal damaged muscles through improved blood circulation. The BlueTouch Pain Relief Patch is intended to be used on the upper or lower back and can be positioned and adjusted on the treatment area by means of a fabric strap. The device is powered by built-in, non-exchangeable rechargeable batteries. The BlueTouch Pain Relief Patch is intended to be only used and operated by persons older than 18. It should always be used in accordance with the safety procedures and operating instructions included in this user manual and for the purpose for which it is designed.

Important

Important safety information

Danger

- Keep the adapter dry.

Warning

- Do not immerse the BlueTouch Pain Relief Patch in water nor rinse it under the tap.
- Do not use the BlueTouch Pain Relief Patch in wet surroundings.
- The BlueTouch Pain Relief Patch is not intended for use by persons younger than 18 and persons with reduced physical, sensory or mental capabilities.
- THIS IS A MEDICAL DEVICE. KEEP THE DEVICE OUT OF THE REACH OF CHILDREN.
- Make sure children do not play with the BlueTouch Pain Relief Patch.
- Always adhere to the treatment schedule in chapter 'Using your BlueTouch Pain Relief Patch'. Do not treat more than twice a day.
- Only charge the BlueTouch Pain Relief Patch by connecting the USB cable and the adapter supplied with the device to a wall socket. Do not connect any other cable with a micro-USB plug to the micro USB socket of the BlueTouch Pain Relief Patch.
- Always check the BlueTouch Pain Relief Patch and the adapter for damage before use.
- Do not use the BlueTouch Pain Relief Patch or the adapter if either is damaged.
- If the adapter is damaged, always have it replaced with one of the original type in order to avoid a hazard. Contact Philips to obtain a new adapter.
- Do not modify the BlueTouch Pain Relief Patch and its accessories.
- Do not intentionally touch the sensor in the centre of the BlueTouch Pain Relief Patch when it is switched on.
- Do not place any object on the sensor in the centre of the BlueTouch Pain Relief Patch when it is switched on.

- Never leave the BlueTouch Pain Relief Patch unattended when it is switched on or charging.
- Always return the BlueTouch Pain Relief Patch to a service centre authorised by Philips for examination or repair. Repair by unqualified people could cause a hazardous situation for the user.
- Use and charge the BlueTouch Pain Relief Patch at a temperature between +10°C and +35°C. Store the BlueTouch Pain Relief Patch at a temperature between -20°C and +50°C.
- Do not use the device if it is closer than 30cm to wireless communication equipment, such as wireless home network routers and walkie-talkies. This wireless communication equipment may cause electromagnetic interference that prevents the device from operating properly.

Caution

- The BlueTouch Pain Relief Patch is specifically designed to treat back pain during normal daily activities, such as working on the computer, reading, watching TV, gardening and performing household tasks, for instance cooking. Do not use it for any other purpose.
- If treatment becomes uncomfortable, switch to a lower treatment mode. If treatment is still uncomfortable in low mode, switch off the BlueTouch Pain Relief Patch and stop using it.
- Never exert excessive pressure on the BlueTouch Pain Relief Patch, for instance by pressing, lying, sitting or standing on it. Do not lean heavily against the BlueTouch Pain Relief Patch when sitting.
- Use the BlueTouch Pain Relief Patch in one of the Philips straps that has been specially designed for it. The straps are not supplied with the patch and have to be obtained separately.
- Do not subject the BlueTouch Pain Relief Patch to heavy shocks or drop it.
- Do not squeeze or twist the BlueTouch Pain Relief Patch.

- Keep the BlueTouch Pain Relief Patch away from sharp, pointed or abrasive objects.
- Do not wear the BlueTouch Pain Relief Patch while it is charging.
- Do not apply any creams or lotions to the skin before treatment.
- Blue light may discolour tattoos in the treatment area.
- If you take the BlueTouch Pain Relief Patch from a very warm environment to a colder environment, wait approximately 3 hours before you use it.
- Do not look directly into the light emitted by the BlueTouch Pain Relief Patch when it is switched on.
- Do not use the BlueTouch Pain Relief Patch if you have a fever or if you have a health condition and/or use medication or cosmetics that may cause adverse skin reactions in combination with the BlueTouch Pain Relief Patch. Read section 'Contra-indications' in this chapter to find out if any apply to you before you start using the BlueTouch Pain Relief Patch.
- To disconnect the BlueTouch Pain Relief Patch from the mains, remove the adapter from the wall socket.
- The BlueTouch Pain Relief Patch is intended for non-continuous use. The exact usage time is stated in chapter 'Using your BlueTouch Pain Relief Patch', section 'Treatment modes, duration and intensity'.
- Always install the BlueTouch Pain Relief Patch according to the instructions in this user manual. Make sure wireless communication devices, such as wireless home network routers, mobile phones, cordless phones and their base stations, and walkie-talkies, are kept at least 2 metres away from the BlueTouch Pain Relief Patch, as these devices may affect its working.
- Clean the infrared sensor with a damp cloth after each treatment to ensure it continues to function properly.
- To prevent damage to the drum of the washing machine or to other laundry in the drum, wash the straps in a fine-mesh washing bag.

Contra-indications

Do not use the BlueTouch Pain Relief Patch:

- If your skin is burnt or sunburnt.
- If you have a fever.
- If you have a known sun allergy.
- If you are allergic to the surface materials of the BlueTouch Pain Relief Patch (polyurethane for the skin contact surface, stainless steel for the infrared sensor and a polyamide/elastane mix for the surface that does not touch the skin) or to the DriRelease polyester/cotton mix of the straps.
- If you have any known disease or idiopathic dermatosis such as porphyria, polymorphic light eruption, chronic actinic dermatitis, actinic prurigo or solar urticaria that causes photosensitivity.
- If you have active skin cancer; if you have a history of skin cancer or any other localised cancer in the areas to be treated, if you have pre-cancerous lesions or large moles in the areas to be treated.
- If you are taking photosensitising agents or medications, check the package insert of your medicine and never use the appliance if it is stated that it can cause photo-allergic reactions, photo-toxic reactions or if you have to avoid sun when taking this medicine.
- If you are on steroids. This medication makes the skin more susceptible to irritation.
- If you have infections, eczema, unhealed tattoos, burns, inflammation of hair follicles, open lacerations, abrasions, herpes simplex, psoriasis, wounds or lesions and haematomas in the areas to be treated.
- If you have severe or uncontrolled diabetes or diabetes with peripheral damage.
- If you have uncontrolled arterial hypertension, a history of stroke, myocardial infarction, angina pectoris, arteriosclerotic vascular disease (ASVD), or peripheral vascular disease.
- If you are pregnant or breastfeeding.
- If you have any inflammatory disease that causes pain or any other chronic disease or infection known to cause pain, such as spondyloarthritis, rheumatoid arthritis, lupus erythematosus, Lyme borreliosis.
- If you have a degenerative central nervous system disease such as multiple sclerosis or Parkinson's disease, or a spinal stenosis that contributes to or causes back pain, or cauda equina syndrome or other neurological symptoms that indicate neuropathy.
- If you have schizophrenia, borderline syndrome or severe depression.
- If you have any sensory deprivation or have been diagnosed with shingles or post-herpetic neuralgia, especially in the mid-trunk area.
- If you have any anatomical pathology or other physical limitation which would prevent you from placing the BlueTouch Pain Relief Patch successfully.
- If you have an active implanted device, such as a cardiac pacemaker; defibrillator; neurostimulator; cochlear implant, or an active drug administration device.
- If you have a history of severe osteoporosis (T score of 2.5 and a history of 1 or more bone fractures), or another severe bone disease.
- If you have had an acute dislocation or fracture within the previous 8 weeks.

- If you have had any failed back surgery or any surgery to torso, head or back within the previous 8 weeks.
- If you have widespread pain.
- If you are in poor general health.

Compliance with standards

- This BlueTouch Pain Relief Patch meets all standards relevant for electrical medical appliances and appliances using optical (LED) radiation for home use.

Electromagnetic emissions and immunity

- The Blue Touch Pain Relief Patch is approved according to EMC safety standard EN 60601-1-2. It is designed to be used in typical domestic or clinical environments.

Explanation of symbols

BlueTouch Pain Relief Patch

- This symbol means: Conforms to EC Directives. CE stands for 'Conformité Européenne'.

- This symbol means: Do not put the BlueTouch Pain Relief Patch in the washing machine.

IP22

- This symbol means that the device is protected against dripping water.

- This symbol means that an applied part that includes a patient connection intended to deliver electrical energy or an electrophysiological signal to or from the patient must be Type BF (Body Floating) applied, according to IEC 60601-1. The applied part is the device not including the charger.

- This symbol means: Do not stare into the beam of light.

- This symbol is a warning to read the enclosed documents before use and take all safety precautions required.

- This symbol means: Read the user manual before you start using the BlueTouch Pain Relief Patch.

- This symbol means: Do not clean under a running tap.

- This symbol means: Do not stare at the light source. Risk Group 1 product (IEC60601-2-57:2011).

- This symbol means: Do not throw away with the normal household waste. For further instructions, see chapter 'Environment'.

- This symbol indicates the areas where the BlueTouch Pain Relief Patch may be positioned.

- This symbol means 'Manufactured by' and provides the address of the legal manufacturer.

Adapter

- This symbol indicates that the adapter is double insulated (Class II) according to IEC 60601-1.

- This symbol indicates that the adapter may only be used indoors.

- This symbol means: Do not throw away with the normal household waste. For further instructions, see chapter 'Environment'.

Straps

- This symbol means: Wash at a temperature of 30°C max.

- This symbol means: Do not bleach.

- This symbol means: Do not tumble-dry.

- This symbol means: Do not iron.

- This symbol means: Do not dry clean.

- This symbol means that the material has been tested for harmful substances according to Oeko-Tex® standard 100.

- This symbol means that this strap is to be used on the lower back.

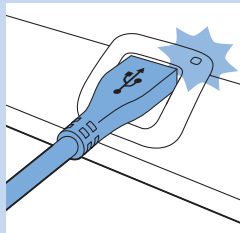
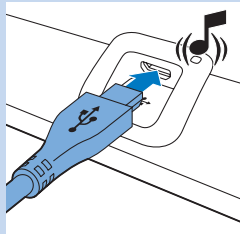
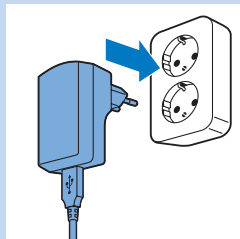
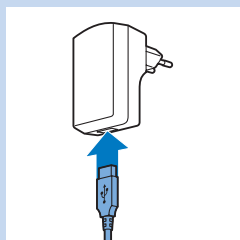
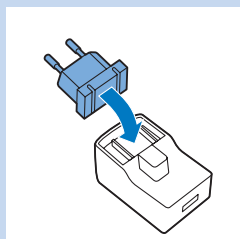
- This symbol means that this strap is to be used on the upper back.

General description (Fig. 23)

- 1 BlueTouch Pain Relief Patch (outside)
- 2 Mode button
- 3 Connection panel
- 4 Charging light
- 5 Micro USB socket
- 6 BlueTouch Pain Relief Patch (skin side)
- 7 Treatment area
- 8 Infrared sensor
- 9 Adapter
- 10 Standard USB plug
- 11 Micro USB plug


Charging

Fully charge the BlueTouch Pain Relief Patch for the first time and when its power is low. When fully charged, the BlueTouch Pain Relief Patch has enough power for 4 or more treatments. You can charge the BlueTouch Pain Relief Patch with the adapter and USB cable supplied. It takes about 3 hours to fully charge the BlueTouch Pain Relief Patch.

Note: Treatment is not possible when the BlueTouch Pain Relief Patch is charging.

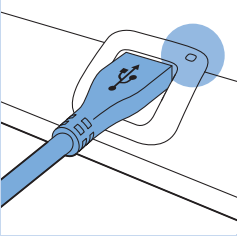- 1 The adapter comes with a removable mains plug. Before you use the adapter, first assemble the mains plug into the adapter and lock it into place with the locking slide.
- 2 Insert the standard USB plug into the socket on the adapter.
- 3 Insert the adapter into a wall socket (100V-240V).
- 4 Insert the micro USB plug into the socket on the BlueTouch Pain Relief Patch.
 - ▶ When the BlueTouch Pain Relief Patch is connected to the mains, it beeps once to indicate that charging has started.
 - ▶ The charging light on the connection panel flashes green slowly and regularly while the BlueTouch Pain Relief Patch is charging.

- 5 When the BlueTouch Pain Relief Patch is fully charged, the charging light turns solid green.
- 6 Remove the adapter from the wall socket and pull the USB plug out of the micro USB socket on the BlueTouch Pain Relief Patch.

Using your BlueTouch Pain Relief Patch

Treatment schedule

Always stick to the treatment schedule in section 'Treatment modes, duration and intensity' in this chapter. Do not treat more than twice a day.

The BlueTouch Pain Relief Patch has been designed for daily use.

You can use it when you feel pain. It is important, however, that you stick to the treatment schedule in section 'Treatment modes, duration and intensity' in this chapter and do not treat more often than twice a day. If you follow this treatment schedule, you do not exceed the maximum daily light dose of 90 J/cm².

Treatment options

The following treatment options are possible:

- two treatments of the lower back per day
- two treatments of the upper back per day
- one treatment of the lower back and one treatment of the upper back per day

Note: You can use the BlueTouch Relief patch twice in a row for two treatments of 15 minutes, 20 minutes or 30 minutes with a short break. To start the second treatment, press the mode button to switch on the device again.

Treatment modes

Treatment modes, duration and intensity

Modes	No. of treatments per day and treatment duration	Intensity
1: low	2 x 30 min.	50%
2: medium	2 x 20 min.	75%
3: high	2 x 15 min.	100%

Selecting the treatment modes

The BlueTouch Pain Relief Patch has three modes and an OFF setting. You have to press the mode button one or more times to select one of the three treatment modes. When you press the mode button a fourth time, you switch off the BlueTouch Pain Relief Patch.

- 1 Mode 1 (low): press once
- 2 Mode 2 (medium): press twice
- 3 Mode 3 (high): press three times
- 4 ○ (OFF): press four times

Note: If you want to switch from mode 3 to mode 1 or 2 or from mode 2 to mode 1, you have to press the mode button as many times as required to reach the mode you want to select.

Note: If you select another mode by switching through the OFF setting within 2 seconds, the treatment timer is not reset to the treatment of the newly selected mode but continues to count down at the point where you switched to the other mode. This means that the remaining treatment time is recalculated for the intensity and the treatment time of the newly selected mode.

For example, if you switch from mode 3 to mode 1 and the remaining treatment time in mode 3 was 5 minutes, the remaining treatment time in mode 1 is 10 minutes. In mode 1 the intensity is 50% of that in mode 3, which means that the treatment in mode 1 lasts twice as long as in mode 3.

Treatment intensity

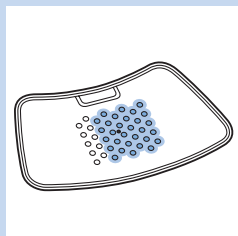
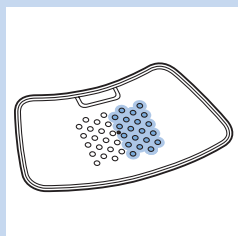
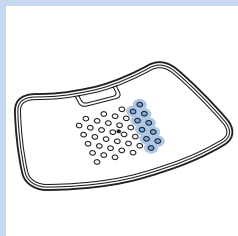
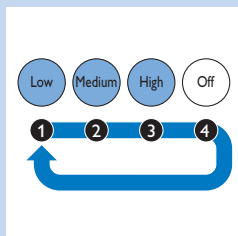
When you select a treatment mode, you get audible and visible feedback.

- ▶ In mode 1 (low), two rows of LEDs light up. The BlueTouch Pain Relief Patch beeps once.
- ▶ In mode 2 (medium), four rows of LEDs light up. The BlueTouch Pain Relief Patch beeps twice.
- ▶ In mode 3 (high), six rows of LEDs light up. The BlueTouch Pain Relief Patch beeps three times.
- ▶ After you select a mode, the BlueTouch Pain Relief Patch switches on in power-save mode. In this mode all LEDs are on at their lowest intensity.
- ▶ When you place the BlueTouch Pain Relief Patch on the skin, treatment starts at the intensity of the mode selected (100% for mode 3, 75% for mode 2 and 50% for mode 1) and you get audible feedback.

Preparing the BlueTouch Pain Relief Patch for use

The preferred method for fastening the Pain Relief Patch to the body is by means of one of the BlueTouch Pain Relief straps. There is a special strap for treatment of the upper back and a special strap for treatment of the lower back.

- Lower back straps are available separately under type number PR3083, depending on your size.


- Upper back straps are available separately under type number PR3081, depending on your size.

Note: Do not wear multiple layers of clothing or tight or thick clothing over the BlueTouch Pain Relief Patch to prevent the device from switching off before the treatment has been completed. If you wear too tight or thick clothing, the infrared sensor will measure a too high temperature and switch off the device.


Positioning the lower back strap

To place the BlueTouch Pain Relief Patch in the strap, follow the steps below.

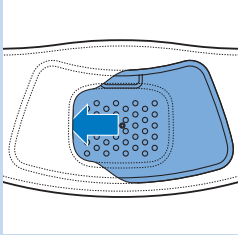- 1 Put the lower back strap and the BlueTouch Pain Relief Patch in front of you on a table. Slide the left side of the BlueTouch Pain Relief Patch behind the elastic edge of the opening in the strap.

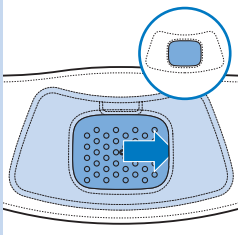
- 2 Then slide the other end of the BlueTouch Pain Relief Patch behind the opposite elastic edge of the opening so that the whole patch is inside the lower back strap. Check to make sure that the LEDs are not covered by the material of the strap. If necessary, shift the BlueTouch Pain Relief Patch a little.

- 3 Remove clothing from the area to be treated.

Note: Make sure the skin in the area to be treated is clean, entirely dry and free from creams, lotions and other cosmetics.

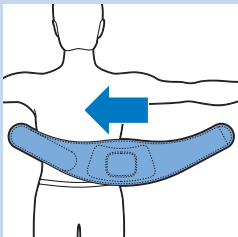- 4 Wrap the lower back strap with the BlueTouch Pain Relief Patch in it around your waist with the opening of the strap facing your back.



- 5 Fasten the lower back strap with the Velcro strips on both ends of the lower back strap.

Note: Make sure the BlueTouch Pain Relief Patch in the strap is located on the area to be treated and the LED area is in contact with the skin.

Positioning the upper back strap

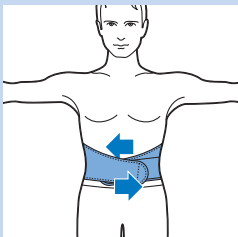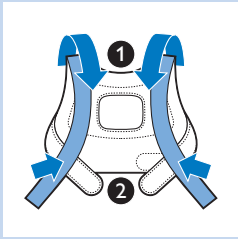- 1 To put the BlueTouch Pain Relief Patch in the upper back strap, follow the same procedure as described in steps 1 and 2 in section 'Positioning the lower back strap' above.


2 Pull the shoulder slings forward (1) and fasten them to the Velcro strips on the waistband (2).


3 Remove clothing from the area to be treated.

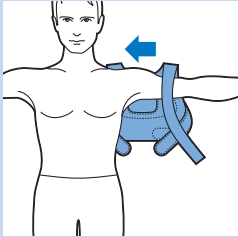Note: Make sure the skin in the area to be treated is clean, entirely dry and free from creams, lotions and other cosmetics.


4 Put the upper back strap on like a jacket. First put one arm through the shoulder sling attached to the waistband on one side and then do the same on the other side.

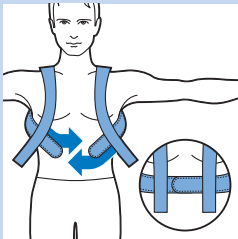
Note: Make sure the BlueTouch Pain Relief Patch in the strap is located on the area to be treated and the LED area is in contact with the skin.

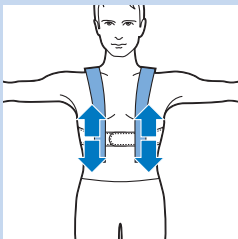5 Fasten the waistband with the Velcro strips.

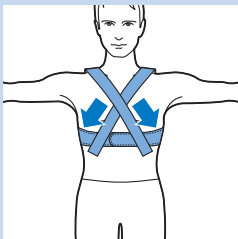6 Adjust the shoulder slings to the preferred length.

7 Alternatively, you can first put the upper back strap on your back, fasten the waistband and then attach the Velcro strips of the shoulder slings crosswise to the waistband. This may provide a better fit for some individuals.

Treatment procedure

- 1 Press the mode button one or more times to select one of the three treatment modes.
- 2 After you have selected the treatment mode (see section 'Selecting the treatment modes' in this chapter), the BlueTouch Pain Relief Patch beeps and all LEDs come on in power-save mode.
- 3 Treatment starts after the BlueTouch Pain Relief Patch has been placed on the skin and the sensor of the patch senses a temperature of at least 33°C for at least 5 seconds.
 - ▶ All LEDs light up at the intensity of the selected mode (see section 'Treatment modes, duration and intensity' in this chapter).

Do not intentionally touch the sensor in the centre of the BlueTouch Pain Relief Patch when it is switched on.

Note: You can also place the patch on your back and then press the mode button to select the treatment mode. In that case, the BlueTouch Pain Relief Patch switches on in the selected mode within five seconds.

- 4 The BlueTouch Pain Relief Patch automatically switches off after the treatment time in the selected mode has elapsed. For the treatment times, see section 'Treatment modes, duration and intensity' in this chapter.

Interrupting treatment

- 1 If you want to interrupt the treatment briefly, simply remove the Pain Relief Patch from the skin.
 - ▶ The LEDs of the BlueTouch Pain Relief Patch automatically switch back to power-save mode.

Caution: Do not look directly into the light emitted by the LEDs, even when the BlueTouch Pain Relief Patch operates in power-save mode and the treatment time is paused.

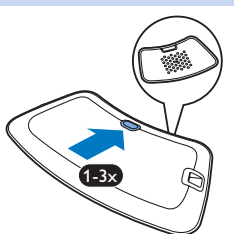
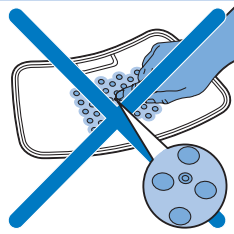
Note: After 10 minutes in power-save mode, the BlueTouch Pain Relief Patch switches off.

- 2 To resume the treatment after a brief interruption, place the BlueTouch Pain Relief Patch back onto the skin.
 - ▶ The LEDs automatically come on at the intensity selected and the treatment time continues to count down.

Switching off

- 1 If you want to switch off the Pain Relief Patch before the treatment time has elapsed, press the mode button one or more times until the LEDs switch off.
 - ▶ A sequence of two different tones indicates that the BlueTouch Pain Relief Patch switches off.

Note: The number of times you have to press the mode button until the BlueTouch Pain Relief Patch switches off depends on the mode you selected.

After treatment

Treatment ends automatically after the treatment time in the selected mode has elapsed, unless you remove the BlueTouch Pain Relief Patch and press the mode button to stop treatment earlier.

Note: A sequence of two different tones indicates that the BlueTouch Pain Relief Patch switches off and treatment has ended.

- 1** Loosen the strap and remove it from your back.
- 2** Remove the BlueTouch Pain Relief Patch from the strap.
- 3** Recharge the BlueTouch Pain Relief Patch if necessary (see section 'Battery level signals').

Possible reactions and side-effects

- It is normal for the skin to become a little red during treatment. This skin redness is caused by increased blood circulation due to the warmth of the BlueTouch Pain Relief Patch. This skin redness is harmless and fades after the treatment has ended.
- You may sometimes notice tanning after treatment. The cause of this tan is the exposure to light - a similar effect to that caused by sunlight - or to warmth. This effect is harmless and fades away by itself after a number of weeks and sometimes a few months. Postpone treatment until the tan has disappeared. To prevent tanning, resume treatment in a lower mode and avoid excessive pressure on the BlueTouch Pain Relief Patch during treatment, for example by sitting or lying on it or by leaning heavily against it.

Light and sound signals during use

Overheating signal

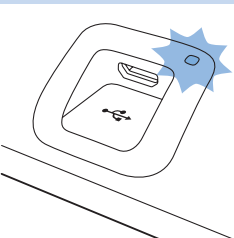
- 1** If the sensor measures a too high temperature, the BlueTouch Pain Relief Patch switches off for safety reasons.
- 2** If you try to restart the BlueTouch Pain Relief Patch within the standard cooling-down time of 10 minutes after switching off, the charging light flashes red three times.
- 3** After the standard cooling-down time of 10 minutes, the BlueTouch Pain Relief Patch can be switched on again and the timer starts counting down where it stopped.

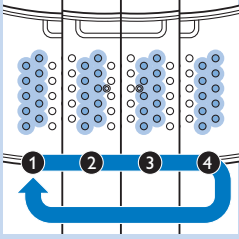
Note: If treatment is not resumed within 2 hours, the timer is automatically reset.

Resetting signal

When the software of the BlueTouch Pain Relief Patch no longer responds or another unexpected failure occurs, you can reset the BlueTouch Pain Relief Patch.

- 1** Connect the BlueTouch Pain Relief Patch to the mains with the adapter.


2 Press the mode button for 5 seconds.

- ▶ First the first two rows of LEDs light up briefly. When they go out, the third and fourth row light up briefly. When the third and fourth row go out, the fifth and sixth row light up briefly. When the fifth and sixth row go out, the seventh and eighth row light up briefly.
- ▶ After this the BlueTouch Pain Relief Patch is reset and works normally again.


Battery level signals

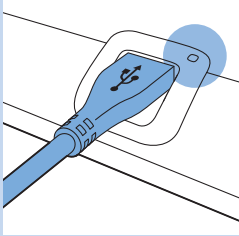When you press the mode button, the charging light shows the battery status. The charging light goes out when the sensor senses skin contact or when the device is switched off.

- 1 When the batteries contain enough energy for two or more treatments, the charging light is solid green.
 - ▶ **You can select a mode and start treatment.**
- 2 When the batteries contain just enough energy for one treatment, the charging light is solid orange.
 - ▶ **You can select a mode and complete just one treatment.**
- 3 When the batteries are empty, the charging light turns solid red for 5 seconds and the BlueTouch Pain Relief Patch produces 3 beeps when you press the mode button.
 - ▶ **You cannot select any mode. You have to charge the batteries before you can start a treatment.**

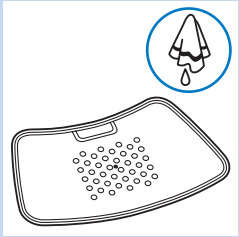Cleaning


Do not immerse the BlueTouch Pain Relief Patch in water or rinse it under the tap.

Always remove the BlueTouch Pain Relief Patch from the strap before you put the strap in the washing machine to clean it.


1 Clean the BlueTouch Pain Relief Patch with a damp cloth.

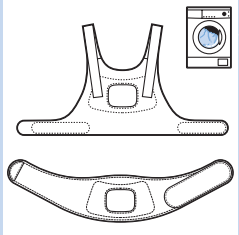Caution: Clean the infrared sensor with a damp cloth after each treatment to ensure it continues to function properly.

2 The BlueTouch Pain Relief straps can be washed in the washing machine.

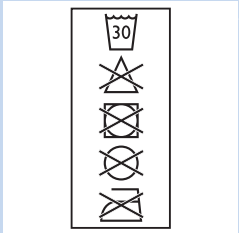Caution: To prevent damage to the BlueTouch Pain Relief straps and other clothes in the washing machine, wash the straps in a fine-mesh washing bag.

Note: Use a gentle cycle and a temperature of 30°C maximum. Dry the BlueTouch Pain Relief straps on the air and not in the tumble dryer. Do not iron the straps.

Recycling

- This symbol on a product means that the product is covered by European Directive 2012/19/EU.

- This symbol means that the product contains a built-in rechargeable battery covered by European Directive 2006/66/EC which cannot be disposed of with normal household waste. Follow the instructions in section 'Removing the rechargeable battery' to remove the battery.
- Inform yourself about the local separate collection system for electrical and electronic products and rechargeable batteries. Follow local rules and never dispose of the product and rechargeable batteries with normal household waste. Correct disposal of old products and rechargeable batteries helps prevent negative consequences for the environment and human health.

Removing the rechargeable battery

Note: Always remove the rechargeable battery before you hand in the device at an official collection point. If you have trouble removing the battery, you can also take the device to a Philips service centre, which will remove the battery for you and will dispose of it in an environmentally safe way.

- 1** Make sure the rechargeable battery is empty. Switch on the device and leave it on until the charging light turns solid red and the device switches off automatically. You may have to repeat this procedure several times until it is no longer possible to switch on the device.
- 2** Use a screwdriver to separate the upper part of the device from the lower part.
- 3** Remove the battery. Dispose of the plastic parts and the battery separately according to local rules

Guarantee and support

Your device has been designed and developed with the greatest possible care to guarantee an expected service life of 5 years.

If you need service or information or if you have a problem, please visit our website at www.philips.de or www.philips.ch, or contact the Philips Consumer Care Centre. Its telephone number is:

- Germany: 0800 0007522 (free of charge for calls within Germany).
- Austria: 0800 880932 (free of charge for landline calls within Austria; costs for mobile phone calls depend on provider).
- Switzerland: 0800 002050 (for landline calls, free of charge); 0848 000292 (for mobile phone calls, costs depend on provider).

Guarantee restrictions

The guarantee does not cover standard wear and tear. The guarantee becomes invalid if the BlueTouch Pain Relief Patch is not used according to the instructions in this user manual.

Manufacturer's legal address

Philips Consumer Lifestyle B.V.
Tussendiepen 4 9206 AD Drachten
The Netherlands

Troubleshooting

This chapter summarises the most common problems you could encounter with the BlueTouch Pain Relief Patch. If you are unable to solve the problem with the information below, contact Philips.

Problem	Possible cause	Solution
The BlueTouch Pain Relief Patch suddenly switches off during treatment.	The preset treatment time is finished.	The BlueTouch Pain Relief Patch has been programmed to switch off automatically after the treatment time in the selected treatment mode has elapsed.
	The infrared sensor is blocked.	Check the position of the patch in the strap. Make sure the strap does not cover the infrared sensor. If the infrared sensor is not covered but the BlueTouch Pain Relief Patch still switches off, it is probably broken. In that case, contact Philips.
The BlueTouch Pain Relief Patch switched off and the charging light flashes red.	The skin temperature is too high.	Check if you do not wear too many or too thick clothes over the BlueTouch Pain Relief Patch during treatment.
	The skin temperature is too high. If the infrared sensor senses a too high temperature, the BlueTouch Pain Relief Patch switches off.	Let the BlueTouch Pain Relief Patch cool down for 10 minutes. After 10 minutes, the LEDs go on in power-save mode. Put the BlueTouch Pain Relief Patch back on your skin and continue your treatment.

Problem	Possible cause	Solution
The charging light turns solid red for 5 seconds, the BlueTouch Pain Relief Patch beeps and it is not possible to select a mode.	The rechargeable batteries are empty.	Charge the BlueTouch Pain Relief Patch (see chapter 'Charging').
The charging light is solid orange.	The batteries contain just enough energy for one treatment.	Complete the treatment and charge BlueTouch Pain Relief Patch after use (see chapter 'Charging').
The adapter becomes warm during charging.	This is normal.	No action required.
The charging light does not start flashing green when the micro USB plug is inserted into the micro USB socket on the connection panel.	The adapter is not inserted or not inserted properly into the wall socket.	Insert the adapter into the wall socket properly.
	The standard USB plug is not inserted or not inserted properly into the adapter.	Insert the standard USB plug into the adapter properly.
	The wall socket into which the adapter is not live.	Connect another appliance to the same socket to check if the socket is live. If the wall socket is live, but the BlueTouch Pain Relief Patch does not charge, contact Philips.
The BlueTouch Pain Relief Patch becomes warm during use.	This is normal.	No action required.

Specifications

Model	PR3092
Rated voltage	100-240V
Rated frequency	50-60 Hz
Rated input	15 W
Class	Medical device Class IIa
Classification	Risk Group 1 (according to IEC60601-2-57)
Ingress of water	IP22
Max. daily dose	< 90 J/cm ² if treatment schedule is followed
Nominal wavelength	453 nm
Average irradiance	21 mW/cm ² (nominal for program 3)
Peak/average irradiance ratio	2.3 (nominal for program 3)
Maximum radiant power	1.4 W (nominal for program 3)
Battery type	Li-Ion
Adapter	The adapter is specified as part of the device.

Operation conditions

Temperature	from +10°C to +35°C
Relative humidity	from 15% to 93%

Storage conditions

Temperature	from -20°C to +50°C
Relative humidity	up to 93% (no condensation)

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