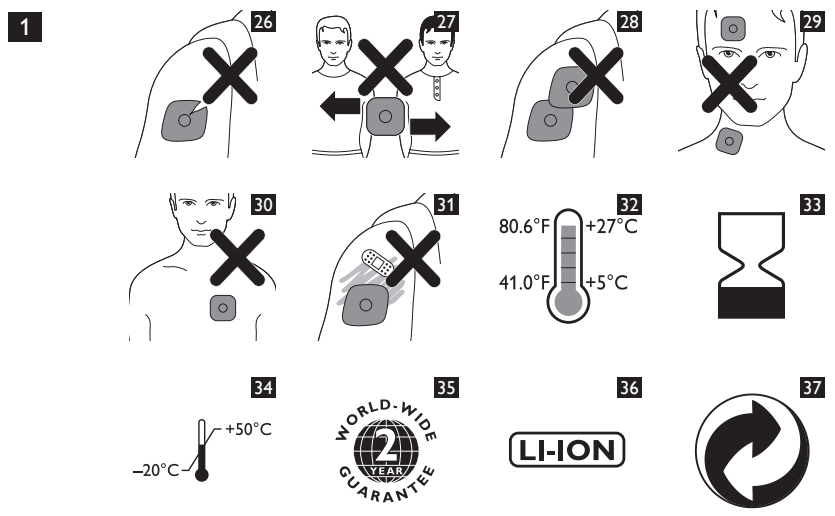
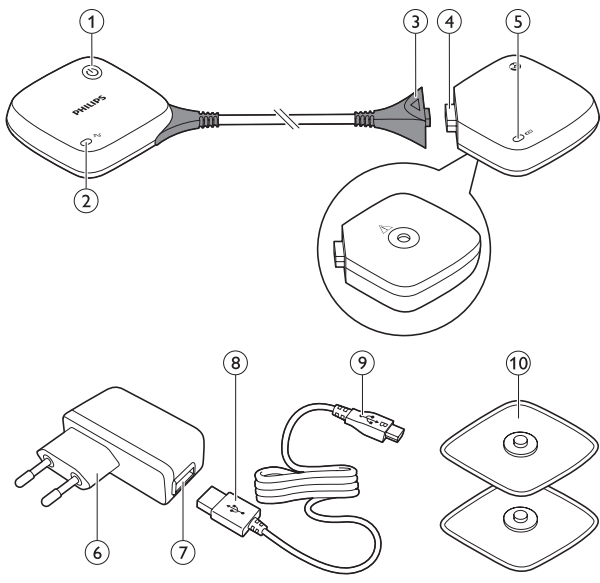
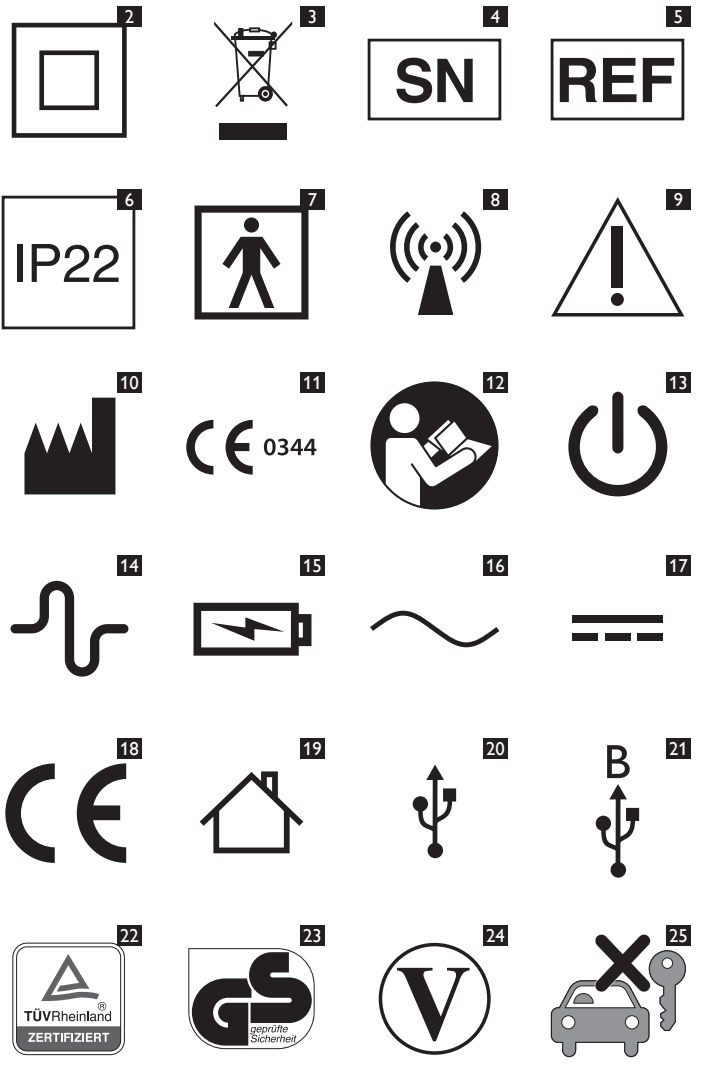
**PHILIPS**

## Introduction

This booklet contains important information on the PulseRelief device. The complete user manual for this device is included in the app and can also be downloaded from [www.philips.com/support](http://www.philips.com/support).

## Intended use

The PulseRelief is intended to be used by adult consumers who experience mild to moderate chronic pain or acute post-surgical pain. The purpose of this device is to provide pain relief through transcutaneous electrical nerve stimulation (TENS). You can also use it for electrical muscle stimulation (EMS). The device is suitable for home use.

## Contra-indications, reactions and side effects

### Contra-indications

Do not use the device if:

- you have a cardiac pacemaker, implanted defibrillator or other implanted metallic or electronic device (e.g. a drug delivery system), as use in these cases could cause electric shock, burns, electrical interference, or even death.
- you have a heart disease.
- you have epilepsy.
- you are in the first trimester of your pregnancy, as the effects of TENS on the development of the foetus are as yet unknown. Avoid use over the uterus during pregnancy to reduce the risk of inducing labour. If you are pregnant, always consult your doctor or midwife before use.
- you have a cognitive impairment.
- you have venous or arterial thrombosis or thrombophlebitis. If you use the device for EMS, there is a risk of mobilising the blood clot.
- you have undiagnosed pain, unless your doctor recommends use of this device.

### Reactions

Stop using the device and consult your doctor if you experience adverse reactions from the device.

Possible adverse reactions may include the following:

- skin irritation beneath the electrodes, although the gel used on the electrodes is not known to cause allergic reactions
- burns beneath the electrodes
- headaches or other painful sensations
- nausea or feeling faint

### Side effects

- After EMS, some muscle soreness may occur some time after the treatment.
- Some reddening of the skin under or around the electrodes may occur during and shortly after EMS treatment. Skin reddening usually disappears within two hours after treatment. If the skin continues to be red for more than a day, please consult your doctor.

## Important safety information

### Warnings

#### General warnings

- Read this user manual carefully and always adhere to the treatment instructions.
- This is a medical device. Keep the device out of the reach of children.
- This appliance is not intended for use by adults with reduced sensory or mental capabilities.
- Do not use the device on children.
- Do not modify the device or the electrodes.
- Only use the adapter supplied to charge the device. Use of another adapter may cause a hazardous situation.
- This device is designed for use by and on a single adult person.

#### Do not apply to or treat the following parts or areas

- Stimulation on the sides of the neck over the carotid sinus may have severe adverse effects on your heart rhythm or blood pressure.
- Stimulation on the front of the neck may cause severe muscle spasms that could close your airway and result in breathing difficulties.
- Stimulation across the chest may increase the risk of ventricular fibrillation, which is an irregular heart rhythm that may be lethal.
- Stimulation across the head or on opposite sides of the head must be avoided, as the effects on the brain are unknown.
- Do not place the electrodes on skin that is not intact, clean or healthy, over or near cancerous lesions or on areas of skin that lack normal sensation.

- Do not place the electrodes over open wounds or rashes, or over swollen, red, infected or inflamed areas or skin eruptions (e.g. phlebitis, thrombophlebitis and varicose veins).
- Do not place the electrodes inside body cavities, e.g. the mouth.

#### Do not use the device in the following conditions

- Do not use the device if you are in the care of a doctor and have not consulted your doctor first.
- Do not use the device if you have a tendency to bleed internally due to any impact or injury.
- As TENS stimulation may have a negative effect on the healing process, do not use the device if you had surgery recently.
- Do not use EMS stimulation to contract a muscle if contraction of the muscle may disrupt healing. For instance, if the muscle or tendon is torn, muscle contraction may exacerbate the tear; just like a voluntary contraction. This may also be the case after a recent surgical procedure or following acute trauma or fracture. Muscle contraction in case of tendinitis may also worsen symptoms.
- Do not use the device while driving, operating machines or performing any other activity in which electrical stimulation can put you at risk of injury.
- Do not use if you are likely to fall asleep during the treatment. If you use the device at bedtime, set the timer to make it switch off automatically.
- Do not use while in the bath, shower or swimming pool.
- Do not apply stimulation in the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), which may not operate properly when the electrical stimulation device is in use.
- Do not use the device when you are connected to high-frequency surgical equipment. This may result in burns on the skin under the electrodes and may damage the device.
- Do not use the device within less than 1 metre from shortwave or microwave medical equipment. Close proximity to this equipment may cause unstable device output.
- Do not use the device if it is closer than 50cm to wireless communication equipment, such as wireless home network routers, mobile phones, cordless phones and their base stations, and walkie-talkies. This wireless communication equipment may cause electromagnetic interference that prevents the device from operating properly.

### Cautions

- Only use this device with the adapter, cord and accessories recommended by the manufacturer.
- Only use the micro-USB plug to connect the device to the mains for charging. Do not connect any other cable with a micro-USB plug to the socket of the device.
- Always check the device and the electrodes for damage before use. Do not use the device or an electrode if it is damaged.
- The device is not waterproof. Do not use the device in wet surroundings and prevent it from getting wet.
- Although you can use the device indoors and outdoors, it does not withstand all weather conditions.
- The device does not withstand high and low temperatures. Please check the operating conditions in chapter 'Specifications'.
- The electrodes have a limited shelf life. Please check the packaging for the use-by date prior to use. Do not use electrodes whose use-by date has expired. For ordering replacement electrodes, see chapter 'Ordering accessories'.
- Do not use plaster or tape to attach the electrodes to the skin.
- Use Burst TENS programs while resting, as these programs can result in muscle contraction in the treatment area.
- Make sure that you end the treatment in the Philips Treatment App or press the on/off button on the device before you remove the device or the electrodes. If you do not end the treatment, you may experience an unpleasant sensation in your fingers when you touch the magnets. This sensation is not harmful, but can be unpleasant.
- Always use and store the electrodes according to the instructions in chapters 'Using the device' and 'After use'.
- If the device does not function as described here, stop using it and contact the Consumer Care Centre. For contact details, see chapter 'Guarantee and support'.

### Compliance with standards

- The device meets the relevant standards for this type of Class IIa electrical medical appliance and appliances using electrical stimulation for home use.
- This Philips appliance complies with all applicable standards and regulations regarding exposure to electromagnetic fields.

## Product overview (Fig. 1)

- 1 On/off button
- 2 Status indicator
- 3 Detachable connector
- 4 Micro-USB socket of device
- 5 Battery indicator
- 6 Adapter
- 7 Socket for standard USB plug
- 8 Standard USB plug
- 9 Micro-USB plug
- 10 Self-adhesive hydrogel electrodes

## Replacement

### Electrodes

Replace the electrodes if:

- they are damaged or torn.
- they are past the use-by date indicated on the resealable bag.
- they have lost their adhesive power. Never use plaster or tape to attach them to your skin.
- stimulation feels less strong.
- when stimulation is uncomfortable, i.e. when you experience an unpleasant stinging or biting sensation.

**Note:** Always replace the electrodes with electrodes recommended for this device by the manufacturer. You can order new electrodes at [www.philips.com/tens-electrodes](http://www.philips.com/tens-electrodes).

## Guarantee and support

Your device has been designed and developed with the greatest possible care to guarantee an expected service life of 5 years.

If you need service or information or if you have a problem, please visit our website at [www.philips.com/support](http://www.philips.com/support). You can also contact the Philips Consumer Care Centre in your country. You can reach this centre at telephone number +971 6 5178727.

### Guarantee restrictions

The terms of the guarantee do not cover the electrodes, as the electrodes are subject to wear and have to be replaced regularly.

### Manufacturer's legal address

Philips Consumer Lifestyle B.V.  
Tussendiepen 4, 9206 AD Drachten, Netherlands

## Recycling

Do not throw away the appliance with the normal household waste at the end of its life, but hand it in at an official collection point for recycling. By doing this, you help to preserve the environment.

## Specifications

Model	PR3840
Rated voltage(V) - adapter	100-240 ~ 150mA
Rated frequency (Hz) - adapter	50-60
Rated output - adapter	5V  500 mA
Class	Medical device Class IIa
Ingress of water - device	IP22
Battery type - device	Li-Ion
Adapter	The adapter is specified as part of the device.

### Pulse output parameters

Frequency range	1-100 Hz
Pulse duration	60-350 $\mu$ s for programs 1 to 20
Maximum current output	60mA at 500-1000 Ohm
Maximum output voltage	60V
Current pulse shape	Biphase symmetrical (net current 0 DC)

### Operating conditions

Temperature	from +5°C to +40°C
Relative humidity	from 15% to 93% (non-condensing)
Atmospheric pressure	from 700hPa to 1060hPa

## Storage and transport conditions

Temperature - electrodes	from 0°C to +40°C
Temperature - device	from -10°C to +50°C
Relative humidity	less than 93% (non-condensing)
Atmospheric pressure	from 700hPa to 1060hPa

**Note:** If you want to store the electrodes for more than a month, keep them at temperatures between +5°C and +27°C.

**Note:** Do not store the electrodes in the freezer or refrigerator. Do not expose them to high temperatures, nor immerse them in water, or leave them outside of the plastic bag.

## Explanation of symbols

### Symbols on the adapter and the device

- This symbol indicates that the adapter is double insulated (Class II), according to IEC 60601-1 (Fig. 2).
- This symbol means: Do not throw away with the normal household waste. For further instructions, see chapter 'Recycling' (Fig. 3).

### Symbols on the device

- This is the symbol for serial number. This symbol is followed by the manufacturer's serial number (Fig. 4).
- This is the symbol for reference number. This symbol is followed by the manufacturer's reference number, which is the type number of the device (Fig. 5).
- This symbol on the device means: protected against access to hazardous parts with a finger and against vertically falling water drops when tilted up to 15 degrees (Fig. 6).
- This symbol means that the part of the device that comes into physical contact with the user (also known as the applied part) is of type BF (Body Floating) according to IEC 60601-1. The applied parts are the electrodes (Fig. 7).
- This symbol means that this device emits non-ionising radiation. All devices and systems that include RF transmitters or that intentionally apply RF electromagnetic energy must have a label with this symbol (Fig. 8).
- This symbol means: caution TENS output. You find this symbol near all electrode connections (Fig. 9).
- This symbol means 'Manufactured by' and appears next to the address of the legal manufacturer (Fig. 10).
- This symbol means: Conforms to EC Directives. CE stands for 'Conformité Européenne'. 0344 is the number of the notified body (Fig. 11).
- This symbol means: read the user manual before you start using the device (Fig. 12).
- This symbol is the mark that identifies the standby button (Fig. 13).
- This is the pulse symbol. It appears next to the status indicator (Fig. 14).
- This is the battery symbol. It appears next to the battery indicator (Fig. 15).

### Symbols on the adapter

- This symbol means: alternating current (Fig. 16).
- This symbol means: direct current (Fig. 17).
- This symbol means: Conforms to EC Directives. CE stands for 'Conformité Européenne' (Fig. 18).
- This symbol means: may only be used indoors (Fig. 19).
- This symbol means: USB connector (Fig. 20).
- This symbol means: micro-USB connector (Fig. 21).
- This symbol is the certification logo that includes identification of the accredited compliance test house. (Fig. 22)
- This symbol is the certification mark for technical equipment. The GS mark is based on the German Equipment and Product Safety Act (Fig. 23).
- This symbol shows that the no-load power CEC (California Energy Commission) Efficiency Level is V (five) in order to meet EU requirements (Fig. 24).

### Symbols on electrode bag

- This symbol means: Do not use while driving, operating machines or performing other activities that may present a risk of injury (Fig. 25).
- This symbol means: Do not use damaged or worn electrodes or electrodes that have lost their adhesive power (Fig. 26).
- This symbol means: The electrodes should be used by one person and not shared with others (Fig. 27).
- This symbol means: Do not place the electrodes on top of each other or so close to each other that they touch each other (Fig. 28).
- This symbol means: Do not place electrodes on your head or on your neck (Fig. 29).
- This symbol means: Do not place the electrodes on your chest (Fig. 30).
- This symbol means: Do not put the electrodes on open wounds, cancerous lesions, rashes and infected or inflamed skin (Fig. 31).

- This symbol indicates the minimum and maximum storage temperature for electrodes if they are stored for more than a month (Fig. 32).
- This is the symbol for 'use by'. It indicates after which date the electrodes must not be used anymore (Fig. 33).

### Symbols on the package

- This symbol means that the package and its contents can be stored at temperatures between -10°C and +50°C (Fig. 34).
- This symbol means that the 2-year worldwide guarantee applies to this device (Fig. 35).
- This symbol means that the device contains a Li-Ion battery (Fig. 36).
- This symbol refers to the Green Dot scheme, which is covered under the European Packaging and Packaging Waste Directive (94/62/EC). Under this scheme, industry has to pay for recycling the packaging materials of consumer goods (Fig. 37)

## Electromagnetic emissions and immunity

### Electromagnetic Compatibility (EMC)

The PulseRelief device is approved according to EMC safety standard EN 60601-1-2. It is designed to be used in typical domestic or clinical environments.

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