

PHILIPS

AVENT

2-in-1 Thermo pad

Warm to stimulate flow; Cool to soothe

Warm or cold use

2 pcs

SCF258/02



Helps to stimulate and soothe

To stimulate breast milk flow before feed

The Philips Avent SCF258/02 helps to stimulate and soothe.

Other benefits

- Developed with breastfeeding expert
- Includes soft sleeves

Soothe and stimulate

- Cold use - to soothe sore or engorged breasts
- Warm use - to stimulate milk flow before breast feed

Highlights

Cold use



To soothe sore or engorged breasts. Keep in fridge or freezer. Always use with sleeves.

Developed with expert


Developed in collaboration with a midwife and breastfeeding advisor who has been supporting mums with breastfeeding for 20 years.

Includes soft sleeves



For greater comfort on skin

Warm use

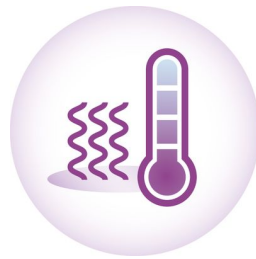

To stimulate milk flow before breast feeding.
Lightly dampen the sleeves, as moist heat gives an extra comfortable feel and helps stimulation.
Heat the pad in hot water for 10 minutes.

Specifications

What is included

2-in-1 Thermopad: 2 pcs

Country of origin

China

Dimensions & weight

Dimensions: 175x175x40 mm

Design

Breast shaped design

Soft feel slave: Comfort on skin

Design by breastfeeding expert

Functions

Cold use: Soothe sore breasts

Warm use: Stimulate milk flow

