PHILIPS

AVVENT


Double
Electric Breast
Pump

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




Plus

SCF338/11
SCF373/11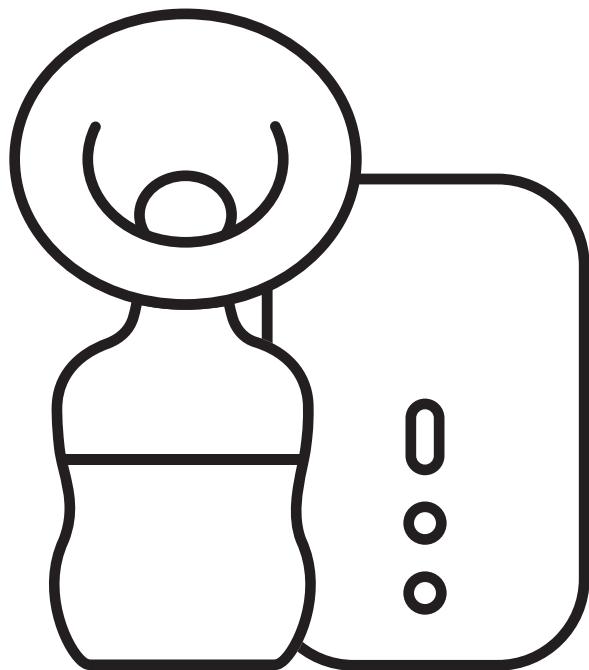

www.philips.com/mybreastpump

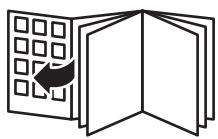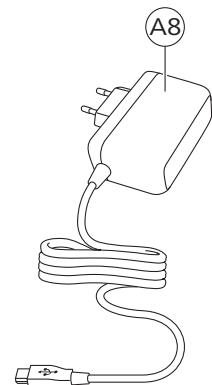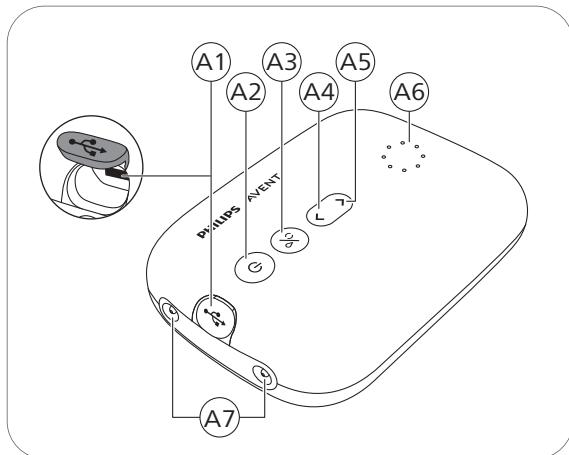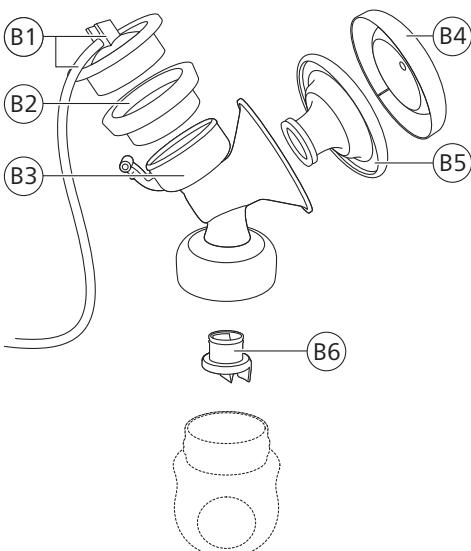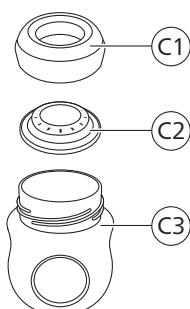A**B****C**

English 6
Deutsch 34
Français 64
Italiano 93
Nederlands 121
Polski 149

Contents

User manual for mother	7
Introduction	7
General description	7
Intended use and indications for use	8
Important safety information	8
Before first use – instruction for mother	10
Cleaning and disinfecting	11
Using the breast pump	14
Using the breast pump when you have symptoms of engorgement	18
After use	19
Compatibility	20
Ordering accessories and spare parts	20
Recycling	20
Support	21
Troubleshooting	21
Undesirable side effects	22
Storage conditions	23
Electromagnetic fields (EMF)	23
Technical information	23
Explanation of symbols	26
User manual for rental agent	29
Introduction	29
General description	29
Important safety information	30
Preparing for rental use	30
Step 1: Register and track	30
Step 2: Instructions for pump return	30
Step 3: Quality checklist	31
Step 4: Factory reset	32
Step 5: Cleaning before renting	32
Step 6: Prepare for the next rental use	33

User manual for mother

Introduction

Thank you for choosing to rent this Philips Avent breast pump. The Philips Avent double electric breast pump Plus is inspired by babies and their natural suction movement. The soft silicone cushion stimulates the nipple with gentle massaging movements, helping you to express milk comfortably and effectively. The combination of suction and nipple stimulation triggers fast milk flow. The no-lean-forward design enables you to sit in a relaxed, upright position while expressing.

Healthcare professionals and the World Health Organization recommend to exclusively feed children with breast milk during the first six months of life and to continue giving them breast milk for two years next to offering additional nutritious foods. This is because your breast milk is specially adapted to your baby's needs and helps to protect your baby against infection and allergies. To breastfeed longer, you can express and store your milk so that your baby can still enjoy the benefits of it, even if you cannot be there to provide it yourself. As the pump is compact, quiet and discreet to use, you can take it with you anywhere, allowing you to express milk at your own convenience and to maintain your milk supply.

If breastfeeding goes well, it is advisable (unless recommended otherwise by your healthcare professional) to wait until your milk supply and breastfeeding schedule are established (normally at least 2 to 4 weeks after giving birth) before you start expressing.

General description

i Note: The numbers mentioned below refer to the numbers in the figure on the front foldout page of this user manual.

Rental items (Fig. A) – motor unit and adapter (to be returned)

- A1 USB power inlet
- A2 On/off button with pause/play function
- A3 Mode selection button
- A4 'Level down' button
- A5 'Level up' button
- A6 Indicator lights for modes and levels
- A7 Tubing ports for double breast pump
- A8 Adapter*

* The adapter supplied may vary for different countries.

Breast pump kit (Fig. B) (for personal use)

B1 Silicone tube and cap	B4 Cover
B2 Silicone diaphragm	B5 Cushion
B3 Pump body	B6 White valve

Philips Avent Natural bottle (Fig. C) (for personal use)

C1 Screw ring
C2 Sealing disc
C3 Bottle

i Note: The Philips Avent teat and bottle cap are not parts of the rental package but they can be purchased separately. We do not recommend using the Philips Avent Natural glass bottles with the breast pump. Explore the Philips Avent range of teats and bottles at www.philips.com/avent.

Accessories (to be returned)

These accessories are not shown in the figure:

- Travel bag
- Insulation pouch

Additional item (to be returned)

This additional item is not shown in the figure:

- User manual

Intended use and indications for use

The Philips Avent double electric breast pump Plus is intended to express and collect milk from the breast of a lactating woman, and indicated to alleviate the symptoms of engorgement. The motor unit and adapter are intended for multiple users. The breast pump kit is intended for a single user.

Important safety information

Read this user manual carefully before you use the breast pump and save it for future reference.

This user manual can also be found online at: www.philips.com/support.

This product is operated by the lactating woman herself.

You can use this breast pump in an everyday environment.

Contraindication

- Never use the breast pump while you are pregnant, as pumping can induce labor.

Warning

Warnings to avoid choking, strangulation and injury:

- Do not allow children or pets to play with the motor unit, the adapter, packaging materials or accessories.

- Always unplug the breast pump after use. Store the breast pump when not in use.

- This breast pump is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge. Such persons can only use this breast pump if they are supervised by or have received proper instruction concerning use of the appliance by a person responsible for their safety.

Warning to prevent transmitting infectious diseases:

- If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), expressing breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.

Warnings to avoid electric shock:

- Inspect the breast pump, including the adapter, for signs of damage before each use. Do not use the breast pump if the adapter or plug is damaged, if it does not work properly or if it has been dropped or submerged in water.
- Only use the adapter (S009AHZ050yyyy) provided with this breast pump.
- Always ensure that it is easy to disconnect the device from the mains in case of malfunction. The adapter is considered the disconnection device.

Warnings to avoid contamination and to ensure hygiene:

- For hygienic reasons, the breast pump kit is only intended for repeated use by a single user.
- Multiple-user breast pumps are designed in such a way that the breast milk can never touch the working parts of the breast pump that are shared. The only parts of a multiple-user breast pump that you can safely share are the motor unit and the adapter.
- Clean and disinfect all parts that come into contact with the breast and breast milk before first use and after each use.
- Do not use antibacterial or abrasive cleaning agents to clean the breast pump parts as this may cause damage.
- Wash your hands thoroughly with soap and water before you touch breast pump parts and breasts to prevent contamination. Avoid touching the inside of containers or lids.

Warnings to avoid cross-contamination:

- Breast pump kits are personal care items intended for use by a single user only and should not be shared between users.
- Clean the motor unit before renting out the device to the next user.

Warnings to avoid breast and nipple problems including pain:

- Do not use the breast pump if the silicone diaphragm appears to be damaged or broken. See 'Ordering accessories and spare parts' for information on how to obtain replacement parts.
- Only use accessories and parts recommended by Philips Avent.
- There are no user serviceable parts inside the motor unit of the breast pump.
- No modification of the breast pump is allowed. If you do so, your warranty becomes invalid.
- Never use the breast pump while you are sleepy or drowsy to avoid lack of attention during use.
- Always switch off the breast pump before you remove the pump body from your breast, in order to release the vacuum.
- If the vacuum is uncomfortable or causes pain, switch off and remove the breast pump from your breast.

10 English

- Do not continue pumping for more than five minutes at a time if you do not succeed in expressing any milk. This can lead to pain and damage to your milk ducts. Try to express at another time during the day.
- If the process becomes very uncomfortable or painful, stop using the breast pump and consult your healthcare professional.

Warnings to prevent damage and malfunction of the breast pump:

- Portable radio frequency (RF) communications equipment (including mobile phones and peripherals such as antenna cables and external antennas) should not be used closer than 30 cm (12 in) to any part of the breast pump, including adapter. This could have a negative effect on the performance of the breast pump.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation.

Warning to avoid increased electromagnetic emissions or decreased electromagnetic immunity of this device, resulting in improper operation:

- Do not use any other adapter or charging cable than supplied with the device.

⚠ Caution

Cautions to prevent damage to and malfunctioning of the breast pump:

- Prevent the adapter and motor unit from coming into contact with water.
- Never put the motor unit or the adapter in water or in a dishwasher, as this causes permanent damage to these parts.
- Keep the adapter and the silicone tubes away from heated surfaces to avoid overheating and deformation of these parts.

Before first use – instruction for mother

Disassemble, clean and disinfect all parts that come into contact with the breast and breast milk before you use the breast pump for the first time and after each use.

Cleaning and disinfecting


Overview

Clean and disinfect the parts that come into contact with the breast and breast milk as described below:

	When	How
	Clean and disinfect before first use and after every use.	Disassemble all parts and then clean them as described in chapter 'Cleaning' and disinfect them as described in chapter 'Disinfecting'.
		

(i) Note: The Philips Avent teat and bottle cap are not parts of the rental package but they can be purchased separately.

The following parts do not come into contact with the breast and breast milk; clean them as described below:

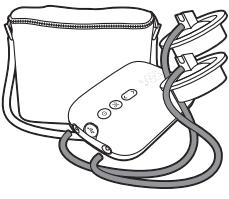	When	How
	Clean when needed.	Wipe with a clean, damp cloth with water and mild detergent.

Step 1: Disassembling

Disassemble the breast pump kit, bottle and storage cup completely.

Also remove the white valve from the pump body (Fig. B3).

Step 2: Cleaning

The parts that come into contact with the breast and breast milk can be cleaned manually or in the dishwasher.

⚠ Warning: Do not use antibacterial or abrasive cleaning agents to clean the breast pump parts as this may cause damage.

⚠ Caution: Never put the motor unit or the adapter in water or in a dishwasher, as this causes permanent damage to these parts.

⚠ Caution: Be careful when you remove and clean the white valve. If it gets damaged, your breast pump does not function properly. To remove the white valve, gently pull at the ribbed tab on the side of the valve. To clean the white valve, rub it gently between your fingers in warm water with some dishwashing liquid. Do not insert objects into the white valve, as this may cause damage.

Step 2A: Manual cleaning

Supplies needed:

- Mild dishwashing liquid
- Drinking-quality water
- Soft, clean brush

- Clean tea towel or drying rack
- Clean sink or bowl


1. Rinse all parts under a running tap with lukewarm water.

2. Soak all parts for 5 minutes in warm water with some mild dishwashing liquid.

3. Clean all parts with a cleaning brush in warm water with some mild dishwashing liquid.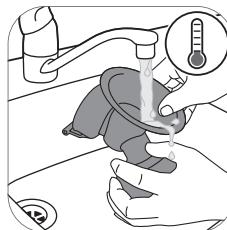

4. Thoroughly rinse all parts under a running tap with cold clear water.

5. Leave all parts to air-dry on a clean tea towel or drying rack.

Step 2B: Cleaning in the dishwasher

Supplies needed:

- Mild dishwashing liquid or a dishwashing tablet
- Drinking-quality water

ⓘ Note: Food colorings may discolor parts.

1. Place all parts on the top rack of the dishwasher.

2. Put dishwashing liquid or a tablet in the machine and run a standard program.

3. Leave all parts to air-dry on a clean tea towel or drying rack.

Step 3: Disinfecting


Supplies needed:

- A household pot
- Drinking-quality water

⚠ Caution: During disinfection with boiling water, prevent the bottle or other parts from touching the sides of the pot. This can cause irreversible product deformation or damage that Philips cannot be held liable for.

1. Fill a household pot with enough water to cover all parts and place all parts in the pot. Let the water boil for 5 minutes. Make sure the parts do not touch the sides of the pot.

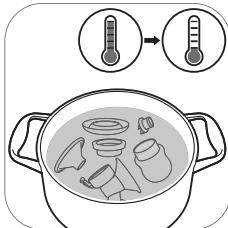2. Allow the water to cool down.

3. Gently remove the parts from the water. Place the parts on a clean surface/tea towel or drying rack to air-dry.

4. Store the dry items safely in a clean, protected area until needed. Ensure that the clean parts have air-dried completely before storing them to help prevent germs and mold from growing.

Using the breast pump

Cushion size

The Philips Avent double electric breast pump Plus has a soft, active cushion. There is only one cushion size. It gently stimulates your nipple to trigger milk flow. The cushion is made of flexible silicone, fitting nipple sizes up to 30 mm.

Assembling the breast pump

⚠ Warning: Be careful, when you have disinfected the breast pump parts by boiling them, they can be very hot. To prevent burns, only start assembling the breast pump when disinfected parts have cooled down.

⚠ Warning: Wash your hands thoroughly with soap and water before you touch breast pump parts and breasts to prevent contamination. Avoid touching the inside of containers or lids.

Make sure you have cleaned and disinfected the parts of the breast pump that come into contact with the breast and breast milk.

Check the breast pump kit parts for wear or damage before use and replace them if necessary.

i Note: Correct placement of the white valve, cushion, silicone diaphragm and silicone tube is essential for the breast pump to form a proper vacuum.

1. Wash your hands thoroughly with soap and water.

2. Insert the white valve into the pump body from underneath. Push the white valve in the pump body as far as possible.

3. Screw the pump body onto the bottle.

4. Place the cushion in the pump body and make sure that the rim covers the pump body.

4A. Push the inner part of the cushion into the funnel against the line (indicated with an arrow).


5. Place the silicone diaphragm into the pump body.


5A. Make sure that the silicone diaphragm fits securely around the rim by pressing it down with your thumbs.

6. Attach the silicone tube and cap onto the silicone diaphragm. Push down the cap until it is securely in place.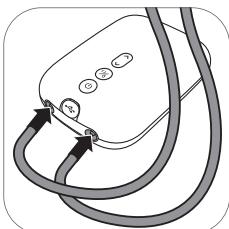

7. Connect the tube(s) to the motor unit.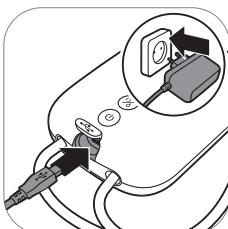

8. Insert the adapter into the wall socket and insert the small plug at the other end into the motor unit.


9. The breast pump is now ready for use.

i Note: You can place the cover over the cushion to keep the breast pump clean when you take it with you and while you are preparing for expressing.

Motor unit part description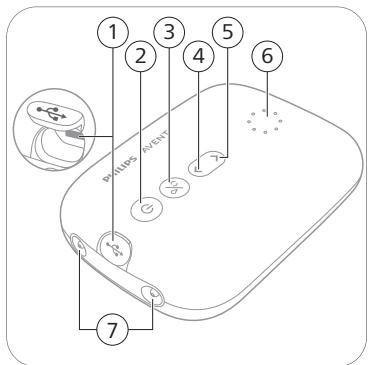

- 1 USB power inlet
- 2 On/off button with pause/play function
- 3 Mode selection button
- 4 'Level down' button
- 5 'Level up' button
- 6 Indicator lights for modes and levels
- 7 Tubing ports for double breast pump

Mode indicator lights

The breast pump has two modes. Below you can find an explanation of these modes.

Indicator lights	Mode	Explanation	No. of suction levels
• • • • • • • • •	Stimulation mode 	Mode to stimulate the breast to start up the milk flow.	8 suction levels
• • • • • •	Expression mode 	Mode for efficient milk removal after milk has started flowing. After 90 sec. of stimulation, the device automatically switches to the expression mode.	16 suction levels

i Note: When you change the suction level, the display shows the selected suction level for a few seconds and then shows the selected mode again.

Suction level indicator lights

Suction level 1: Bottom light flashes	Suction level 2: Bottom light steady on	Suction level 3: Bottom light steady on + next light flashes	Suction level 4: 2 lights steady on	Suction level 5: 2 lights steady on + next light flashes.	Suction level 16: all lights are on

i Note: Every time the level increases, the next light starts flashing or the flashing light becomes solid.

Pumping instructions

⚠ Warning: Always switch off the breast pump before you remove the pump body from your breast, in order to release the vacuum.

⚠ Warning: Do not continue pumping for more than five minutes at a time if you do not succeed in expressing any milk. This can lead to pain and damage to your milk ducts. Try to express at another time during the day.

⚠ Warning: If the process becomes very uncomfortable or painful, stop using the breast pump and consult your healthcare professional.


i Note: If you regularly express more than 125 ml/4 fl oz per session, you can purchase and use a 260 ml/9 fl oz Philips Avent Natural bottle to prevent overfilling and spillage.


1. Wash your hands thoroughly with soap and water and make sure that your breast is clean.


2. Place the breast pump kit onto your breast. Make sure that you center your nipple.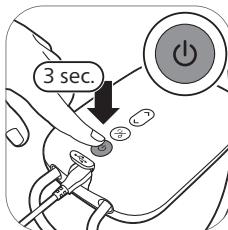


3. Press and hold the on/off button until the device switches on. The indicator lights go on to indicate that the device is in stimulation mode.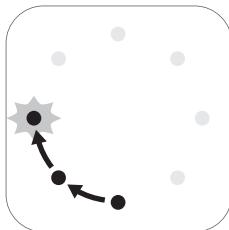


4. The breast pump starts in the stimulation mode and slowly increases its vacuum to the last used stimulation suction level (or level 5 when used for the first time).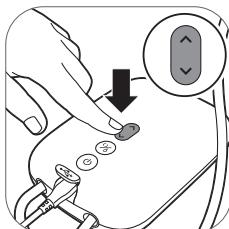


5. Use the 'level up' and 'level down' buttons to change the suction level to your preference.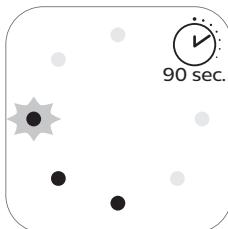


6. After 90 seconds, the breast pump will switch smoothly to expression mode and slowly increase its vacuum to the last used expression suction level (or level 11 when used for the first time).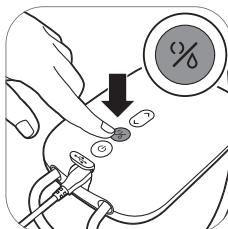


7. If your milk starts flowing earlier or if you want to switch from expression to stimulation mode, you can switch modes with the mode selection button.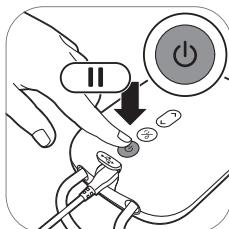

8. Briefly press the on/off button if you want to pause. To continue, briefly press the on/off button again.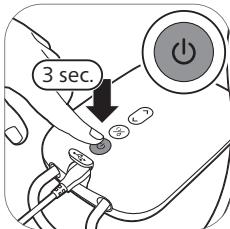

9. Press and hold the on/off button to switch off.

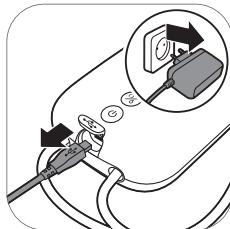10. Unplug the adapter from the wall socket and pull the small plug out of the motor unit.

Tip: For optimal expression, choose the highest setting that still feels comfortable. This setting may differ across different sessions.

Tip: You can use the double breast pump as a single breast pump by connecting only the tube of one pump kit to the motor unit.

Note: The breast pump will switch off automatically after 30 minutes in order to prevent overheating.

Using the breast pump when you have symptoms of engorgement

Breast engorgement is the swelling of the breasts, often caused by increased milk production. It occurs when the breasts become overly full and swollen, leading to discomfort. Symptoms may include hard, tight, or warm breasts, difficulty with milk ejection reflex, and redness, tenderness, or pain.

To reduce the risk of overstimulation and engorgement, follow these instructions:

- Gently massage the breasts before and during pumping to encourage milk flow.
- Use the lowest effective suction level and avoid prolonged pumping sessions. High suction levels or extended sessions may cause discomfort or potential breast tissue damage.
- Pump only the amount that your baby needs.

If you experience symptoms of engorgement, consider the following measures:

- Pumping gently for a short duration can help relieve discomfort. Pump just enough to relieve the discomfort as excessive pumping may worsen engorgement.
- If discomfort persists or if you experience redness, warmth, fever, or flu-like symptoms, consult a healthcare professional.

⚠ Warning: Overstimulation of the breasts through high suction levels or prolonged pumping may lead to discomfort or potential breast tissue damage.

⚠ Warning: Excessive or prolonged pumping may increase breast swelling, tenderness, or worsen engorgement. If you experience these symptoms, stop pumping and consult a healthcare professional.

After use


1. Unscrew the bottle.


2A. To store the breast milk: close the bottle with the sealing disc and screw ring.

3. Remove the silicone tube and cap from the breast pump kit.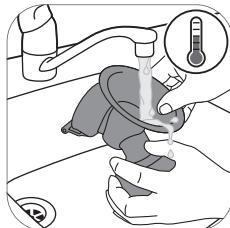

4. Disassemble, clean and disinfect all parts that have been in contact with the breast and breast milk (see 'Cleaning and disinfecting').

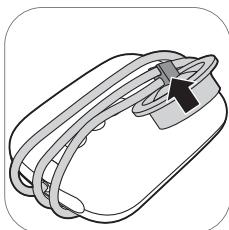5. For easy storage, wrap the silicone tube around the motor unit and clip the cap onto the tube.

⚠ Warning: Refrigerate or freeze expressed breast milk immediately or keep it at a temperature of 16-29 °C for a maximum of 4 hours before you feed your baby.

ⓘ Note: For more information about the breast pump and tips for expressing milk, visit www.philips.com/support.

Storing breast milk

Below you find guidelines for storing breast milk:

Storage location	Temperature	Max. storage duration
Room	16-29 °C	4 hours
Refrigerator	4 °C	4 days
Freezer	< -4 °C	6 months

⚠ Warning: Never refreeze thawed breast milk to avoid a decrease in milk quality.

⚠ Warning: Never add fresh breast milk to frozen breast milk to avoid a decrease in milk quality and unintended thawing of the frozen milk.

Compatibility

The Philips Avent double electric breast pump Plus is compatible with all baby bottles in the Philips Avent Natural range and Philips Avent storage cups with adapter. We do not recommend using the Philips Avent Natural glass bottles with the breast pump.

ⓘ Note: The motor unit of the Philips Avent double electric breast pump Plus (SCF373/11) can only be used in combination with the Philips Avent single/double electric breast pump kit (SCF335/12, SCF337/01, SCF338/11) or Philips Avent hands-free collection cups (SCF439/01). Do not combine it with other Philips models or different brands.

⚠ Warning: For hygienic reasons, the breast pump kit is only intended for repeated use by a single user.

Ordering accessories and spare parts

To buy accessories or spare parts, visit www.philips.com/support or go to your Philips dealer. You can also contact the Philips Consumer Care Center in your country.

When you order spare teats, make sure that you use a teat with the correct flow rate for your baby and do not mix Philips Avent Anti-colic bottle parts and teats with parts of the Philips Avent Natural bottles. They do not fit and could cause leakage or other issues.

Recycling

- This symbol means that electrical products shall not be disposed of with normal household waste.
- Follow your country's rules for the separate collection of electrical products.
- Correct disposal helps prevent negative consequences for the environment and human health.

Support

For all product support, power consumption details and the EU Declaration of Conformity, please visit www.philips.com/support.

Troubleshooting

This chapter summarizes the most common problems you could encounter with the breast pump. If you are unable to solve the problem with the information below, visit www.philips.com/support for a list of frequently asked questions or contact the Consumer Care Center in your country.

Problem	Solution
I experience pain when I use the breast pump.	<ul style="list-style-type: none"> - Choose a suction level that is comfortable for you (see 'Pumping instructions'). - Check that the silicone diaphragm is not damaged (does not have small cracks, holes etc.). - Make sure that your nipple is placed in the center of the cushion. - If the pain persists, stop using the breast pump and consult your healthcare professional.
The breast pump does not switch on (indicator lights do not go on).	<ul style="list-style-type: none"> - Check if you are using the adapter that was supplied with the device and that the adapter is connected correctly to the motor unit and wall socket. - Connect the adapter to another wall socket. Then press and hold the on/off button to check if the breast pump can be switched on. - If the problem persists, contact the Consumer Care Center via www.philips.com/support.
All indicator lights on the motor unit are flashing and device shuts off automatically.	<ul style="list-style-type: none"> - Try to switch on the motor unit again. - Check if you are using the adapter that was supplied with the device. - If the problem persists, contact the Consumer Care Center via www.philips.com/support.
I do not feel any suction/the suction level is too low.	<ul style="list-style-type: none"> - If possible, try a higher suction level. - Check if the silicone diaphragm is assembled correctly (see 'Assembling the breast pump'). Check that the silicone diaphragm is not damaged (does not have small cracks, holes etc.). - Check that the white valve is not damaged, the opening in its pointed part closes fully, and that the valve is assembled correctly (see 'Assembling the breast pump'). - Make sure that the breast pump is positioned correctly on the breast to allow a seal between breast pump and breast. - If you still do not feel any suction, please contact the Consumer Care Center via www.philips.com/support.

Problem	Solution
The suction of the breast pump is too strong.	<ul style="list-style-type: none"> - If possible, try a lower suction level. When you express for the first time, start with the default suction level (level 5 in stimulation mode, level 11 in expression mode) and increase/decrease the level when needed. During different expression sessions, different levels may feel most comfortable for you. - Make sure that you only use the breast pump parts provided with this breast pump. - Check that the silicone diaphragm is not damaged (does not have small cracks, holes etc.). - If the problem persists, contact the Consumer Care Center via www.philips.com/support.
I express little/no milk when I use my breast pump.	<ul style="list-style-type: none"> - If you do not feel enough suction, please check Troubleshooting item 'I do not feel any suction/the suction level is too low'. - If you are using the Philips Avent breast pump for the first time, you may need some practice before you are able to express any milk. More expressing tips can be found on the Philips website www.philips.com/mybreastpump.
When I switch on the breast pump after storage, it does not start working immediately.	Perhaps you stored the breast pump at a temperature below the minimum storage temperature or above the maximum storage temperature. Let the breast pump acclimatize in a room of 20 °C for 30 minutes to adjust its temperature to the operating range (5 °C to 40 °C).

Undesirable side effects

Any serious incident that has occurred in relation to this device should be reported to Philips via www.philips.com/support and to the competent authority of the Member State in which the user and/or patient is established.

When using the breast pump, the below undesirable side effects may occur. If you experience any of these symptoms, contact a healthcare professional or breastfeeding specialist.

Undesirable side effect	Description
Pain	Pain felt on the breast or nipple, or felt as a response to the suction being above a comfortable level.
Sore nipples	Persistent pain in the nipples at the beginning of the pumping session, or that lasts throughout the entire pumping session, or pain between sessions.
Engorgement	Swelling of the breast. The breast may feel hard, lumpy and tender. May include erythema (redness) of the breast area and fever. Usually occurs during the first days of lactation.
Erythema (redness)	Redness of the skin of the breast.
Bruise, thrombus	A reddish-purple discolouration that does not fade when pressed. When a bruise fades, it may become green and brown.

Undesirable side effect	Description
Nipple trauma	<ul style="list-style-type: none"> - Fissures or cracked nipples. - Skin tissue peeling off the nipple. Normally occurs in combination with cracked nipples and/or blisters. - Nipple tear. - Blister. Looks like small bubbles on the surface of the skin. - Bleeding. Cracked or torn nipples can lead to bleeding of the affected area.
Clogged mammary ducts	A red, tender lump on the breast. May include erythema (redness) of the breast area. Can lead to mastitis (breast inflammation) and fever if left untreated.
Rash or allergic reaction	Redness or irritation of the skin of the breast, it can be itchy, scaly, painful or blistered.

Storage conditions

Keep the breast pump out of direct sunlight as prolonged exposure may cause discoloration. Store the breast pump and its accessories in a safe, clean and dry place.

If the device has been stored in a hot or cold environment, place it in an environment with a temperature of 20 °C for 30 minutes to let it reach a temperature within the usage conditions (5 °C to 40 °C) before you use it.

Electromagnetic fields (EMF)

This Philips device complies with all applicable standards and regulations regarding exposure to electromagnetic fields.

Technical information

	Double electric breast pump
Vacuum level	Stimulation: -60 to -200 mbar (-45 to -150 mmHg) Expression: -60 to -360 mbar (-45 to -270 mmHg) Adjustable in steps of 20 mbar Intermittent vacuum
Cycle speed	Stimulation: 105-120 cycles/min, depending on vacuum level Expression: 53-85 cycles/min, depending on vacuum level
Motor unit rated input	5 V direct current / 1.8 A
Mains adapter rated input	100-240 V  50/60 Hz 400 mA
Mains adapter rated output	5 V direct current / 1.8 A

	Double electric breast pump
Adapter type number	S009AHZ050yyyy The letters "yyyy" represent the output current from 0100 (1000 mA) to 0180 (1800 mA), increments in steps of 100 mA. The letter "z" denotes type of plug, it can be: U, V, B, S, C, A, D, K, T, I, E, F.
Power port	Micro-USB
Operating conditions	Temperature: 5 °C to 40 °C Relative humidity: 15 to 90 % (non-condensing)
Storage and transportation conditions	Temperature: -25 °C to 70 °C Relative humidity: up to 90 % (non-condensing)
Operating and storage pressure	700-1060 hPa (<3000 m altitude)
Net weight	310 g
External dimensions	Motor unit: 145 mm x 95 mm x 45 mm (LxWxH)
Device classification	Applicable classifications: Degree of protection against electric shock: - Class II ME equipment - Type BF applied part The breast pump and the adapter are specified as medical electrical system according to IEC 60601-1.
Materials	ABS, silicone (motor unit) Polypropylene, silicone (other parts)
Service life	500 hours
Mode of operation	Continuous operation
Automatic power-off	After 30 minutes (after last user interaction)
Ingress protection	IP22 (motor unit), IP20 (adapter)

Electromagnetic compatibility - compliance information

This device has been tested and found to comply with IEC 60601-1-2:2014 Ed 4.0 for electromagnetic compatibility (EMC) as Class B according to CISPR 11:2009. Tests levels are listed in the accompanying tables. Only use this Philips Avent double electric breast pump Plus in combination with the adapter supplied.

Emissions	Compliance	Electromagnetic environment – guidance
RF emissions	Group 1	The device does not use RF energy for its intended function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
CISPR 11		

Emissions	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Class B	The device is suitable for use in locations in residential environments and in establishments directly connected to a low voltage power supply network which supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Complies Class A Category	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration – electromagnetic immunity

The Philips Avent double electric breast pump Plus is intended for use in the electromagnetic environment listed below. The user of the device should ensure that it is used in such an environment to prevent improper operation.

Immunity test	Immunity test level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	Floors can be a source of static electricity. When the air is very dry (<20 % RH) the user is advised to prefer to use the device in a room with concrete or wooden floor (instead of using it in a room with a synthetic floor).
Radiated RF EM fields IEC 61000-4-3	10 V/m 80 MHz – 2.7 GHz 80 % AM at 1 kHz	
Proximity fields from RF wireless communications equipment IEC 61000-4-3	385 (27), 450 (28), 710 (9), 745 (9), 780 (9), 810 (28), 870 (28), 930 (28), 1720 (28), 1845 (28), 1970 (28), 2450 (28), 5240 (9), 5500 (9), 5785 (9) MHz (V/m)	The device is immune against any RF wireless communication device such as smart phone, WiFi or Bluetooth device at a distance of 30 cm or more. Avoid a closer distance as this may result in improper operation.
Rated power frequency magnetic fields IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	
Electrical fast transient/burst IEC 61000-4-4	±2 kV 100 kHz repetition frequency	Mains power quality expected of a typical commercial or residential environment.
Surges IEC 61000-4-5	±1 kV Line-to-line ±2 kV Line-to-ground	Mains power quality expected of a typical commercial or residential environment.

Immunity test	Immunity test level	Electromagnetic environment - guidance
Conducted disturbances induced by RF fields IEC 61000-4-6	3 Vrms, before modulation is applied 0.15 MHz – 80 MHz 6 Vrms, before modulation is applied, in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80 % AM at 1 kHz	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles single phase at 0° 0 % UT; 250/300 cycle	Mains power quality expected of a typical commercial or residential environment. At power interruptions the breast pump will switch off.

Note: UT is the AC mains voltage prior to application of the test level. If in the unlikely event the Philips Avent double electric breast pump Plus is locked inoperative in a function by disturbances from other devices beyond the levels stated in the table above, the device will shut-off completely and restart. In that case make sure the distance to the other device causing this disturbance is enlarged.

Explanation of symbols

The warning signs and symbols are essential to ensure that you use this device safely and correctly and to protect you and others from injury. Below you find the meaning of the warning signs and symbols on the label and in the user manual.

Symbol	Explanation
	Indicates to follow the instructions for use.
	Indicates important information such as warnings and cautions.
	Indicates usage tips, additional information or a note.
	Indicates the manufacturer.
	Indicates the manufacturing date.
	Indicates the compliance with the general safety and performance requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

Symbol	Explanation
MD	Indicates that the device is a medical device.
	Indicates that the part of the appliance that comes into physical contact with the user (also known as the applied part) is of type BF (Body Floating) according to IEC 60601-1. The applied part is the breast pump kit.
	Indicates 'Class II Equipment'. The adapter is double-insulated (Class II).
REF	Indicates manufacturer's catalog number of the breast pump.
	Indicates 'direct current'.
	Indicates 'alternating current'.
	Indicates that electrical products shall not be disposed of with normal household waste.
	Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
IP22	Indicates the protection against ingress of solid foreign objects greater than 12.5 mm and against harmful effects due to the ingress of dripping water when tilted at 15 degrees.
IP20	Indicates the protection against ingress of solid foreign objects greater than 12.5 mm and no protection against ingress of water.
LOT	Indicates the batch number of the medical device.
	Indicates USB.
	Indicates the relative humidity upper limits to which the device can be safely exposed: up to 90 %.
	Indicates the storage and transportation temperature limits to which the device can be safely exposed: -25 °C to 70 °C.
	Indicates Forest Stewardship Council. The FSC trademarks enable consumers to choose products that support forest conservation, offer social benefits, and enable the market to provide an incentive for better forest management.
	Indicates 2-year Philips worldwide guarantee.

28 English

Symbol	Explanation
	Indicates that the breast pump must be kept out of direct sunlight.
	Indicates that the breast pump must be kept dry.
	Indicates the operating pressure range: 700 to 1060 hPa.
	Indicates that the material used in the product is safe for food contact.
	Identifies the UDI carrier, including the AIDC and human readable information.
	Indicates that the material used for the packaging is corrugated board.
	Indicates that the material used for the packaging is non-corrugated cardboard/fiberboard.
	Indicates that the material used for the plastic bag is low density polyethylene.
	Indicates that the part is certified with six level energy efficiency rating.
	Indicates that the product should not be used after the period accompanying the symbol. The symbol indicates the expiry date of x years from the manufacturing date (FAB).

User manual for rental agent

Introduction

Thank you for choosing to rent out this Philips Avent double electric breast pump Plus to your customers. These instructions describe how you can prepare the breast pump for rental use. For all warnings and general specifications of the pump, please refer to section 'User manual for mother' or refer to your Philips sales agent.

(i) Note: The motor unit of the Philips Avent double electric breast pump Plus (SCF373/11) can only be used in combination with the Philips Avent single/double electric breast pump kit (SCF335/12, SCF337/01, SCF338/11) or Philips Avent hands-free collection cups (SCF439/01). Do not combine it with other Philips models or different brands.

⚠ Warning: For hygienic reasons, the breast pump kit is only intended for repeated use by a single user.

General description

(i) Note: The numbers mentioned below refer to the numbers in the figure on the front foldout page of this user manual.

Rental items (Fig. A) – motor unit and adapter (to be returned)

- A1 USB power inlet
- A2 On/off button with pause/play function
- A3 Mode selection button
- A4 'Level down' button
- A5 'Level up' button
- A6 Indicator lights for modes and levels
- A7 Tubing ports for double breast pump
- A8 Adapter*

* The adapter supplied may vary for different countries.

Breast pump kit (Fig. B) (for personal use)

- | | |
|--------------------------|----------------|
| B1 Silicone tube and cap | B4 Cover |
| B2 Silicone diaphragm | B5 Cushion |
| B3 Pump body | B6 White valve |

Philips Avent Natural bottle (Fig. C) (for personal use)

C1 Screw ring

C2 Sealing disc

C3 Bottle

Accessories (to be returned)

These accessories are not shown in the figure:

- Travel bag
- Insulation pouch

Additional item (to be returned)

This additional item is not shown in the figure:

- User manual

Important safety information

Read the important safety information (see 'User manual for mother') before you perform the quality checklist with the breast pump.

Preparing for rental use

Follow the 6 steps below to successfully prepare the Philips Avent double electric breast pump Plus for rental use.

Step 1: Register and track

Before renting out the breast pump for the first time, please visit www.philips.com/welcome to register the product.

Step 2: Instructions for pump return

(i) Note: Before unpacking returned items and before performing the checks on returned items, put on personal protective equipment (disposable gloves).


2A Clean the **motor unit** (the buttons and overall surface). You can do this by using alcohol wipes (isopropanol).

2B Clean the **adapter**. You can do this by using alcohol wipes (isopropanol).

2C Check if the customer returned **only**:

- Motor unit
- Adapter
- Travel bag
- Insulation pouch
- User manual

2D It is important to check whether the customer has not accidentally also returned other items, including the breast pump kit which is for personal use only. Please dispose of all other items in case they are returned to you.

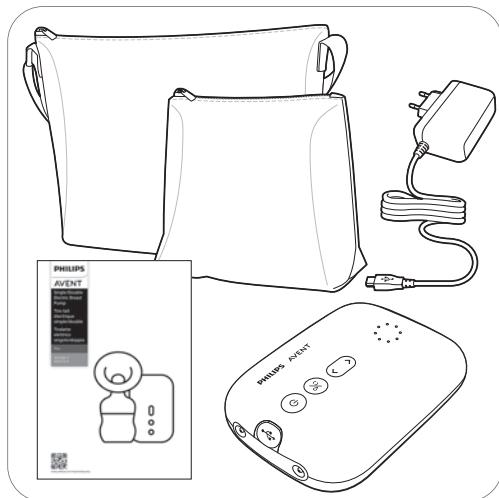Keep**Throw away**

Step 3: Quality checklist

Follow steps 3A to 3I to inspect the returned items for any damage or malfunction. In case of damage or malfunction, please contact your Philips sales agent.

- 3A**
 - Check whether the returned items include a motor unit.
 - Also check whether the correct adapter (labeled with type number S0009AHZ050yyyy) has been returned.

- 3B** Check whether any damage or malfunction is visible from the outside.

- 3C** Gently shake the motor unit to listen for any loose parts inside.

- 3D**
 - Insert the adapter into the wall socket and insert the small plug at the other end into the motor unit.
 - Press and hold the on/off button until the device switches on.
 - The indicator lights go on.

- 3E** Check whether the indicator lights for modes and levels (A6 in Fig. A) light up.

- 3F** Listen whether the device starts pumping.

- 3G** Listen to the sound of the pumping; it should be a rhythmic pumping sound.

3H Check whether the two outlets draw in air. In order to check the performance of the pump, you should have a spare breast pump kit to be able to assemble the complete pump.

How to check the performance of the pump:





- Assemble all parts of the breast pump kit: silicone tube and cap, pump body with white valve, Natural bottle and diaphragm (Fig. B).
- Press and hold the on/off button (A2 in Fig. A) until the device switches on and press the 'Level up' button (A5 in Fig. A) several times.
- Check if the silicone diaphragm (B2 in Fig. B) moves gradually to the top of the cap and all the way back down in each pumping cycle.
- The above-described checks must be performed for both diaphragms.
- If any of diaphragms does not move according to the description above, please contact your Philips sales agent.

3I

- Check the travel bag and insulation pouch to make sure they are not damaged.
- Check the user manual for completeness.
- If the travel bag, insulation pouch or the user manual is damaged, please replace.

Step 4: Factory reset

Reset the motor unit to factory settings as follows:

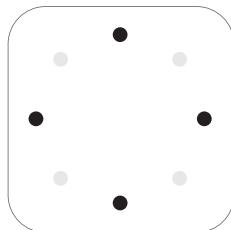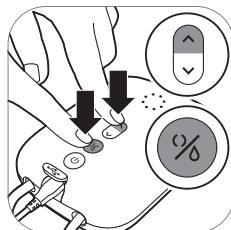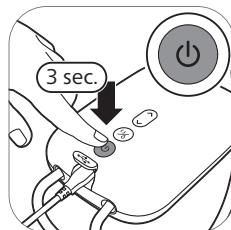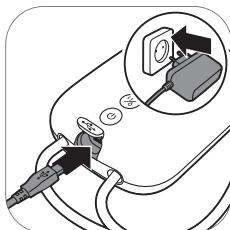4A. Make sure that the adapter is inserted into the wall socket and that the small plug at the other end is inserted into the motor unit.

4B. Press and hold the on/off button to switch off.

4C. Simultaneously press the mode selection button (see A3 in Fig. A) and the 'Level up' button (see A5 in Fig. A).

4D. Wait until the indicator lights for modes and levels light up (A6 in Fig. A).

Step 5: Cleaning before renting

5A Motor unit Adapter It is important to repeat cleaning of the **motor unit** (the buttons and overall surface) and the **adapter** before the breast pump is rented out to the next customer. This can be done by using alcohol wipes (isopropanol).

5B Travel bag Insulation pouch Clean the travel bag and the insulation pouch by wiping them with a clean, damp cloth with water and mild detergent.

⚠ Warning: Do not use alcohol wipes (isopropanol) to clean the travel bag and the insulation pouch.

5C User manual Do not clean the user manual. If the user manual is dirty or damaged, please replace it.

Step 6: Prepare for the next rental use

Make sure that the following 6 items are included in an assembled package for the next customer:

- Motor unit
- Adapter
- Breast pump kit – Please include **two new** kits for each new customer for a **double** electric breast pump.
- Travel bag
- Insulation pouch
- User manual

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Trademarks are the property of Koninklijke Philips N.V. / Les marques commerciales sont la propriété de Koninklijke Philips N.V. / I marchi commerciali e depositati sono di proprietà di Koninklijke Philips N.V.
All rights reserved. / Tous droits réservés. / Tutti i diritti riservati.
Philips Consumer Lifestyle B.V.