www.philips.com

4222.002.7557.2

Register your product and get support at

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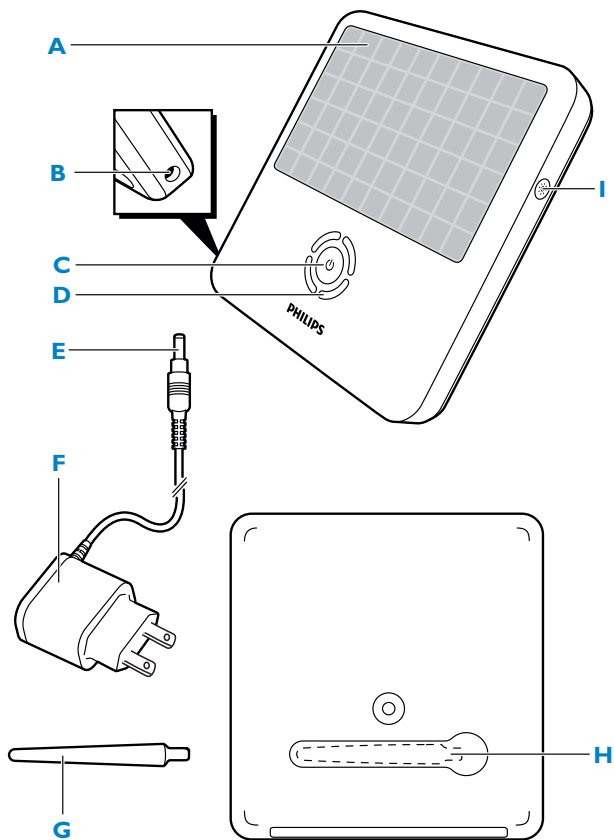
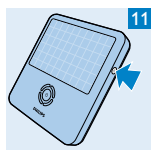
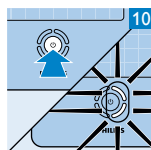
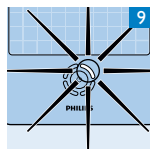
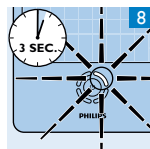
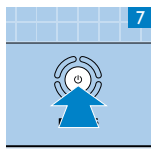
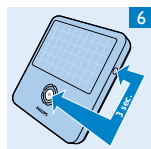
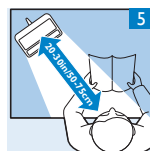
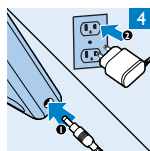
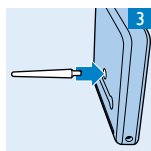
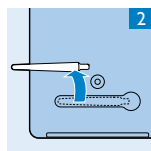
IMPORTANT SAFETY INSTRUCTIONS

Read all warnings and cautions before you use your goLITE BLU energy light.

Observe these safety instructions to avoid potential hazards that could result in personal injuries or damage to your goLITE BLU energy light.

WARNING (a warning indicates the possibility of injury to the user or operator)

- Use the goLITE BLU energy light in vertical position with the stand in place in the recess on the back of the unit. Do not use this appliance laying flat in horizontal position without the stand.
- Do not expose the appliance to rain or moisture. Do not use the appliance near a water source, while bathing, while washing hands or while you are in contact with water.
- If you suffer from an eye disease or have a history of eye disease in your family, consult your ophthalmologist before you use this appliance.
- If you are experiencing any temporary eye problems, wait until the problems are resolved before using this appliance.
- Light emitted by this appliance may interact with photosensitizing medications. Be sure to review all medications that you are currently taking with your doctor before using this appliance.
- As with any bright light source, do not stare into the appliance.
- Do not use this appliance where the light might compromise your ability to perform essential tasks such as air traffic control.
- Do not use this appliance where the light might distract other individuals and compromise their ability to perform essential tasks such as air traffic control.
- Do not increase intensity by sitting closer to the light than recommended. Increased intensity does not improve response and may result in increased side effects.
- The metal stand for the appliance could present a choking hazard.

- Individuals who use photosensitizing medications, who have had cataract surgery, or who have pre-existing eye conditions should avoid unnecessary exposure to light. They should consult their ophthalmologist before using the goLITE BLU.
- The goLITE BLU should never be used adjacent to or stacked with other equipment. However, if it is necessary, the appliance should be observed to verify normal operation.
- Do not position the goLITE BLU or the attached power adaptor in such a way that makes it difficult to disconnect from the power source.
- No modification of the goLITE BLU device is allowed.
- Not suitable for use in the presence of a flammable anaesthetic mixture with air, oxygen, or nitrous oxide.

Caution (a caution indicates the possibility of damage to the appliance)

- Do not operate this appliance near any heat sources such as radiators, heat registers, or other heating devices.
- Never use damaged or worn cords or plugs; this could result in electric shock, burns, and/or fire.
- Use our provided AC/DC adaptor only. The use of power adaptors other than those specified and provided by Philips may result in increased electromagnetic emissions or decreased immunity of the goLITE BLU.
- Do not place heavy objects on top of the power cord or the appliance.
- When unplugging the appliance, grasp the adaptor directly to avoid damaging the cord. Never pull on the power cord to remove the adaptor from the outlet.
- If this appliance is damaged in any way or is not working, it should not be operated. Please visit philips.com/support or call 1-866-832-4361 for assistance.
- There are no user serviceable parts inside the goLITE BLU. No modification of this equipment is allowed.

SAVE THESE INSTRUCTIONS

IMPORTANT: The power plug contains a transformer. Do not cut off the power plug to replace it with another plug, as this will cause a hazardous condition.

Contraindications

The goLITE BLU energy light may not be suitable for use on individuals who use photosensitizing medications, who have had cataract surgery, who have pre-existing eye conditions, or who have bipolar disorder.

Thank you for purchasing the goLITE BLU energy light.

- To fully benefit from the support that Philips offers, register your goLITE BLU at www.philips.com/welcome
- Learn more about your goLITE BLU at www.philips.com/golite

Safety and compliance

- This appliance meets the requirements of the safety standard for medical devices ANSI AAMI ES60601-1:2005 and CAN/CSA-C22.2 No. 60601-1:2008.
- This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Symbol Key

The following symbols may appear on the device:

Symbol	Description
	Consult accompanying instructions for use
	DC Power
	DC Jack Polarity
	Serial Number
HF3321	Serial Number
	UL Listing Mark for Canada and the United States
	Compliant with the Waste Electrical and Electronic Equipment/Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment (WEEE) recycling directives.
	Philips Shield
	In US Manufactured for: Philips Consumer Lifestyle, A Division of Philips Electronics North America Corporation, P.O. Box 10313, Stamford, CT 06904. In Canada Imported for: Philips Electronics LTD, 281 Hillmount Road, Markham, Ontario L6C 2S3
	Date of Manufacture

Introduction

Intended use

The goLITE BLU energy light is intended to make people feel more energetic and to provide mood relief from the winter blues. If you suffer from severe mood swings or depression make sure to consult a professional and do not use this device for self treatment.

10 ENGLISH

Light and health

The human body uses light cues, such as those provided by the sun, to time certain functions. Light exposure can boost mood and energy. Light has direct alerting and energizing effects and can improve our performance. Bright light therapy is also successfully used for mood relief, especially for seasonal mood swings.

In countries with long winters, many people experience loss of energy and low spirits due to lack of light. These symptoms are referred to as the winter blues. They typically start in the fall and may last for several weeks. Exposure to bright light sources can help alleviate these symptoms and restore the summer mood and summer energy level.

The blue light of the summer sky is well known. We do not get as much of this light in the fall and winter, and indoor light does not emit enough of this essential color. The goLITE BLU energy light helps to deliver the wavelength, color and intensity of light that we see in summer.

General description (Fig. 1)

- A LED panel
- B Socket for appliance plug
- C On/off button and timer button
- D Time indicator (15, 30, 45 and 60 min.)
- E Small plug
- F Adaptor
- G Stand
- H Stand Storage cavity
- I Light intensity button

Preparing for use

It is very important that you know how long and how often you should use your Philips goLITE BLU energy light. This knowledge can mean the difference between successfully responding to light and not responding at all. Visit www.lighttherapy.com for a free assessment and to customize a light therapy schedule just for you.

When to use the goLITE BLU energy light

Knowing what time to use the goLITE BLU energy light can mean the difference between feeling better in just a few days instead of weeks, or perhaps not responding at all. Follow these guidelines to understand how light can boost mood and energy and when to use the goLITE BLU for the best results.

Duration and intensity

Most people get beneficial results from 15 to 45 minutes of usage each day. You can start using the goLITE BLU at medium intensity and increase the light intensity if you feel no response after a week.

Timing

Using the goLITE BLU energy light at any time of day directly affects your alertness. You can use the goLITE BLU energy light as a light bath for the winter blues throughout the day. The best time to use it is in the morning. Do not use the goLITE BLU late in the evening. If you use it within 2 hours of going to bed, it could disrupt your sleep.

- If you find it difficult to get up each day without an alarm clock or if you often sleep in, you need morning light. Use the goLITE BLU within the first hour or two after waking up.
- If you find it difficult to stay awake in the late evening and wake up too early in the morning, you need evening light. Use the goLITE BLU before you start to feel drowsy.
- If mornings or evenings do not seem to be the problem, but you feel generally tired during the day, then the time at which you use the goLITE BLU is not as important. You can use the goLITE BLU in the morning or around midday. For example, when you wake up in the morning, use it for 15 to 30 minutes and then again at midday or in the evening for approximately the same amount of time. Place the goLITE BLU further away from you during the evening session. Do not use the goLITE BLU within two hours of your desired bedtime or you may not be able to fall asleep.

12 ENGLISH

Installation

- 1** Remove the stand from the stand storage cavity (Fig. 2).
- 2** Insert the bevelled end of the stand into the hole on the back of the appliance. Magnets hold the stand in place (Fig. 3).
- 3** Put the adaptor in the wall outlet and insert the appliance plug into socket on the appliance (Fig. 4).
- 4** Place the goLITE BLU approximately 20-30 in./50-75cm from your eyes in such a way that the light bathes your face from the side (Fig. 5).
 - You can read, eat, work at the computer, watch TV or exercise while you use your goLITE BLU energy light.
 - For the best results, place the goLITE BLU at the same level as your midriff (stomach area), e.g. place it on the table at which you are sitting.

Note: Do not stare into the light of the LED panel. It is sufficient that the blue light reaches your eyes indirectly from the side.

- Use your goLITE BLU in a well-lit room to minimize eye strain.

Demo mode

The demo mode is used to demonstrate the goLITE BLU energy light in stores. When the appliance is turned on in demo mode, customers of the store can:

- adjust the light intensity
- turn the appliance on and off
- set the light time

If the appliance is not operated for 3 minutes, it reverts to its demo mode default settings.

To activate the demo mode:

- 1** Put the adaptor in the wall outlet and insert the appliance plug into socket on the appliance (Fig. 4).
- 2** Press and hold both the on/off button and light intensity button for 3 seconds (Fig. 6).

- ▶ The appliance turns on and the segments of the timer indicator go on and off in a rotating pattern.
- ▶ The LED panel goes on at the low light intensity level.

Note: The appliance remains turned on in demo mode until it is turned off.

To deactivate the demo mode, press and hold the on/off button and light intensity button together for 3 seconds.

Note: If the appliance has been in DEMO mode for 10 hours, or it is unplugged, the appliance will turn off, and the DEMO mode will be deactivated.

Using the appliance

Use this appliance for its intended household use as described in this manual.

- 1** Put the adaptor in the wall outlet and insert the appliance plug into the socket on the appliance (Fig. 4).
- 2** Press the on/off button briefly to turn on the appliance (Fig. 7).
 - ▶ The first segment of the time indicator goes on and blinks for 3 seconds (Fig. 8).
 - ▶ When the time indicator stops blinking, the LED panel goes on and the timer starts counting down to the set time (Fig. 9).

Note: The timer is automatically set to 15 minutes.

Note: The LED panel only goes on after the time indicator stops blinking. This prevents the light from shining into your eyes while you are setting the time.

- 3** If you want to switch off the appliance before the set time has elapsed, press the on/off button 2 seconds.

Note: You can also switch off the appliance by briefly pressing the on/off button a few times.

Note: The appliance will automatically switch off after the set time duration.

- 4** Unplug the appliance after use.

14 ENGLISH

Setting the Timer

- 1** Press the on/off button briefly to switch on the appliance (Fig. 7).
- 2** Press the on/off button one or more times to select the preferred time (15, 30, 45 or 60 minutes) (Fig. 10).
 - ▶ The corresponding segment or segments of the time indicator will blink for 3 seconds.
 - ▶ When the time indicator stops blinking, the LED panel goes on and the timer starts counting down the set time.

Note: If you want to set the timer to 0 minutes, press the on/off button until all segments of the time indicator are off.

Note: The time indicator only shows the time remaining.

Setting the Light Intensity

- 1** Press the on/off button briefly to switch on the appliance (Fig. 7).
- 2** Press the light intensity button repetitively and the light intensity options of low, medium and high will be displayed (Fig. 11).
- 3** Choose the light intensity desired.

Cleaning

Never immerse the appliance in water or any other liquid, nor rinse it under the tap.

Never use scouring pads, abrasive cleaning agents or aggressive liquids such as petrol or acetone to clean the appliance.

- 1** Clean the appliance with a dry cloth.

This appliance has no other user-serviceable parts. For assistance call 1-866-832-4361.

Storage

- If the appliance will not be used for an extended period of time, remove cord from outlet and store in a safe, dry location where it will not be crushed, banged, or subject to damage.
- Do not wrap cord around the appliance when storing.

Disposal

- Dispose of the device in accordance with local regulations.
- Your local or national recycling organizations may also have disposal information.
- For assistance visit www.recycle.philips.com or call 1-866-832-4361.

Assistance

For assistance visit our website www.philips.com/support or call toll free: 1-866-832-4361.

45-Day Money-Back Guarantee

If you are not fully satisfied with your Philips goLITE BLU energy light, send the product back along with the original dated sales receipt and we will refund you the full purchase price.

The goLITE BLU must be shipped prepaid by insured mail, insurance prepaid, and have the original sales receipt, indicating purchase price and the date of purchase, enclosed. We cannot be responsible for lost mail.

The goLITE BLU must be postmarked no later than 45 days after the date of purchase. Philips reserves the right to verify the purchase price of the goLITE BLU and limit refunds not to exceed suggested retail price.

To obtain Money-Back Guarantee Return Authorization Form, call 1-866-832-4361. Please allow 4-6 weeks for delivery of the check.

Full Two-Year Warranty

Philips Electronics North America Corporation (USA) and Philips Electronics Ltd (CANADA) warrant each new Philips product, model HF3321 against defects in materials or workmanship for a period of two years from the date of purchase, and agree to repair or replace any

16 ENGLISH

defective product without charge. IMPORTANT: This warranty does not cover damage resulting from accident, misuse or abuse, lack of reasonable care, or the affixing of any attachment not provided with the product. NO RESPONSIBILITY IS ASSUMED FOR ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES. In order to obtain warranty service, simply call toll-free 1-866-832-4361.

In US Manufactured for: Philips Consumer Lifestyle, A Division of Philips Electronics North America Corporation, P.O. Box 10313 Stamford, CT 06904

In Canada Imported for: Philips Electronics LTD, 281 Hillmount Road, Markham, Ontario L6C 2S3

Trademarks owned by the Philips Group.

© 2010 Koninklijke Philips Electronics N.V. All Rights Reserved.

Frequently asked questions

This section lists the questions most frequently asked about the appliance. If you cannot find the answer to your question, visit www.philips.com/support or call 1-866-832-4361 for assistance.

Question	Answer
Do I have to take off my glasses when I use the goLITE BLU?	No, you can use the goLITE BLU energy light with your glasses on. Do not wear tinted glasses or sunglasses when you use the goLITE BLU, as these glasses reduce its effectiveness.
Does the appliance have any side effects?	The goLITE BLU is safe if used according to the instructions. Temporary headaches or nausea have been reported. These effects are usually solved by sitting further away from the appliance.

Question	Answer
Can I get sunburn from using the goLITE BLU?	No, the technology used in the goLITE BLU eliminates all UV light.
I have heard that light can be dangerous. Can the goLITE BLU damage my eyes?	The goLITE BLU emits safe levels of blue light, according to current standards. However, if you suffer from an eye disease or have a history of eye disease in your family, consult your ophthalmologist before you use the goLITE BLU. If you have a temporary eye problem, wait until it is resolved before you use the goLITE BLU.
	The goLITE BLU energy light is safe if used according to the instructions. The appliance is free from any ultraviolet light risk to the eye and it produces a safe, diffused field of light. The appliance has been tested for ocular safety.
	As with any bright light source, do not look into the light too long. You get a better result if you place the goLITE BLU at a slight angle to your face.
Should I use the goLITE BLU every day?	You can use the goLITE BLU energy light daily, especially during the winter months. Many people find that they do not need the light in summer. Others use it year-round because they spend most of their time indoors and do not get the right kind of light signals to boost their mood and energy.
Can the light cause harm to children or pets?	No. The light is safe to be used around children and pets. However, do not let children play with the appliance.

Question	Answer
Can I use the goLITE BLU more than once a day?	For most people, it is sufficient to use the goLITE BLU once a day. Keep in mind that you may have trouble falling asleep if you use the goLITE BLU too late in the evening.
How long do the LEDs last?	If you use the appliance 30 minutes a day throughout the year, the LEDs have a lifetime of over 50 years.
What if one or more LEDs stop working prematurely?	In the unlikely event that one or two LEDs fail prematurely, you can continue to use your goLITE BLU normally because its effectiveness is not affected. However, if several LEDs stop functioning, call 1-866-832-4361 for assistance.
What is the demo mode?	This mode is used to demonstrate the goLITE BLU energy light in stores. To turn off this mode, press and hold both the on/off button and the light intensity button together for 3 seconds or unplug the appliance from the outlet.

Specifications**Model** HF3321**Electrical**

Rated voltage appliance (VAC) 100-240

Rated voltage adaptor (V DC) 15

Rated frequency (Hz) 50/60

Rated input power adaptor (W) 9

Power consumption appliance (W) 5.4

Power consumption adaptor (W) 0.15

IEC 60601-I classification

- Insulation class II

- Ingress of water IPX0

Model HF3321

- Mode of operation Continuous

Operating conditions

Temperature from 0°C (32°F) to +35°C (+95°F)

Relative humidity from 15% to 90% (no condensation)

Storage conditions

Temperature from -20°C (-4°F) to +50°C (+122°F)

Relative humidity from 15% to 90% (no condensation)

Physical characteristics

Dimensions 14cm x 14cm x 2.5cm
(5.5in x 5.5in x .98in)

Weight 0.27kg (9.52 ounces)

Circuit diagrams, component part lists and other technical descriptions are available on request.

As user of this appliance, you should take measures to prevent the occurrence of electromagnetic interference or other types of interference between this appliance and other appliances.

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Au Canada, importée pour : Philips Electronics LTD, 281 Hillmount Road, Markham, Ontario L6C 2S3