**PHILIPS**

**ZOOM!**

WhiteSpeed  
Light-Accelerated  
Tooth Whitening  
Procedure Kit  
牙齿美白手术套装

Philips Zoom! WhiteSpeed  
Light-Accelerated  
Tooth Whitening Procedure Kit  
牙齿美白手术套装

**Instructions for use**  
使用说明书

— |

| —

— |

| —

— |

| —

— |

| —

— |

| —

— |

| —

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## Symbols

Consult instructions for use

Reference to enclosed or attached information

Caution

Keep away from sunlight, keep away from heat

Temperature limits

Manufacturer

Use by date

Batch code

Catalogue number

Do not re-use. This product is intended for use on a single patient during a single procedure.

Keep refrigerated

## Important information

Read and understand the information in these instructions for use provided with the Philips Zoom! Whitespeed Light-Accelerated Tooth Whitening Gel (whitening gel) prior to initiating whitening treatment.

The whitening gel must be removed from refrigeration and warmed to room temperature so it will thicken before it is applied to the patient's teeth. Either:

- Remove the syringe from refrigeration at least 6 hours before use (preferably the night before) or
- If removed just prior to use, place the syringe in a cup of hot water (49 °C) for 10 minutes. Remove the syringe from the water and wait 5 minutes before extruding gel.

## Conditions of use

This product is to be distributed to and used by dental professionals exclusively. This product is intended for use on healthy patients aged 18 years and older who have no underlying oral health issues, are not perio-involved and do not exhibit failing restorations.

## Contents

**Philips Zoom! WhiteSpeed Light Accelerated Tooth Whitening Procedure Kits (tooth whitening procedure kits) (for 1-patient use) include:**

- Light guide (1)\*
- Light guide instructions for use (1)

## 8 English

- Relief ACP oral care gel syringe, 2.4 ml (1)\*
- Relief ACP oral care gel instructions for use (1)
- Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Kit (tooth whitening kit) (1)

\* Refer registration information to the appendix.

### **Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Kit (1)\* includes:**

- Face bib (2)
- Gauze squares (6)
- Liquidam Dental Dam (Liquidam) Syringe, 2.9 g (1)
  - Liquidam needle point tip (metal tip) (1)
  - Liquidam black tip (plastic tip) (1)
- Cotton rolls (2)
- Surgical suction tip (1)
- IsoPrep retractor (retractor) (1)
- IsoPrep small retractor (small retractor) (1)\*\*
- Application brush (1)\*\*\*

\* Refer filing information to the appendix.

\*\*Only available in the 2-patient kit.

\*\*\*Non-medical device.

### **Packaged separately:**

- Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Gel (1)\*
  - Instructions for use (1)
  - 25 % Hydrogen Peroxide Whitening Gel syringe, 4.6 g (1)
  - Mixing tip (1)
- Philips Zoom! WhiteSpeed Whitening LED Accelerator (LED Accelerator)\*

\*Refer registration information to the appendix. Outside Philips Zoom! Tooth Whitening Procedure Kit offering.

## Product description

Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Procedure Kits are professional level (in-office) tooth whitening kits and are for single use only.

## Indications for use – candidate qualification

A screening exam is recommended to ensure the patient has no underlying oral health issues. You may want to treat existing issues before recommending this whitening treatment and disqualify any patient who is perio-involved, exhibits failing restorations or is otherwise in an unhealthy oral state.

## Patient's expectations

Set realistic expectations about potential results based on the oral exam and explain the following:

- Results vary by patient.
- Anatomy, calcifications, age, tooth condition, hygiene and starting shade affect the results.
- Teeth with yellow and brown stains will achieve the most noticeable results.
- Previous orthodontic treatments may cause teeth to whiten unevenly either due to residual resin remaining on the teeth or overpolishing upon removal.
- Crowns and other tooth-colored restorations will not whiten.
- Tooth-colored restorations may require replacement following whitening to maintain a uniform color.
- Tooth whitening may cause temporary dehydration of teeth resulting in white spots or striations. Uneven whitening will fade to normal within 2-7 days of the treatment.

## Pre-treatment for reducing tooth sensitivity

It is recommended to prescribe the following prior to the treatment:

- Treat with a desensitizing toothpaste with 5 %  $\text{KNO}_3$  twice a day for 14 days before the procedure.

- Nonsteroidal anti-inflammatory drug (NSAID) adhering to the respective drug's instructions for use 1 hour before the procedure.

## Warnings

- Do not place a composite/bonded restoration within seven days after whitening treatment to ensure color stability of the composite/bonded restoration and bond strength to enamel and dentin.
- Do not use tooth whitening gel on anesthetized teeth.
- Tooth whitening is not recommended when there are previous injuries in the gingival tissues.
- Avoid contact of whitening gel with eyes and soft tissue. If contact with eyes occurs, rinse eyes immediately with plenty of water and seek medical advice.
- Do not swallow the whitening gel.

## Intended use

Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Procedure Kits are for use with Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Gel and Philips Zoom! WhiteSpeed Whitening LED Accelerator for dental professional tooth whitening treatment. The tooth whitening procedure kit contents are intended to be used as follows.

Procedure Kit contents	Intended use
IsoPrep retractor	Moves the lips and cheeks away from the teeth, protecting the lips, cheeks and tongue from coming into contact with the whitening gel.
Face bib	Help protect the patient's face from the light source.

**Procedure Kit contents****Intended use**

Liquidam Dental Dam (gingival barrier material)	Used as an isolation material to prevent the whitening gel from contacting gingival tissues.
Cotton rolls	Separate the upper and lower vestibule during the procedure and help isolate the patient's lips from the teeth.
Gauze squares	Help isolate the patient's lip from the teeth and provide additional separation between the upper and lower vestibule.
Surgical suction tip	Remove excess liquid/saliva that may accumulate during the procedure.
Application brush	Apply whitening gel to the patient's teeth.
Light guide	An accessory to the LED Accelerator used to help align the head to a specified distance between the LED Accelerator and the patient.
Relief ACP oral care gel	Relieve discomfort from dentin sensitivity.

**Contraindications**

Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Gel is not for use on or by:

- Pregnant or lactating women.
- Patients currently being treated for a serious illness or disorder, e.g. immune compromised, AIDS, etc.
- Patients under 18 years of age.

## Possible side effects

- Teeth sensitivity or hypersensitivity.
- Gingival irritation or burn if Liquidam is not placed in accordance with the instructions in the tooth whitening procedure kit.

## Frequency of use

Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Gel is intended for a single-patient treatment. A single treatment typically consists of three to four 15-minute applications. A patient may be treated more than once at the discretion of the dental professional, allowing seven days between treatments.

## Safety

- Dental professional and patient need suitable protective covering and Zoom! protective eyewear.\*
- Avoid whitening gel contact with soft tissue and eyes.

\* Supplied with the LED Accelerator.

## First aid instructions

- Contact a Poison Information Center or medical doctor immediately if a large quantity (over 25 % of the syringe) is swallowed. Provide the ingredient label if possible. Do not induce vomiting. Immediately give a glass of water.
- If whitening gel gets into the eyes, hold the eyelids apart and flush the eyes continuously with running water for at least 15 minutes.
- If improper isolation leaves a patient's soft tissue exposed to light, administer a nonsteroidal anti-inflammatory drug (NSAIDs) adhering to the respective drug's instructions for use.

## Precautions

The whitening gel must be removed from refrigeration and warmed to room temperature so it will thicken before it is applied to the patient's teeth. Either:

- Remove the syringe from refrigeration at least 6 hours before use (preferably the night before) or

- If removed just prior to use, place the syringe in a cup of hot water (49 °C) for 10 minutes. Remove the syringe from the water and wait 5 minutes before extruding gel.

Note: Improper soft tissue isolation may result in irritation or burning of gingiva or swollen lips due to the LED Accelerator light coming into contact with tissue. Improper soft tissue isolation may also result in chemical burn due to whitening gel coming into contact with tissue.

Note: Wait one week after whitening to place restorations. Patients with porcelain fused to metal crowns, amalgams, lingual bars or implants will feel excessive heat.

## In-office whitening treatment

### Before the in-office whitening treatment

Complete diagnosis of patient, treat for sensitivity (see 'Pre-treatment for reducing tooth sensitivity') and fabricate trays\* prior to whitening treatment.

\* Outside Philips Zoom! Tooth Whitening Kit offering.

### In-office whitening treatment

- 1 Pumice teeth just prior to whitening.
- 2 Remove the light guide from the packaging and place it on the LED Accelerator.
- 3 Switch on the LED Accelerator using the power button located on the side of the power pack. When the light guide is attached, the display reads 'Guide Status-Guide Attached' followed by 'X of X Sessions'.
- 4 Place the surgical suction tip on the high vacuum suction.
- 5 Place the Zoom! protective eyewear on the patient.

Note: The retractor provided is necessary to ensure proper positioning of the LED Accelerator head and additional protection from light.

## 14 English

- 6** Insert the retractor at an angle, retracting one side at a time using a dental mirror to assist in placement.

Note: The retractor has been specifically engineered to work with the Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening System (whitening system). Do not substitute.

- 7** Take a retracted 'before' photo using a matched shade tab.
- 8** Fully recline the patient.

- 9** Place cotton rolls in the center of the upper and lower vestibules. If the cotton rolls are too large, unfold the gauze and twist it like a cotton roll.

Note: Do not substitute gauze. The gauze provided has been tested and proven to provide superior protection.

- 10 Fully open the gauze squares and fold them into triangles. Place the apex of the triangle into the posterior cheek, tuck the balance of material into the cheek, tucking the ends between the cotton rolls and the retractor.

- 11 Carefully place one face bib around the retractor, one side at a time. Add an additional bib for heavy salivators.
- 12 Remove the Liquidam syringe from the packaging, twist to remove the cap and attach the metal tip.

Note: Exposed dentin and tissue should be covered by barrier material. Use the 'no pink' rule.

- 13 Dry soft tissue (failure to do so may create imperfect seal and allow whitening gel to migrate onto soft tissue) and begin by scalloping barrier material to CEJ area (gingival margins) on the upper teeth slightly overlapping enamel and interproximal spaces to form enamel seal.

## 16 English

- 14** Cure the application of Liquidam using a curing light.\*

\* Outside Philips Zoom! Tooth Whitening Procedure Kit offering.

Note: Liquidam can become warm upon curing. Move the light back and forth across the arch using a sweeping motion for about 10 seconds.

- 15** Change to the plastic tip. Fill in from the cotton roll to just formed line of Liquidam and cure. Be sure to cover all interproximal areas leaving no soft tissue exposed.

On the lower arch, apply Liquidam using the same method.

Use the tip of the syringe to press lightly on the application to ensure that it is completely cured. The material should be solid and have no give.

If necessary, cure the arch again for an additional 5 seconds.

The application should extend distal at least one tooth beyond the area receiving the whitening gel and should be 2 mm thick.

- 16** Remove the cap from the room temperature whitening gel syringe. Attach the mixing tip by aligning the marking and secure it by turning it one quarter turn clockwise. Using the brush provided, apply whitening gel to the patient's teeth (1-2 mm thick).

Use caution not to disturb the barrier material. Gingival irritation can occur with prolonged exposure to the whitening gel.

Note: A small air gap or bubbles in the syringe due to degassing of the whitening gel is normal. If the air gap is near the front of the syringe, purge until both sides of the barrel extrude gel.

Note: A slight yellow or pink coloring of the whitening gel is a normal occurrence from the manufacturing process. There is no impact on the peroxide content, other chemical properties, or performance of the whitening gel. It will not discolor the patient's teeth.

- 17** Once the whitening gel application is complete, raise the back of the chair so the patient is in a slightly reclined position (~45 degree angle) and able to swallow comfortably while maintaining light pressure on the bite block.

Note: The light guide is designed to ensure proper distance and alignment to maintain correct light exposure on the smile zone.

- 18** Slide the LED Accelerator into place next to the patient. The arm of the LED Accelerator should be movable, but stable. Position the LED Accelerator head by aligning the slots on the light guide with retractor.
- 19** Follow the prompts on the display panel to switch on the light and to activate the timer.

Once the light is on, a countdown is displayed. Light emits from the LED accelerator head to shine on the patient's teeth.

It is normal for the light to flicker for the first few seconds.

The LED Accelerator beeps once when three minutes of the session are remaining and beeps three times in the final three seconds of the 15-minute session.

When the session is finished, the light goes out and you hear a long beep.

Note: If a session must be interrupted, press the 'Pause' button. The remaining session time flashes on the display. To reactivate the session, press either the 'Pause' or 'Start' button.

- 20** Inform the patient that the progress of each session are tracked by four white lights on top of the LED Accelerator head. Each light indicates one quarter of a session.

Note: During each 15-minute session, make sure the patient has a means of communication (e.g. a bell) or do not leave the patient unattended.

- 21** After each session, gently pull the LED Accelerator away from the patient, taking care not to dislodge isolation material.

## 18 English

Note: At the end of each 15-minute session, the display indicates the number of remaining sessions left on the light guide. Press 'Start' to activate the light for the next session.

- 22** Carefully remove the whitening gel with the surgical suction tip and/or wipe the surface of enamel clean with damp gauze. Do not irrigate as the force of the water may pull the dam loose and saturate isolation materials.
- 23** Check the isolation materials to make sure 'no pink' tissue is visible. Replace or add isolation if needed.
- 24** Follow the same instructions as above for the remaining sessions.
- 25** When all sessions on a light guide are finished, the display reads 'Sessions Complete' for three seconds, followed by 'Guide Status - Replace Guide'. Detach the light guide from the IsoPrep retractor and pull the LED Accelerator away from the patient.

Note: For sensitive patients, lower the intensity of the LED Accelerator. However, please note this could result in a fourth 15-minute session.

Tip: If the patient notes tooth sensitivity, place Relief ACP (found in the tooth whitening procedure kit) on the lingual surface(s) for the remaining sessions. After the last session is complete, detach the light guide from the retractor and pull the LED Accelerator away from the patient. Suction or wipe the whitening gel from the teeth. Moisten gauze and cotton rolls and remove the isolation materials.

If barrier material remains interproximally, remove it with floss.

- 26** Reinsert the retractor and take a retracted 'after' photo with the lighter shade tab in place. Measure the shade change by counting the shade shift according to the value-ordered shade guide.\*

\* Outside Philips Zoom! Tooth Whitening Procedure Kit offering.

## Maximizing results

- Advise the patient not to consume coffee, tea, red wine and tobacco for two hours after the whitening treatment, until the salivary pellicle reforms on the teeth.
- All tooth whitening requires maintenance. The patient may choose to have the whitening treatment again or to use a maintenance product, depending on their habits.

## At home post care and maintenance

Philips Relief ACP oral care gel has been shown to decrease tooth sensitivity. Place Relief ACP oral care gel into fabricated trays and set on patient. Normal wear time is 30 minutes/time. Use Relief ACP twice a day until symptoms are relieved or as directed by your dental professional. For the best results, do not drink or eat for 30 minutes after application. See the Relief ACP oral gel instructions for use for complete instructions.

## At home whitening maintenance

24 hours after the whitening treatment, the patient may use Philips Zoom! NiteWhite or Philips Zoom! DayWhite Tooth Whitening Gel\* to maintain or further enhance their results. See the Philips Zoom! NiteWhite/DayWhite patient instructions for complete instructions.

\*Refer registration information to the appendix. Outside Philips Zoom! Tooth Whitening Procedure Kit offering.

## Storage instructions

The tooth whitening gel should be stored at 2 °C - 10 °C. The tooth whitening procedure kit should be stored at 15 °C - 25 °C.

Storage temperature outside the recommended ranges above will result in reduced product shelf life.

## Appendix

The medical device product registration information referenced within the IFU is listed as follows :

Product name: Tooth Whitening Gel

Model name: Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening

Product specifications: 4.6g/syringe

Medical Device Registration Certificate No./Product Technical Requirement No. : GXZJ 2020170135

---

Product name: Whitening LED Accelerator (including light guide)

Model and specification: Philips Zoom! WhiteSpeed

Medical Device Registration Certificate No./Product Technical Requirement No.: GXZJ2020170375

---

Product name: Oral Care Gel

Model name: Relief ACP

Product specifications: (2.4g/syringe x1)/box, (2.4g/syringe x4)/box

Medical Device Registration Certificate No./Product Technical Requirement No.: GXZJ 2020170435

---

Product name: Tooth Whitening Kit

Model name: Philips Zoom! WhiteSpeed Light-Accelerated

Product specifications: 1 set/box, 2 sets/box

Medical device filing certificate No./ Product Technical Requirement No.: GXB 20200672

---

Product name: Tooth Whitening Gel

Model name:

Philips Zoom! DayWhite 6% Hydrogen Peroxide

Philips Zoom! DayWhite 9.5% Hydrogen Peroxide

Philips Zoom! NiteWhite 10% Carbamide Peroxide

Philips Zoom! NiteWhite 16% Carbamide Peroxide

Philips Zoom! NiteWhite 22% Carbamide Peroxide

Product specifications: (2.4ml/syringe x6 syringes)/box,  
(2.4ml/syringe x30 syringes)/box

Medical Device Registration Certificate No./Product Technical  
Requirement No.: GXZJ20203170310

---

Registrant/manufacturer name: Discus Dental LLC.

Registrant/Manufacturer: 1700 A South Baker Avenue, Ontario, CA  
91761, United States Of America

Manufacturer address: 1700 A South Baker Avenue, Ontario, CA 91761,  
United States Of America

Registrant contact information: +1 (310) 845 8260

After-sales service agent: Philips (China) Investment Co., Ltd.

After-sales service agent address: Building A1, No. 718, Lingshi Road,  
Jing'an District, Shanghai

Contact information of after-sales service unit: 4008 800 008

Agent name: Philips (China) Investment Co., Ltd.

Agent address: Building A1, No. 718, Lingshi Road, Jing'an District,  
Shanghai

Agent contact information: 4008 800 008

Manufacturing date: see product label

Shelf-life: see product IFU

Transport conditions: see product IFU

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All rights reserved