

**PHILIPS**

**AVENT**



## Nipple Shield

Shaped to allow skin contact

Protect sore nipples

Medium(21mm)

2 pcs



SCF153/03

**Helps you breastfeed longer, comfortably\*\***

May help baby to latch and gentle on your skin\*\*

Nipple shields are designed to help you breastfeed when you are experiencing nipple issues or latch on challenges. The ultra-thin butterfly shaped nipple shield allows breast contact for baby.

### **Easy latch on for baby\***

- Designed to help with latch on challenges

### **Product features**


- Designed to fit
- Shaped to allow skin contact with the baby

### **When to use a nipple shield**

- Combining breast and bottle feeding
- Babies can drink at their own pace
- Made for sensitive, sore or painful nipples


# Highlights

## Unique Butterfly shape

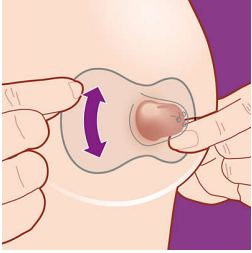

Our Nipple Shields ultra-thin butterfly shape is designed for skin-to-skin contact for baby. Thanks to the shape not only the nose is in contact with the breast but also the chin allowing the baby to smell mom`s scent and touch mom`s skin. Just position the shield over the center of your nipple to make sure your baby can properly latch on the entire areola. Enjoy bonding with your little one while protecting your nipples.


## Designed to support latch on\*\*

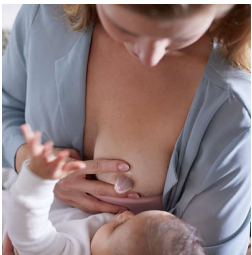

We designed our Nipple Shields to help babies with early latch on challenges and help you breastfeed longer\*\*. For mothers with flat and

inverted nipples and babies with weak sucking technique or oral anomalies, the Nipple Shield can provide a nipple shape for latch on, keeping and maintaining a protruded position during baby`s feeding pauses.

## Protects nipples while feeding

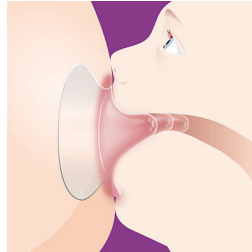

Nipple shields are designed to cover and protect the nipples of lactating women to support breastfeeding. It is indicated for use when the mother has sore, sensitive, cracked nipples or suffers from nipple pain.

## Feed at baby's pace




If mom`s milk flow needs controlling or the baby lacks strength, nipple shields are also a great support. The three holes in the tip of the shield

can help rate the breastmilk flow for the baby in case there is an oversupply of milk. This way the baby can drink at his own pace in case the mother is experiencing an overactive let down.

## Combining breast & bottle feed

The ultra-thin, soft silicone material of the Philips Avent Nipple Shield is closer to our infant feeding solutions to help seamless transition between breast and bottle feeding when combining both.

## Two sizes to suit your need

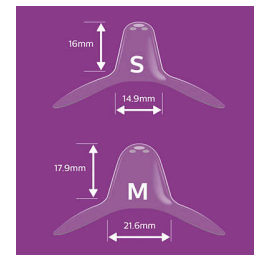

Nipple shields are available in two sizes to better fit your breast. Small shields fit from 12mm nipple diameter and medium up to 21mm. To find your size, get comfortable and place one of the punched rings over your nipple. Rotate shield clockwise and guide your nipple into the tunnel. Then stretch the shield`s base around the areola. If there is 2mm of space between the nipple and silicone, this is the right fit. Tip: Size are based on the mamilla and not the areola. Not sure? Ask your Health Care Professional for support.

# Specifications

## Design

Butterfly shaped design

## Material

Nipple shield: BPA Free\*, Silicone

## What is included

Nipple shields 21mm: 2 pcs

Hygienic carrying case: 1 pcs



\* \*0% BPA, following EU regulation 10/2011

\* \*\*in case of flat or inverted, sore, sensitive, cracked, painful nipples and some babies with latch on challenges

\* ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE

\* Warning to avoid choking and swallowing - Never leave the nipple shield and the hygiene case unattended and keep out of reach of children. - Inspect before each use. Throw away at the first signs of damage or weakness or after 2 months of usage.