

# PHILIPS

Massage Gun


Mini Massage Gun

PPM7302



## Quick Start Guide


1. Use the preinstalled massage head for massage with hot compress. If you do not need hot compress, select an appropriate massage head to fit on the heating plate support according to your massage demand. After installation, check that the massage head works normally.

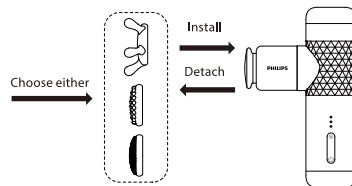

2. Hold the power button to turn on the device in level 1 by default. When the hot compress mode is off, the indicator light will be illuminated in blue.



3. Press the power button and the device will switch to level 1/level 2/level 3/slow waveform mode/fast waveform mode/standby mode successively. The level indicator light varies with the level. Level 0 is standby mode, in which all level indicators are on and change in breathing mode.

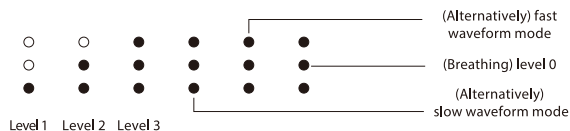


4. Press the hot compress button to turn on/off the hot compress function. When the hot compress function is turned on, the gear indicator light is illuminated in orange.



5. The device will shut down after being used continuously for 10 min. If you need to use again, please hold the power button to start the device.



6. Before initial use, please fully charge the product for 4 hours.

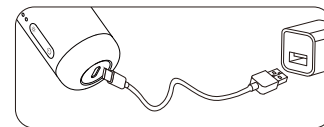



Visit us

[www.philips.com](http://www.philips.com)

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

©2022 AquaShield

All rights reserved

The Philips trademark and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. used under license.

Rev A JUN 22



7. When the power indicator is flashing in red, it indicates the current power is less than 10% and needs immediate charging.



- 
- ❗ 1. Do not start the device while charging.
  - 2. Always remove the pendant and decorations before use to avoid affecting the massage part.
  - 3. Please install the massage head in place before use.
  - 4. It is recommended to operate the device from the lowest level for the initial use to avoid stimulation.
  - 5. When using the finger massage head to massage the scalp, please do not press hard.