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This product has been manufactured by and is sold under the responsibility of Hongkong AquaShield Health Technology Company Limited and Hongkong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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User Manual
使用說明書

Table of Contents

1	Important Information	1
	Symbol Description	1
	Precautions	1
	Safety Warning	2

2	Product Features	3
---	------------------	---

3	Parts Name	3
---	------------	---

4	How to Use	5
---	------------	---

5	Description of Indicator Light	7
---	--------------------------------	---

6	Main Specification	8
---	--------------------	---

7	Service Guideline for Warranty	9
---	--------------------------------	---

8	Packing List	10
---	--------------	----

1 Important Information

Please carefully read the operation manual before use and keep it properly.

Symbol Description

The following contents must be abided by so as to avoid personal injury or property loss.

· The following diagrams are used to illustrate the contents that you should abide by.

 It means forbidden

 It means to operate according to the instructions

Precautions

Please use the product with caution for the following conditions after being approved by doctors.

Pregnant, diabetic complication (such as, neuropathy or retinal damage). People who wear pacemaker and who suffer from seizures or migraines, disc herniation, spinal prolapse, loss of vertebrae or stiffness of joints of immediate vertebrae as well as people who have perform surgeries recently. In case of such conditions, you can use the product, but please consult with doctors in advance.

According to practices, under many cases, it is a treatment method to

combine the beating training with other therapeutic regime, however, such method should be suggested or accompanied by doctor, expert or therapist.


Safety Warning

-  It is forbidden to apply on wet and filthy skin surface. You can use it with your clothes on. Gently press and move it on your skin;
-  It is forbidden to use it above the neck and areas with less muscle coverage;
-  It is forbidden to use it on the weakened parts of the spine, neck, head, joints, etc.;
-  It is forbidden to apply on part with bruise and discomfort;
-  It is forbidden to put any articles on the heat part of the product;
-  It is forbidden to soak the product into water and protect it from water;
-  It is forbidden to throw the product in fire or keep it close to fire source;
-  It is forbidden to fall off the product deliberately;
-  It is forbidden to dismantle the product in any method;
-  It is forbidden to insert finger, hair or other body part into the mounting hole of the massage head, which may be clamped;
-  People who aged 12-18 year's old should be accompanied by parents for use and should not use it on their own. It is forbidden for children under 12 to use;
-  When there is no pain or discomfort, use this product only on the soft tissues of the body, and pay attention to avoiding nearby bones and joints to avoid damage;
-  In case of muscle injury, please consult doctors in advance or please use it in the company of a professional.
-  The battery can be taken out from the bottom cabin (users are not allowed to replace battery on their own) :
 - The battery must be taken out from the cabin before disposing the device
 - The device must be disconnected from power while taking the battery out
 - The battery must be disposed in a safe manner

2 Product Features

The product can relax the soft tissue of body through the high-frequency impact, which can help relax the tense muscles and release the sour and fatigue after exercise. Different massage heads can be matched to massage the muscles. It is compact and portable.

3 Parts Name

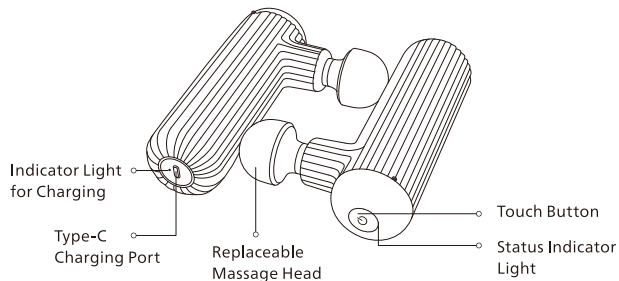Rubber Spherical Head
Suitable for daily massage and relaxation

Titanium Flat Head
Suitable for muscle relaxation after high-intensity exercise

Bullet
Used to impact deep tissues and quickly relieve muscle soreness

U-shaped Head
Suitable for concentrated massage on specific sore points

4 How to Use

1. Product Installation

According to the massage needs, please select the massage head and insert it into the front round hole of the fascial gun.


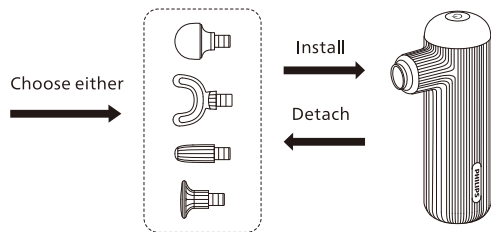
Illustration as shown in the figure:

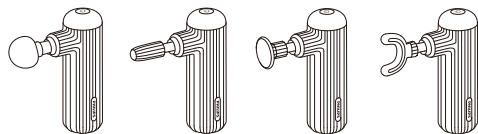2. On/Off


Turn the device on by double clicking the operation button. Hold it to turn it on or switch to forth level and then press the operation button to turn it off.

- ⚠ It is advised to use it after operating 5 seconds.
- ⊘ The device will switch to charging mode while charging and the product will not turn on by pressing the touch button.
- ⚠ In order to avoid massaging the same position for a long time, the device will be automatically off after using 10 minutes for each level.

3. Level Switch

The default level is 1 after power-on and press the operation button to switch from level 1-4.

- ⚠ As the moving parts are made of metal, they may heat after a long-time use, which is normal.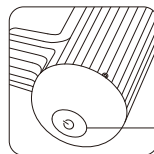


The lamp ring will be on after start and the current level will be displayed according to colour.

- Level 1: Blue
- Level 2: Cyan
- Level 3: Green
- Level 4: Orange

4. Charging

It takes about 3.5 hours for the battery to be fully charged. After charging, please unplug the charging cable in time.

❗ During charging, the fascial gun should lay flat.

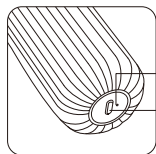○ Charging Indicator Light

○ Type-C Charging Port
5V== 2A

Note: While charging, the fascial gun should lay flat.

It takes about 3.5h for the battery to be fully charged.

After charging, please unplug the charging cable in time.

5 Description of Indicator Light

Charging Indicator	Status	Description
Red Light	Flicker	Battery:<20%
Orange Light	Flicker	Battery:20%-70%
Green Light	Flicker	Battery:70%-99%
Green Light	On	Battery:100%

Level Indicator	Status	Description
Blue Light	On	Level 1
Cyan Light	On	Level 2
Green Light	On	Level 3
Orange Light	On	Level 4

6 Main Specification

Product Name	Philips Mini Fascial Gun
Product Model	PPM7301
Power Supply Method	Rechargeable Lithium Battery
Charging Voltage	5V==
Charging Current	2A
Battery Capacity	2600mAh
Working Voltage	7.4V==
Working Power	20W
Power Supply Port	USB Type-C
Power Adjustment	4 Levels
Product Size	78x142 x48mm
Product Net Weight	328g

7 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

HongKong

Hotline: 852-3188-4987

Service Email: Philips.Massage.hk@centuryfield.com

Taiwan

Hotline: 0800-661-600

Service Email: Philips.Massage.hk@centuryfield.com

Warranty Card

Product Model		Customer Name	
Seller		Contact No.	
Date of Purchase		Serial No.	
Customer Address			
Maintenance Content	Date	Maintenance Personnel	

8 Packing List

Main Unit	1x	Storage Bag	1x
Massage Head	4x	User Manual	1x
Charging Cable	1x	Certificate of Conformity	1x

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All rights reserved
The Philips trademark and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. used under license.