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This product has been manufactured by and is sold under the responsibility of Hongkong AquaShield Health Technology Company Limited and Hongkong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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User Manual
使用說明書

Catalogue

1	Instructions for User	1
	Symbol Description	1
	Applicable Persons	1
	Use Precautions	3
	Notes for the Use Place	3
	Maintenance Precautions	4
2	Product Features	4
3	Parts Name	5
4	How to Use	6
	Precautions Before Use	6
5	Maintenance	7
	Cleaning Methods	7
6	Diagnosis	7

7	Main Specification	9
8	Service Guideline for Warranty	10
	Warranty and Service	10
	Warranty Card	11
9	Packing List	11

1 Instructions

Please read this manual carefully and keep it properly before using the product.

Symbol description

To avoid personal harm or property damage, be sure to follow the followings.

- Below icons marks what should be noted.

 Prohibit

 Must follow the instructions before use

Applicable Persons

 Product should be used with caution by following users:

User under medical treatment or has any of the symptoms, please be sure to consult with a doctor or physiotherapist before use.

- People with medical appliances implanted in pacemakers
- People with malignant tumors or heart disease
- Pregnant or maternity
- People with diabetes and nerve damage
- People with skin trauma

- People with a body temperature above 38 degrees
- People who are tingling in hands or feet
- People with breathing disorders problems
- People with visceral diseases (gastritis, hepatitis, gastroenteritis) and other acute symptoms
- People using recovery device for health
- People with abnormal physical paresthesia

 The following persons / conditions are prohibited

For example: thrombosis, severe aneurysm, acute venous aneurysm, various skin infections, because it is easy to cause deterioration, so prohibited to use.

Do not use on body parts that have been operated on;

Do not use when having skin injury, redness, inflammation, congestion and other uncomfortable symptoms;

Patients with a history of heart disease, acute disease, skin disease, malignant tumor and physical abnormalities are not allowed to use;

Patients with bleeding tendency, osteoporosis, soft tissue diseases, stroke are not allowed to use.

- People who cannot express their will or who cannot operate the machine
- This product is scheduled for working 15 minutes once, and please do not use exceed 15 minutes each time. After use, please let the machine rest for 30 minutes to ensure the extension of the machine life cycle.

· The following cases are not allowed to use

- During severe fatigue
- Before and within an hour after meals
- After got drunk

Use Precautions

- ❗ Remove the massage accessories or other items that may affect the massage before use;
- ❗ Do not let children play with the products as toys;
- ❗ Before use, please confirm that the product is in good condition, please stop using it when have inquiries.
- ⊘ Do not Step on the product, and do not use the product as a toy, in order to avoid accidents.
- ⊘ Do not break it into pieces, remodel, or repair your product yourself.

Notes for the Using Place

- ⊘ Do not use the product near the stove, heater and electric blanket and other heating appliances, it may cause a fire..
- ⊘ Do not cover the product with towels or cloth.

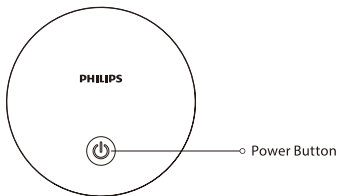
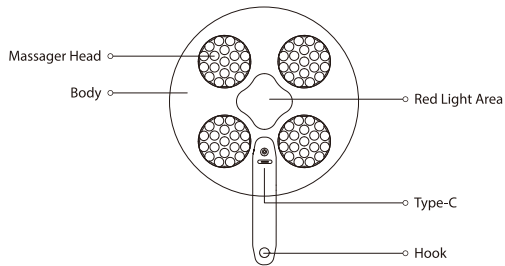
Maintenance Precautions

- ❗ Do not clean the machine during charging; ensure the product is off before cleaning to avoid danger.
- ❗ Wipe with soft cloth in daily maintenance or clean directly with water; do not use wire mesh, iron brush, sharp appliance, etc., volatile oil, thinner, alcohol, strong alkaline detergent, bleach and so on.
- ❗ Before using with water or cleaning with water, please use silicone hook to protect the charging port to prevent water or other liquid from entering. Before charging, Do not put the charging port near water and get wet, to avoid the risk of short circuit and electric shock.
- ❗ Products with simple earthquake prevention facilities, suitable for air railway, road and ship transportation. Rain, inversion and collision should be avoided during transportation.
- ❗ Please place this product away from children.
- ❗ When storing the appliance, cool it down first.
- ❗ Please check the product if any tearing or damage issue, and if so, Or if it has been misused, please return to the local distributor for checking

2 Product Features

This product adopts manual synchronous grasping movement, and uses 76 food grade silicone soft teeth, which can effectively knead and massage the scalp and promote the blood circulation of the scalp. When washing hair, it can be used with shampoo, which can deeply clean the scalp and thoroughly dredge the hair follicles.

3 Parts name

4 How to Use

Preparation items before use

Please remove the pendants and any accessories to avoid affecting the massage parts.

How to Use

1. Long press the power button for 1 second before the product starts working. The indicator light displays in cyan and the product enters the level 1 by default;
2. Short press the power button to switching the level from 1 to 3 in loop, the indicator light will flash corresponding times.
3. Red light function: start by default When the product power on.

Kindly reminder:

1. Power shortage reminder

When the remaining power is less than 10%, the button indicator and red light at bottom will flash to remind charging, and the product will automatically shut down after the battery power is exhausted.

2. Charging reminder

When charging, the indicator cyan light breath flashes away, and the charging time is about 3 hours. The product does not work when charging. When fully charged, the cyan indicator light is always on.

3. Automatic shutdown

The default time to automatically shut down is 15 minutes.

5 Maintenance

Cleaning Methods

- (1) Clean cloth can be used to wipe the dirty surface;
 - (2) For deep stains, wet the rag with detergent, and then gently wipe it off;
 - (3) Dry naturally.
- Do not place it under direct sunlight or in places with great moisture after use. If you do not use it for a long time, remove the dirt, and cover it with a cloth, and do not be contaminated with dust.
 - Waste method: In order to protect the environment, please do not waste at will, please waste according to the waste method of each part.
- !** Do not charge it during storage.

6 Diagnosis

This chapter lists the most common problems about this product. If you cannot find solutions to the problems you encountered in, contact the service hotline or the authorized service center in your area.

Problems encountered	Possible cause	Solutions
The product cannot power on	Low charging battery	Charge the product
	The product is charging	Use after charging
The product cannot be charged	The charging cable or the adapter is not well connected	Reconnect it
Stop Working suddenly	Low battery of the product	Please charge the product in time
	The product shut down regularly	Reboot
	The product is being blocked with something and turn off	Reboot under safe condition

After not having used for a long time, please do check the product is working properly before use.		
Regular product check	Is there any of the following problems	If those problems keep occurring, please stop to use it immediately. Please send it to the local service center for repairing.
	<ul style="list-style-type: none">· The charging cable or plug is overheat· The charging cable has scratch, and got burnt smell· There is an abnormal sound or vibrate· The product doesn't work properly after turn on· There are other anomalies or faults	

Pay attention to:

If the problem is still not solved, or the product is not working properly, Please contact the after-sales service personnel, do not repair it yourself.

7 Main Specification

Product Name	Philips Mini Head Massager
Product Model	PPM1301
Operating Voltage	3.7V ==
Operating Power	5W
Charging Voltage	5V ==
Charging Current	1A
Waterproof Grade	IPX7
Product Dimensions	90×90×60mm
Net Product Weight	0.18kg
Length of Charging Cable	50cm
Manufacture Date	See the product qualification certificate

8 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

Hongkong

Hotline: 852-3188-4987

Service Email: Philips.Massage.hk@centuryfield.com

Taiwan

Hotline: 0800-661-600

Service Email: Philips.massager@xander.com.tw

Warranty Card

Product Model		Customer Name	
Seller		Contact Number	
Date of Purchase		Serial Number	
Service Center		Customer Address	
Maintenance Content		Date	Service Personnel

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All rights reserved
The Philips trademark and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. used under license.