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©2022 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Hongkong AquaShield Health Technology Company Limited and Hongkong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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User Manual
使用說明書

Catalogue

1	Instructions for User	1
	Symbol Description	1
	People Restricted or Prohibited from Use	1
	Precautions for Use	3
	Precautions for Operation Place	3
	Precautions for Maintenance	4
2	Product Features	4
3	Parts Name	5
4	How to Use	5
	Preparations for Use	5
	Description of Indicator Lights	6
	Operation	6
5	Maintenance	7
	Cleaning Method	7
6	Troubleshooting	9

7	Main Specification	10
8	Service Guideline for Warranty	10
	Warranty and Service	10
	Warranty Card	12
9	Packing List	12

1 Instructions for User

Please read this instruction manual carefully before use and keep it properly

Symbol Description

In order to avoid personal injury or property damage, be sure to observe the followings.

- The following symbols respectively illustrate the items that you must observe.
-

 Indicate prohibition

 Indicate that you should do as instructed

People Restricted or Prohibited from Use

-  The following groups shall use the product with caution.

People under treatment at a medical institution or in the following conditions, should consult with a doctor or physiotherapist before using the product.

- For contact lens wearers, the contact lenses must be removed before using this product.
- Please keep this product away from water, fire and corrosive environment.
- If used in a specific environment, please comply with the corresponding regulations.
- Children between 6 and 12 years old should be supervised and guided by the guardian when using the product.

- People with sensitive skin, facial allergic dermatitis, and facial implantation of cosmetic plastic materials should use this product under the guidance of a physician. This product is a household electronic health care product other than a medical device, and is not intended for diagnostic, therapeutic or other commercial purposes.
Please charge the product only with the charging equipment provided by our company. Pay attention to the safety of using electricity.
- The product has a heating surface, so people who are not sensitive to heat should use with caution.

-  The following groups are prohibited from using the product

People whose doctors advise against exercising or stretching

Such as: thrombosis, severe aneurysm, acute venous aneurysm, various dermatitis, various skin infections. It is prohibited to use because it is easy to cause deterioration.

- People who have had eye surgery or have heart disease are forbidden to use the product.
- People with eye diseases such as glaucoma, cataract, retinal detachment, pink eye, etc. are not allowed to use the product.
- Pregnant or menstruating women are not recommended to use.
- Do not use the product while driving.
- Do not use this product when you are drunk or in poor health.
- Do not apply the product to other parts of your body other than the eyes.
- Children under the age of 6 are not allowed to use the product.

Precautions for Use

- ❗ People who wear glasses, hair accessories, etc. should take them off before using.
- ❗ If the product has not been used for a long time, be sure to confirm whether the switch, etc. are normal before using it again.
- ❗ Please close your eyes when using the product. Blurred vision may occur when you remove the product after use, which can be relieved by closing your eyes for 1-3 minutes.
- ❗ In case of any abnormality during use, please power off immediately.
- ❗ This product is not intended for diagnostic or therapeutic purposes.
- ❗ The product is not waterproof, please be carefully prevent water from entering the product, and do not directly immerse the product in water.
- ⊘ Do not deliberately put heavy pressure on the product.
- ⊘ Do not supply the product at a non-rated voltage, otherwise the circuit of the product will be damaged.
- ⊘ Do not drop the product or hit it with strong force, otherwise the product will be damaged.
- ⊘ Do not disassemble, modify or repair the product by yourself.
- ⊘ Do not pull the product with force, or stab it with a knife or sharp object.
- ⊘ Do not use the product in direct sunlight or at places where the room temperature is above 35°C.
- ⊘ Do not use the product in bathrooms or places with high humidity.

Precautions for Operation Place

- ⊘ Do not use it near the stove, or use it on heating appliances such as electric quilts and electric blankets, which may cause fire or failure.
- ⊘ Do not cover the product with a towel, cloth or other objects.
- ⊘ Do not use this product near water-filled bathtubs, showers, washbasins, or other high-humidity vessels.

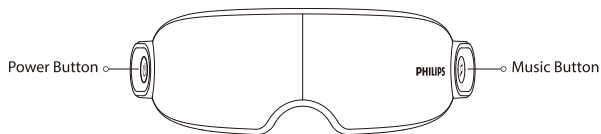
Precautions for Maintenance

- ❗ Turn off the product before maintenance.
- ❗ When the product will not be used for a long time, be sure to unplug the charger. Do not spray water on the product.
- ⊘ During maintenance, do not clean the product with volatile oil, diluent, strong alkaline detergent, bleach and other cleaning agents.
- ⊘ To clean the surface of the product, please use alcohol cotton. Do not directly clean with water.

2 Product Features

This product could promote blood circulation and relieve eye fatigue by imitating human hand pressure and hot compress function. There are multiple massage modes, suitable for different people for eye massage. With the innovative wireless music function, you can relax your eyes while enjoying your favourite music.

3 Parts Name

4 How to Use

Preparations for Use

- (1) Check that the device and accessories are complete.
- (2) Remove the pendant and decorations around the eyes before use to avoid affecting the massage part.
- (3) Adjust the elastic headband to the size that fits your head.
- (4) Turn on the Bluetooth function of wireless devices to connect the massager.

Description of Indicator Lights

Power Button Indicator			
	Colour	Status	Description
Normal working status	Orange	Always on	Hot compress function - on
	Green	Always on	Hot compress function - off
Charging status	Red	Breathing light	0%-20%
	Green	Breathing light	20%-99%
	Green	Always on	Fully charged
Low battery reminder	Red	Flashing frequently	Low battery

Music Button Indicator			
	Colour	Status	Description
Normal working status	Blue	Always on	Wireless device connected
	Cyan	Always on	Wireless device not connected
Searching new wireless device	Blue	Flashing frequently	Searching new wireless device

Operation

Button	Button Function	Operation Method	Description
⏻	Power on/off	Hold for 2 seconds	Turn on/off the massager
	Hot compress on/off	Double click	Double click the Power Button (twice within 0.5 seconds) to turn on/off the hot compress function.
	Mode switch	Short press	This product will enter the Eye Energetic Mode by default when it is turned on. Press the Power Button, and the massage mode will cycle through Eye Energetic Mode, Eye Care Mode, Eye Warming Mode, Sleep Aid Mode.

Button	Button Function	Operation Method	Description
	Searching new wireless device	Hold for 2 seconds	Hold the Music Button for 2 seconds to search for a new wireless device, and the indicator will flash frequently in blue. After successful pairing, the indicator will remain on in blue.
	Music on/off	Double click	Double click the Music Button (twice within 0.5 seconds) to turn on/off the music.
	Next track	Short press	Short press to play the next song.

5 Maintenance

Cleaning Method

- (1) The dirt on the product housing can be wiped off with a clean cloth;
- (2) The dirt on the eye mask can be removed with a clean cloth stained with alcohol;
- (3) For deep dirt, dampen a cloth with a neutral detergent, then wipe gently.
- (4) Prevent water from entering the product;
- (5) Place the product and dry it naturally.

- ! Do not put the product in direct sunlight or at places with high humidity after use.
- ! If it will not be used for a long time, cover it with a cloth after removing the dirt.
Disposal: Do not discard the product arbitrarily. Please dispose of it according
- ! to the disposal method for each part to protect the environment.
During maintenance, the product should be turned off.
Before discarding the appliance, the battery must be removed.
- ! The device must be powered off when removing the battery.
- ! Batteries should be disposed of safely.
- !
- !

Battery Removal Steps

- (1) Cut off the external power supply and remove the product housing.
- (2) Unplug the power supply of the main board
- (3) Remove the battery.
- (4) Please place the waste batteries at a fixed recycling site.
- (5) Please comply with local environmental protection laws and regulations and disposal guidelines.

6 Troubleshooting

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center at the region where you are located.

Basic Fault	Possible Cause	Basic Troubleshooting
Unable to power on	Low lithium battery	Charge and restart to check the problem
Weak airbag compression	loose headband	Adjust the length of the headband according to individual demands, so that the massager can be worn comfortably around the eyes.
Unable to connect wireless devices	The Music Button is pressed by mistake, and the music function is turned off.	Hold the Power Button for 2 seconds to turn off, and then restart the music function.
Low playback volume of music	Low playback volume of cell phone	Adjust the playback volume of cell phone

Note:

If the fault cannot be eliminated or the product still cannot be operated ideally, please contact the after-sales service personnel, and do not disassemble or repair by yourself.

7 Main Specifications

Product Name	Philips Eye Massager
Product Model	PPM2701
Rated Voltage	3.7V ---
Operating Power	4W
Charging Voltage	5V ---
Charging Current	500mA
Battery Capacity	1200mAh
Product Dimensions	215×114×80mm
Net Weight	301g
Manufacture Date	See the product qualification certificate

8 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

HongKong

Hotline: 852-3188-4987

Service Email: Philips.Massage.hk@centuryfield.com

Taiwan

Hotline: 0800-661-600

Service Email: Philips.massager@xander.com.tw

Warranty Card

Product Model		Customer Name	
Seller		Contact Number	
Date of Purchase		Serial Number	
Service Center		Customer Address	
Maintenance Content		Date	Service Personnel

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