

PHILIPS

頭部按摩器


迷你頭部按摩器

PPM1301




## Quick Start Guide 快速入門指南

1. Long press the power button, the product begins to work, the indicator light displays in cyan, and the product starts from level 1 as default.

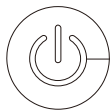


Long press the power button,  
the indicator light shows cyan

2. When the product is on, long press the button to switch off the product.

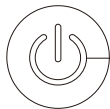


Long press the power button,  
and the indicator light turns off

3. Short press the power button to switch level from 1/2/3 in loop. The indicator lights will flash corresponding times.

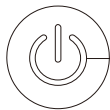

Short press the power button

4. Red light function: start by default when the product is on.

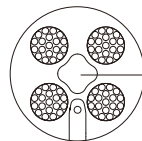

Red light module

- ❗ 1. Please remove hair clips and other accessories before massage to avoid skin injury and machine damage.
- 2. It is recommended to start from low level to avoid uncomfortable feeling by strong stimulation.
- 3. When using the 4 massage heads, please gently fit to the scalp to avoid hair hanging around massage heads.
- 4. Do not put your fingers between the massage heads to avoid injuries.
- 5. The suggested using time of the same skin area in a single day is not more than 10 minutes.
- 6. Do not put the charging port near water or get wet, to avoid the risk of short circuit and electric shock.
- 7. Slightly hair loss may happen at first use, and it will decrease gradually after regular use. If heavy hair loss occurs, stop using the product and consult your doctor immediately.
- 8. Dandruff may appear at first use, it is a normal phenomenon, it will decrease gradually after regular use.



請在以下網站連接中了解詳情  
[www.philips.com](http://www.philips.com)

本手冊內容與實物可能存在差異，請以實物為準，如產品有任何更新恕不另行通知。

©2022 AquaShield

All rights reserved

The Philips trademark and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. used under license.


This product has been manufactured by and is sold under the responsibility of Hongkong AquaShield Health Technology Company Limited and Hongkong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

水護盾是飛利浦淨飲水解決方案、熱水解決方案、衛浴解決方案、水杯及個人按摩器唯一品牌官方授權運營主體。

Rev A JUL 22




1. 長按電源按鈕，產品開始操作，指示燈顯示青色，產品設定進入一段。

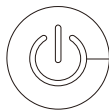


長按電源按鈕，  
指示燈顯示青色

2. 產品在操作狀態下，長按按鈕操作停止。

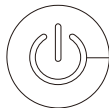


長按電源按鈕，  
指示燈熄滅

3. 短按電源按鈕，青色指示燈閃爍切換模式提示，依次按照一段/二段/三段循環切換。

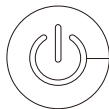

短按電源按鈕

4. 紅光功能：在操作狀態下，啟動。

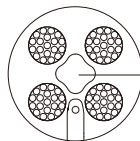

產品底部紅光亮起

- ❗ 1. 請在按摩前除下頭飾等物品，避免皮膚及機器損傷。
- 2. 首次使用時建議從低段位開始，避免刺激。
- 3. 使用時請將4個按摩頭輕柔貼合頭皮，避免按摩頭懸空纏繞頭髮。
- 4. 使用時請勿將手指伸入按摩頭之間，避免夾傷。
- 5. 每天同一部位使用時間建議不超過10分鐘。
- 6. 充電前需確保充電口乾燥無異物，避免短路及觸電風險。
- 7. 初次使用可能會有脫髮增加的感覺，規律按摩調理後會逐漸減少；若出現大量脫髮，請停止使用並諮詢醫生。
- 8. 初次使用可能會有頭皮屑增多的情況，屬於正常現象，規律按摩調理後會逐漸減少。