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This product has been manufactured by and is sold under the responsibility of HongKong AquaShield Health Technology Company Limited and HongKong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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User Manual
使用說明書

Catalogue

1	Instructions for User	1
	Symbol Description	1
	Applicable Persons	1
	Use Precautions	2
	Maintenance Precautions	3

2	Product Features	4
---	------------------	---

3	Parts Name	5
---	------------	---

4	How to Use	6
	Preparations before Use	6
	Operation Method	7
	Charging Method	8
	Description of Indicator Light	9

5	Maintenance	10
	Cleaning Methods	10

6	Diagnosis	11
---	-----------	----

7	Main Specification	12
---	--------------------	----

8	Service Guideline for Warranty	13
	Warranty and Service	13
	Warranty Card	14

9	Packing List	14
---	--------------	----

1 Instructions for User

Please read and keep this specification carefully before use.

Symbol description

To avoid causing personal harm or property damage, be sure to follow the following.

- Use the following illustration to separately illustrate what you want to follow.

Prohibit

Represents needs to be done as instructed

Applicable Persons

The following populations should be used with caution

- People with leg nerve impairment or skin sensory disorders.
- Pour metal, silicon or plastic into the hand for fracture or plastic surgery.
- The appliance has a hot surface function and must be used with care by people who are not sensitive to heat.
- People with leg or knee sequelae due to traffic accidents.
- Patients with or have had leg diseases (tendon injury, arthritis deformity, fractures, varicose veins, etc.).
- People with implanted medical electronic devices who are susceptible to electromagnetic failures such as pacemakers.

- People deemed to have lost a temperature sensation.
 - People with a malignancy.
 - People with a heart disease.
 - Unease in the beginning of pregnancy People who regularly or shortly after giving birth.
 - Diabetes causes people with perceptual disorders.
 - People with scars on their skin.
 - People need quiet and people in poor health.
 - People with osteoporosis, people with severe fractures, sprains, muscle pain, acute (head) pain.
 - Body temperature is 38°C (fever).
 - With acute inflammatory symptoms, burnout, cold, blood pressure changes and other symptoms of the strong, weak time.
 - Others who are treated at a medical facility.
-

Use Precautions

Each use time is 30 minutes, please check your knee condition, and then adjust the number of uses appropriately.

It is recommended to use the level 1 strength at the beginning of this product stage.

When using this product, your knees may turn red in a short time. This does not mean that there is a problem. If this symptom continues to resolve, consult the surgeon.

When wearing silk stockings, the strap on the product may break through your silk stockings, and you can remove them to use.

If you feel pain around your legs or knees, please stop using the product immediately.

- ❗ If you have any problems with your skin or legs during use, please stop using this product immediately and consult with your physician.
 - ❗ It is recommended not to overuse the massage function.
 - ⊘ No use when sleeping.
 - ⊘ Do not cover it with a quilt or blanket.
 - ⊘ Do not lie down or sit on the product.
 - ⊘ Prohibit to bring huge impact and load to products.
 - ⊘ Do not use the products in the bathroom or in the shower room.
 - ⊘ Do not wear the body for any other event.
-




Maintenance Precautions

- ❗ When cleaning, always unplug the adapter from the home power outlet. Please properly handle the packaging bag of this product to keep it away from infants and young children.
- ⊘ This product contains lithium battery, do not remove, modify, water immersion product (controller) or throw into the fire, otherwise it may cause product damage.
- ⊘ Do not keep it within the contact range of children and infants. In addition, do not let them contact or operate this product. Otherwise, it may cause burns, electric shock, or personal injury.
- ⊘ Do not damage or modify, or vigorously bend or pull the power cord out. Also, do not place heavy objects or squeeze the power cord on the power cord.

2 Product Features

This product through the imitation of hand flap with hot compress function, can promote knee blood circulation, relieve muscle and joint pain, a variety of temperature and massage strength is available, suitable for different groups of people for knee massage maintenance. With the extension belt for use, it can also be used for other parts of the body maintenance.


3 Parts Name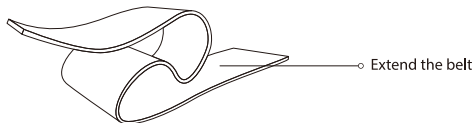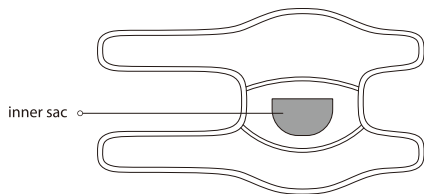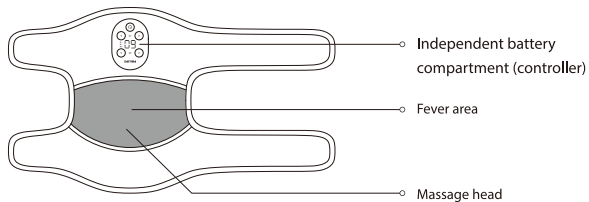


4 How to Use

! Preparations before Use

1. Take the magnetic suction buckle behind the battery compartment together and the magnetic suction buckle on the body, so that the battery compartment is firmly installed on the body;

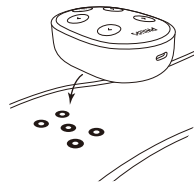
2. Put the massager into your knee and adjust the tightness through the Velcro on the strap.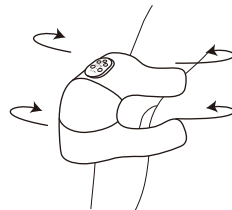

! When wearing silk stockings, the strap on the massager may break through your silk stockings. Please remove the silk stockings and use them again.

Use the operation: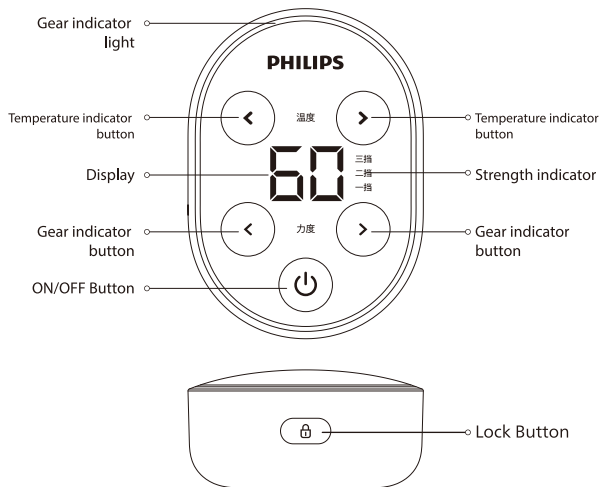

Use the operation

Switch button

1. In standby state, press the switch button for 2 seconds to turn on / off, and the default heating is 50°C, and the vibration strength is Level 1. Voice broadcast "Hello, Philips";
2. Short press the switch button to turn on / off the heating function, and voice broadcast "on heating" / "off heating";

3. The machine is scheduled for 30 minutes by default, and automatically shut down after 30 minutes to play shutdown music.

Adjust the keys

1. Short press the strength "<" and ">" keys to adjust the vibration strength (Level 1, Level 2, Level 3, close the vibration cycle switch);
2. When adjusting, the corresponding strength indicator light is on, the display screen is on synchronously (L1, L2, L3), and automatically switch to the temperature display after 2 seconds. Broadcast "1", "2", "3" and "Close vibration" each time.

Temperature adjustment button


1. Short press the temperature "<" and ">" key to adjust the temperature (40°C, 45°C, 50°C, 50°C, 55°C, 60°C cycle switch), the middle display shows the current temperature; each accompanying voice broadcast "40°C", "45°C", "50°C", "50°C", "60°C".

Lock screen button

When the machine is working normally, long press the 2-second lock screen button to lock / open the screen, each operation accompanied by voice broadcast "lock screen" and "open screen".

Charging Method

1. Plug the charging plug into the charging hole on the side of the battery compartment;
2. Insert the charger plug into the 220V~ / 50Hz power socket;
3. When charging, the indicator lamp ring is blue breathing light until the blue light is fully charged;
4. It takes about 3 hours to fully charge.
5. Please use the 5V/1A charger of the authorized manufacturer for charging.

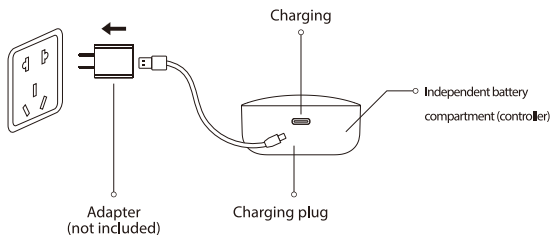- ❗ When the power is less than 10%, the indicator ring shows a blue flash for 5 seconds, and the voice broadcast "battery is low, please charge in time", and shut down.
- ❗ Please turn it off and charge it again. You cannot use it while charging.

Description of Indicator Light

Battery coil lamp	State	Description
Orange	Long bright	Turn on heating
Blue	Long bright	Turn off the heating
Orange	Breathing light	Turn on the heating and lock in the screen
Blue	Breathing light	Turn on the heating and lock in the screen
Blue	Flash for 5 seconds	Power < 10%
Blue (in power-off state)	Breathing light	In charging
Blue (in power-off state)	Long bright	The power is full

5 Maintenance

Cleaning Method

1. Battery compartment (controller) nursing: the battery compartment (controller) display screen with touch control function should be properly protected, and please wipe with soft cloth when cleaning.
2. Cloth area care: water washing is not supported in the cloth area. When needing cleaning and maintenance, please use alcohol sterilization and sweat stains or wipe with a clean and wet towel.

- ❗ When maintenance, be sure to pull out the power cord, but do not use wet hands to remove the wire.
- ❗ When not in use for a long time, it is recommended to charge once a month.

6 Diagnosis

This chapter lists the most common questions about this product. If you cannot find answers to the questions you encountered in them, please contact the service hotline or the authorized service center in your area.

Battery coil lamp	State	Description
Unable to start	The product is still connected to the power outlet	Remove the charging cable from the body
	Battery power is exhausted	Please charge
	The power button is not firmly pressed	Press the power key firmly until it is in place
The charging indicator light is not lit	The USB charging cable is not properly plugged into the power outlet	Please insert the USB charging cable into the power outlet correctly
Even if it is just overcharged, it only lasts for a few minutes	Battery life exhausted (approximately 5 years)	Please consult with dealer (do not replace battery)
The body is not heated	This product may not be positive when used outside of the recommended ambient temperature range often run	Use in the recommended operating temperature range of 0 to 40 °C
The product stops working during the operation process	The battery power is exhausted	Please charge if charge
	After 30 minutes, the product will automatically shut down	To continue using, please reboot

After not using it for a long time, be sure to confirm that the function is normal before you can use it again.		
Regular product check	Is there any of the following faults	If the above faults occur, in order to prevent accidents, please stop the use immediately, and cut off the power supply. Please send it to our repair site.
	<ul style="list-style-type: none"> The power cord or plug is hot The power cord is deeply scarred or deformed and burnt There is an abnormal sound or vibration at work With the switch on, it does not work properly There are other abnormalities or faults 	

pay attention to:

If the fault cannot be eliminated or the product is still not ideal operation, contact the after-sales service personnel, do not remove or try to repair.

7 Main Specification

Product name	Philips Knee Massager
Product Model	PPM5301
Rated Voltage	5V ---
Power Rating	7W
Charging Voltage	5V ---
Charging Current	1A
Product Size	500×290×30mm
Net Weight of Product	310g
Battery Capacity	2500mAh
Manufacture Date	See the product qualification certificate

8 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

HongKong

Hotline: 852-3188-4987

Service Email: Philips.Massage.hk@centuryfield.com

Taiwan

Hotline: 0800-661-600

Service Email: Philips.Massage.hk@centuryfield.com

Warranty Card

Product Model		Customer Name	
Seller		Contact Number	
Date of Purchase		Serial Number	
Service Center		Customer Address	
Maintenance Content		Date	service personnel

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All rights reserved
The Philips trademark and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. used under license.