PHILIPS

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PPM2501


Quick Start Guide 快速入門指南


1. Remove your contact lenses, eyeglasses, and accessories around the eye.

2. Adjustable strap to make your eyes feel comfortable before turn it on.

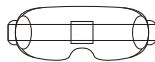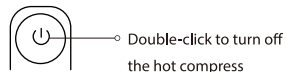
3. Long press the  key for 2s to turn on the massager. Switch to beauty mode by default when the product is on.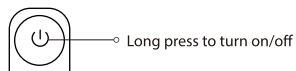


4. Press  to switch to Beauty mode / Comfort mode / Sleep mode / Energizing mode

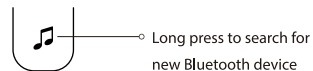
5. Turn on or any massage mode by default. Double-click  to turn off the hot compress.


6. Press the short button  switch to the next song.

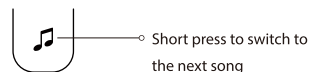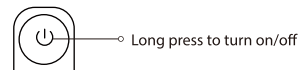
7. Long press  search for new Bluetooth device / disconnect the currently connected Bluetooth device, search for new Bluetooth device, after the match voice broadcast "Bluetooth connection success", then play the Bluetooth device music.

8. Double-click  turn off / restart the music function.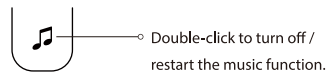

9. The default time to shutdown is in 15 minutes, if you want to continue, long press button  to reboot.

- 1. Please close your eyes when using the product. After using the product, blurred vision may appear. Resting with your eyes closed for 1-3 minutes.
- 2. Use alcohol wipes to clean the product surface after using it. Do NOT wash it in water.

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This product has been manufactured by and is sold under the responsibility of HongKong AquaShield Health Technology Company Limited and HongKong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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All rights reserved
The Philips trademark and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. used under license.