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This product has been manufactured by and is sold under the responsibility of HongKong AquaShield Health Technology Company Limited and HongKong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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User Manual
使用說明書

Catalogue

1	Important Information	1
	Symbol Description	1
	Applicable Persons	1
	Use Precautions	3
	Notes for the use place	3
	Maintenance Precautions	4
2	Product Features	4
3	Parts Name	5
4	How to Use	5
	Precautions before use	5
	Operation Methods	6
5	Maintenance	7
	Cleaning Methods	7
6	Diagnosis	7

7	Specification	9
8	Service Guideline for Warranty	10
	Warranty and Service	10
	Warranty Repair	11
	Warranty card	12
9	Packing List	12

1 User must read

Please read and keep this specification carefully before use.

Symbol description

To avoid causing personal harm or property damage, be sure to follow the below.

- Use the following illustration to separately illustrate what you want to follow.
-

 Prohibit

 Must follow the instructions before use

Use Precautions

 The following persons/ conditions should be used with caution

During treatment, or with the following conditions, please consult a doctor or physiotherapist before use.

- Please remove the contact lenses before use.
- Please keep the product away from the water source, fire source and corrosive environment.
- For use in specific environments, please comply with the appropriate regulations.
Children between the ages of 6 and 12 should be used under the care of their guardian.

- For sensitive skin, facial allergic dermatitis and those who have been injected with cosmetic materials on their face, please use this product under the guidance of doctors.
- This product is a household electronic health care product, not a medical device, not suitable for diagnostic, therapeutic and other commercial purposes.
Please only use the charging equipment configured by our company for charging.
- Please pay attention to use electricity safely.
The device has a heating surface. People who are insensitive to heat need to be aware.

 The following persons/ conditions are prohibited

Doctors recommend banning people who exercise or stretch

For example: thrombosis, severe aneurysm, acute venous aneurysm, various skin infections, because it is easy to cause deterioration, so prohibited to use.

- Do not use this product while driving.
- Do not use this product while get drunk or feeling unwell.
- People whoever had eye surgery or patients with heart disease are banned to use.
People with glaucoma, cataract, retinal detachment, pinkeye or any other similar symptoms are banned to use.
Pregnant women and menstrual women are not recommended to use.
- Do not use on the other parts of the body other than the eyes.
- Do not wear this product during sleeping.
- Children under 6 years old are prohibited.
-

Use Precautions

- ❗ Remove glasses, hair accessories and other things around your eyes before use.
Be aware to check the switch is in normally working before use each time.
- ❗ Please close your eyes when using the product. After using the product, blurred vision may appear. Resting your eyes and closed for 1-3 minutes.
If the product is not working properly, please turn off immediately.
- ❗ This product is not suitable for diagnostic and therapeutic purposes.
- ❗ The product is not designed in water-resistant, do not use the product near water, and do not directly soak the product in water.
Do not put pressure on the product.
- ⊘ Do not use non-specified voltage power supply, it may cause different product circuit damage.
Do not throw the product.
- ⊘ Do not break the products into pieces, or repair them yourself.
- ⊘ Do not pull the product hard or stab it with a knife or a sharp object.
- ⊘ Do not use it at room temperature above 35°C or under direct sunlight.
- ⊘ Do not use the product near bathtubs, showers, basins or other vessels containing water.

Notes for the use place

- ⊘ Do not use the product near the stove, heater, electric blanket and other heating appliances, it may cause a fire.
- ⊘ Do not cover towels or cloth on the product.
- ⊘ Do not use this product near bathtubs, showers, basins or other vessels containing water.


Maintenance Precautions

- ❗ Please always turn off the product for maintenance.
- ❗ Please always unplug the power cord for not using it for a long time.
- ⊘ Do not pour water on the product.
- ⊘ Please do not use volatile oil, thinner, strong alkaline detergent, bleach and other cleaner to clean the product.
- ⊘ Use alcohol wipes to clean the product surface after using it and should not clean with water.


2 Product Features

This product uses air pressure massage, vibration, hot compress to give you brighter and healthier eyes. Preset a variety of care modes, suitable for different groups of people for eye relaxation. Innovative Bluetooth music playback function can comfort your eyes and enjoy your favorite music.

3 Parts name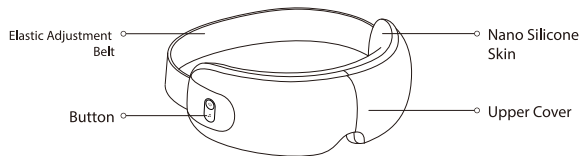


Eye massager 7 massage area distribution map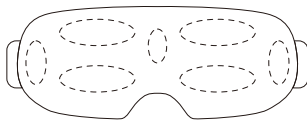

4 How to Use

! Notes before use:

- (1) Please check that the machine and accessories are complete.
- (2) Please remove contact lenses, eyeglasses or any other accessories around the eye to avoid affecting the massage effect.
- (3) Adjust the elastic band to fit your head, and make your eyes feel comfortable before turn it on.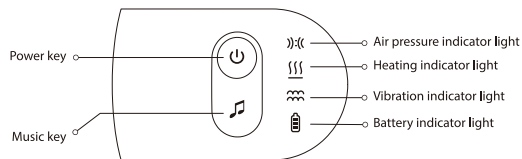

⏻ Power Key

1. Long press 2s to turn the power on / power off.
2. Short press to switch to the 4 different modes.
 - (1) Beauty Mode: air pressure (moderate), vibration, hot compress is on, and the function display lights of air pressure, hot compress, vibration and battery show in cyan.
 - (2) Protection Mode: air pressure (moderate), hot compress is on, and the function display lights of air pressure, hot compress and battery show in cyan.
 - (3) Sleeping-aid Mode: air pressure (soft), hot compress is on, and the function display lights of air pressure, hot compress and battery show in cyan.
 - (4) Warm Mode: hot compress is on, and the function display lights of hot compress, battery show in cyan.
3. Each mode will turn on heating by default, press twice the power key to turn off heating, voice broadcast: "turn off heating", The hot compress green light will be off.
4. The default working time is 15 minutes, and then automatically shut down. If you need to continue, press the power key again.

🎵 Music key

1. Built-in Music Mode:

Short press the key to switch the next music.

Double-click to turn off/on music (The light is off when the music is off, the light is cyan when the music is on)

2. Bluetooth Music Mode:

Long press to search for new Bluetooth device/ disconnect current Bluetooth device and search for new Bluetooth device.

Short press to switch the next song.


Double-click to turn off/on music (The light is off when the music is off, the light is cyan when the music is on)

Charging Method

1. Fold the product to the angle below to expose type-c, charge the interface, and then insert the charging plug into the charging hole of the product;

2. After the charger power is turned on, the battery indicator light shows in red;

3. It takes about 2.5 hours to fully charge the battery. After the full charge, the battery indicator light shows in cyan.

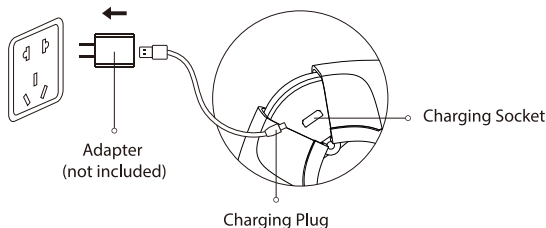❗ When the power is low, the voice prompts "the battery is low, please charge in time", and then the red light flashes before playing the shutdown music. When charging, the battery light is always on, and the battery light is always on.

During the charging process, the product does not work.

❗ Use your native data cable and a self-matching 5V 1A adapter.

❗ Do not squeeze the power cord, so as to avoid damage.==

5 Maintenance

Cleaning Method

1. After using the product, if there are oil stains, please wipe with a dry soft cloth;

2. If the product is particularly dirty, alcohol cotton can be gently wiped on the holster;

3. After use, please put the product to the safety place, and put the bag in the packaging box;

4. Please do not place heavy objects on the product to avoid damp products and contamination.

6 Diagnosis

This chapter lists the most common questions about this product. If you cannot find answers to the questions you encountered in them, please contact the service hotline or the authorized service center in your area.

Fault description	The possible causes of the cause	Basic exclusion method
Unable to boot	The on / off button is not on	Press the on / off button again
	The battery runs out	charge
	Set the time to	Press on / off again/push-button
Unable to charge	The plug is not inserted	Plug in
The indicator light is dark or flashing	The battery power is insufficient	Please charge
Air bag pressure strength is weak	The straps are too loose	Adjust the length of the head wear strap according to the individual to wear the massage machine comfortably in the eyes
Bluetooth is not connected	The music key is mistaken	Long press the shutdown button for 2 seconds, then restart the massage machine again, long press the music button, you can resume into the Bluetooth pairing mode
The Bluetooth has been connected successfully, and it still cannot be played	Not connected to the corresponding Bluetooth system	Long press the music to disconnect the current Bluetooth device and search for new devices
Bluetooth play music sound is small	The cell phone playback volume is too low	Adjust the mobile phone playback volume
The use time is short	Unnormal temperature	Please use it at ambient temperatures from 10°C to 35°C

After not using it for a long time, be sure to confirm that the function is normal before you can use it again.		
Regular product check	Is there any of the following faults	If the above faults occur, in order to prevent accidents, please stop the use immediately, and cut off the power supply. Please send it to our repair site.
	<ul style="list-style-type: none"> • Is there any of the following faults • The power cord or plug is hot • The power cord is deeply scarred or deformed and burnt • There is an abnormal sound or vibration at work • With the switch on, it does not work properly • There are other abnormalities or faults 	

Pay attention to:

If the fault cannot be eliminated or the product is still not ideal operation, contact the after-sales service, do not remove or try to repair.

7 Main Specifications

Product Name	Philips Eye Massager
Product Model	PPM2501
Rated Voltage	3.7V 
Power Rating	4.5W
Product Size	215.5×113×78.5mm
Net Weight of Product	About 0.325kg
Manufacture Date	See the product qualification certificate
Operation Norm	GB4706.1-2005 , GB4706.10-2008

8 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

HongKong

Hotline: 852-3188-4987

Service Email: Philips.Massage.hk@centuryfield.com

Taiwan

Hotline: 0800-661-600

Service Email: Philips.Massage.hk@centuryfield.com

Warranty Card

Product Model		Customer Name	
Seller		Contact Number	
Date of Purchase		Serial Number	
Service Center		Customer Address	
Maintenance Content		Date	service personnel

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All rights reserved
The Philips trademark and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. used under license.