PHILIPS

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
PPM7502


Quick Start Guide 快速入門指南


1. It is recommended to charge the device for 6 hours before first use. Do not start the device while charging.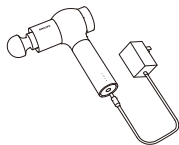


2. When the fourth power indicator remains on, it indicates that the device is fully charged. When the first indicator flashes (indicating the remaining power is less than 10%), please charge in time.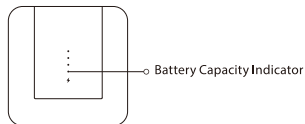


3. To install the massage head, insert it into the front round hole of the massage gun until the head is aligned with the mark line.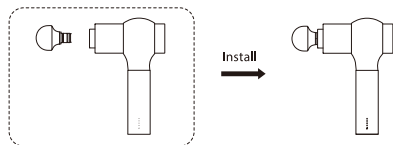


4. Hold the button to turn on the device, the light ring will light up in a section and enter level 1. Each time you press the button, the gear will increase by 1 level. If you continue pressing the button at level 5, the device will turn off.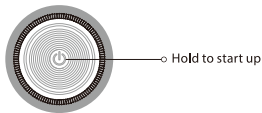


5. Intelligent intensity sensor indicator: 3 seconds after key operation, the light ring will enter into intensity indication mode and the color changes with the intensity; the light changes with the intensity. The blue indicator indicates a pressure of 0-2 kg, the green indicator indicates a pressure of 2-4 kg, the yellow indicator indicates a pressure of 4-6 kg and the orange indicator indicates a pressure of >6 kg.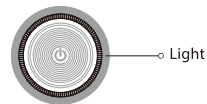

6. The device will shut down automatically after continuous use for 10 minutes. If you need to use again, please restart it.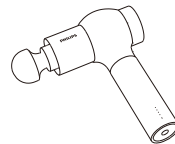

7. The massage head will not output power if it is stalled. This is to protect the device against abnormal operation and switch from operation mode to standby mode. Short press the button to restart the device.

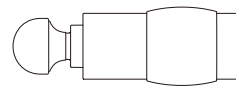! Please choose the massage intensity reasonably according to your own physical condition, and each single use should not exceed 10 minutes.

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This product has been manufactured by and is sold under the responsibility of Hong Kong AquaShield Health Technology Company Limited and Hong Kong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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All rights reserved
The Philips trademark and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. used under license.