Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

©2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

PHILIPS

Massage Gun

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PPM7303

User Manual

Contents

- 1 Readme
 - Symbol Description
 - People Restricted or Prohibited from Use
 - Precautions for Use
 - Precautions for Maintenance
-

- 2 Product Features
-

- 3 Components
-

- 4 How to Use
 - Preparations Before Use
 - Installation
 - Operation
 - Charging
-

- 5 Indicator Description
-

- 6 Maintenance
 - Disposal
-

-
- 7 Troubleshooting
-

- 8 Technical Parameters
-

- 9 Service Guide for Product Warranty
 - Warranty and Service
 - Warranty Card
-

- 10 Packing List
-

1 Readme

Please carefully read the operation manual before use and keep it properly. The manual is available at www.philips.com

Symbol Description

To avoid personal injury or property damage, be sure to observe the contents below.

- The following symbols are used to illustrate what you should observe.

 Indicates prohibition

 Indicates the actions that need to be subject to instructions

People Restricted or Prohibited from Use

 Important Information

This product cannot be used under any of the following circumstances without the prior consent of your doctor.

The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

Children should be supervised to ensure that they do not play with the appliance.

If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.

This appliance contains batteries that are only replaceable by skilled persons.

Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.

- The occurrence of pregnancy, diabetic complications (e.g. neuropathy or retinal damage), pacemaker wearing, recent surgery, epilepsy or migraine, disc herniation, anterior vertebra, vertebrae detachment, or ankyloses of vertebra, recent arthroplasty surgery or IUD, Kirschner wire or steel plate and other health-related issues. These contraindications do not mean that you are not allowed to use the Product. It is recommended to consult your doctor before use.

The underway research is about the impact of the vibration training on specific

- medical diseases. This stands a good chance to reduce various contraindications mentioned above. The practice proves that under many circumstances, the incorporation of vibration training into a therapeutic regimen is also a form of therapy, but it must be done at the suggestion or in the company of a doctor, expert or physical therapist.

! The product is not allowed to be used by the following groups/under the following circumstances

- It is forbidden for minors to use the product. If injured, it is not allowed to use it. Please consult your doctor before using it in case of muscle injury.
- Do not use the device on non-dry and non-clean skin surface. You can use this product with clothes on. Gently press and move it on the skin for about 60 seconds per area.
- Do not use the device on the part above the neck and with less muscle coverage. It is forbidden to use it on the bones. Under the circumstance of not causing pain or discomfort, only use the device on the soft tissues of the body. Do not use the device on the occurrence of bruising, pain or discomfort.
- Do not place any object in the vent of the device.
- Do not immerse the device in water, or let water get into the vent of the device.
- Do not put the device into the fire or place it near the fire source. Do not drop the device intentionally.
- Do not charge the device with a charger with a voltage exceeding 9V.
- Disassembly of the device in any way is prohibited. Do not place your fingers, hair or other body parts near the shaft or back of the massage head due to the risk of being clamped. It is forbidden for people who are unable to express their will or who cannot operate the machine themselves to use this Product. Children are not allowed to use the product. Keep children away from it when using it.

- Please stop immediately when you feel lower back pain, dizziness, vomiting, palpitation and other abnormal body reactions during use.
- This product is set for 10-minute timing at a time. Do not use it for no more than 20 minutes each time. After each use, stop the device for 30 minutes to ensure the extension of its service life.
- Do not use the product under the following circumstances:
 - When you are severely tired
 - Within one hour before and after meals
 - When you are drunk

Precautions for Use

- !** Do not wear excessive clothes during use.
- !** Keep away from liquid or heat source.
- !** Do not use any products other than the original binding charging base and DC charger. Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.
- !** Do not use the product during charging.
- !** After charging, disconnect the power supply in time.
- !** Stop the device immediately if it cannot work properly or you feel abnormal.
- !** Follow the instructions when using the product.
- !** Before each use, carefully check whether the product can be used normally according to the operation instructions in this manual.
- ⊘** Do not load and ballast the power cord or cause damage to it.
- ⊘** Do not stand on the device or use the device as a toy so as to avoid accidents.
- ⊘** Do not disassemble, modify or repair the product by yourself.
- ⊘** Do not use the device if your body is wet.


Precautions for Maintenance

- ⚠ Be sure to disconnect the power supply before maintenance.
- ⊘ Do not drop the main unit and its charging base. Do not strike it strongly.
- ⊘ Do not pour water on the product.
- ⊘ Do not use the product in a bathroom or in places with high humidity.
- ⚠ Be sure to confirm whether the switch is normal in advance before it is used again if it has not been used for a long time.
- ⊘ Do not use volatile oil, thinner, alcohol, strong alkaline detergent, bleach or other cleaning agents for maintenance.
- ⚠ The product is equipped with a simple shockproof facility, and is suitable for air, railway, road and ship transportation. Keep away from rain and snow splashing, inversion and collision during the transportation.
- ⚠ Keep out of reach of the children.
- ⚠ Make the device cool down before storing.
- ⚠ Check the device for any sign of wear or damage on a regular basis. If there is such sign, or if the device has been misused, contact the supplier for maintenance before reuse.

2 Product Features

This product is available for relaxing the tense muscles and releasing the post-exercise soreness and fatigue by massaging the soft tissues of the body with high frequency. A variety of massage heads can be combined to massage muscles all over the body. Equipped with a portable storage bag, it is easy to carry out.

3 Components

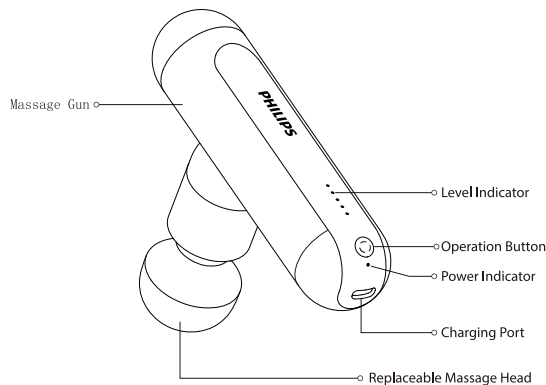Metal Hot Compress Head

Apply stainless steel surface with hot compress to relieve muscle tension and enhance the massage impact.

Rubber Spherical Head Pro

Made of sports equipment material, it is soft touch, elastic, skin friendly and easy to clean.

Pointed Tip Head

Suitable for impacting deep tissues and relieve muscle soreness quickly.

Claw-shape Attachment

Installed on the pointed tip head. Suitable for gentle massage with its 6 silicone nodes.

U-shaped Head

Suitable for concentrated massage of specific sore areas.

Wedge-shape Head

Suitable for muscle edge tissues massage and making better muscle shape.


4 How to Use

Preparations Before Use


1. Check if the device and its accessories are complete.
2. Remove the pendants and decorations during use, so as to avoid affecting the massage part.
3. Install the massage head in place.

Installation

According to the message demand, insert the appropriate message head into the front round hole of main unit, and then press it firmly to the end, as shown in the figure below, indicating that it is installed in place.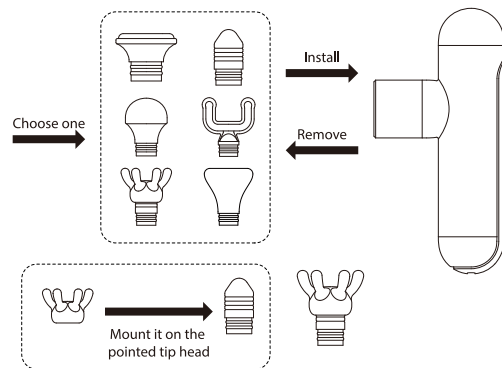

Installation effect is shown in the figure below: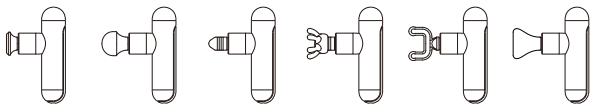

Operation

1. Start the device. Press and hold the operation button for 1 second to start the device, then the device will enter Level 0 by default. Press the button once to increase one level, with the maximum of 5 levels. When it goes to Level 5, press the button to enter Level 0 again, and repeat in this way.

2. Press and hold the operation button to switch off the device at any level.

3. The device will automatically switch off after 10 continuous use for 10 minutes.

Restart the device for use again.

4. The massage head is suspended after being blocked. This is to protect the device against abnormal operation. Press the button to restart the device.

Power Capacity Indicator

① Power-on State:

Red lamp: current power

0-10%

Yellow lamp: current power

11-50%

Green lamp: current power

51-100%

② Charging State:

Red breathing lamp: current

power 0-10%

Yellow breathing lamp:


current power 11-50%

Green breathing lamp:

current power 51-99%

Green remains on: fully

charged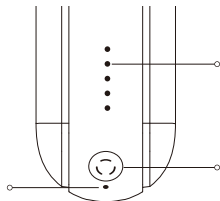

Level Indicator

1 lamp indicates Level 1,

2 lamps indicate Level 2,

And so forth

Operation Button

Press and hold to start/ be

on standby

Press to switch the levels

Charging

1. Before the first use, charge the device for 4 hours until it is fully charged.


2. When charging, use the TYPE-C charging cable provided in the accessories, and connect the end of USB to the 5V2A output port of the charger;

3. The LED lamp on the device flashes regularly to display the battery remaining capacity, and indicating that it is being charged;

4. The battery can be charged at any time and at any power level. Do not start and use the device when charging;

5. The battery needs to be charged when it is exhausted to less than 5% (the power indicator flashes red).

6. The fully-charged device can be operated for about average 2-3 hours, depending on the working level and strength.

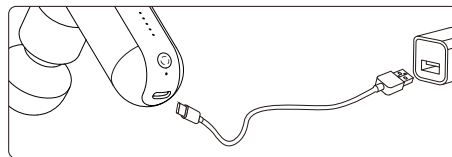5 Indicator Description

Level Indicator	Status	Description
5 lamps on	Flash	Level 0
1 lamp on	Remain on	Level 1
2 lamp on	Remain on	Level 2
3 lamp on	Remain on	Level 3
4 lamp on	Remain on	Level 4
5 lamp on	Remain on	Level 5

Power Capacity Indicator	Status	Description
Red lamp	Flash	Current power at <5%
Red lamp	Remain on / breathing	Current power at 0% - 10%
Yellow lamp	Remain on / breathing	Current power at 11% - 50%
Green lamp	Remain on / breathing	Current power at 51% - 99%
Green lamp	Remain on	Current power at 100%

6 Maintenance

1. Clean the surface of the device with a slightly wet towel and then dry it with a soft cloth.
2. Put the device in the appropriate portable bag for storing or traveling.

3. Store the device in a dry and well-ventilated place without corrosive gas.
4. Clean and pack the device if it is not used for a long time. It is recommended to charge it every 1 month.
5. This product is suitable for transportation by car, train, ship and airplane. During transportation, the product shall be fixed to against violent vibration, collision, tumbling and falling, and shall be placed upward and be handled with care and rain proof.
6. In order to protect the environment, do not discard it at random. Please discard each part according to their disposal methods.

! During maintenance, the power cord must be unplugged, but do not unplug it with wet hands.

Disposal

The lithium-ion battery used in this product is a valuable resource that can be recycled. Take out the lithium-ion battery when it is used up and discarded, and then hand it over to a professional institution that can recycle the rechargeable batteries for disposal.

Battery removal:

Pry open the bottom of main unit, and then remove the battery from the device (The battery cannot be replaced by users arbitrarily):

- The battery must be taken out from the product before the device is discarded;
- The product must be powered off when the battery is removed;
- The battery shall be disposed of in a safe manner.

7 Troubleshooting

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center in your region.

Fault Description	Troubleshooting
Massage head slips out or falls off	Check whether the massage head is installed in place
Start-up failure	Check whether the power capacity is sufficient (charge for more than 8 hours if not used for a long time)
Charging failure	1. Check whether the voltage and current of the charger are in line with the product requirements 2. Check whether the charging connector and the power supply are plugged in place
Sudden stop after continuous operation for 10 minutes	This is the function setting of protection against long time operation. The device is switched from operation mode to standby mode, which is normal. Press the button to restart the device.
Massage head is blocked and stops running	This is the function setting of protection against abnormal operation. The device is switched from operation to standby mode, which is normal. Press the button to restart the device.
Reasonable use of the product can effectively prolong its service life. It is recommended that: 1. Do not toss or drop the device during use. 2. Do not use the device continuously for more than 30 minutes. Keep it away from the humid environment when using or storing it. 3. If it is left unused for a long time, please charge it once every 1 month.	

If the product has been left unused for a long time, be sure to confirm that its functions are normal before using it again

	Check for the following faults	
Regular check	<ul style="list-style-type: none">Whether the power cord or plug is hotWhether the power cord is deeply scarred or deformed, or has a burnt smellWhether there is abnormal sound or vibration during workWhether the product can work properly after switch onWhether there are other abnormalities or faults	In the event of the above faults, stop using the device immediately and cut off the power supply to prevent accidents. Please send it to our maintenance center.

8 Technical Parameters

Product Name	Philips Mini Massage Gun
Product Model	PPM7303
Operating Voltage	7.4V ---
Operating Power	10W
Charging Voltage	5V ---
Charging Current	2A
Battery Capacity	1300mAh
Interface Type	USB Type-C
Charging Time	4 hours
Product Net Weight	303g
Product Size	75×38×153mm
Intensity Level Adjustment	5 Levels

9 Service Guide for Product Warranty

Warranty and Service

If you need information or have any questions, please visit www.philips.com/support or contact consumer service center in your country/region. If there is no consumer service center in your country/region, please contact your local distributor.

Within two years from the date of purchase, you will receive free warranty service for any damage caused by manufacturing process problems, or any damage to components under normal use conditions confirmed by our maintenance service personnel.

Warranty service excludes frequently replaced consumables, auxiliary equipment, shipping cost and door-to-door service. Please show proof of purchase to service personnel when servicing.

Malaysia
Consumer Care: +60-1112984248
Email: Philips.MY@quadforcemy.com

Singapore
Consumer Care: +65-80385589
Email: Philips.SG@quadforcesg.com

Philippines
Consumer Care: +63-960411314
Email: Philips.massager.ph@orrohome.com

Thailand
Consumer Care: +66-20051888
Email: Philips_massager@orrohome.co.th

Indonesia
Consumer Care: +021-39715012 / +021-39715013
Email: Philips_Massager.ID@orrohomeindonesia.com

Vietnam
Consumer Care: +84-909800829
Email: Philips.massager.VN@orrohome.vn

Warranty Card

Product Model		User Name	
Sales Agency		Phone Number	
Date of Purchase		Serial Number	
Special Technical Service Center		User Address	
Items of Maintenance	Date	Service Personnel	

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