

Lumify iOS | 5.0

Patient Data Transfer

Global Education Services
Ultrasound Services and Solutions Deliverability
2023


Before we get started

Connectivity Profiles

Non-default settings and customization selections do not transfer between Lumify 2.0 and 5.0 applications. All customized settings must be selected again on the new 5.0 application.

DICOM Services

Lumify 2.0 and Lumify 5.0 use the same port for DICOM services. If both applications are running concurrently during set-up a "Failed to start DICOM Server" message will appear. Select "Continue" to ignore this message.

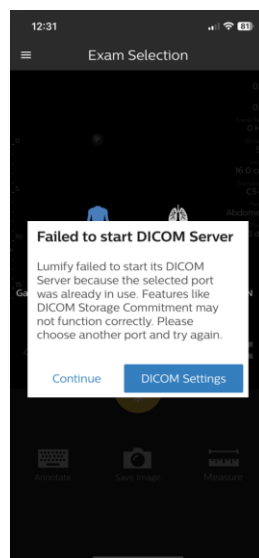

Lumify Power Modules



Do not plug the Lumify Power Module into the device during the data back up or transfer process. Doing so may automatically launch the Lumify 2.0 application.



Exporting data



Lumify 2.0

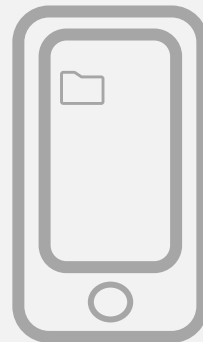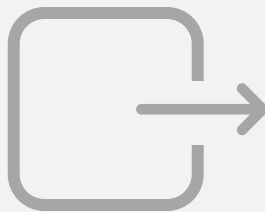


Exporting data

1. Open the application on the device by selecting the green Lumify 2.0 icon.




2. The Lumify application will launch and open the home screen.

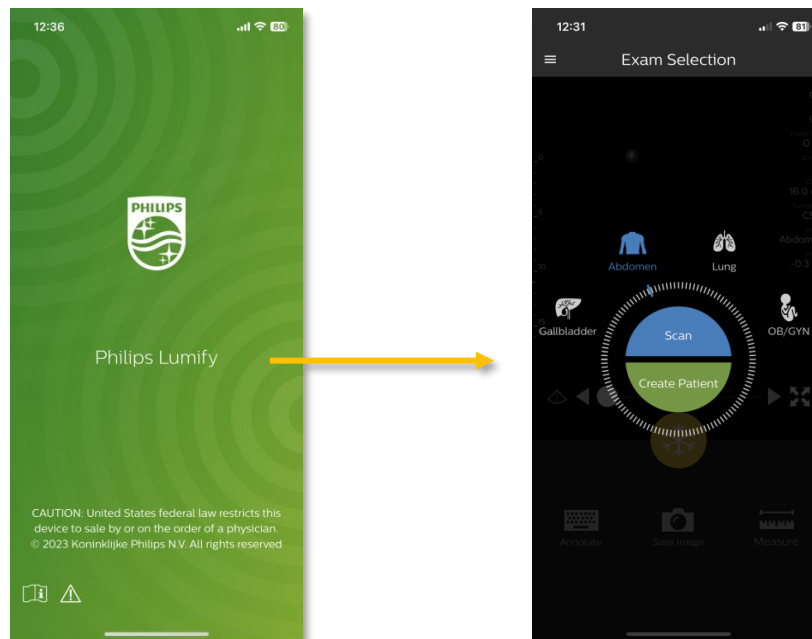


Exporting data

3. Open the menu by touching the hamburger menu (three horizontal lines) in the upper left corner of the screen.

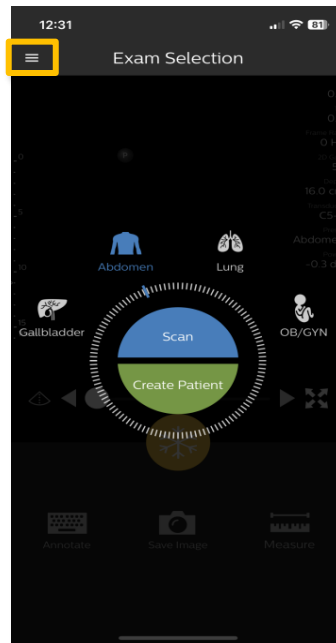


4. Select **Settings**.

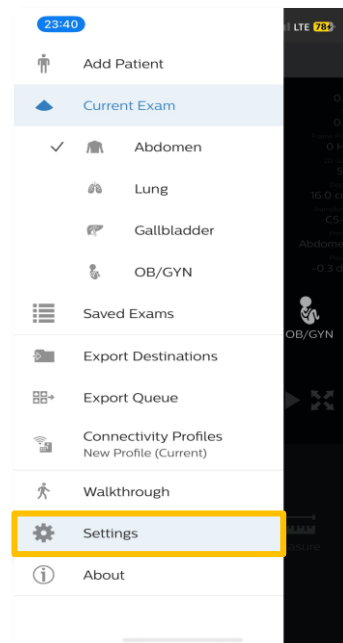


Exporting data

5. Select **Patient Database**.

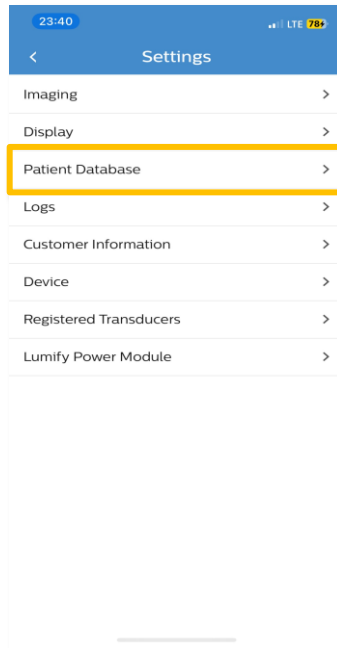

6. Select **Export Database**.

This menu is where patient database file names are entered, and their folder destination is selected.

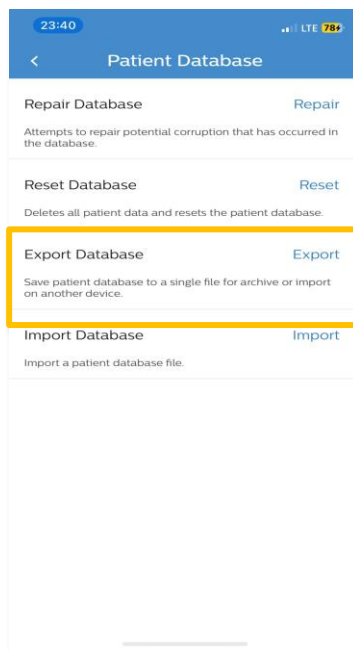

Exporting data

7. Follow the password requirements. Enter and confirm a password to encrypt the database. This action is required to move forward.

23:41 LTE 70%

< Export Export

Filename: Lumify Patient Data

Password*:

Confirm Password*:

1. Please enter a Filename for the patient database export
2. Enter a Password and Confirm Password, in order to encrypt the patient database export
3. Select a Destination for the patient database export

This password is required to import the patient database and cannot be recovered.
Password must be at least 8 characters and include the following types of characters:

- Uppercase characters
- Lowercase characters
- Numeric numbers

8. Review the safety prompt and press **Okay**.

23:41 LTE 70%

< Export Export

Filename: Lumify Patient Data

Warning


If the images contain patient identifiable information under applicable privacy laws and regulations, then you are responsible to make sure that the patient authorizes you to transmit his or her medical images to third parties, as may be required by such laws and regulations or that transmission is otherwise permitted under such laws and regulations.

Philips does not represent that information transmitted via this feature meets applicable security laws and standards. Please consult with your IT department before using this feature.


Okay

Exporting data

9. The export will begin, and a pop-up will automatically close upon completion. The data transfer time will vary depending on the size and number of exams in the database.

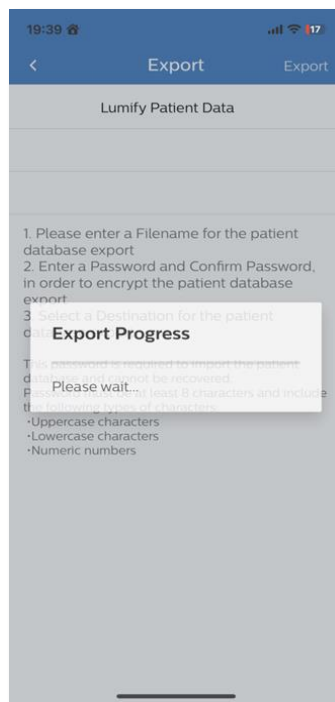


10. When the export is complete the Export menu will be visible. Select the export destination.

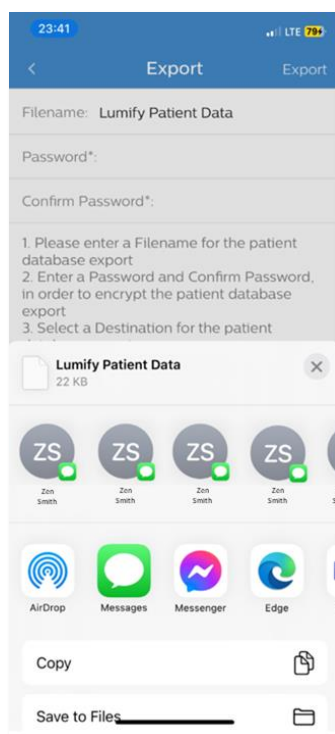


Note: Patient data can be sent to the new Lumify 5.0 application by saving the data directly in the files on the device or it can be transferred to another device.

Exporting data

11. To save the data to the same device, select **Save to Files**.

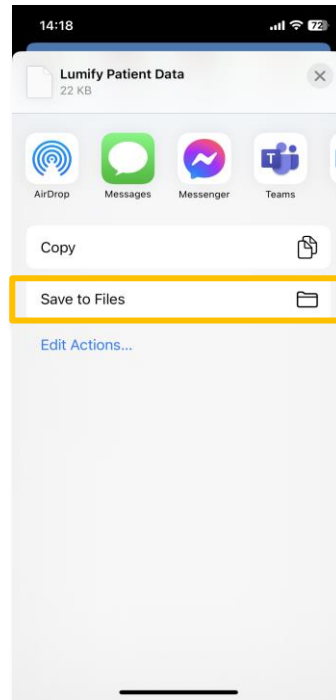


12. A default destination folder will appear. Select the default destination.

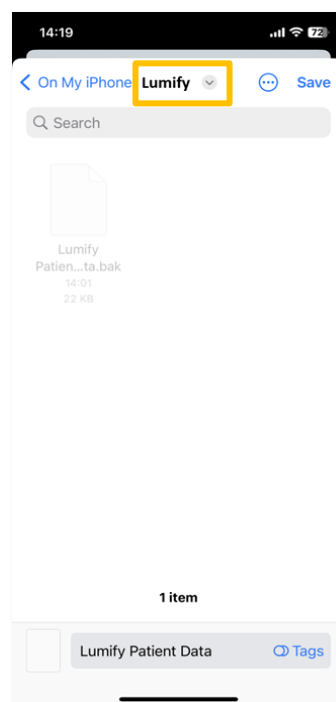


Note: Users can also create a new destination or an existing destination folder.

Exporting data

13. To verify the export saved to the correct destination go to the home screen and select the **Files** app.

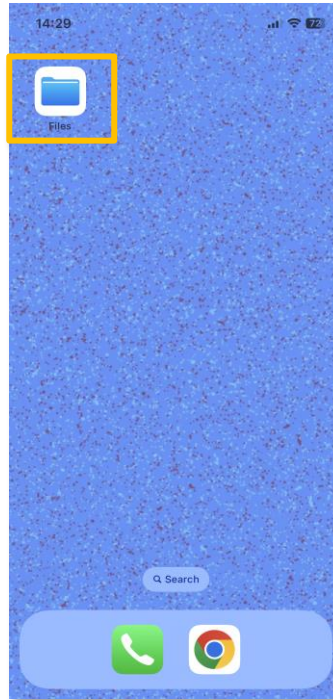


14. If the Patient Data files are not located in the Files app, browse the device and select **On My iPhone**.


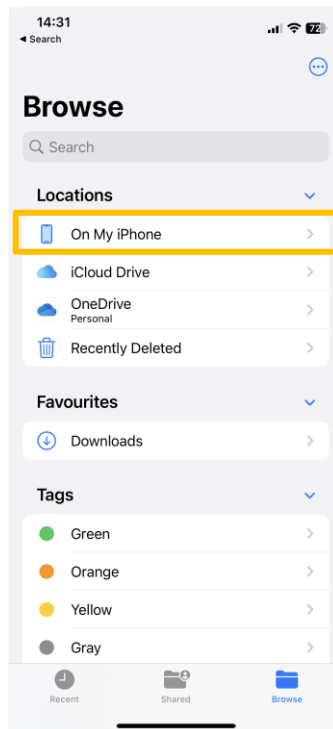


Exporting data

15. Select the green Lumify 2.0 folder. All exams that have been successfully imported will be visible.

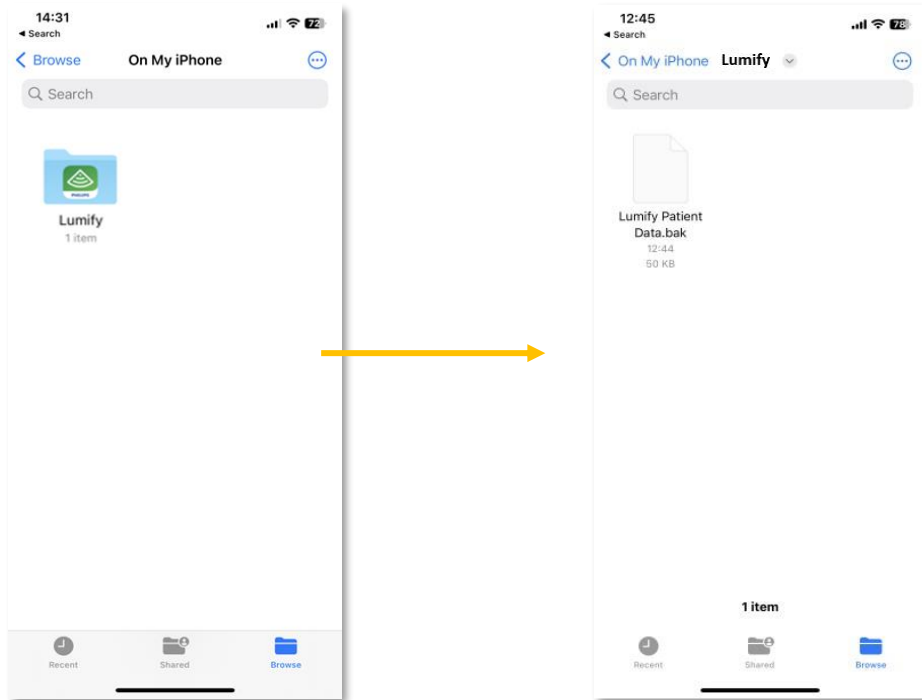


Importing data

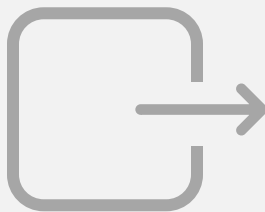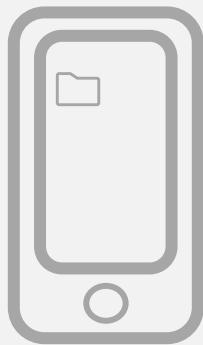


Lumify 5.0

Importing data

1. Select the blue Lumify 5.0 folder.

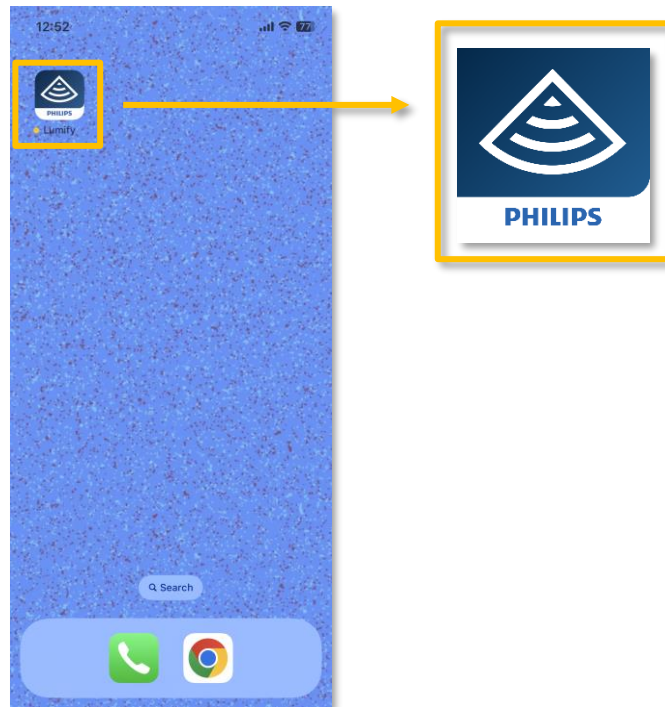


2. The application will launch. Review the data migration alert and press **Close** to proceed.

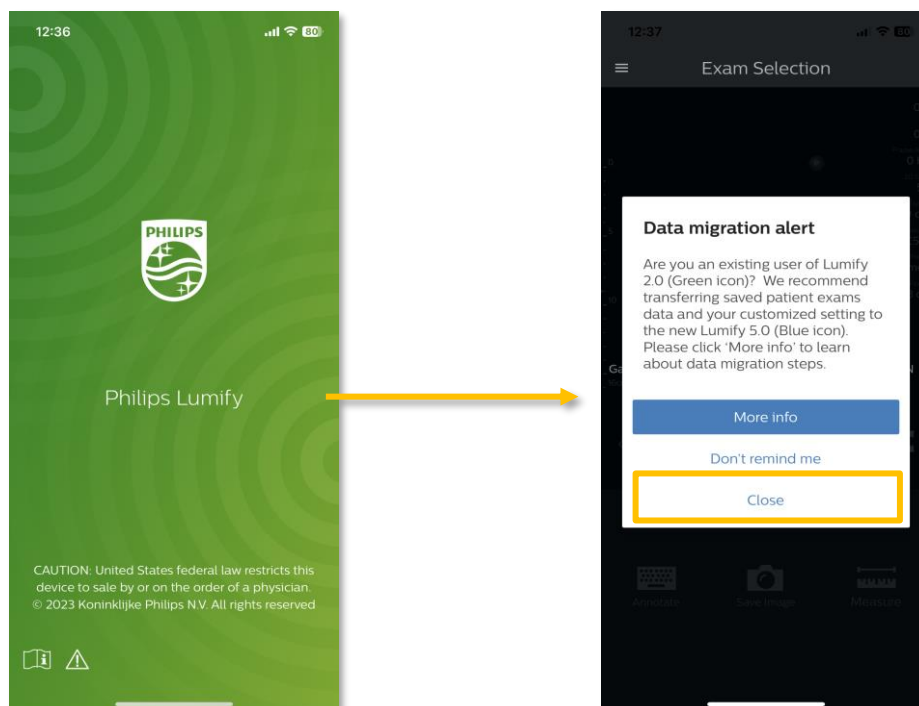


Importing data

3. Open the menu by touching the hamburger menu (three horizontal lines) in the upper left corner of the screen.

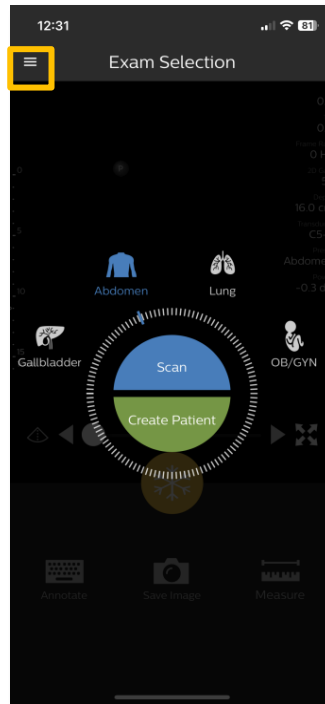


4. Select **Settings**.

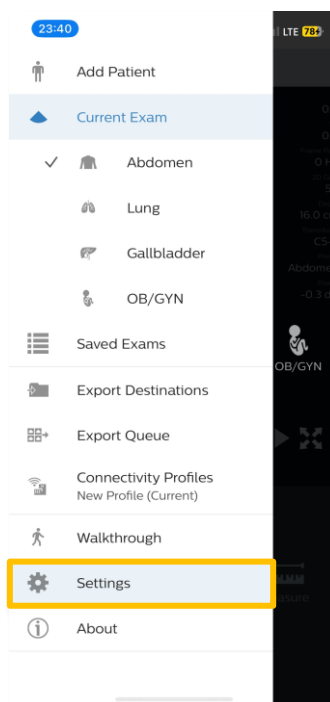


Importing data

5. Select **Patient Database**.

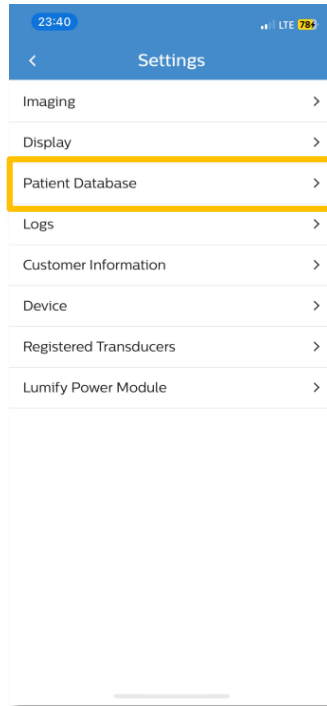


6. Select **Import Database**.

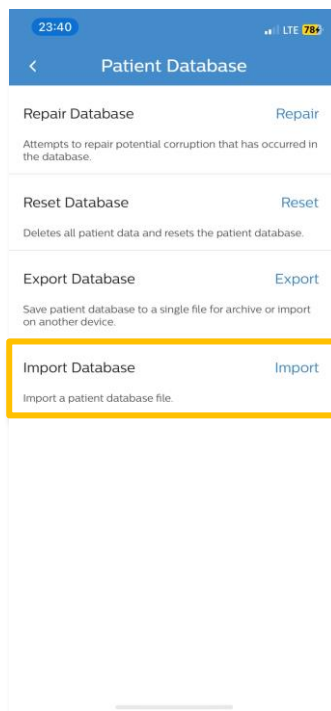


Importing data

7. In the Import Database menu, click **Select a file**.

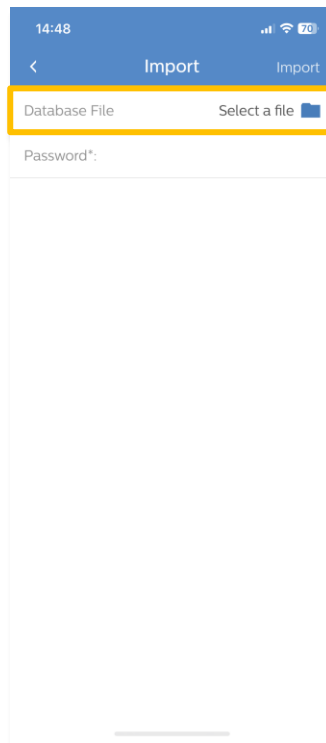

8. Most devices will launch the **Recent** directory by default. Within the **Recent** directory the data files will be visible.

Select the file then select **Open**.

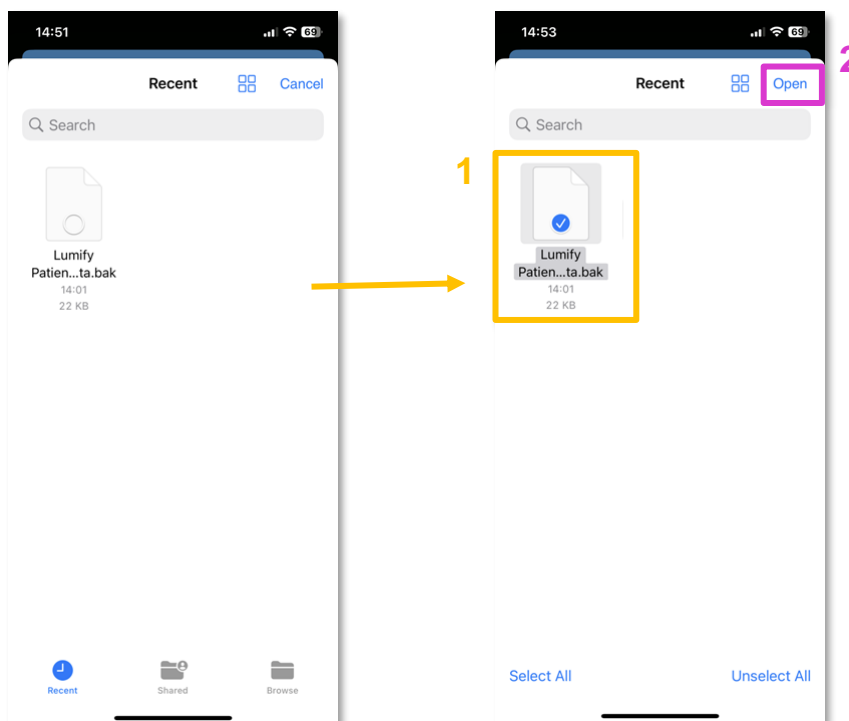


Importing data

9. If data is not visible in the **Recent** directory, select the **Browse** icon on the bottom right of the screen.




10. Select **On My iPhone/iPad**

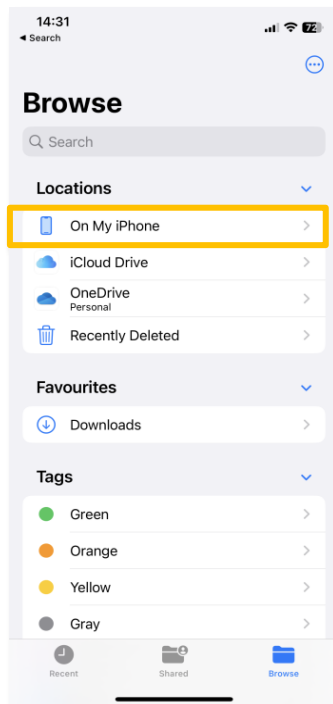


Importing data

11. The export destination folder will be visible. In this example, the Lumify 2.0 Folder is the specified destination folder. Select the folder.

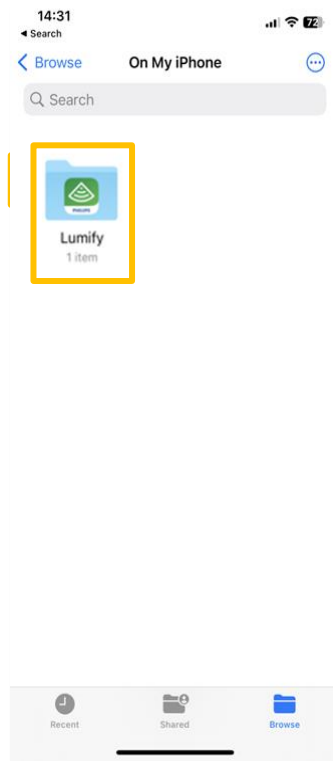

12. The patient data file will be visible. Select the file then select **Open**.

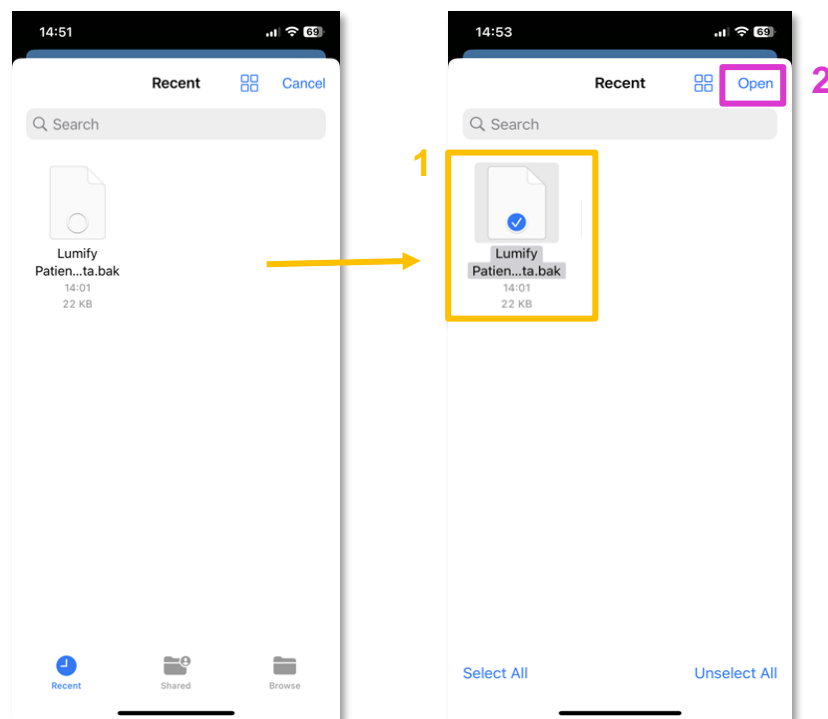

Importing data

13. The data file selected will be visible in the **Database File** field.




14. Enter the password chosen during the export process in the **Password** field. Select **Import** on the top right of the screen.



Importing data


15. Select **Confirm**. The import process will begin.

Note: The file size will impact the duration of the import process.

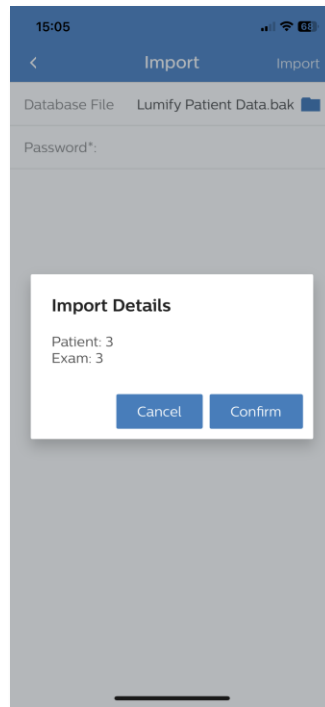

16. Upon completion of the import, an **Import Summary** prompt will appear.


Select **Okay**.

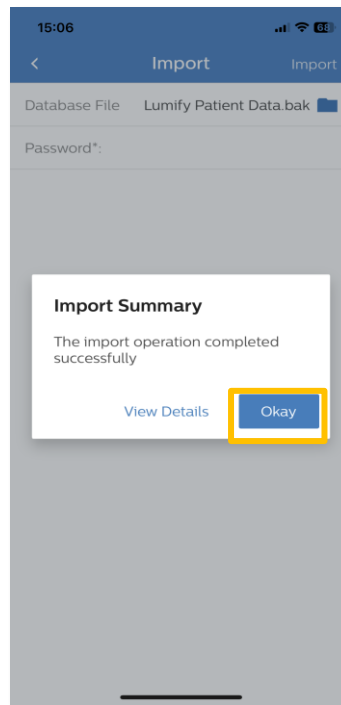


Importing data

17. The application will redirect to the Patient Database menu.

To return to the previous **Settings** menu, select the arrow on the top left of the screen.

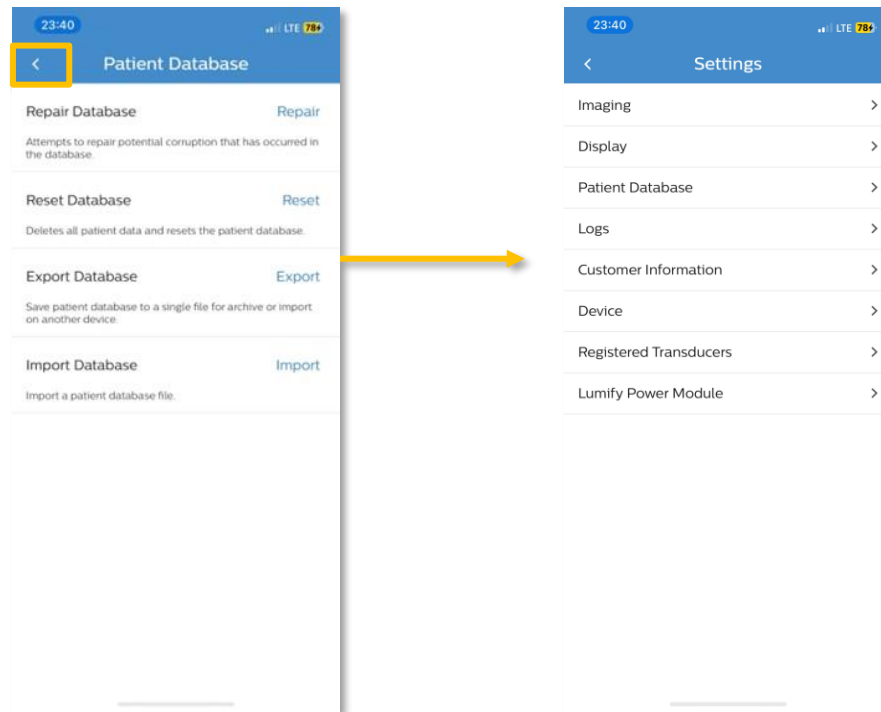


18. Select the arrow on the top left to return to the home screen.

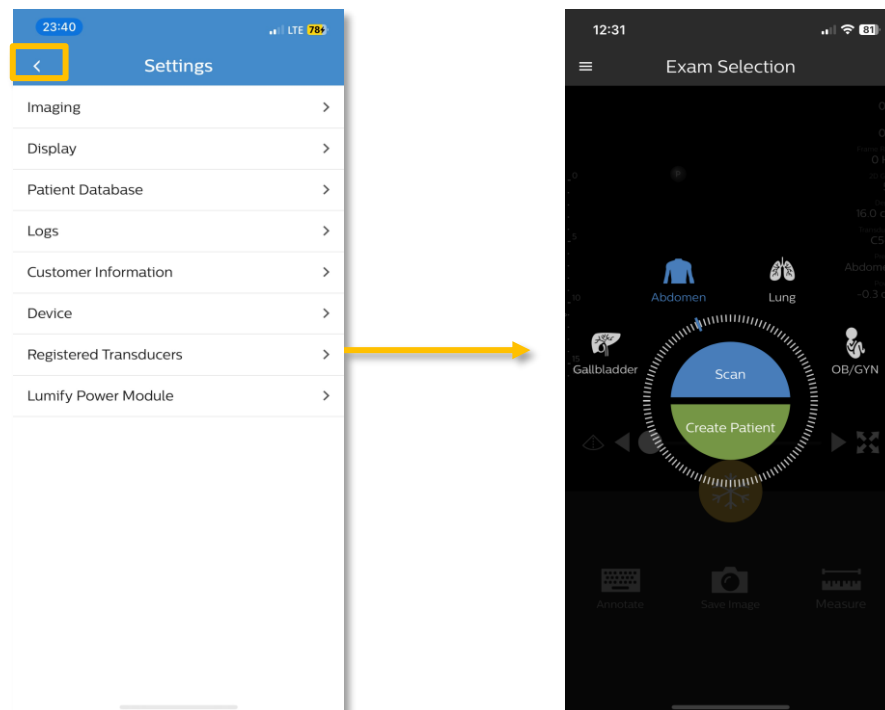


Importing data

19. To verify the upload was successful, select the hamburger menu on the top left of the screen.

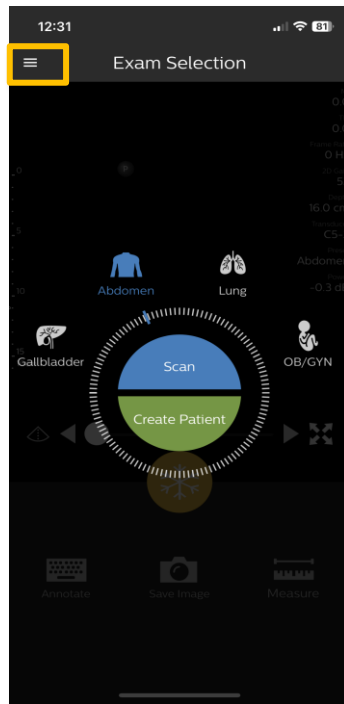


20. Select **Saved Exams**.

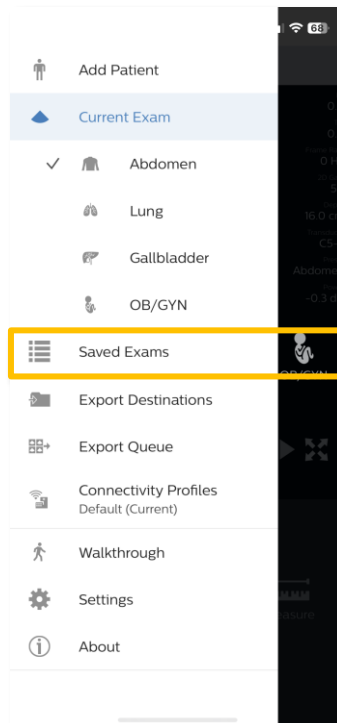

Importing data

21. All exams that have been successfully imported will be visible.

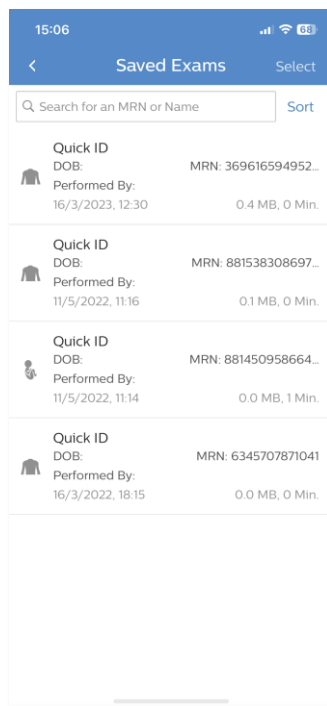

22. Remove the Lumify 2.0 application from the device.

Note: The steps to remove applications varies based on the version of iOS running on the device. Refer to the [Apple Support iPhone User Guide](#) to access detailed, version-specific instructions.

