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©2023 AquaShield
All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Hongkong AquaShield Health Technology Company Limited and Hongkong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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1 Readme

Symbol Description

People Restricted or Prohibited from Use

Precautions for Use

Precautions for Maintenance

2 Product Features

3 Components

4 How to Use

Preparations Before Use

Installation

Operation

Charging

5 Indicator Description

6 Maintenance

Disposal

7 Troubleshooting

8 Technical Parameters

9 Service Guide for Product Warranty

Warranty and Service

Warranty Card

10 Packing List

1 Readme

Please carefully read the operation manual before use and keep it properly. The manual is available at www.philips.com

Symbol Description

To avoid personal injury or property damage, be sure to observe the contents below.

- The following symbols are used to illustrate what you should observe.

 Indicates prohibition

 Indicates the actions that need to be subject to instructions

People Restricted or Prohibited from Use

 Important Information

The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

Children should be supervised to ensure that they do not play with the appliance.

If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.

This appliance contains batteries that are only replaceable by skilled persons. Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.

This product cannot be used under any of the following circumstances without the prior consent of your doctor.

- The occurrence of pregnancy, diabetic complications (e.g. neuropathy or retinal damage), pacemaker wearing, recent surgery, epilepsy or migraine, disc herniation, anterior vertebra, vertebrae detachment, or ankyloses of vertebra, recent arthroplasty surgery or IUD, Kirschner wire or steel plate and other health-related issues. These contraindications do not mean that you are not allowed to use the Product. It is recommended to consult your doctor before use.
- The underway research is about the impact of the vibration training on specific medical diseases. This stands a good chance to reduce various contraindications mentioned above. The practice proves that under many circumstances, the incorporation of vibration training into a therapeutic regimen is also a form of therapy, but it must be done at the suggestion or in the company of a doctor, expert or physical therapist.

! The product is not allowed to be used by the following groups/under the following circumstances

- It is forbidden for minors to use the product. If injured, it is not allowed to use it. Please consult your doctor before using it in case of muscle injury.
- Do not use the device on non-dry and non-clean skin surface. You can use this product with clothes on. Gently press and move it on the skin for about 60 seconds per area.
- Do not use the device on the part above the neck and with less muscle coverage. It is forbidden to use it on the bones. Under the circumstance of not causing pain or discomfort, only use the device on the soft tissues of the body.
- Do not use the device on the occurrence of bruising, pain or discomfort.
- It is forbidden for people who are unable to express their will or who cannot operate the machine themselves to use this Product.
- Children are not allowed to use the product. Keep children away from it when using it.

- Please stop immediately when you feel lower back pain, dizziness, vomiting, palpitation and other abnormal body reactions during use.
- This product is set for 10-minute timing at a time. Do not use it for no more than 20 minutes each time. After each use, stop the device for 30 minutes to ensure the extension of its service life.
- Do not use the product under the following circumstances:
 - When you are severely tired
 - Within one hour before and after meals
 - When you are drunk

Precautions for Use

- !** Do not wear excessive clothes during use.
- !** Keep away from liquid or heat source.
- !** Do not use the product during charging.
- !** After charging, disconnect the power supply in time.
- !** Stop the device immediately if it cannot work properly or you feel abnormal.
- !** Follow the instructions when using the product.
- !** Before each use, carefully check whether the product can be used normally according to the operation instructions in this manual.
- ⊘** Do not load and ballast the power cord or cause damage to it.
- ⊘** Do not disassemble, modify or repair the product by yourself.


Precautions for Maintenance

- ❗ Be sure to disconnect the power supply before maintenance.
- ⊘ Do not use volatile oil, thinner, alcohol, strong alkaline detergent, bleach or other cleaning agents for maintenance.
- ❗ The product is equipped with a simple shockproof facility, and is suitable for air, railway, road and ship transportation. Keep away from rain and snow splashing, inversion and collision during the transportation.
- ❗ Check the device for any sign of wear or damage on a regular basis. If there is such sign, or if the device has been misused, contact the supplier for maintenance before reuse.

2 Product Features

This product is available for relaxing the tense muscles and releasing the post-exercise soreness and fatigue by massaging the soft tissues of the body with high frequency. A variety of massage heads can be combined to massage muscles all over the body. Equipped with a portable storage bag, it is easy to carry out.

3 Components

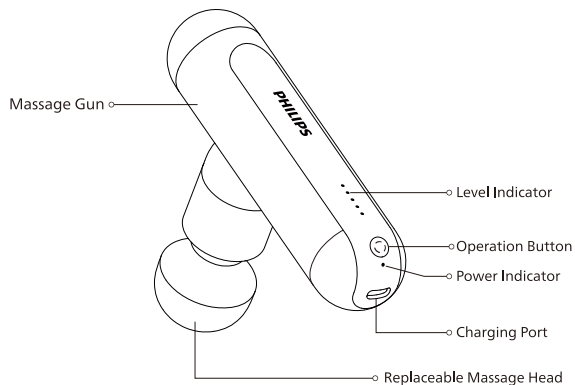Metal Hot Compress Head

Apply stainless steel surface with hot compress to relieve muscle tension and enhance the massage impact.

Closed-cell Rubber Ball Head

Made of sports equipment material, it is soft touch, elastic, skin friendly and easy to clean.

Pointed Head

Suitable for impacting deep tissues and relieve muscle soreness quickly.

Finger-simulated Head

Installed on the pointed tip head. Suitable for gentle massage with its 6 silicone nodes.

U-shaped Head

Suitable for concentrated massage of specific sore areas.

Wedge Head

Suitable for muscle edge tissues massage and making better muscle shape.


4 How to Use

Preparations Before Use


1. Check if the device and its accessories are complete.
2. Remove the pendants and decorations during use, so as to avoid affecting the massage part.
3. Install the massage head in place.

Installation

According to the message demand, insert the appropriate massage head into the front round hole of main unit, and then press it firmly to the end, as shown in the figure below, indicating that it is installed in place.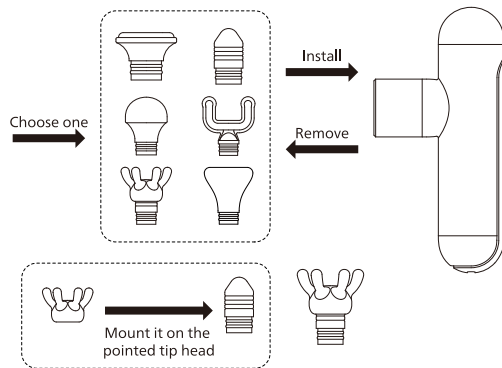

Installation effect is shown in the figure below: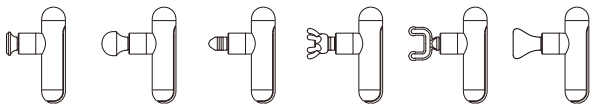

Operation

1. Start the device. Press and hold the operation button for 1 second to start the device, then the device will enter default level by default. Press the button once to increase one level, with the maximum of 5 levels. When it goes to Level 5, press the button to enter default level again, and repeat in this way.
2. Press and hold the operation button to switch off the device at any level.
3. The device will automatically switch off after 10 continuous use for 10 minutes. Restart the device for use again.
4. The massage head is suspended after being blocked. This is to protect the device against abnormal operation. Press the button to restart the device.

Power Capacity Indicator

①Power-on State:

Red light: current power

0-10%

Yellow light: current power

11-50%

Green light: current power

51-100%

②Charging State:

Red flashing light: current

power 0-10%


Yellow flashing light:

current power 11-50%

Green flashing light:

current power 51-99%

Green remains on: fully charged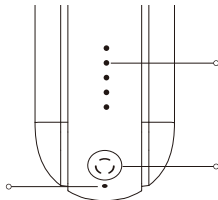

Level Indicator

1 light indicates Level 1,

2 lights indicate Level 2,

And so forth

Operation Button


Press and hold to start/ be

on standby

Press to switch the levels

Charging

1. Before the first use, charge the device for 4 hours until it is fully charged.
2. When charging, use the TYPE-C charging cable provided in the accessories, and connect the end of USB to the 5V2A output port of the charger;
3. The LED light on the device flashes regularly to display the battery remaining capacity, and indicating that it is being charged;
4. The battery can be charged at any time and at any power level. Do not start and use the device when charging;
5. The battery needs to be charged when it is exhausted to less than 5% (the power indicator flashes red).
6. The fully-charged device can be operated for about average 2-3 hours, depending on the working level and strength.

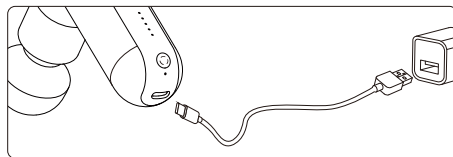5 Indicator Description

| Level Indicator | Status | Description |
|-----------------|-----------|---------------|
| 5 lights on | Flash | Default level |
| 1 light on | Remain on | Level 1 |
| 2 lights on | Remain on | Level 2 |
| 3 lights on | Remain on | Level 3 |
| 4 lights on | Remain on | Level 4 |
| 5 lights on | Remain on | Level 5 |

| Power Capacity Indicator | Status | Description |
|--------------------------|----------------------|----------------------------|
| Red light | Flashing | Current power at <5% |
| Red light | Remain on / Flashing | Current power at 0% - 10% |
| Yellow light | Remain on / Flashing | Current power at 11% - 50% |
| Green light | Remain on / Flashing | Current power at 51% - 99% |
| Green light | Remain on | Current power at 100% |

6 Maintenance

1. Clean the surface of the device with a slightly wet towel and then dry it with a soft cloth.
2. Put the device in the appropriate portable bag for storing or traveling.

3. Store the device in a dry and well-ventilated place without corrosive gas.
4. Clean and pack the device if it is not used for a long time. It is recommended to charge it every 1 month.
5. This product is suitable for transportation by car, train, ship and airplane. During transportation, the product shall be fixed to against violent vibration, collision, tumbling and falling, and shall be placed upward and be handled with care and rain proof.
6. In order to protect the environment, do not discard it at random. Please discard each part according to their disposal methods.

! During maintenance, the power cord must be unplugged, but do not unplug it with wet hands.

Disposal

The lithium-ion battery used in this product is a valuable resource that can be recycled. Take out the lithium-ion battery when it is used up and discarded, and then hand it over to a professional institution that can recycle the rechargeable batteries for disposal.

Battery removal:

Pry open the bottom of main unit, and then remove the battery from the device (The battery cannot be replaced by users arbitrarily):

- The battery must be taken out from the product before the device is discarded;
- The product must be powered off when the battery is removed;
- The battery shall be disposed of in a safe manner.

7 Troubleshooting

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center in your region.

| Fault Description | Troubleshooting |
|--|--|
| Massage head slips out or falls off | Check whether the massage head is installed in place |
| Start-up failure | Check whether the power capacity is sufficient (charge for more than 8 hours if not used for a long time) |
| Charging failure | 1. Check whether the voltage and current of the charger are in line with the product requirements 2. Check whether the charging connector and the power supply are plugged in place |
| Sudden stop after continuous operation for 10 minutes | This is the function setting of protection against long time operation. The device is switched from operation mode to standby mode, which is normal. Press the button to restart the device. |
| Massage head is blocked and stops running | This is the function setting of protection against abnormal operation. The device is switched from operation to standby mode, which is normal. Press the button to restart the device. |
| Reasonable use of the product can effectively prolong its service life. It is recommended that: 1. Do not toss or drop the device during use. 2. Do not use the device continuously for more than 30 minutes. Keep it away from the humid environment when using or storing it. 3. If it is left unused for a long time, please charge it once every 1 month. | |

If the product has been left unused for a long time, be sure to confirm that its functions are normal before using it again

| | Check for the following faults | |
|---------------|---|--|
| Regular check | <ul style="list-style-type: none">Whether the power cord or plug is hotWhether the power cord is deeply scarred or deformed, or has a burnt smellWhether there is abnormal sound or vibration during workWhether the product can work properly after switch onWhether there are other abnormalities or faults | In the event of the above faults, stop using the device immediately and cut off the power supply to prevent accidents. Please send it to our maintenance center. |

8 Technical Parameters

| | |
|----------------------------|------------------------------|
| Product Name | Philips Mini Massage Gun |
| Product Model | PPM7303 |
| Operating Voltage | 7.4V --- |
| Operating Power | 10W |
| Charging Voltage | 5V --- |
| Charging Current | 2A |
| Battery Capacity | 1300mAh |
| Interface Type | USB Type-C |
| Charging Time | 4 hours |
| Product Net Weight | 303g |
| Product Size | 75×38×153mm |
| Intensity Level Adjustment | 5 Levels |
| Power Supply | Rechargeable Lithium Battery |
| Production Date | See Product Certificate |

9 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

HongKong

Hotline: 852-3188-4987

Service Email: philips.massage.hk@centuryfield.com

Taiwan

Hotline: 0800-661-600

Service Email: Philips.massager@xander.com.tw

Warranty Card

| | | | |
|---------------------|--|------------------|-------------------|
| Product Model | | Customer Name | |
| Seller | | Contact Number | |
| Date of Purchase | | Serial Number | |
| Service Center | | Customer Address | |
| Maintenance Content | | Date | Service Personnel |
| | | | |
| | | | |
| | | | |
| | | | |

10 Packing List

| | | | |
|-------------|----|--------------------|----|
| Massage Gun | 1x | Massage Head | 6x |
| Storage Bag | 1x | USB Charging Cable | 1x |
| User Manual | 1x | Certificate | 1x |

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