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This product has been manufactured by and is sold under the responsibility of Hong Kong AquaShield Health Technology Company Limited and Hong Kong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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User Manual
使用說明書

Table of Contents

1	Instructions for User	1
	People restricted from use	1
	Precautions for Use	2
	Precautions for Maintenance	5
<hr/>		
2	Product Features	5
<hr/>		
3	Parts Name	6
<hr/>		
4	How to Use	7
<hr/>		
5	Maintenance	9
	Cleaning	9
	Disposal	10
<hr/>		
6	Diagnosis	11
<hr/>		
7	Main Specification	13
<hr/>		

<hr/>		
8	Service Guideline for Warranty	13
	Warranty and Service	13
	Warranty Card	14
<hr/>		
9	Packing List	14
<hr/>		

1 Instructions for User

Please read this instruction manual carefully before use and keep it properly.

Symbols

In order to avoid personal injury or property damage, be sure to observe the followings.

- Meanings of the symbols.
-

 Prohibit

 Represents needs to be done as instructed

People restricted from use

 Take caution for the following groups or in the following situations

Anyone who is under treatment in a medical institution, or has the following conditions, please consult a doctor or a physical therapist before use.

- People with implanted pacemakers and other related medical electrical products
- Patients who are being treated by doctors and feel unwell
- Patients with cardiac and cranial nerve abnormalities
- Patients with body temperature over 38 degrees (fever)
- Patients with infectious diseases
- Patients with skin sensory disturbances or skin abnormalities
- Patients taking medication

- Patients with abnormal blood pressure
- Patients who cannot express their will
- Patients with perceptual disturbances due to high peripheral circulatory disorders such as diabetes mellitus
- People who are not sensitive to heat must use the product with caution, because the product surface can be heated.

 Do not use the product for the following groups or in the following situations

People whose doctors advise against exercising or stretching

For example, thrombosis, severe aneurysm, acute venous aneurysm, various dermatitis, and various skin infections, because it is likely to cause deterioration, the use of the product is prohibited.

- Patients with malignant tumors
 - Patients with bleeding tendency
 - Pregnant and maternal women
 - Patients with acute suppurative inflammation
 - People with implanted cardiac pacemakers
 - Patients with metal implants
-

Precautions for Use

 Please confirm that the buttons are normal before using the product.

 Before the first use, or after the product has Not been used for a long time, the battery power is insufficient, so please charge it according to the instruction manual, to ensure that it can work normally.

 If you feel unwell because of the use of this product, please stop using it immediately.

- ❗ If you have any strange feeling on your body or skin, please consult your doctor.
- ❗ In the event of a failure, please power off the product and take it off immediately, otherwise it may cause high temperature or short circuit, resulting in an accident, Then contact after-sales service.
- ❗ Please wear this product on the cervical spine symmetrically and correctly, otherwise it may cause an accident or physical discomfort.
- ❗ When the product operates for a long time, the surface will heat up, which is a normal phenomenon.
- ❗ When disposing of this product, please observe the city's relevant environmental protection regulations, otherwise it may cause environmental pollution.
- ❗ People aged 14 and under must be used under adult supervision.
- ⊘ Never use the product with the following medical electronic devices:
 - implantable medical electronic devices such as cardiac pacemakers;
 - artificial heart-lung and other life-sustaining medical electronic devices;
 - electrocardiograph and other medical electronic devices;* If these medical electronic devices are used together with the product, the devices may malfunction, which may be life-threatening.
- ⊘ If you feel unwell during or after use, Please stop using it and consult a doctor.
- ⊘ This product has Built-in lithium battery, do not disassemble, modify or immerse the product in water or throw it into fire, otherwise it may cause damage or explosion.
- ⊘ Do not use this product near the heart, above the neck, on the head, near the mouth or genitals, in the skin disorder areas, or on the soles of both feet. Do not compress the internal organs during use, otherwise it may cause an accident or physical discomfort.

- ⊘ Do not use this product with other medical devices or use more than 2 units of this product, otherwise it may cause nausea or physical discomfort.
- ⊘ Do not use this product for purposes other than massage described in the instruction manual, otherwise it may cause an accident and malfunction.
- ⊘ Do not disassemble, repair or modify this product without authorization, otherwise it may cause malfunction or electric shock.
- ⊘ Do not plug or unplug the USB charging cable with wet hands, as this may result in electric shock or an accident.
- ⊘ Do not use this product for children or people who cannot express their will, as it may cause an accident or physical discomfort.
- ⊘ Do not use this product on the same part for a long time (more than 30 minutes), otherwise it may fatigue the muscles in the massage area and cause physical discomfort.
- ⊘ When unplugging the USB charging cable from the product, do not pull the cable but pull the cable plug, otherwise it may cause electric shock or damage to the product.
- ⊘ Do not use this product in gas stations, high-dust workshops, gas and other flammable and explosive places.
- ⊘ Do not use this product in high-humidity areas such as bathrooms. Do not use the product while bathing, or you may experience a severe electric shock and damage the product.
- ⊘ Do not use this product when sleeping, it may cause the product to malfunction, and the massage head may be moved to other parts and cause physical discomfort.

- ⊘ Do not use this product when driving a vehicle, it may cause physical discomfort due to excessive stimulation, resulting in an accident.
- ⊘ Do not use or store this product in the following places:
 - Places with direct sunlight
 - Places with high temperature and humidity
 - Places with water
 - Dusty places
 - Near fire sources
 - Places susceptible to vibration shocks
 - Near strong electromagnetic fields



Precautions for Maintenance

- ❗ This product will be automatically turned off after 15 minutes of operation. It is recommended to use it for no more than 30 minutes a day and use it as needed every week.
- ❗ This product shall be stored in a ventilated and dry place after use.
- ⊘ Never place heavy objects on the product.

2 Product Features

This product has 4 built-in 3D massage heads, imitating human hands to knead the trigger points of the neck. Its ergonomic design can provide 360° support for the neck, and its double-layer memory foam is soft outside and hard inside, ensuring good, soft and skin-friendly support. It adopts hot compress at a constant temperature of 42°C, 5 massage modes, micro-porous aroma diffuser, and built-in soothing music, helping you relax your whole body, and inducing relaxation through tactile, olfactory and auditory stimuli.

3 Parts Name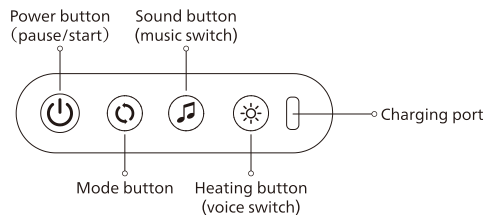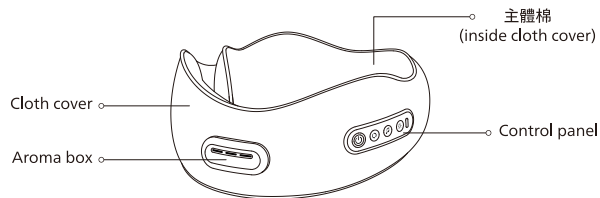

4 How to Use

User instructions

1. Press and hold the power button for 2s to turn on the massager, device starts in low power mode by default, with heating function activated, indicator light in orange, and music playing;
2. You can short press the power button to pause the roller massage, and short press again to start the roller massage;
3. You can short press the heating button to deactivate the heating function, and in this case, the indicator light turns blue; short press again to activate the heating function, and the indicator light turns orange; long press the heating function button to deactivate/activate the voice reminder;
4. You can short press the mode button to switch from low power mode to energetic mode, high power mode, neck-care mode, and wake-up mode in turn;
Low power mode: the massage head rotates inward at low speed for 30s, and outward at low speed for 30s, reciprocating in a cycle of 60s;
Energetic mode: the massage rotates inward at low speed for 15s, outward at low speed for 15s, inward at high speed for 15s, and outward at high speed for 15s, reciprocating in a cycle of 60s;
High power mode: the massage head rotates inward at high speed for 30s, and outward at high speed for 30s, reciprocating in a cycle of 60s;
Neck-care mode: the massage head rotates inward at high speed for 30s, inward at low speed for 30s, outward at low speed for 30s, and outward at high speed for 30s, reciprocating in a cycle of 120s;
Wake-up mode: the massage head rotates inward at low speed for 60s, and outward at low speed for 60s, reciprocating in a cycle of 120s;

5. You can short press the sound button to switch music background in sequence: fireplace, wind chimes, stream, spring rain, morning, ocean waves; long press this button to deactivate/activate the music;
6. In power-on state, if the battery power is no more than 10%, the indicator light will flash for 5s (in blue) and the massager will be turned off; in the power-off state, if the battery power is no more than 10%, the indicator light will flash for 5s (in blue) and the massager cannot be turned on;
7. When the massager is being charged, the blue light ring will be breathing and flashing, and when fully charged, the blue light ring will stay on;
8. The massager will be automatically turned off after 15 minutes of operation;
9. Press and hold the power button for 2s to turn off the massager (when the voice function is active, the turn-off music will sound, and when the voice function is inactive, there will be no prompt);
10. The voice setting has a memory function. If it is set to activate voice or music before the last turn-off, the voice/music will be activated by default after the next turn-on; if it is set to deactivate voice or music before the last turn-off, the voice/music will be deactivated by default after the next turn-on;
11. How to use aroma box: Open the aroma box to drop the essential oil on the foam, and close the aroma box; the aroma diffuses through the pores while massaging.

Tips:

1. Please remove the personal accessories when using the massager, so as not to affect the massage part.
2. If you feel discomfort during the massage, please stop operation immediately.
3. When the massager is not in use, please turn it off. In case of a malfunction, please turn off the power switch to avoid accidents.


5 Maintenance

Cleaning

Before cleaning the neck massager, be sure to power it off, and do not touch the plug with wet hands.

Before cleaning, please remove the cloth cover to completely separate it from the non-cleanable components such as the circuit board. How to remove cloth cover:

- Fully unzip
- Gently lift up the control panel positioning frame
- Slowly pull out the inner components from one end so as to remove the cloth cover

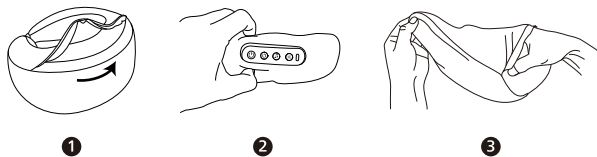Precautions for cleaning:

(1) Only the cloth cover can be removed and cleaned separately, other massage components and control panel must not get wet;

(2) The cloth cover is not machine washable, but can be hand washed gently.

Please wash it separately from other colored fabrics, choose mild detergent, and the water temperature should not exceed 30°C. Do not bleach or spin.

Hang and dry naturally. Avoid sun exposure.

(3) Install the other components only after the cloth cover is completely dry. Always keep away from fire sources, oil pollution, corrosive liquids and other unfavorable environments.

- ⊘ Please keep the product out of the reach of children.
- ⊘ Do not store the product in a place exposed to direct sunlight, high temperature and humidity.
- ⊘ Please store the product in a dry and ventilated place.
- ⊘ Do not disassemble, repair or modify the product without authorization, otherwise it may cause malfunction.

Disposal

The lithium-ion battery of this product is a valuable resource that can be recycled and regenerated. When disposing of the product, please take out the lithium-ion battery and hand it over to a professional organization that can recycle and regenerate rechargeable batteries.

Precautions for battery disposal:

1. Do not replace the battery without authorization during normal use.
2. Before disposing of the product, take out the battery. How to take out the battery:
 - Remove the cloth cover of the product;
 - Remove the battery box assembly and then its bottom cover;
 - Remove the internal battery cables, and take out the battery.
3. Be sure to power off the product before taking out its battery.
4. Always dispose of the battery safely.

6 Diagnosis

The most common problems of this product are described here. If you still cannot solve the problem according to this, please call the hotline or contact a local authorized service center.

If you encounter problems when using the product, please refer to the following tips and solutions.

If the product is still faulty or cannot be used, please power it off.

Do not disassemble or repair the product without authorization, otherwise the resulting damage to the product will invalidate the warranty, and the company will not be responsible for any consequences arising therefrom.

Problem	Possible cause/solution
Failure to turn on neck massager	1. Check if the battery power is low. 2. Check if the button has been pressed (the indicator light becomes blue or stays on in orange when the product is turned on).
Charging failure or no display of charging indicator light	Check if the USB cable is well connected to the product or the mobile phone power adapter.
Sudden stop when operation	1. Check if the battery power is low. 2. 15 minutes set by the automatic timer has been reached. 3. If the problem still exists, please stop using the product, and contact the company's service center.

After not using it for a long time, be sure to confirm that the function is normal before you can use it again.

Regular product check	Is there any of the following faults	If the above faults occur, in order to prevent accidents, please stop the use immediately, and cut off the power supply. Please send it to our repair site.
	<ul style="list-style-type: none">The power cord or plug is hotThe power cord is deeply scarred or deformed and burntThere is an abnormal sound or vibration at workWith the switch on, it does not work properlyThere are other abnormalities or faults	

Note:

If the fault cannot be troubleshot or the product still fails to operate ideally, contact the after-sales service, and do not disassemble or attempt to repair it without authorization.

7 Main Specification

Product name	Philips neck massager
Product model	PPM3501
Operating voltage	3.7V \approx
Operating power	6W
Charging voltage	5V \approx
Charging current	1A
Battery capacity	2500mAh
Product dimensions	272×267×119mm
Net weight	0.85kg
Date of manufacture	See product certificate

8 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

HongKong

Hotline: 852-3188-4987

Service Email: Philips.Massage.hk@centuryfield.com

Taiwan

Hotline: 0800-661-600

Service Email: Philips.massager@xander.com.tw

Warranty Card

Product Model		Customer Name	
Seller		Contact Number	
Date of Purchase		Serial Number	
Service Center		Customer Address	
Maintenance Content		Date	service personnel

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All rights reserved
The Philips trademark and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. used under license.