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©2023 AquaShield
All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of HongKong AquaShield Health Technology Company Limited and HongKong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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User Manual
使用說明書

Contents

1	Symbol Description	1
	People Suitable to Use	1
	Precautions for Use	2

2	Product Features	3
---	------------------	---

3	Components	4
---	------------	---

4	How to Use	5
	Preparation before Use	5
	Operation	5
	Charging	7

5	Maintenance	8
	Disposal	8

6	Troubleshooting	10
---	-----------------	----

7	Technical Parameters	12
---	----------------------	----

8	Service Guideline for Warranty	13
	Warranty and Service	13
	Warranty Card	14

9	Packing List	14
---	--------------	----

1 Readme

Please read this manual carefully and keep it properly before using the product. Manual can be downloaded by www.philips.com

Symbol description

To avoid personal injury or property damage, be sure to observe the contents below.

- The following symbols are used to illustrate what you should observe.
-

 Indicates prohibition

 Indicates the actions that need to be subject to instructions

People Suitable to Use

 The product should be used with caution by the following groups

- Do not use the product and other medical electronic instruments at the same time, such as cardiac pacemaker, artificial heart and lung and other life-sustaining medical electronic instruments, as well as electrocardiograph and other medical electronic instruments, otherwise it is likely to cause danger;
- If the high-frequency surgical equipment and this product are used on the user at the same time, burns may be caused at the electrode position of the massager, or the massager is likely to be damaged;

- If this product is used in the vicinity (1m) of the short-wave or micro-wave therapeutic equipment, the output of this product may become unstable;
 - Pregnant women and women during menstruation, patients having sensitive skin and suffering from heart disease, abnormal blood pressure, malignant tumors, cerebrovascular disease, and acute diseases, or other people under medical treatment should consult a doctor before using this product.
-

Precautions for Use



-  The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
-  This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
-  Children should be supervised to ensure that they do not play with the appliance.
-  If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.
-  This appliance contains batteries that are only replaceable by skilled persons.
-  Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.
-  Do not move the massager during use. If you want to change the position to be massaged, please shut it down first, otherwise there will be a strong stimulation.
-  Do not use the product in places with high temperature, inflammable electromagnetic radiation and humidity.

- ❗ Stop using the device if you feel uncomfortable due to the use of the product.
- ❗ Do not disassemble, repair or modify this massager without authorization, otherwise it is likely to result in failure or electric shock accidents.

2 Product Features

This product integrates vibration massage, constant temperature warm compress, and three-dimensional elastic booster. The vertical burden of the shoulder and neck muscle can be eased through its 3D tailored shoulder-and-neck boosting structure. Its vibration massage and warm compress function can effectively relax muscles and relieve the pressure on shoulders and neck. In the back hunching reminder mode, the built-in sensor will vibrate to remind the user when the user is continuous hunching back may cause the fatigue of the shoulder and neck muscle, so as to avoid injury due to fatigue accumulation. Its physical booster combined with the intelligent reminder and the effective relaxation massage can help the users to lessen their shoulder and neck fatigue and the resulting bad posture.

3 Components

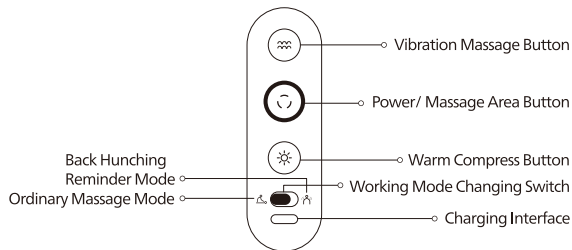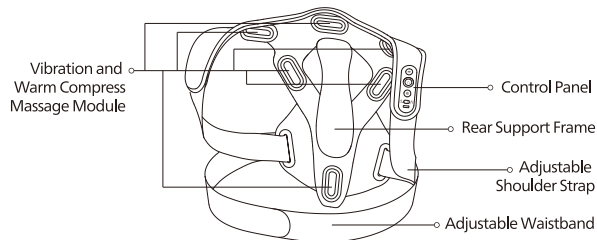
4 How to Use


Preparations before Use

1. Check whether the device and accessories are complete.
2. Wear the massager on the back and adjust it to a comfortable state.

Operation

 Select the working mode

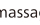Slide the working mode selection switch " to switch to the required working mode. " on the left side is the ordinary massage mode, " on the right side is the back hunching reminder mode.

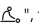In the ordinary massage mode "", the device has the default setting of vibration level 1 and warm compress level 1 for the entire area when it is turned on. The device will automatically shut down after 10 minutes of no button operation.

In the back hunching reminder mode "", the massager vibrates to remind when the user hunches the back. Both the vibration massage and the warm compress are turned off by default. The warm compress can be manually turned on and the heating area can be selected. The massager will not shut down regularly.

 Power/massage area button

1. Press and hold the " button for 1 second, then the massager is powered on and enters the corresponding working mode according to the state of the working mode selection switch. The button icon and the lamp ring lights on.

2. Press the button to select the massage area, which is in order as follows: full back, shoulders and neck, back center, shoulders, and spine.

 Vibration adjusting button

The default vibration massage intensity is in level 1 after power-on, and the " button turns green; press the button to switch to intensity level 2, and the button turns yellow; press the button again to switch to intensity level 3, and the button turns orange; press the button again to turn off the vibration massage, and the button lamp dims.

 Warm compress adjusting button

The default warm compress is in level 1 after power-on, and " button turns green; press the button to switch to warm compress level 2, and the button turns yellow; press the button again to switch to warm compress level 3, and the button turns orange; press the button again to turn off the warm compress, and the button lamp dims.

 Low battery reminder and charging

When the battery remaining capacity is less than 5% under the working state, the " button working indicator will flash red to remind the user. Please connect the charging cable to charge the device. To avoid battery damage, do not start and use the device during charging. After the device is fully charged, the lamp ring will turn green and remain on.

Charging

1. During the use of the massager, when the working indicator lamp of the “” button ashes red, it indicates that the built-in lithium battery is low, and you need to charge it with the USB power cord.

2. Plug the output end of Type-C USB cable into the USB interface of the device, and then plug the input end into the mobile phone charger or mobile power supply. The maximum charging power supported by the device is 5V $\overline{=}$ 2A.

3. When charging, the status indicator lamp ring will display the current battery remaining capacity in different colors:

Red lamp ashes <5%,

Yellow lamp ashes 6%-50%,


Green lamp ashes 51-99%.

When the battery is fully charged, the charging indicator turns green and remains on. It takes about 4 hours to fully charge the device.

Warm Tips:

1. The product can be used for about 4 days (twice a day) after being fully charged.

2. If the rechargeable battery is left unused for a long time, the service life of the rechargeable battery will be shortened. It is recommended to fully charge the battery at least once a month.

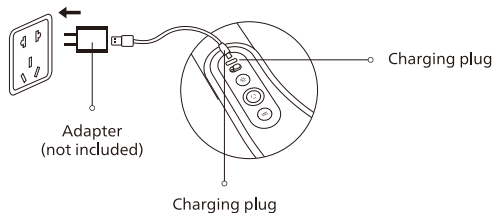5 Maintenance

Cleaning

Wipe the device with a dry soft cloth or a dry towel when it is dirty.

Storage

- Keep out of reach of children.
- Do not put it in the places with direct sunlight, high temperature and humidity.
- Keep it in a dry and ventilated place.

 During maintenance, be sure to unplug the power cord, but do not unplug the cord with wet hands.

Disposal

The lithium-ion battery used in this product is a valuable resource that can be recycled. Take out the lithium-ion battery when it is used up and discarded, and then hand it over to a professional institution that can recycle the rechargeable batteries for disposal.


Precautions for battery:

1. Do not replace the battery by yourself during normal use.
 2. Waste battery pack should be safely recycled and should not be discarded at will;
- Be sure to remove the battery from the device before disposal;
 - The product must be powered off before the battery is removed;
 - The battery must be disposed of in a safe manner;

Steps before disposal


1. Cut the fabric at the dotted line with scissors.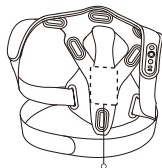


Battery holder

2. Cut the fabric, and you can find a battery holder.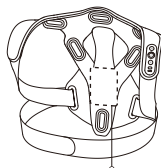

Wires

3. Shear off all the external wires of the battery holder with scissors.

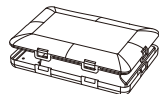4. Unscrew the two screws on the battery holder with a dedicated screwdriver, loosen the four buckles on the side, then you can find the battery. Shear off the red and black electric wires connecting the battery. In this way, the battery can be removed for disposal.

6 Troubleshooting

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center in your region.

Fault Description	Possible Cause	Troubleshooting
Startup Failure	Low battery of the rechargeable battery inside the main unit?	In case of low battery, please charge it in time.
	Is the charging normal?	Check whether the USB cable is plugged in place. After it is connected correctly, the massager's charging indicator will flash.
Normal display, but the user feels nothing at the time of massage	Is the massage intensity adjusted?	The default state after power-on is intensity level 1. If you need to adjust the intensity, please turn it up appropriately.
	Is the warm compress area of the massager tight to the back?	Adjust the length of elastic band and the back strap to adjust the wearing position, so that the tailbone fits with the thoracic spine, and the massage module is tightly attached to the massage area.

Fault Description	Possible Cause	Troubleshooting
Normal display, but the user feels nothing at the time of warm compress	Is the warm compress area heating?	After power-on, wait half a minute, then touch the silicone sheet with your hand. If you feel the heat, it is normal; if not, please contact the after-sales service.
	Is the warm compress temperature too low?	The default warm compress is in level 1 after power-on, please try to turn up the warm compress temperature moderately.
Warm compress button lamp is on, but the user feels nothing at the time of massage and warm compress	Lower battery of the rechargeable battery?	Charge the device in case of low battery.

Note:

If the fault cannot be eliminated or the product still cannot be operated satisfactorily, contact the after-sales service personnel, and do not disassemble or attempt to repair it by yourself.

7 Technical Parameters

Product Name	Philips Massage Back Brace
Product Model	PPM4561
Power Supply	Rechargeable lithium battery
Working Voltage	3.7V ---
Working Power	15W
Charging Voltage	5V ---
Charging Current	2A
Product Size	468x377x30mm
Net Weight	Approx. 590g
Battery Capacity	5000mAh
Interface Type	USB Type-C
Charging Time	Approx. 4 hours
Date of Manufacture	See the product certificate

8 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

HongKong

Hotline: 852-3188-4987

Service Email: Philips.Massage.hk@centuryedi.com

Taiwan

Hotline: 0800-661-600

Service Email: Philips.massager@xander.com.tw

Warranty Card

Product Model		Customer Name	
Seller		Contact Number	
Date of Purchase		Serial Number	
Service Center		Customer Address	
Maintenance Content		Date	service personnel

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