PHILIPS

sonicare

Teeth Whitening Kit

Instructions for use Mode d'emploi

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4 English

Contents

Introduction	5
Product contents	5
Intended use.....	5
Conditions of use	6
Warnings	6
Potential adverse effects	8
First aid instructions.....	10
Precautions	10
Forming upper and lower trays.....	11
Whitening instructions.....	16
Storage instructions	20
Achieving and maintaining whitening results	21
Gel refills.....	21
Whitening expectations.....	21
Disposal instructions	22
Frequently Asked Questions (FAQs).....	23
Symbols.....	25
Support	26

Introduction

Congratulations on your purchase of a Philips Sonicare **Teeth Whitening Kit**! You are on your way to brighter teeth and a healthy looking smile. For best results, use the product for nine consecutive days.

Product contents

- 2 Sets of Teeth Whitening Trays (1 extra set has been provided as a backup)
- 1 Tray storage case
- 1 Instruction for use
- 1 Quick start guide
- 3 Whitening gel syringes (6% or 9.5% Hydrogen Peroxide gel) 0.08 fl. oz. U.S./ 2.4 mL
- 3 Reusable mixing tips
- 1 Shade guide
- 1 Practice Card

Each 2.4 mL (0.08 fl. oz. U.S.) syringe contains enough gel for 3 whitening treatments for the whole mouth, so the 3 syringes can be used for 9 treatments.

Intended use

The Philips Sonicare Teeth Whitening Kit is a consumer use tooth whitening treatment intended for whitening discolored natural teeth. The gel should be used only with Philips Sonicare teeth whitening trays.

Conditions of use

Current research has not evaluated the possible effects of whitening procedures on all user types. Consult your dental professional before use especially if you:

- Are pregnant or lactating
- Have tooth or gum pain or sensitivity
- Have gum disease or are undergoing periodontal treatment
- Have loose teeth or fillings
- Have a history of severe allergic reactions
- Are currently under treatment for a serious illness e.g. immune compromised, etc.
- Are under 18 years of age

Warnings

- It is highly recommended to have a dentist perform a pre-treatment dental screening before starting the use of this product.
- Ask your dentist before using this product, particularly if you have sensitive teeth or gum disease that makes your gums bleed, if you require periodontal treatment or if you have damaged or broken fillings, crowns, veneers etc. or if you require crown and bridge work and have one or more temporary fillings or crowns.
- If you are younger than 18 years, pregnant, or if you have unfilled cavities, gum disease, or other

dental problems, please consult your dentist before exploring tooth whitening options.

- Products containing peroxides are not recommended for use by children under 13 years of age.
- Consult your oral health professional before prolonged use of this product.
- Do not use the product within two weeks prior to or immediately after dental restoration.
- Use of tobacco and/or alcohol can negatively affect your gums, and any pre-existing gum, throat, tongue, cheek or tooth injury may become worse when using a whitening product in combination with tobacco and/or alcohol.
- Do not use hot water to clean your trays and do not put in the in the dishwasher. Doing so will deform the trays.
- Do not use damaged trays as gel could leak, resulting in gum irritation.
- Product contains hydrogen peroxide. The active ingredient: 6% hydrogen peroxide (6% concentration) or 9.5% hydrogen peroxide (9.5% concentration) is present in the gel. Check the marking on the syringe.
- Do not use this product when sleeping, or when you may fall asleep.
- Keep out of reach of children.

8 English

- Syringes may be under slight pressure. Remove the tip protector carefully and point away from your face.
- Do not touch the gel if a syringe is broken or leaking.
- If irritation (such as redness, swelling, soreness) of the gums or the mouth occurs, discontinue use and consult an oral health professional.
- Avoid swallowing the cosmetic or part thereof.
- Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow.
- Avoid contact of the product (whitening gel) with the eye.
- Avoid contact of whitening gel with skin.
- The Philips Sonicare teeth whitening gel should only be used with Philips Sonicare teeth whitening trays.

Potential adverse effects

Adverse effects that may be caused by the use of a whitening product containing peroxide include, but are not limited to:

Gum irritation

When whitening gel comes in contact with your gums, it may cause gum irritation. This includes redness, a burning feeling, or white spots on the gums.

If you see any gel on your gums, or notice a burning feeling:

- Use a cotton swab or soft-bristled toothbrush to remove any excess gel from your gums.
- Refer to the quick start guide and the practice card to determine the correct amount of gel per tooth impression. Each syringe contains enough gel for at least 3 treatments (using top and bottom trays)
- If you still have gum irritation or sensitivity, treatment should be paused until the irritation is gone. Resume treatment with shorter, less frequent treatments until the desired whitening effect has been achieved.
- If the discomfort continues, discontinue the treatment and consult your dentist.

Sensitivity

Some users may experience increased tooth sensitivity to cold during or after whitening treatment, while others may have non-specific sensitivity or pain in their teeth, gums, tongue, lips or throat.

If you experience tooth sensitivity:

- Try whitening every other or every third day.
- If you are currently using 9.5% hydrogen peroxide gel, consider purchasing a 6% hydrogen peroxide gel refill kit for a more gradual and comfortable whitening experience. Gel refills can be purchased separately.
- An over-the-counter desensitizing toothpaste with 5% potassium nitrate (KNO₃) can be used to treat the tooth sensitivity, adhering to the respective toothpaste instructions.

10 English

- A Non-Steroidal Anti-Inflammatory Drug may be taken for pain, adhering to the respective drug's instructions for use as well as your doctor's recommendation for acceptable use.

Symptoms should subside within 1 to 3 days.

First aid instructions

- Contact a Poison Information Center or medical doctor at once if a large quantity (over 25% of syringe contents) is swallowed. Provide the ingredient label if possible. Do not induce vomiting and immediately consume a glass of water.
- If whitening gel gets into the eyes, hold the eyelids apart and flush the eyes continuously with running water for at least 15 minutes.
- If clothing, skin or hair contact occurs, remove and rinse contaminated clothing and flush the skin or hair with running water.
- If an allergic reaction to the whitening gel occurs, remove the whitening trays, wipe off teeth, rinse out mouth, and seek immediate medical attention.

Precautions

- Ensure trays have a snug and comfortable fit and do not have rough edges.
- Do not use any household products for teeth whitening.

- Rinse trays thoroughly between uses. Use cold water only, to avoid damage to the trays. If necessary, use a soft-bristled toothbrush to remove residual gel.
- Avoid exposure of hypersensitive teeth to whitening gel.
- Do not overfill trays with whitening gel as gum irritation may occur.
- Exceeding specified wear-times may result in tooth sensitivity and/or gum irritation, and will not lead to better results.

Forming upper and lower trays


Before first use, the tray needs to be fitted to your teeth. This ensures a custom fit, giving you the best whitening results. This process only needs to be completed once, after that the fitted trays can be re-used.

Note: Do not use trays if damaged.

To fit the trays, you will need the following:

- A large cup or medium-size bowl filled with cold water
- A large cup or medium-size bowl for boiling water
- Boiling water
- Scissors (optional)


12 English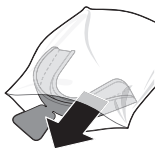

- 1 Remove two trays from the packaging. Rinse the trays before beginning the fitting process. Place them on a clean surface.

Note: The upper and lower trays are identical (before fitting).

Note: This teeth whitening kit includes two sets of trays. The second set of trays can be used if you need an extra pair.

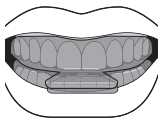- 2 Practice the fitting process before you start the actual fitting process:
 - a Place one tray over your upper teeth.
 - b Gently bite down on the tray.
 - c Suck the air from the tray.
 - d Use your fingers to apply pressure on the outside of the tray.
 - e Use your tongue to apply pressure on the tray from the inside, pressing the tray against your teeth.

- 3 Fill a bowl or a large cup with at least 4 inches (10 cm) of boiling water, so the tray can be fully submerged.

Warning: Be careful not to touch the hot water with your fingers.

- 4 Holding the tab, carefully submerge **one tray** in hot water holding the tab for 5 seconds. Make sure that the front surface of the tray is also submerged.

Note: Only fit one tray at a time.


- 5 Lift the tray vertically from the water using the tab.



- 6 Carefully and quickly place the tray around your teeth, ensuring that the sides don't fold in and stick together. If needed, use your fingers to spread the sides apart, to create space for your teeth.

14 English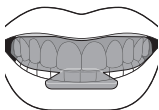


7 Make sure that the tray is properly centered and immediately perform the following steps at the same time:

- a Gently bite down on the tray.
- b Suck the air from the tray.
- c Use your fingers to apply pressure on the outside of the tray.
- d Use your tongue to apply pressure on the tray against your teeth.
- e Hold for 20-30 seconds.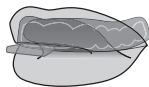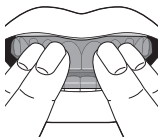

8 Remove the tray and dip it into the large cup or medium-size bowl filled with cold water or rinse it under the tap with cold water.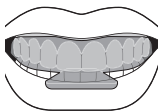

- 9 Place the tray back in your mouth. The tray has been correctly fitted if it fits snugly on your teeth and does not move when you open your mouth.

If you are satisfied with the fit, repeat steps 3-9 to fit the other tray to your lower teeth. Use new boiling water for best results.


Note: If the trays are too loose, you can try dipping the trays in the hot water again and repeat steps 3-9 to get a better fit, or start over with a new set of trays. The starter kit comes with two sets of two trays in case you need an extra pair.

When you have successfully fitted both trays, you can remove the tabs by cutting them off with scissors. You may also leave the tabs on to make it easier to insert and remove the trays.


Whitening instructions

Preparing for use and applying gel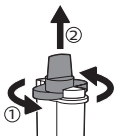


- 1 Floss and brush your teeth.
- 2 Remove the orange protective cap from the syringe by twisting it in a counterclockwise motion, and pulling it off.


Warning: Point the tip away from yourself while removing the cap, as the contents may be under pressure.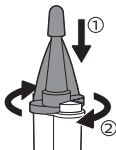

- 3 Place the mixing nozzle on the syringe and secure it by twisting it in a clockwise motion. Remove the tip protector from the tip of the nozzle.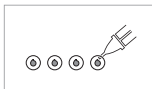

- 4 Use the included card to practice dispensing the correct amount of gel per tooth impression. Dispose card in trash after dispensing gel on it.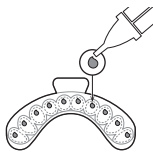


- 5 If the trays have been fitted correctly, you should see impressions of your teeth in the trays. Place a small amount of gel in each tooth impression, where the bottom and front surfaces of the trays meet.

Note: Image is not scale, refer to the Quick Start Guide or Practice Card for correct dot size.


Note: The whitening gel is absorbed into the tooth and whitens from the inside, so it is not important to cover the entire tooth with whitening gel.

Warning: Do not overfill your tray with gel, as this may cause gum irritation when excess gel gets pushed out of the trays and onto your gums.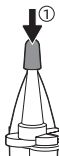


- 6 Place the small orange mixing nozzle cap back on the mixing nozzle.


18 English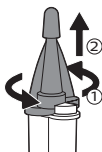

- 7 Rotate the mixing nozzle counterclockwise and pull it off the syringe.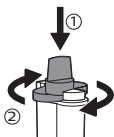

- 8 Place the orange protective cap back onto the syringe and rotate it clockwise to ensure that the ingredients in each side of the syringe stay separate during storage. Place the syringe and mixing nozzle back into the box.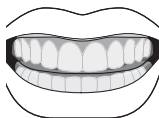


- 9 Place both trays with gel in your mouth. As you insert the trays, be careful not to apply too much pressure which may push the gel out of the trays.
- Use extra care to avoid getting gel on your gums. Remove excess gel with a cotton swab or soft dry toothbrush, to avoid causing gum irritation.
 - Bubbling within the trays is normal.
 - Do not eat, drink or smoke while wearing trays.

Wearing the whitening trays

The recommended wear time is 30 minutes per day. You may continue to wear the trays for more than 30 minutes.

Do not wear your whitening trays for more than 60 minutes, as this will not improve the whitening results and may lead to tooth sensitivity and/or gum irritation.


After use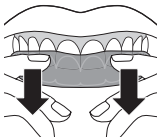

- 1 When you are done with your whitening treatment, remove the trays from your mouth, rinse your mouth with water and brush your teeth with a soft-bristled toothbrush to remove any excess gel.
- 2 Clean the trays by rinsing them under cold tap water. Do not use any soap or other cleaning agents. If necessary, use a toothbrush to remove residual gel from the trays.

Warning: Do not use hot water or the dishwasher to clean trays. Doing so will deform them.

20 English

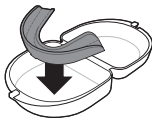- 3 Place the trays in the storage case and allow the trays to air-dry before closing the storage case. Store the trays and the syringes in a cool, dry place between 59°F and 77°F (15°C and 25°C). Do not refrigerate or freeze the gel.
- 4 Avoid staining foods, drinks (such as coffee, tea and red wine) and tobacco for 30 minutes after each treatment.
- 5 Continue good oral hygiene by brushing twice a day and flossing.

Note: It is normal to see dark colors in trays where you have amalgam (silver) fillings.

Storage instructions

Allow the trays to air dry before closing the tray case and storing. Store them in a cool, dry place between uses.

Whitening gel should be stored between 59 °F and 77 °F (15 °C and 25 °C) in a dark, dry place. Do not refrigerate or freeze. Storage temperatures above 77 °F (25 °C) will result in reduced shelf life. After opening a syringe, it can be kept for 30 days.

Achieving and maintaining whitening results

For best results, use your Philips Sonicare Teeth Whitening Gel at least 30 minutes daily for 9 days in a row.

To keep your white smile, the full 9-day whitening process can be repeated as needed every 4-6 months. The frequency will depend on a number of factors, such as your current dental condition, your natural teeth color and consumption of staining food and drinks (such as coffee, tea and red wine).

Gel refills

Gel refills can be purchased separately. Two different options are available: 6% hydrogen peroxide and 9.5% hydrogen peroxide.

If you are currently using 6% and are experiencing no sensitivity, consider 9.5% for your next refill, for better results.

If you are currently using 9.5% and experience tooth sensitivity, consider 6% for your next refill, for a more gradual and comfortable whitening experience. Refills can be used as needed every 4-6 months.

Whitening expectations

Whitening results vary from person to person, and depend on many factors, including: anatomy, age, tooth condition, hygiene and your current tooth shade.

22 English

- Your Philips Sonicare Teeth whitening gel refills comes with a shade guide, which you can use to track your progress over the 9 days of use
- Note that crowns, veneers, fillings, dentures and other restorations will not whiten.
- Tooth whitening may cause temporary dehydration of teeth resulting in white spots or striations. Uneven whitening will fade to normal within 2-7 days of using the whitening product.
- Final results will be achieved about 1 week after completion of your 9-day whitening treatment series.
- The area of the tooth closest to the gums may take longer to lighten than the biting edge. It will also remain the darkest part of the tooth.

Disposal instructions

Follow your country's rules for the separate collection of products. Correct disposal helps prevent negative consequences for the environment and human health.

Frequently Asked Questions (FAQs)

Question	Solution
The trays are too loose.	<p data-bbox="439 333 868 459">Practice the fitting process before submerging the trays in hot water. See step 2 under 'Tray fitting instructions'.</p> <p data-bbox="439 477 853 603">Use water that has just boiled, don't let it cool down for more than a minute before using it to form the trays.</p> <p data-bbox="439 621 860 775">When fitting the tray, use your fingers to apply pressure to the outside of the tray, and use your tongue to press the tray against your teeth.</p> <p data-bbox="439 793 840 954">Repeat the fitting process for a tray that is still too loose after fitting. The trays are correctly fitted when they don't move when you open your mouth.</p>

24 English

Question	Solution
The sides of the tray have folded in.	<p>Do not submerge the trays in hot water for more than 5 seconds.</p> <p>When removing the trays from the water, use your fingers to spread the sides apart, to create space for your teeth.</p> <p>When placing the trays in your mouth, be careful not to bite down on the sides. If needed, look into a mirror for better visualization.</p> <p>If the sides of the trays are stuck together, you may need to use a new tray. The starter kit comes with 4 trays.</p>
The trays are uncomfortable to wear.	<p>You can carefully trim away any excess material that extends far beyond the gum line if trays are uncomfortable. Use scissors to carefully cut away excess material. Make small adjustments and evaluate the fit and comfort when trimming away material. Ensure there are no sharp edges on the tray after cutting.</p>

Question	Solution
I am experiencing a burning feeling, redness and/or white spots on my gums.	When whitening gel comes in contact with your gums, it may cause gum irritation. This includes redness, a burning feeling, or white spots on the gums. See section "Potential adverse effects" for more information.
My teeth are sensitive to cold after a whitening treatment.	Some users may experience increased tooth sensitivity to cold during or after whitening treatment, while others may have non-specific sensitivity or pain in their teeth, gums, tongue, lips or throat. See section "Potential adverse effects" for more information.

Symbols

Reference to enclosed or attached information

Keep away from sunlight, keep away from heat

26 English

Temperature limit

Keep out of reach of children.

Manufacturer

Use by date

Batch code

Actual gel droplet size per tooth

Support

If you need information or support, please visit **www.philips.com/support** or (in the US or Canada only) call 1-800-682-7664 for assistance.

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All rights reserved
Discus Dental, LLC.