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PHILIPS ZOOM!

WhiteSpeed
Light-Accelerated
Tooth Whitening
牙齒美白膠

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Introduction

Philips Zoom! WhiteSpeed Light-Accelerated (25% hydrogen peroxide) Tooth Whitening Gel is a professional product used by dentists in their offices. This whitening gel product is composed of a dual-cylinder syringe filled with tooth whitening gel and a detachable mixing nozzle. One cylinder of the syringe contains a 31% hydrogen peroxide whitening gel, and the other cylinder contains a catalyst. When the two ingredients are mixed together they form a 25% hydrogen peroxide formula used for tooth whitening.

Philips Zoom! WhiteSpeed Light-Accelerated (25% hydrogen peroxide) Tooth Whitening Gel is used with the Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Kit and Philips Zoom! WhiteSpeed LED Accelerator. This product must be used under the guidance of a medical professional.

Included

- 1 Instructions for use
- 1 syringe containing 4.6 g of 25% hydrogen peroxide tooth whitening gel
- 1 detachable mixing nozzle for the syringe

Model and specifications

Model: Philips Zoom! WhiteSpeed Light-Accelerated Tooth Whitening Gel, specifications: 4.6g per syringe

Structure and composition

This product is composed of a tooth whitening gel and a catalyst. The main ingredients of the whitening gel are hydrogen peroxide and poloxamer 407, and the main ingredients of the catalyst are deionized water, poloxamer 407, propylene glycol, glycerin, and potassium hydroxide. The product is a dual-cylinder syringe with a filling ratio of 4:1, including a detachable mixing nozzle. The product's active whitening ingredient is hydrogen peroxide at a concentration of 25%.

Scope of application

This product is suitable for healthy people over the age of 18, and it is used for whitening exogenous teeth stains caused by tobacco, soda, tea, coffee, soy sauce, and other colored substances.

Contraindications

- 25% hydrogen peroxide tooth whitening gel cannot be used by the following people:
- Women who are pregnant or nursing.
 - Those suffering from serious diseases or disorders, such as immune disorders or AIDS
 - Minors under 18 years of age

Method of use

- This product can only be purchased by dentists.
- This product may only be used by dentists.
- Patients must be over 18 years of age.
- Patients shall not have other oral diseases and shall not have previously used tooth whitening methods to no effect.

Frequency of use

One syringe (4.6 g) of tooth whitening gel is intended for a single treatment. A single treatment typically consists of three to four times of 15-minute applications, about 1.15 g per time. A patient may be treated more than once at the discretion of the dental professional, allowing more than seven days between treatments.

Application

Philips Zoom! WhiteSpeed Light-Accelerated (25% hydrogen peroxide) tooth whitening gel is to be applied to patients' healthy natural teeth. This gel can be used on non-living teeth under the guidance of a dentist. This gel cannot be used on the gums, face, or oral tissues, and an appropriate covering is required to avoid injury or irritation to gums and other tissues.

Warnings

- Do not use the whitening gel on teeth that had fillings more than 7 days ago, because the color stability, as well as the bonding strength with the enamel and dentine may be compromised.
- Do not use on anesthetized teeth.
- It is not recommended for use when there is a wound on the gum tissue.
- Avoid contact with eyes and soft tissues. If the gel comes into contact with your eyes, wash with water and seek medical attention.

Possible side effects

- Tooth sensitivity or hypersensitivity
- Inflammation of the gums if protective coverings are not used correctly

Safety

- Dentists and patients must properly wear safety protection film and the Zoom! protective eyewear.
- Avoid contact of the whitening gel with soft tissue and eyes.

First aid instructions

- Contact a Poison Information Center or medical doctor immediately if a larger quantity (over 25% of the syringe) is swallowed. Provide the ingredient label if possible. Do not induce vomiting or drink large quantities of water.
- If whitening gel gets into the eyes, hold the eyelids apart and flush the eyes continuously with running water for at least 15 minutes.

Precautions

The whitening gel must be removed from refrigeration and warmed to room temperature before application:

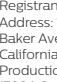
- Remove the syringe from refrigeration at least 6 hours before use (preferably the night before).
- If it was removed from refrigeration just prior to use, place the syringe in a cup of hot water (49 °C) for 10 minutes. Remove the syringe from the water, take the cap off, and wait 5 minutes before extruding gel.
- A single whitening gel syringe can only be used on one patient.

Please see the instructions for use of the Philips Zoom! WhiteSpeed Procedure Kit for complete treatment instructions.

Storage conditions

Philips Zoom! WhiteSpeed Light-Accelerated (25% hydrogen peroxide) Tooth Whitening Gel should be stored at 2°C - 10°C. Storage temperatures above 10°C will result in reduced shelf life.

Symbols

	Consult instructions for use
	Caution
	Keep away from sunlight, keep away from heat
	Temperature limits
	Use by date
	Batch code
	Catalogue number
	Do not re-use (for use on a single patient during a single procedure)
	Keep refrigerated

REFRIGERATE Keep refrigerated

Manufacturer

Production date

Registrant/Manufacturer Name: Discus Dental, LLC.

Registrant/Manufacturer Address: 1700A South Baker Avenue, Ontario, California 91761, USA

Production Address: 1700A South Baker Avenue, Ontario, California 91761, USA

Registrant Contact Info: +1 (310) 845 8260

After Sales Service Company: Philips (China) Investment Co., Ltd.

After Sales Service Company Address: Building A1, No. 718 Lingshi Road, Jing'an District, Shanghai

After Sales Service Contact Info: 4008 800 008

Agent Name: Philips (China) Investment Co., Ltd.

Agent Address: Building A1, No. 718 Lingshi Road, Jing'an District, Shanghai

Agent Contact Info: 4008 800 008

Medical Device Registration Certificate No.: GXZJ20203170315

Product Technical Requirements No.: GXZJ20203170315

Production Date: See product label

Shelf Life: 24 months

Shipping Conditions: Ship at ambient temperature

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All rights reserved