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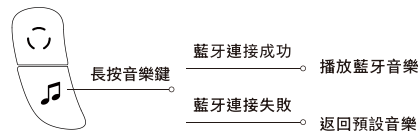
© 2024 AquaShield  
All rights reserved  
Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.  
This product has been manufactured by and is sold under the responsibility of Hong Kong AquaShield Health Technology Company Limited and Hong Kong AquaShield Health Technology Company Limited is the warrantor in relation to this product.  
Made in China

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PPM2321

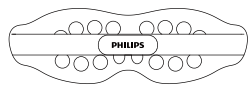
Quick Start Guide  
快速入門指南


1. Remove contact lenses, frame glasses and pendants around the eyes.


2. Adjust the elastic headband, ensuring that the product fits comfortably around the eyes.

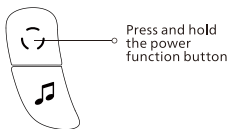
3. Press and hold the power function button for 1.5 seconds to turn on/off the device, with voice prompts of "massage on/ massage off".

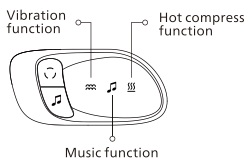
4. Press and hold the button for 1.5 seconds to turn on the product. It will be in [Energizing Mode] by default with vibration and hot compress on. After the voice announces "Energizing Mode", the preset music will automatically play.

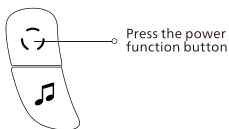5. Press the button for 0.5 seconds to cycle through the massage modes in an order of [Energizing Mode], [Eye Care Mode], [Sleep-aid Mode], [Relaxing Mode] and [Eye Warming Mode].

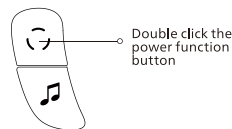
6. Double click the button within 1 second to turn off/on the hot compress function, with the voice prompt of "turn off heating" or "turn on heating".

1


7. Press and hold the music button for 2 seconds, there will be a voice prompt of "search for new bluetooth device". Users can connect accordingly Bluetooth device on electronic devices.

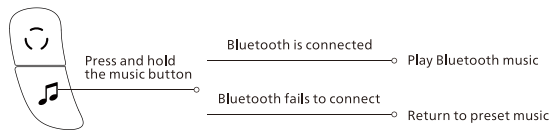To disconnect the current Bluetooth device and search for a new one, press and hold the music button for 2 seconds. After turning on the product, if Bluetooth is connected previously, it enters Bluetooth music mode by default. If Bluetooth is not connected, it returns to play the preset music.


a. After the electronic device successfully connects to Bluetooth, there will be a voice prompt "the bluetooth device is connected successfully".  
b. If the connection fails, there will be a voice prompt "the bluetooth device fails to connect", and the product will automatically return to play the preset music.

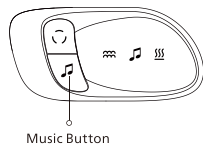8. Press the button for 0.5 seconds to switch to the next music track. Double click the music button within 1 second to turn off/on music, with the voice prompt of "turn off music" or "turn on music".

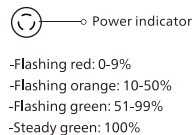9. When battery power is less than 10%, the indicator of the product will flash quickly in red, with a voice prompt "the battery power is low, please charge it in time". When charging, the indicator will flash slowly. After charging is completed, the indicator turns green.

2

10. The preset message time is 15 minutes, and the product will automatically shut down after 15 minutes, with the voice prompt of "massage off". There is no voice prompt when shutting down in [Sleep-aid Mode].

1. It is recommended to use the low level during the first use to avoid irritation.  
2. Close your eyes when using the product. After use and removal of the product, blurred vision may occur. Closing your eyes for 1-3 minutes can relieve the symptoms.  
3. After use, wipe and clean the surface of product with a damp cloth. Do not rinse it directly with water.

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