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©2024 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Hong Kong AquaShield Health Technology Company Limited and Hong Kong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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1 Readme

Symbol Description

People Restricted or Prohibited from Use

Precautions for Use

Precautions for Use Places

Precautions for Maintenance

2 Product Features

3 Product Overview

4 How to Use

Preparations Before Use

Operation

Charging Method

Description of Indicator Lights

5 Maintenance

Cleaning

Disposal

6 Troubleshooting

7 Technical Parameters

8 Service Guideline for Warranty

Service and Support

Warranty Card

9 Packing List

1 Instructions

Please read this manual carefully before using the product and keep it properly.

The manual is available at www.philips.com

Symbol description

To avoid personal harm or property damage, be sure to follow the followings.

- The following symbols respectively illustrate the items that you must observe.

 Indicates prohibition

 Indicates the actions that need to be subject to instructions

People Restricted or Prohibited from Use

 The product should be used with caution by the following groups:

People under treatment at a medical institution or in the following conditions, should consult with a doctor or physical therapist before using the product.

- People with fever or body temperature exceeding 38°C.
- People with loss of skin temperature sensation.
- People who have experienced low-temperature burns.
- People with abnormal body sensations.
- Children aged 6-12 should use it under the guidance of an adult guardian.

- People with sensitive skin, facial allergic dermatitis, and facial areas with cosmetic and plastic materials should be guided by a physician to use the product.

 The following groups are prohibited from using the product

People who are prohibited from exercising or stretching activities as advised by doctor. For example, people with thromboembolism, severe aneurysm, acute venous aneurysm, various skin inflammations, or various skin infections. Because they are easy to worsen, these people are prohibited from using this product.

- Do not use while driving.
- Do not use while sleeping.
- Children under the age of 6 are not allowed to use.
- Patients with infectious diseases are not allowed to use.
- Pregnant and menstrual women are not allowed to use.
- Patients in recovery after eye surgery are not allowed to use.
- Patients and those without operational skills are not allowed to use.
- Do not use it for massage on other parts of the body except for eyes.
- Patients who have eye surgery or have heart disease are not allowed to use.
- Patients with high fever, malignant tumors, and osteoporosis are not allowed to use.
- Patients who are being treated and feel physically abnormal are not allowed to use.
- Patients with hypertension, high intraocular pressure, arteriosclerosis, or cerebral thrombosis are not allowed to use.
- People with symptoms of eye diseases such as glaucoma, cataracts, retinal detachment, and pinkeye are not allowed to use.

Precautions for Use

- ❗ The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
- ❗ This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- ❗ Children should be supervised to ensure that they do not play with the appliance.
- ❗ If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.
- ❗ This appliance contains batteries that are only replaceable by skilled persons.
- ❗ Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.
- ❗ Power off the product immediately in case of abnormality occurs.
- ❗ The product is not suitable for diagnostic or therapeutic purposes.
- ❗ Contact lens wearers should remove their contact lenses before use.
- ❗ If the product has not been used for a long time, be sure to confirm in advance whether the switches and other units work properly before using it again.
- ❗ The product is not waterproof. When using it, do not allow the water to enter the interior of product, nor immerse the product in water.
- ❗ Close your eyes when using the product. After use and removal of the product, blurred vision may occur. Closing your eyes for 1-3 minutes can relieve the symptoms.

- ⊘ Do not use the product for two persons simultaneously.
 - ⊘ Do not intentionally exert forces on the product.
 - ⊘ It is strictly prohibited to disassemble, modify or repair the product by yourself.
 - ⊘ Do not fall off the product, nor beat it strongly, as it can easily damage the product.
 - ⊘ Do not forcefully pull the product, nor use knives or sharp objects to stab the product.
-

Precautions for Use Places

- ⊘ Do not use the product in bathrooms or places with high humidity.
 - ⊘ Do not cover the product with towels, cloth or other objects for use.
 - ⊘ Do not use the product in areas with a room temperature above 35°C or in direct sunlight.
 - ⊘ Do not use the product near bathtubs, showers, washbasins, or other high humidity utensils that hold water.
 - ⊘ Do not use the product near a stove, inside an electric heating quilt, or on heating appliances such as an electric blanket, which is easy to cause fire or failure.
-

Precautions for Maintenance



- ❗ Turn off the product before maintenance.
- ❗ When not in use for a long time, be sure to unplug the charger.
- ❗ If the surface of product needs to be cleaned, please clean it with a damp cloth and do not rinse it directly with water.

- ❗ If the product has not been used for a long time, be sure to confirm in advance whether the switches and other units work properly before using it again.
- ⊘ Do not drop the product or pour water on the product.
- ⊘ Do not clean the product with volatile oil, diluent, alcohol, strong alkaline cleaner or bleaching agent for cleaning during the maintenance.

2 Product Features

The product is designed to relieve eye fatigue through various methods of vibration massage and hot compress. A variety of preset massage modes are suitable for different groups of people for eye relaxation. Bluetooth connection function allows you to relax your eyes while enjoying your favorite music.

3 Product Overview

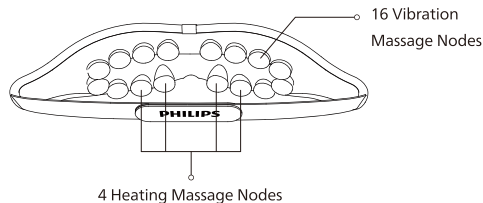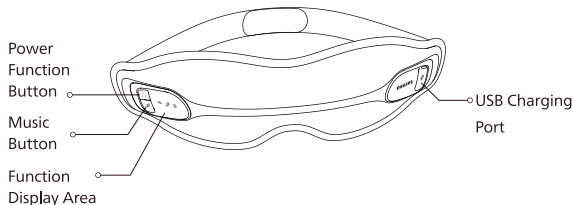4 How to Use

Preparations Before Use

1. Check whether the device and accessories are complete.
2. Check whether the product is charged. If the battery capacity is too low, the device may not turn on or its functionality may be turned off, please charge it before use.
3. Remove the pendants and decorations around the eyes before use to avoid affecting the massage effect.
4. Adjust the elastic headband to a size suitable for your head shape, ensuring that the product fits comfortably around the eyes before turning it on.

Operation

Power Function Button

1. Press and hold the button to turn on the product. It will be in [Energizing Mode] by default with vibration and hot compress on. After the voice announces "Energizing Mode", the preset music will automatically play.

2. Press the button for 0.5 seconds to cycle through the massage modes in an order of [Energizing Mode], [Eye Care Mode], [Sleep-aid Mode], [Relaxing Mode] and [Eye Warming Mode].

- [Energizing Mode] High speed vibrating + temperature level 1 (40°C) + music track (Waves). The product will automatically shut down after 15 minutes, with the voice prompt of "massage off".

- [Eye Care Mode] Rhythmic vibrating + temperature level 2 (42°C) + music track (Stream). The product will automatically shut down after 15 minutes, with the voice prompt of "massage off".

- [Sleep-aid Mode] Soft vibrating + temperature level 1 (40°C) + music track (Dripping Water). The product will automatically shut down after 15 minutes, without voice prompt.

- [Relaxing Mode] Dynamic point-vibrating + temperature level 1 (40°C) + music track (Early Morning). The product will automatically shut down after 15 minutes, with the voice prompt of "massage off".

- [Eye Warming Mode] Temperature level 2 (42°C) + music track (Spring Rain). The product will automatically shut down after 15 minutes, with the voice prompt of "massage off".

3. Double click the button within 1 second to turn off/restart the hot compress function, with the voice prompt of "turn off heating" or "turn on heating".

4. The preset massage time is 15 minutes, and the product will automatically shut down after 15 minutes, with the voice prompt of "massage off". There is no voice prompt when shutting down in [Sleep-aid Mode].

Music Button

Preset Music Mode

1. Press the button for 0.5 seconds to switch to the next music track.

2. Double click the music button within 1 second to turn off/on music, with the voice prompt of "turn off music" or "turn on music".

3. Press and hold the music button to search for a new Bluetooth device (when used for the first time)/disconnect the current Bluetooth device and search for a new one.

Bluetooth Music Mode

1. Bluetooth Name: Philips PPM2321

2. Bluetooth pairing:

- After powering on, if a Bluetooth device has been connected before, it will enter Bluetooth music mode by default. If a Bluetooth device is not connected, it will automatically return to playing the preset music. During the use of the product, users can search for devices by long pressing the music button, pair the devices in the electronic device, and then enter the Bluetooth music mode.

- After the electronic device successfully connects to Bluetooth, there will be a voice prompt "the bluetooth device is connected successfully".

- If the connection fails, there will be a voice prompt "the bluetooth device fails to connect", and the product will automatically return to play the preset music.

Press the music button/operate in the corresponding electronic device to switch to the next music track.

Double click the music button to turn off/on music, with corresponding voice prompts.

Press and hold music button to search for a new Bluetooth device (when used for the first time)/disconnect the current Bluetooth device and search for a new one. To return to built-in music: after the user disconnects the Bluetooth device on their electronic device, it will return to play the preset music.

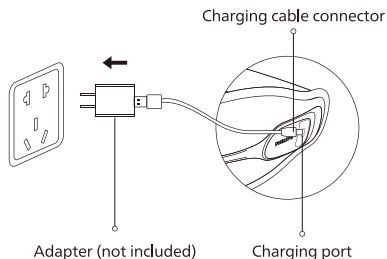
Search for new
Bluetooth devices

Bluetooth is connected —○ Bluetooth music

Bluetooth fails to connect —○ Preset music

Charging Method

1. Connect the charging cable to the product charging port.
2. When charging, the indicator will  flash slowly. The indicator will remain on green when charging is complete.
3. During the charging process, the product cannot be turned on.
4. The product uses rechargeable batteries, and the charging time is about 3 hours.
5. During use, if a low-battery voice reminder appears, please suspend use. Please use it after it is fully charged.
6. Multiple charging methods are available: computer USB interface/mobile power/5V \equiv 1A power adapter.

-  During charging, when the power is below 10%, the indicator  flashes red slowly; when the power is between 10% and 50%, the indicator  flashes orange slowly; when the power is between 51% and 99%, the indicator  flashes green slowly. The indicator remains green when fully charged.
-  When the product is low on power, there will be a voice prompt "the battery power is low, please charge it in time".
-  Please turn off the device before charging. Do not use it while charging.

Description of Indicator Lights

1. Power Function Button

- 1) Battery status display during use

Power Indicator	Status	Description
Red	Flash	Current battery capacity 0%-9%
Orange	Remain on	Current battery capacity 10-50%
Green	Remain on	Current battery capacity 51%-100%

- 2) Battery status display during charging

Power Indicator	Status	Description
Red	Flash slowly	Current battery capacity <10%
Orange	Flash slowly	Current battery capacity 10-50%
Green	Flash slowly	Current battery capacity 51%-99%
Green	Remain on	Current battery capacity 100%

2. Music Button:

When the preset music is on, the music button turns green. When Bluetooth music is on, the music button turns blue. When searching for Bluetooth devices, the music button  flashes in blue.

3. Function status display

- 1)  :When vibration is turned on, the vibration indicator lights up in green; when vibration is turned off, the vibration indicator does not light up.
- 2)  :When preset music is turned on, the music indicator lights up in green; when Bluetooth music is turned on, the music indicator lights up in blue.
- 3)  :Heating on: green light is on at hot compress level 1, and orange light is on at hot compress level 2;
Heating off: the hot compress indicator does not light up.

5 Maintenance

Cleaning

1. After using the product, if there is any oil stain, wipe it with a dry clean soft cloth.
2. If the product is particularly dirty, gently wipe it with a damp cloth or alcohol cotton on the leather cover.
3. Do not allow water to enter the interior of product. Place and dry it naturally.
4. Do not place heavy objects on the product to avoid moisture and dirt.

- ! During maintenance, be sure to unplug the charging cable with dry hands.
 - ! After use, do not place the product in direct sunlight or in areas with high humidity. If it is not used for a long time, after removing the dirt, place it in a storage bag and keep it away from dust.
 - ! It is recommended to charge the product at least once every month if it is not used for a long time.
-

Disposal

The lithium-ion battery used in this product is a valuable resource that can be recycled. Take out the lithium-ion battery when it is used up and discarded, and then hand it over to a professional institution that can recycle the rechargeable batteries for disposal.

Steps to remove the battery:

1. Pry open the shell decoration on the right side.
2. Loosen the screws on the circuit board and take out the circuit board.
3. Unplug the battery terminals and remove the battery.

- ! It is prohibited to replace the battery by users themselves.
- ! Be sure to remove the battery from the device before disposal.
- ! The device must be powered off before the battery is removed.
- ! The battery must be disposed of in a safe manner.

6 Troubleshooting

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center at the region where you are located

Fault Description	Possible Cause	Solution
Start-up failure	Power function button is not turned on.	Press the power function button again.
	Battery capacity is low.	Charge the product.
	A 15-minute massage completed.	Press the power function button again.
Charging failure	The charging connector is not properly inserted.	Insert the charging connector properly again.
	The original charging cable is not used.	Replace with the charging cable that matches the product.
Charging time is too long	The corresponding adapter is not used.	Replace with a 5V::: 1A adapter.
No music	The music is paused.	Double click the music button to restart music function.

Weak vibration and compression intensity	The headband is too loose.	Adjust the length of headband according to personal condition to ensure the massager can be comfortably worn around the eyes.
Bluetooth connection failure	The distance between the paired electronic device and the massager is out of limit.	Shorten the distance between the electronic device and the product.
	Music button is accidentally pressed, and Bluetooth function is turned off.	Press the power function button to shut down, then restart the Bluetooth function.
Unable to play music even though bluetooth is connected successfully	The correct electronic device is not connected.	Press and hold the music button for 2 seconds to disconnect the current device, search for a new one and pair again.
Low Bluetooth music sound	Electronic device playback volume is too low.	Adjust the playback volume of the electronic device.
Short usage time	Does not use under normal temperature environment.	Use the product in an ambient temperature of 10-35°C

After the product is unused for a long time, be sure to confirm that the function is normal before reuse.		
Regular check	Check for the following faults	In the event of the above faults, stop using the device immediately and cut off the power supply to prevent accidents. Please send it to our maintenance center.
	<ul style="list-style-type: none"> · Whether the charging cable or connector is hot · Whether the charging cable is deeply scratched or deformed, or has a burnt smell · Whether there is abnormal sound or vibration during work · Whether the product can work properly after powering on · Whether there are other abnormalities or faults 	

Note:

If the fault cannot be eliminated or the product still cannot operate satisfactorily, please contact the after-sale service personnel, and do not disassemble or attempt to repair it by yourself.

7 Technical Parameters

Product Name	Philips Eye Massager
Product Model	PPM2321
Operating Voltage	3.7V ===
Operating Power	4W
Charging Voltage	5V ===
Charging Current	550mA
Interface Type	USB Type-C
Product Dimension	298x93x75mm
Battery Capacity	1200mAh
Net Weight	220g
Date of Production:	See the product certificate

8 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

HongKong

Hotline: 852-3188-4987

Service Email: philips.massage.hk@centuryxi.com

Taiwan

Hotline: 0800-661-600

Service Email: Philips.massager@xander.com.tw

Warranty Card

Product Model		Customer Name	
Seller		Contact Number	
Date of Purchase		Serial Number	
Service Center		Customer Address	
Maintenance Content		Date	Service Personnel

9 Packing List

Massager	1x	Storage Bag	1x
USB Charging Cable	1x	User Manual	1x
Quick Start Guide	1x	Product Certificate	1x

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