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©2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Hongkong AquaShield Health Technology Company Limited and Hongkong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

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6 Troubleshooting

7 Technical Parameters

8 Environmental Protection Description

9 Service Guide for Product Warranty

Warranty and Service

Warranty Card

10 Packing List

1 Readme

Please carefully read the operation manual before use and keep it properly. The manual is available at www.philips.com

Symbol Description

To avoid personal injury or property damage, be sure to observe the contents below.

- The following symbols are used to illustrate what you should observe.

 Indicates prohibition

 Indicates the actions that need to be subject to instructions

People Suitable to Use

 The product should be used with caution by the following groups/ under the following circumstances

- People who have a pacemaker or other related medical electrical products implanted in their bodies
- People who have metal, silicone or plastic materials injected into their bodies as a result of bone fractures or plastic surgery
- People who have ever had skin trauma
- People whose body temperature is above 38 degrees Celsius

- People who are easy to suffer from numbness of body
- People who have difficulty with respirators
- People suffering from hypertension
- People having acute symptoms such as visceral diseases (gastritis, hepatitis, gastroenteritis)
- People who have disturbances of skin sensation
- People suffering from body paresthesia
- People suffering from lumbar disc herniation, spondylitis
- People suffering from deformed arthritis, arthritis, and gout
- People suffering from spinal fractures and swelling of bones

 The product is not allowed to be used by the following groups/ under the following circumstances

- People who are unable to express their will or who cannot operate the machine themselves to use the product
- This product is not suitable for people who have physical, sensory or mental capacity decline and lack experience and knowledge (including children)
- Please stop using immediately when you feel lower back pain, dizziness, vomiting, palpitation and other abnormal body reactions during use.
- This product is set for 15-minute timing at a time. Do not use it for no more than 30 minutes each time. After each use, stop the device for 15 minutes to ensure the extension of its service life.
- Do not use the product under the following circumstances:
 - When you are severely tired
 - Within one hour before and after meals
 - When you are drunk

Precautions for Use

- ❗ The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
- ❗ This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- ❗ Children should be supervised to ensure that they do not play with the appliance.
- ❗ If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.
- ❗ This appliance contains batteries that are only replaceable by skilled persons.
- ❗ Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.
- ❗ Check whether the product is intact before use. If any damage is found, regardless of its extent, stop using the product immediately.
- ❗ Remove the dust on the charger on a regular basis. The accumulated dust on the plug is likely to cause poor insulation due to humidity, and eventually lead to fire.
- ❗ If you cannot use it or feel abnormal, stop using it immediately
- ❗ In case of power cut, please unplug the USB charging cable.
- ❗ When the product is not in use, the USB charging cable must be unplugged.
- ⊘ Do not disassemble, modify or repair the product by yourself, nor unzip the product by yourself.
- ⊘ Do not use the device when your body is wet, otherwise it is easy to get an electric shock.
- ⊘ Do not pull the product with force, nor stab it with a knife or a sharp object.

Precautions for Maintenance

- ❗ Be sure to unplug the charging cable during maintenance.
- ❗ Please properly dispose of the packaging bag of the product and keep it away from the infants.
- ⊘ The product has built-in lithium batteries. Do not disassemble or modify the product by yourself, immerse the product (controller) in water or toss it into the fire, otherwise the product is likely to be damaged.
- ⊘ Do not damage, modify, bend or pull the power cord with force. Do not place heavy objects on the power cord or squeeze the power cord.
- ⊘ Do not use volatile oil, thinner, alcohol, strong alkaline detergent, bleach or other cleaning agents for maintenance.
- ⊘ Keep the product out of reach of children and infants. Do not let them touch or operate the product, otherwise it is likely to lead to burns, electric shocks or personal injuries.
- ❗ Make the device cool down before storing.

Class III appliance

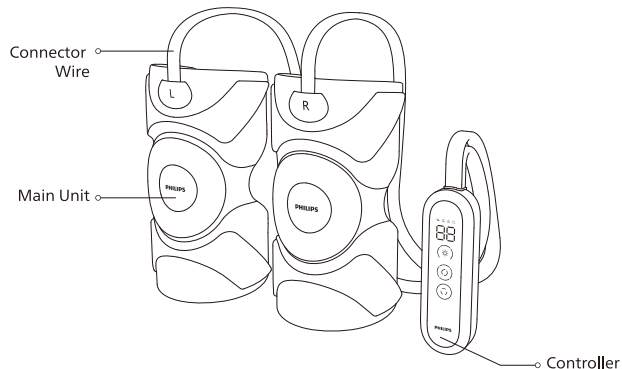
R3A020
RoHS

2 Product Features

Through the airbag and warm compress functions, and equipped with multiple optional massage modes and levels, this product is suitable for different people for knee relaxation.

3 Components

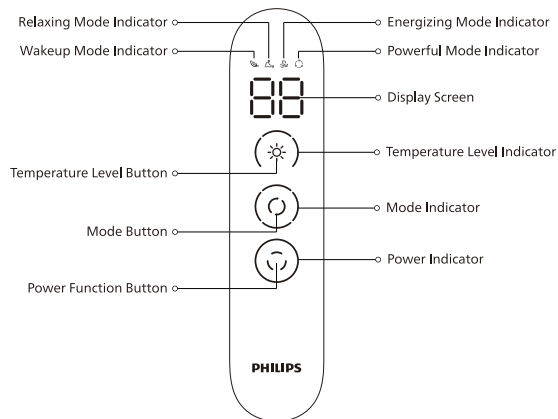
4 How to Use

Preparations Before Use

1. Check whether the device and its accessories are complete.
2. Wear the massager on the knees and adjust it to a comfortable state.

! Since everyone has different sensitivity to temperature, it is recommended to wear clothes when the temperature is above 45°C to avoid burns.

Operation

Power Function Button

1. Press and hold the power function button for 2 seconds to turn on/off the massager. After power-on, the massager will enter the wakeup mode by default, enabling the warm compress at 40°C with a voice prompt "Massage On".
2. Double click the power function button to lock/unlock the screen with the voice prompts "Lock screen" and "Turn on screen". After the screen is locked, the status indicator is in the state of the orange breathing lamp, the screen displays the current warm compress temperature (it displays 00 when the warm compress is off), and the mode lamp turns on to indicate the current mode. Press and hold the power function button to turn off the massager.
3. The default timing of the product is 15 minutes. It is set to shut down automatically after 15 minutes of operation and enter the standby state with the voice prompt: "Turn off massage".

Mode Button

Press the mode button to switch the massage mode (Wakeup Mode - Relaxing Mode - Energizing Mode - Powerful Mode - Turn off air pressure) circularly. During adjustment, the mode level indicator turns blue progressively, the corresponding mode indicator lights on, and there is a voice prompt of the corresponding mode for each adjustment.

Temperature Level Button

Press the temperature level button to switch the temperature (40°C, 45°C, 50°C, heating off) circularly, and the display screen in the middle will display the current selected temperature. When adjusting, the corresponding temperature indicator is illuminated progressively (orange), and there is a voice prompt (40°C, 45°C, 50°C) of the corresponding temperature for each adjustment.

Low Battery Reminder

When the battery remaining capacity is less than 10%, the indicator lamp ring flashes orange for 5 seconds to prompt, and then the massager shuts down. At the same time, there is a voice prompt "The battery power is low, please charge it in time".

Charging State

The indicator lamp ring flashes blue slowly when charging, and stays on blue when fully charged.

Charging

1. Insert the charging plug into the charging port on the side of the battery holder.
2. Plug the charger plug into a 100-240V~, 50/60Hz power socket.
3. The indicator lamp ring flashes blue slowly when charging, and stays on blue when fully charged.
4. It takes about 5 hours to fully charge the product.

Indicator Description

Indicator Type	State Indication	Voice Prompt
Power indicator	It turns orange by default after power-on. If the battery is low, it flashes quickly for 5 seconds to prompt and then shuts down. While charging, it flashes blue slowly, and remains blue after fully charged.	Massage On / Turn off massage / The battery power is low, please charge it in time (No voice prompt while charging)
Temperature level indicator	Display in orange for three levels, the first level light on by default after power-on. Press the button again, the second level light will turn on.	40°C/45°C/50°C/ Turn off heating
Wakeup mode indicator	Enter this mode by default after power-on, the first green icon light on. Press the mode button to switch to the next mode.	Corresponding voice prompts of Wakeup Mode/ Relaxing Mode/ Energizing Mode/ Powerful Mode/ Turn off air pressure
Relaxing mode indicator	Press the mode button to enter relaxing mode, and the second icon turns blue.	
Energizing mode indicator	Press the mode button to enter energizing mode, and the third icon turns orange	

Indicator Type	State Indication	Voice Prompt
Powerful mode indicator	Press the mode button to enter powerful mode, and the fourth icon turns red	Corresponding voice prompts of Wakeup Mode/ Relaxing Mode/ Energizing Mode/ Powerful Mode/ Turn off air pressure
Digital tube display	The set temperature is displayed in the heating state, and 00 is displayed after the heating is off	/

5 Maintenance

Cleaning

- 1.Remove the dust from the cloth cover of the product with a vacuum cleaner rather than by patting.
- 2.Wipe off the dirt on the surface with a clean rag.
- 3.For the heavy dirt, dampen a rag with the neutral detergent and then gently wipe it. Prevent the water from entering the product. Let it air dry.

! During maintenance, be sure to unplug the power cord, but do not unplug the cord with wet hands.

Storage

- Keep out of reach of children.
- Do not put it in places with direct sunlight, high temperature and humidity.
- Keep it in a dry and ventilated place.

Disposal

The lithium-ion battery used in this product is a valuable resource that can be recycled. Take out the lithium-ion battery when it is used up and discarded, and then hand it over to a professional institution that can recycle the rechargeable batteries for disposal.

1. Do not replace the battery by yourself during normal use.
2. The battery must be removed from the appliance before the it is discarded.
3. The appliance must be powered off before the battery is removed.
4. The battery must be disposed of in a safe manner.

Steps:

1. Pry open the cover with a tool and then unscrew with a screwdriver;
2. Take out the PCB screws on the lamp board support and main control board, and remove all the corresponding terminals;
3. Remove the screws on the cover with a screwdriver, and take out the cover;
4. Remove the battery from the lower case for safe disposal.

6 Maintenance

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center in your region.

Fault Description	Possible Cause	Solution
Startup Failure	Low battery	Connect the charger and use the product after charging
	The power function button is not on	Press and hold the power function button to power on
	Short circuit of the circuit board	Contact the service center
Stop working during use	The battery is low	Connect the charger and use the product after charging
	The 15-minute timed massage ends, and it shuts off.	Press and hold the power function button to power on
There is a sound coming from the product	The sound of air in u ation and bleeding of the internal air pump, which is a normal working state	No treatment required

Fault Description	Possible Cause	Solution
Product smokes	Short circuit of internal wires	Stop using it immediately and contact the service center

If the product has been left unused for a long time, be sure to confirm that its functions are normal before using it again

Are there any of the following faults?		
Regular check	<ul style="list-style-type: none"> Whether the power cord or the plug is hot Whether the power cord is deeply scratched or deformed, or has a burnt smell Whether there is abnormal sound or vibration during work Whether the product can work properly after switch on Whether there are other abnormalities or faults 	In the event of the above faults, stop using the device immediately and cut off the power supply to prevent accidents. Please send it to our maintenance center.

Note:

If the fault cannot be eliminated or the product still cannot operate satisfactorily, contact the after-sale service personnel, and do not disassemble or attempt to repair it by yourself.

7 Technical Parameters

Product Name	Philips Knee Massager
Product Model	PPM5521
Operating Voltage	3.7V
Operating Power	15W Max
Charging Voltage	5V
Charging Current	2A
Battery Capacity	4800mAh
Interface Type	Type-C
Charging Time	≤5hours
Net Weight	About 0.78kg
Product Dimension	1020x543x40mm
Date of manufacture	See the product certificate

8 Environmental Protection Description

Equipment name: Philips knee massager, type designation (type): PPM5521						
Unit	Restricted substances and its chemical symbols					
	Lead (Pb)	Mercury (Hg)	Cadmium (Cd)	Hexavalent chromium (Cr+6)	Polybrominated biphenyls (PBB)	Polybrominated diphenyl ether (PBDE)
Circuit board	—	○	○	○	○	○
Cover	○	○	○	○	○	○
Motor	○	○	○	○	○	○
Power cable	○	○	○	○	○	○

Note 1: "Exceeding 0.1 wt %" and "exceeding 0.01 wt %" indicate that the percentage content of the restricted substance exceeds the reference percentage value of presence condition.

Note 2: "○" indicates that the percentage content of the restricted substance does not exceed the percentage of reference value of presence.

Note 3: The "—" indicates that the restricted substance corresponds to the exemption.

9 Service Guideline for Warranty

Warranty and Service

The product is a household appliance with exquisite design and high-quality component. Within two years from the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal operation confirmed by our maintenance service.

The warranty service does not include consumable components, transportation fee and door-to-door service. Please show the proof of purchase to the service personnel during maintenance.

You cannot receive free warranty service under the following conditions:

1. Damage by improper operation, such as improper components connection, improper installation, deprived of manual use, mis-operation.
2. Negligence and damage because of transportation or other accident.
3. Maintenance and modification without approval.
4. Other damage caused by force majeure (such as natural disaster, abnormal voltage, etc.).
5. Damage caused by general use outside home (e.g. industrial and commercial use).
6. Aging, wearing and tearing caused by normal operation of the product without affecting normal operation of the product.

This statement serves as a publicity for free maintenance within the specified time limit and conditions, and does not limit the legal rights of consumers.

Please contact our local maintenance service center for any information or difficulties or you can consult through the hotline.

Hong Kong Importer: Century Field International Ltd.

Address: Room 505, 5/F, Nam Wo Hong Building, 148 Wing Lok Street,
Sheung Wan, Hong Kong

Hotline: 852-3188-4987

Service Email: philips.massage.hk@centuryfield.com

Taiwan Importer: Xander International Corp.

Address: No. 5-2, Section 2, Fenliao Road, Linkou District, New Taipei City,
Taiwan

Hotline: 0800-661-600

Service Email: philips.massager@xander.com.tw

Warranty Card

Product Model		Customer Name	
Seller		Contact Number	
Date of Purchase		Serial Number	
Service Center		Customer Address	
Maintenance Content		Date	Service Personnel

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