Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

©2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

PHILIPS

Neck and Shoulder
Massager

Neck Massager

PPM3304

Quick Start Guide

1. Wrap the U-shaped neck pillow around your neck (or other parts that need to be massaged) and fix it with the lock catch.

2. Press and hold the power function button for 2 seconds to start the device (voice: "Massage on"), then the device will enter the soothing mode by default with hot compress function on. The Mild Mode begins to work, and the indicator turns orange.

Press and hold the button for 2 seconds to start the device

3. Press the power function button to turn off the hot compress function (voice: "Turn off heating"), and the indicator lamp turns blue; Press the button again to turn on hot compress function (voice: "Turn on heating"), and the indicator lamp turns orange. Circle in this way.

Press the button to turn on /off heating

4. Press the mode button to switch to the Energizing Mode (voice: "Energizing Mode") in turn. Press the button again to switch to the Kneading Mode (voice: "Kneading Mode"). Circle in this way.

Press the button to switch the modes

5. Press the intensity button to switch to the Mild Mode (voice: "Mild Mode"), and Powerful Mode (voice: "Powerful Mode") in turn. Press and hold the intensity button for 2 seconds to pause the massage (voice: "Stop massage"). Press this button again for 2 seconds to start the massage (voice: "Turn on massager");

Press the button to switch the intensity level

6. The product will be automatically powered off after working for 15 minutes. To discontinue halfway, press and hold the power button to turn it off.

7. At the time of charging, the indicator lamp flashes blue, and the blue light remains on when the device is fully charged. Reminder of low power capacity: The indicator lamp flashes blue for 5 seconds as an alarm (voice: "The battery power is low, please charge it in time").

- ! 1. If you feel uncomfortable during the massage, please stop the device immediately.
- 2. When not using the massager, please turn it off and let it stop working. In case of failure, turn off the power to avoid accidents.

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