Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

©2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

PHILIPS

Back and
Waist Massager

Massage Back Brace

PPM4361

User Manual

Contents

1 Readme

Symbol Description

People Suitable to Use

Precautions for Use

2 Product Features

3 Components

4 How to Use

Button Description

Operation

Charging

5 Maintenance

Cleaning

Storage

Disposal

6 Troubleshooting

7 Technical Parameters

8 Service Guide for Product Warranty

Warranty and Service

Warranty Card

9 Packing List

1 Readme

Please read this manual carefully before using the product and keep it properly. The manual is available at www.philips.com

Symbol Description

To avoid personal injury or property damage, be sure to observe the contents below.

- The following symbols are used to illustrate what you should observe.

 Indicates prohibition

 Indicates the actions that need to be subject to instructions

People Suitable to Use

 The product should be used with caution by the following groups/ under the following circumstances

- Do not use the product and other medical electronic instruments at the same time, such as cardiac pacemaker, artificial heart and lung and other life-sustaining medical electronic instruments, as well as electrocardiograph and other medical electronic instruments, otherwise it is likely to cause danger;
- If the high-frequency surgical equipment and this product are used on the user at the same time, burns may be caused at the electrode position of the massager, or the massager is likely to be damaged;

• Pregnant women and women during menstruation, patients having sensitive skin and suffering from heart disease, abnormal blood pressure, malignant tumors, cerebrovascular disease, and acute diseases, or other people under medical treatment should consult a doctor before using this product.



Precautions for Use

-  The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
-  This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
-  Children should be supervised to ensure that they do not play with the appliance.
-  If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.
-  This appliance contains batteries that are only replaceable by skilled persons.
-  Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.
-  Do not use the product in places with high temperature, flammables, electromagnetic radiation and humidity;
-  Stop using the device if you feel uncomfortable due to the use of the product;
-  Do not disassemble, repair or modify this massager without authorization, otherwise it is likely to result in failure

2 Product Features

This product integrates vibration massage, constant temperature warm compress, and 3D fitness designed back brace with elastic material. The vertical burden of the shoulder and neck muscle can be eased through its 3D tailored shoulder-and-neck assistance structure. Its vibration massage and warm compress function can effectively relax muscles and relieve the pressure on shoulders and neck. In the daily shoulder and neck care mode, the built-in sensor will vibrate to remind the user before the user's arched back causes shoulder and neck muscle fatigue, so as to avoid injury due to fatigue accumulation. Its physical assistance combined with the intelligent reminder and the effective relaxation massage can help the user to lessen their shoulder and neck fatigue and the resulting bad posture.

3 Components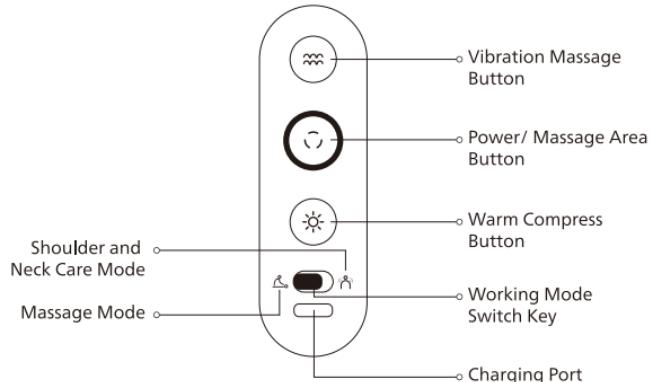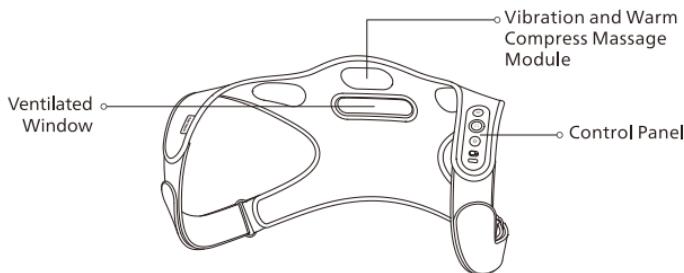

4 How to Use

I. Button Description

Preparations Before Use

1. Check whether the device and accessories are complete.
2. Wear the massager on the back and adjust it to a comfortable state.

II. Operation

Select the Working Mode

Slide the working mode selection switch “

In the massage mode “

In the shoulder and neck care mode “

Power On

Press and hold the “

In the shoulder and neck care mode, when the massager is powered on, only the arched back reminder is enabled by default. The user can manually enable the warm compress function and adjust the warm compress position and level.

Select the Massage Area

After power-on, the default massage and warm compress area is the shoulders and the cervical vertebrae. “

Adjust the Vibration Massage Level

In the massage mode, the default vibration massage intensity is in level 1 after power-on, and the “

In the shoulder and neck care mode, the vibration massage cannot be enabled, and the button lamp remains off.

Adjust the Warm Compress Level

In the massage mode, the default warm compress is in level 1 after power-on, and “

In the shoulder and neck care mode, the warm compress is turned off by default after power-on. Press the button as per the above steps to adjust the levels of warm compress in turn.

Low Battery Reminder and Charge

When the battery remaining capacity is less than 5% under the working state, the “” button working indicator will flash red to remind the user. Please connect the charging cable to charge the device. To avoid battery damage, do not start and use the device during charging. After the device is fully charged, the lamp ring will turn green and remain on.

III. Charging

1. During the use of the massager, when the working indicator lamp of the “” button flashes red, it indicates that the built-in lithium battery is low, and you need to charge it with the USB power cord.
2. Plug the output end of Type-C USB cable into the USB interface of the device, and then plug the input end into the mobile phone charger or mobile power supply. The maximum charging power supported by the device is 5V= 2A.

3. When charging, the status indicator lamp ring will display the current battery remaining capacity in different colors:

Flash in red: <5%,


Flash in yellow: 6%-50%,

Flash in green: 51-99%.

When the battery is fully charged, the charging indicator turns green and remains on. It takes about 4 hours to fully charge the device.

Warm Tips:

- 1.The product can be used for about 4 days (twice a day) after being fully charged.
- 2.Under normal use, the service life of the rechargeable battery is ≥ 300 times of charging and discharging, therefore, it can be used for about 2-3 years (30 minutes per day).
- 3.If the rechargeable battery is left unused for a long time, the service life of the rechargeable battery will be shortened. It is recommended to fully charge the battery at least once a month.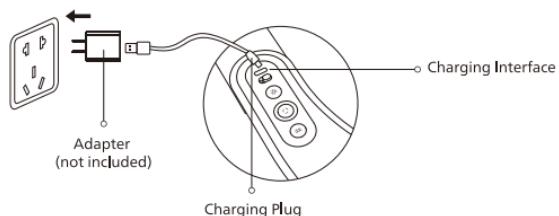

5 Maintenance

Cleaning

Wipe the device with a dry soft cloth or a dry towel when it is dirty.

Storage

- Keep out of reach of children.
 - Do not put it in the places with direct sunlight, high temperature and humidity.
 - Keep it in a dry and ventilated place.
- !** During maintenance, be sure to unplug the power cord, but do not unplug the cord with wet hands.


Disposal

The lithium-ion battery used in this product is a valuable resource that can be recycled. Take out the lithium-ion battery when it is used up and discarded, and then hand it over to a professional institution that can recycle the rechargeable batteries for disposal.


Precautions for battery:

1. Do not replace the battery by yourself during normal use.
2. Waste battery pack should be safely recycled and should not be discarded at will;
 - Be sure to remove the battery from the device before disposal;
 - The product must be powered off before the battery is removed;
 - The battery must be disposed of in a safe manner;


Disposal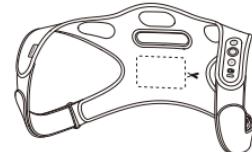

1. Cut the fabric at the dotted line with scissors.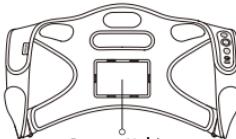

2. Cut the fabric, and you can find a battery holder.

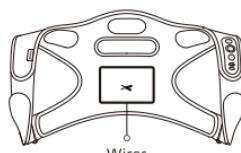3. Shear off all the external wires of the battery holder with scissors.

4. Unscrew the two screws on the battery holder with a dedicated screwdriver, loosen the four buckles on the side, then you can find the battery. Shear off the red and black electric wires connecting the battery. In this way, the battery can be removed for disposal.

6 Troubleshooting

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center in your region.

| Fault Description | Possible Cause | Troubleshooting |
|---|---|---|
| Startup Failure | Low battery of the rechargeable battery inside the main unit? | In case of low battery, please charge it in time. |
| | Is the charging normal? | Check whether the USB cable is plugged in place. After it is connected correctly, the massager's charging indicator will  flash. |
| Normal display, but the user feels nothing at the time of massage | Is the massage intensity adjusted? | The default state after power-on is intensity level 1. If you need to adjust the intensity, please turn it up appropriately. |
| Normal display, but the user feels nothing at the time of warm compress | Is the warm compress area of the massager tight to the back? | Adjust the length of the elastic belt and back strap to adjust the wearing position, so that the massage module is tight to the shoulders and neck. |

| Fault Description | Possible Cause | Troubleshooting |
|--|---|---|
| Normal display, but the user feels nothing at the time of warm compress | Is the warm compress area heating? | After power-on, wait half a minute, then touch the silicone sheet with your hand. If you feel the heat, it is normal; if not, please contact the after-sales service. |
| | Is the warm compress temperature too low? | The default warm compress is in level 1 after power-on, please try to turn up the warm compress temperature moderately. |
| Warm compress button lamp is on, but the user nothing at the time of massage and warm compress | Is it in the massage interval of daily shoulder and neck care mode? | Slide the "■" working mode selection switch to the left (massage mode), or wait half an hour for the enabling of the periodic massage. |
| | Lower battery of the rechargeable battery? | In case of low battery, the status lamp ring turns red. Please charge the product in time. |

Note:

If the fault cannot be eliminated or the product still cannot be operated satisfactorily, contact the after-sales service personnel, and do not disassemble or attempt to repair it by yourself.

7 Technical Parameters

| | |
|-------------------|----------------------------|
| Product Name | Philips Massage Back Brace |
| Product Model | PPM4361 |
| Operating Voltage | 3.7V== |
| Operating Power | 12W |
| Charging Voltage | 5V== |
| Charging Current | 2A |
| Product Size | 454x367x20mm |
| Net Weight | 330g |
| Battery Capacity | 5000mAh |
| Charging Time | <4 hours |

8 Service Guide for Product Warranty

Warranty and Service

If you need information or have any questions, please visit www.philips.com/-support or contact consumer service center in your country/region. If there is no consumer service center in your country/region, please contact your local distributor.

Within two years from the date of purchase, you will receive free warranty service for any damage caused by manufacturing process problems, or any damage to components under normal use conditions confirmed by our maintenance service personnel.

Warranty service excludes frequently replaced consumables, auxiliary equipment, shipping cost and door-to-door service. Please show proof of purchase to service personnel when servicing.

Malaysia

Consumer Care: +60-128868274
Email: philips.massage@real✉tmy.net

Singapore

Consumer Care: +65-89420233
Email: philips.massage@real✉tsg.net

Philippines

Consumer Care: +63-9604111314
Email: Philips.massager.ph@orrohome.com

Thailand

Consumer Care: +66-20051888
Email: Philips_massager@orrohome.co.th

Indonesia

Consumer Care: +021-39715012 / +021-39715013
Email: Philips_Massager.ID@orrohomeindonesia.com

Vietnam

Consumer Care: +84-909800829
Email: Philips.massager.VN@orrohome.vn

Warranty Card

| Product Model | | User Name | |
|----------------------------------|------|-------------------|--|
| Sales Agency | | Phone Number | |
| Date of Purchase | | Serial Number | |
| Special Technical Service Center | | User Address | |
| Items of Maintenance | Date | Service Personnel | |
| | | | |
| | | | |
| | | | |
| | | | |

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