Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

PHILIPS

Head Massager

Massage Headband


PPM1321


Quick Start Guide

1. Remove the pendants and accessories on your head during use so as to avoid affecting the massage experience.


2. Press and hold the power function button to start the device, the default option is relaxing mode and hot compress on with level 1.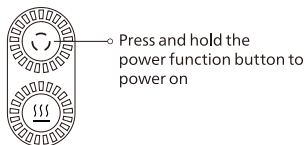


3. Under the power-on state, press and hold the power function button to shut down the device.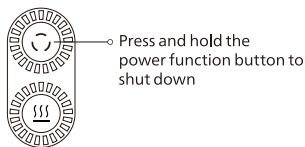

4. Under the power-on state, press the power function button to switch the massage modes circularly, namely, Relaxing mode: one third of the lamp ring will light on; Sleep-aid mode: two thirds of the lamp ring will light on; Energizing mode: the whole lamp ring will light on. When the air pressure massage stops: the lamp ring will be off.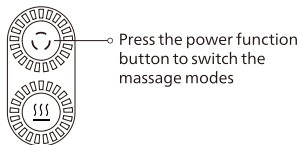

5. Under the power-on state, press the hot compress button to adjust the levels for hot compress; Hot compress in level 1: one third of the lamp ring will light on; Hot compress in level 2: two thirds of the lamp ring will light on; Hot compress in level 3: the whole lamp ring will light on.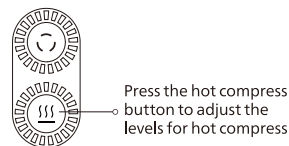


6. When the product is powered on or charged, the power icon will light on, and the lamp color will show the current battery remaining capacity:

Red: <10%

Orange: 11-50%


Green: 51-100%

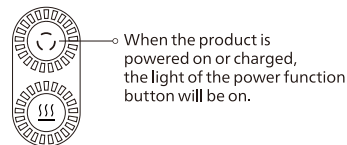7. The attached gel bag is available for cold and warm compress. Before the warm compress, cover the gel bag with a waterproof plastic bag, and then put it into hot water for heating. Before the cold compress, place the gel bag in the refrigerator for more than 30 minutes.

8. After heating or cooling, fix the gel bag on the headband, adjust the position of cold and warm compress, and then buckle up.

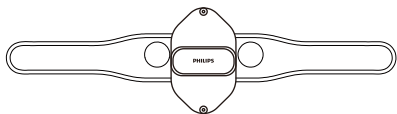- ❗ 1. Please remove the accessories on your head before massage.
- 2. It is recommended to start at low level for the first time, so as to avoid any discomfort.
- 3. Pay attention to controlling the temperature of cold or warm compress, so as to ensure the comfort. When you feel the temperature is too high or too low, remove the headband immediately and stop the cold or warm compress.

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