Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Made in China

Rev A AUG 23

PHILIPS

Foot Massager

Foot Massager

PPM6331

User Manual

User Manual

1 Readme

Symbol Description

People Restricted or Prohibited From Use

Precautions for Use

Precautions for Maintenance

2 Product Features

3 Components

4 How to Use

Preparations Before Use

Operation

5 Maintenance

Cleaning

6 Troubleshooting

7 Technical Parameters

8 Service Guide for Product Warranty

Warranty and Service

Warranty Card

9 Packing List

1 Readme

Please carefully read the operation manual before use and keep it properly.
The manual is available at www.philips.com

Symbol Description

To avoid personal injury or property damage, be sure to observe the contents below.

- The following symbols are used to illustrate what you should observe.

 Indicates prohibition

 Indicates the actions that need to be subject to instructions

People Restricted or Prohibited From Use

 The product should be used with caution by the following groups/ under the following circumstances

- People who have a pacemaker or other related medical electrical products implanted in their bodies
- People who have metal, silicone or plastic materials injected into their bodies as a result of bone fractures or plastic surgery
- People who have ever had skin trauma
- People whose body temperature is above 38 degrees Celsius

- People who are easy to suffer from numbness of body
- People who have difficulty with respirators
- People suffering from hypertension
- People having the acute symptoms such as visceral diseases (gastritis, hepatitis, gastroenteritis)
- People who have skin sensation disorders
- People suffering from body paresthesia
- People suffering from deformed arthritis, arthritis, and gout

 The product is not allowed to be used by the following groups/ under the following circumstances

- People who are unable to express their will or who cannot operate the machine themselves to use the product.
- This product is not suitable for the people (including children) with reduced physical, sensory or mental abilities and lack of experience and knowledge.
- Stop using it immediately when you feel lower back pain, dizziness, vomiting, palpitation and other abnormal body reactions during use.
- This product is set for 15-minute timing at a time. Do not use it for more than 30 minutes each time. After each use, stop the device for 30 minutes to ensure its service life.
- Do not use the product under the following circumstances:
 - When you are severely tired
 - Within one hour before and after meals
 - When you are drunk

Precautions for Use

- ❗ The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
- ❗ This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- ❗ Children should be supervised to ensure that they do not play with the appliance.
- ❗ If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.
- ❗ Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.
- ❗ Check whether the product is intact before use. If any damage is found, regardless of its extent, stop using the product immediately.
- ❗ Remove the dust on the adapter on a regular basis. The accumulated dust on the plug is likely to cause poor insulation due to humidity, and eventually lead to fire.
- ❗ If you cannot use it or feel abnormal, stop using it immediately.
- ❗ Unplug the adapter in case of power cut.
- ❗ When the product is not in use, the adapter must be unplugged.
- ⊘ Do not disassemble, modify or repair the product by yourself, nor unzip the product by yourself.
- ⊘ Do not use the device when your body is wet, otherwise it is easy to get an electric shock.
- ⊘ Do not pull the product with force, nor stab it with a knife or a sharp object.


Precautions for Maintenance

- ❗ Be sure to unplug the charging cable during maintenance.
- ❗ Please properly dispose of the packaging bag of the product and keep it away from the infants.
- ⊘ The product has built-in lithium batteries. Do not disassemble or modify the product by yourself, immerse the product (controller) in water or toss it into the fire, otherwise the product is likely to be damaged.
- ⊘ Do not damage, modify, bend or pull the power cord with force. Do not place heavy objects on the power cord or squeeze the power cord.
- ⊘ Do not use volatile oil, thinner, alcohol, strong alkaline detergent, bleach or other cleaning agents for maintenance.
- ⊘ Keep the product out of reach of children and infants. Do not let them touch or operate the product, otherwise it is likely to lead to burns, electric shocks or personal injuries.
- ❗ Make the device cool down before storing.

2 Product Features

With kneading relaxation and comfortable hot compress, this product can help relieve the discomfort in the legs and feet. Its 2 massage modes paired with 2 massage intensities can meet different massage needs, providing a rich variety of massage enjoyment.


3 Components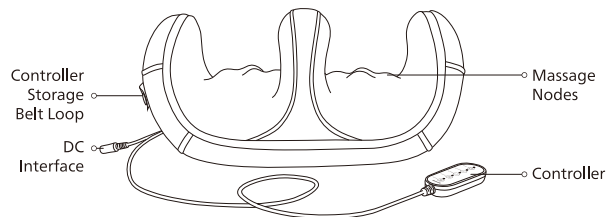

4 How to Use

Preparations Before Use

1. Check if the device and its accessories are complete.
2. Remove the pendants and decorations, so as to avoid affecting the massage area.
3. Fix the massager on the ground or on the sofa for better massage experience.
4. Place the massager comfortably in the area that needs to be massaged (legs or soles of feet).

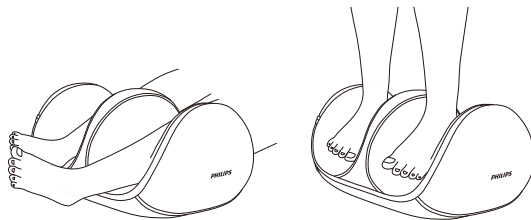Note: It is not allowed to stand on the product to avoid the product from being crushed and causing possible danger.

Operation

Power Button

1. Press the power button to turn on the massager, and the massager enters the foot massage mode and turns on the hot compress function. The corresponding indicator light remains on.
2. Automatic shutdown time of the massager is set to be 15 minutes. If you need to stop halfway during the massage, press the power button to turn it off.

Foot Mode Button

Press this button in the power-on state to enter the foot massage mode, and the corresponding indicator light remains on.

Leg Mode Button

Press this button in the power-on state to enter the leg massage mode, and the corresponding indicator light remains on.

Intensity Button

The intensity is in low speed level by default when the massager is powered on, and the indicator light  flashes slowly. Press this button to switch to the high speed level and the indicator light remains on.

Heating Button

1. The hot compress function turns on by default when the massager is powered on, and the corresponding indicator light remains on.
2. Press this button to turn off the hot compress function and the corresponding indicator light goes out.

5 Maintenance

Cleaning

1. Remove the dust from the cloth cover of the product with a vacuum cleaner with clean cloths gently.
2. Wipe off the dirt on the surface with clean cloths.
3. For the deep stains, wipe off with a cloth dampened with neutral detergent gently.
4. Prevent the water from entering the product.
5. Place the product and dry it naturally.

- ❗ The product cover cannot be removed for cleaning.
- ❗ Do not place the product in direct sunlight or places with high humidity after use.
- ❗ During maintenance, be sure to unplug the adapter, but do not remove the adapter with wet hands.
- ❗ When the product is not in use for a long time, turn off the power and unplug the adapter, and then store the massager in its original package with a dust-proof cover.
- ⊘ Do not use alcohol, paint thinner, gasoline or nail polish remover, which may damage the product or cause parts to crack or discolor.

- ❗ This product can only be used with the included power supply unit.

6 Maintenance

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center in your region.

Fault Description	Possible Cause	Solution
Startup Failure	The power button is not turned on.	Press the power button to turn it on.
	This product has just completed a 15-minute massage and automatically switches to the shutdown state.	If you want another massage, please turn it on again.
	Excessive load are placed on the product (e.g. someone standing on the product or heavy objects are placed on the top of the product).	Remove the excess weight.
It suddenly stops during use.	The product is not plugged in or powered on.	Plug in the power adapter properly.
There is a sound during operation.	This is the internal mechanical sound.	This is the internal mechanical sound.
The product is smoking.	The internal wires are short circuited.	Stop using it immediately and contact the service center.

If the product has been left unused for a long time, be sure to confirm that its functions are normal before using it again		
Regular check	Check for the following faults	In the event of the above faults, stop using the device immediately and cut off the power supply to prevent accidents. Please send it to our maintenance center.
	<ul style="list-style-type: none"> Whether the power cord or the plug is hot Whether the power cord is deeply scratched or deformed, or has a burnt smell Whether there is abnormal sound or vibration during work Whether the product can work properly after switch on Whether there are other abnormalities or faults 	

Note:

If the fault cannot be eliminated or the product still cannot operate satisfactorily, contact the after-sales service personnel, and do not disassemble or attempt to repair it by yourself.

7 Technical Parameters

Product Name	Philips Foot Massager
Product Model	PPM6331
Rated Voltage	12V ---
Rated Power	24W
Product Dimension	395×255×196mm
Product Net Weight	About 1.84kg

8 Service Guide for Product Warranty

Warranty and Service

If you need information or have any questions, please visit www.philips.com/-support or contact consumer service center in your country/region. If there is no consumer service center in your country/region, please contact your local distributor.

Within two years from the date of purchase, you will receive free warranty service for any damage caused by manufacturing process problems, or any damage to components under normal use conditions confirmed by our maintenance service personnel.

Warranty service excludes frequently replaced consumables, auxiliary equipment, shipping cost and door-to-door service. Please show proof of purchase to service personnel when servicing.

Malaysia

Consumer Care: +60-128868274

Email: philips.massager@realx.tmy.net

Singapore

Consumer Care: +65-89420233

Email: philips.massager@realx.tsg.net

Philippines

Consumer Care: +63-9604111314

Email: Philips.massager.ph@orohome.com

Thailand

Consumer Care: +66-20051888

Email: Philips_massager@orohome.co.th

Indonesia

Consumer Care: +021-39715012 / +021-39715013

Email: Philips_Massager.ID@orohomeindonesia.com

Vietnam

Consumer Care: +84-909800829

Email: Philips.massager.VN@orohome.vn

Warranty Card

Product Model		User Name	
Sales Agency		Phone Number	
Date of Purchase		Serial Number	
Special Technical Service Center		User Address	
Items of Maintenance	Date	Service Personnel	

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