


# PHILIPS

Magnetic Resonance


## Caring for your coils, caring for your patients.

### Safe handling

Always lift the coil from the bottom.

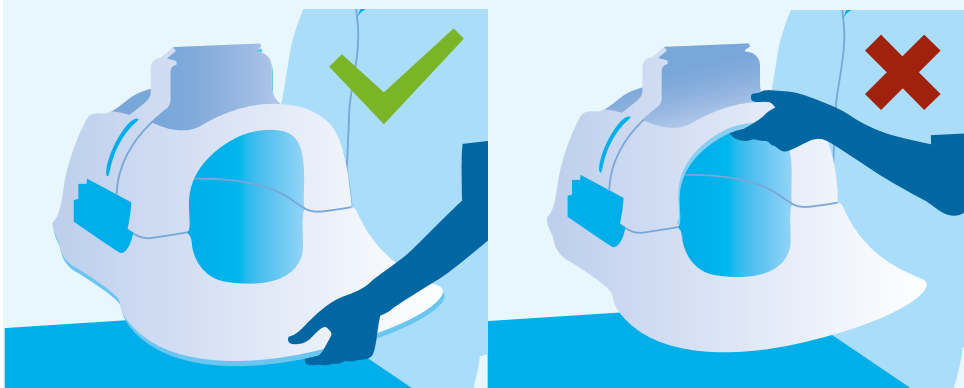


Prevent sharp bends in the coil cables.

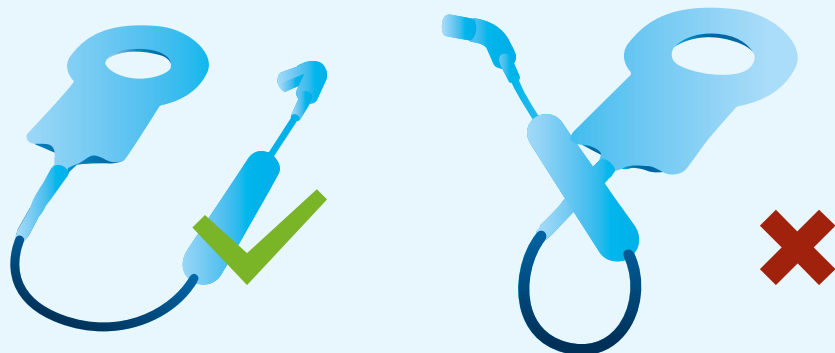

### Safe and trouble-free connections


Insert connector at a 90° angle to prevent damage.

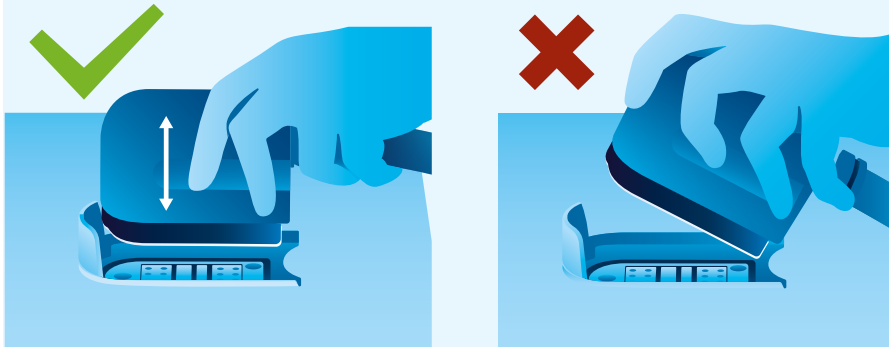

Use connector covers.




Never use coils with damaged cables or cable connectors.

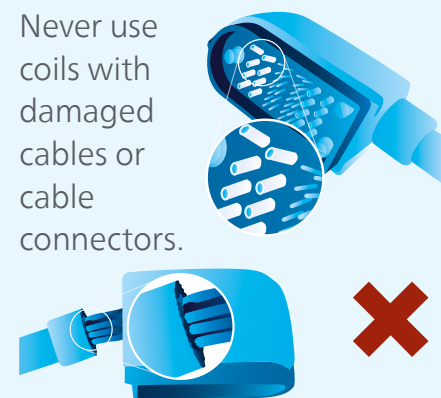

### Safe storage

Practice good coil and cable storage.

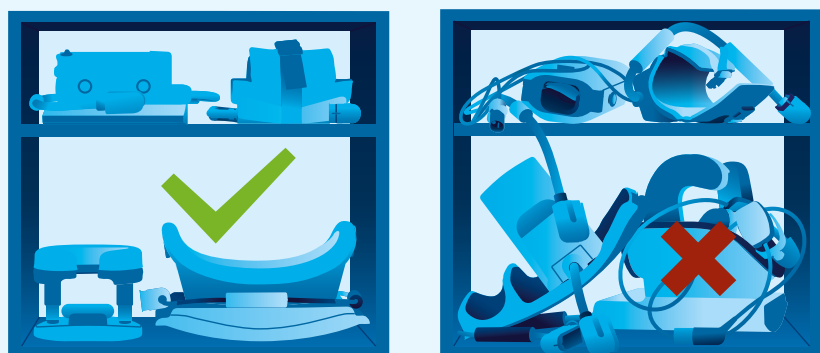

Unplug coil cables when system is not in use.



Store coil cables correctly after unplugging to avoid coil cables from hanging or being stretched.

