Visit us
www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2024 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Made in China

Rev A JAN 24

PHILIPS

Leg Massager

Knee Massager

PPM5301

User Manual

Catalogue

1 Readme

Symbol Description

People Restricted or Prohibited from Use

Precautions for Use

Precautions for Maintenance

2 Product Features

3 Components

4 How to Use

Preparations Before Use

Operation

Charging Method

Description of Indicator Light

5 Maintenance

Cleaning

Disposal

6 Troubleshooting

7 Technical Parameters

8 Service Guideline for Warranty Warranty and Service Warranty Card

9 Packing List

1 Readme

Please read this manual carefully before using the product and keep it properly.

The manual is available at www.philips.com

Symbol description

To avoid personal injury or property damage, be sure to observe the contents below.

- The following symbols are used to illustrate what should be observed.
-

 Indicates prohibition

 Indicates the actions that need to be subject to instructions

People Restricted or Prohibited From Use

 The following populations should be used with caution

- People with leg nerve impairment or skin sensory disorders.
- People who have had metal, silicon or plastic injected into their feet/hands due to fractures or plastic surgery.
- People with leg or knee sequelae due to traffic accidents.
- Patients with or have had leg diseases (tendon injury, arthritis deformity, fractures, varicose veins, etc.).
- People with implanted medical electronic devices who are susceptible to electromagnetic failures such as pacemakers.

- People deemed to have lost a temperature sensation.
 - People with a malignancy.
 - People with a heart disease.
 - Pregnant women or women who have just given birth.
 - People with sensory impairment caused by diabetes.
 - People with scars on their skin.
 - People need quiet and people in poor health.
 - People with osteoporosis, severe fractures, sprains, muscle pain, and acute (head) pain.
 - People with a body temperature of 38°C or above (fever).
 - People with symptoms of acute inflammation, people with strong symptoms such as fatigue, aversion to cold, unstable blood pressure, or people who feel weak.
 - Others who are treated at a medical facility.
-

Precautions for Use

-  The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
-  This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
-  Children should be supervised to ensure that they do not play with the appliance.
-  If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.
-  This appliance contains batteries that are only replaceable by skilled persons.

- ❗ Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.
- ❗ Each use time is 30 minutes. Please check the condition of your knees , and then adjust the number of uses appropriately.
- ❗ It is recommended to use the level 1 intensity when you first start using this product.
- ❗ When using this product, your knees may turn red in a short time. This does not mean that there is a problem. If this phenomenon persists and does not improve, please consult your surgeon.
- ❗ If you use it while wearing stockings, the straps on the massager may scratch the stockings. Please avoid using the massager while wearing stockings.
- ❗ If you feel pain around your legs or knees, please stop using the product immediately.
- ❗ If you have any problems with your skin or legs during use, please stop using this product immediately and consult with your physician.
- ❗ It is recommended not to overuse the massage function.
- 🚫 Do not use it while sleeping.
- 🚫 Do not cover it with a quilt or blanket.
- 🚫 Do not lie or sit on the product.
- 🚫 Do not impose huge impact or load on the product.
- 🚫 Do not use the products in the bathroom or in the shower room.
- 🚫 Do not wear the product to any other activities.

Maintenance Precautions




- ❗ When cleaning, be sure to unplug the adapter from the household electrical outlet..

- ❗ Please dispose of the packaging bag of this product properly and keep it away from infants and young children.
- 🚫 This product contains a lithium battery. Please do not disassemble, modify, immerse the product (controller) in water, or throw it into a fire. Otherwise, the product may be damaged.
- 🚫 Do not store the product within the reach of children and infants. Also, do not allow them to touch or operate this product. Otherwise, burns, electric shock, or personal injury may occur.
- 🚫 Do not damage or modify, or vigorously bend or pull the charging cable out. Also, do not place heavy objects on the charging cable or squeeze the charging cable.

2 Product Features

This product through the imitation of hand tapping with hot compress function, can promote blood circulation in the knee joint and relieve muscle and joint pain. Multiple levels of temperature and massage intensity are available, and it is suitable for different people for knee massage care. With the extension belt, it can also be used to massage other parts of the body.


3 Components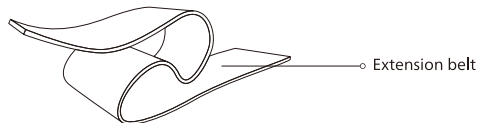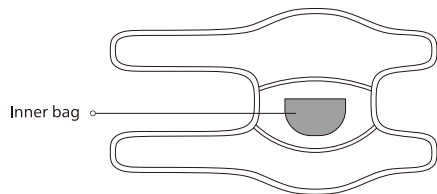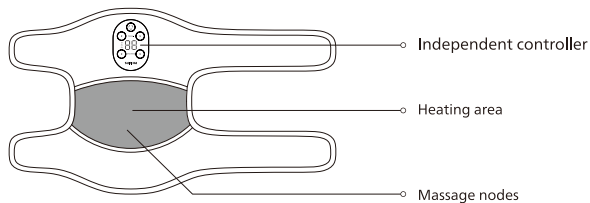


4 How to Use

Preparations before Use

1. Connect the controller and the main device through the magnetic buckles to ensure that the controller is firmly installed.

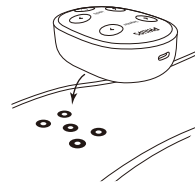
2. Put the massager on the knee and adjust the tightness through the Velcro on the strap.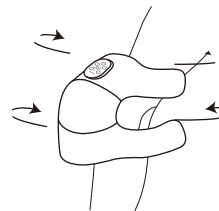

! When using a massager while wearing stockings, the straps on the massager may scratch the stockings. Please avoid using the massager while wearing stockings.

Operation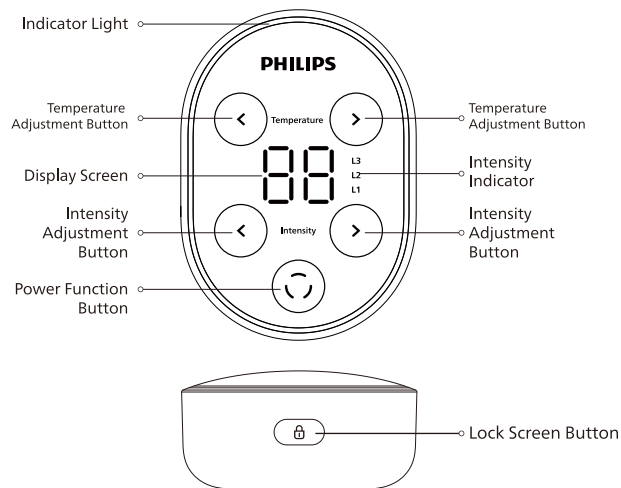

Power Function Button

1. In standby state, press and hold the power function button for 2 seconds to turn on / off. The device starts heating at 50°C and vibration intensity level 1 by default.
2. Short press the power function button to turn on / off the heating function.
3. The device is preset to shut down automatically after 30 minutes of operation, accompanied by a beep sound.

Intensity Adjustment Button

1. Short press the intensity "<" and ">" buttons to adjust the vibration intensities (level 1, level 2, level 3 and turn off vibration).
2. When adjusting, the corresponding intensity indicator light is on. The display lights up L1, L2 or L3 simultaneously, and automatically switch to the temperature display after 2 seconds.

Temperature Adjustment Button


Short press the temperature "<" and ">" buttons to adjust the temperature (40°C, 45°C, 50°C, 50°C, 55°C, 60°C). The center display shows the current heating temperature.

Lock Screen Button

When the machine is working normally, press and hold the lock screen button for 2 seconds to lock / unlock the screen. Each operation is accompanied by a beep sound.

Charging Method

1. Insert the connector of the charging cable into the charging port on the side of the controller.
2. Connect the USB port of the charging cable to a 5V \equiv 1A adapter, and then plug the adapter into a 100-240V~, 50/60Hz power socket.
3. The indicator light ashes blue slowly when charging, and stays on blue when fully charged.
4. It takes about 4.5 hours to fully charge the device.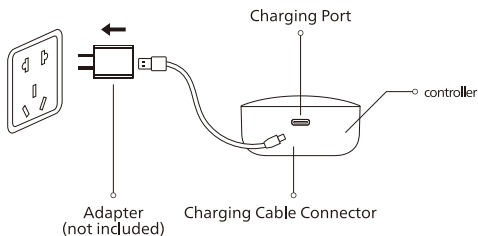

- ❗ When the remaining power is less than 10%, the indicator light will flash blue for 5 seconds as a reminder, and then the device will shut down.
- ❗ Please turn off the device before charging. It is prohibited to use the device while charging.


Description of Indicator Light

Indicator Light of the Controller	State	Description
Orange	Stay on	Hot compress function on
Blue	Stay on	Hot compress function off
Orange	Flash slowly	Hot compress function on, and the screen operation locked
Blue	Flash slowly	Hot compress function off, and the screen operation unlocked
Blue	Flash for 5 seconds	Power <10%
Blue (in power-off state)	Flash slowly	In charging
Blue (in power-off state)	Stay on	Fully charged

5 Maintenance

Cleaning

1. Controller care: controller displays with touch controls should be properly protected. Please use a soft cloth when cleaning.
2. Cloth area care: The cloth area cannot be washed. Please use alcohol to sterilize, wipe away sweat stains or wipe with a clean wet towel.

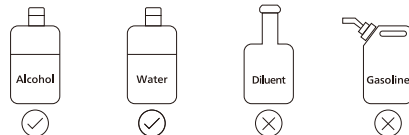- ❗ When maintenance, be sure to pull out the charging cable.
- ❗ If the product has not been used for a long time, it is recommended to charge it once a month.

Disposal

The lithium-ion battery used in this product is a valuable resource that can be recycled. Take out the lithium-ion battery when it is used up and discarded, and then hand it over to a professional institution that can recycle the rechargeable batteries for disposal.

- Be sure to remove the battery from the device before disposal.
- The device must be powered off before the battery is removed.
- The battery must be disposed of in a safe manner.

6 Troubleshooting

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center in your region.

Charge the device	Possible Cause	Solution
Startup failure	The product is still connected to the power outlet.	Remove the product from the socket and hold the power function button to start.
	Battery exhausted	Charge the device.
	The power function button is not pressed	Press the power function button for 2 seconds to turn the device on.
The charging indicator light is not on.	The USB charging cable is not properly plugged into the power socket.	Insert the USB charging cable into the power socket correctly.
Even if it has just been charged, it can only be used for a few minutes.	Battery life is exhausted (about 5 years).	Please consult the dealer (Do not replace the battery by yourself).
The device is not heating.	The device may not function properly when used outside the recommended ambient temperature range.	Use it within the recommended temperature range of 0-40 °C.
The product stops working during operation.	Battery exhausted	Low battery, please charge it.
	The product automatically shuts down when the 30-minute timing is over.	Restart the device if you need to use it again.

If the product has been left unused for a long time, be sure to confirm that its functions are normal before using it again		
Regular Check	Are there any following faults	In the event of the above faults, stop using the device immediately and cut off the power supply to prevent accidents. Please send it to our maintenance center.
	<ul style="list-style-type: none"> · Whether the charging cable or the connector is hot · Whether the charging cable is deeply scratched or deformed, or has a burnt smell · Whether there is abnormal sound or vibration during work · Whether the product can work properly after power on · Whether there are other abnormalities or faults 	

Note:

If the fault cannot be eliminated or the product still cannot be operated ideally, contact the after-sales service personnel, and do not disassemble or attempt to repair it by yourself.

7 Main Specification

Product name	Philips Knee Massager
Product Model	PPM5301
Rated Voltage	5V ---
Power Rating	7W
Charging Voltage	5V ---
Charging Current	1A
Product Size	500×290×30mm
Net Weight of Product	310g
Battery Capacity	2500mAh

8 Service Guideline for Warranty

Warranty and Service

If you need information or have any questions, please visit www.philips.com/-support or contact consumer service center in your country/region. If there is no consumer service center in your country/region, please contact your local distributor.

Within two years from the date of purchase, you will receive free warranty service for any damage caused by manufacturing process problems, or any damage to components under normal use conditions confirmed by our maintenance service personnel.

Warranty service excludes frequently replaced consumables, auxiliary equipment, shipping cost and door-to-door service. Please show proof of purchase to service personnel when servicing.

Malaysia
Consumer Care: +60-128868274
Email: philips.message@realxmy.net

Singapore
Consumer Care: +65-89420233
Email: philips.message@realxsg.net

Philippines
Consumer Care: +63-960411314
Email: Philips.massager.ph@orrohome.com

Thailand
Consumer Care: +66-20051888
Email: Philips_massager@orrohome.co.th

Indonesia
Consumer Care: +021-39715012 / +021-39715013
Email: Philips_Massager.ID@orrohomeindonesia.com

Vietnam
Consumer Care: +84-909800829
Email: Philips.massager.VN@orrohome.vn

Warranty Card

Product Model		User Name	
Sales Agency		Phone Number	
Date of Purchase		Serial Number	
Special Technical Service Center		User Address	
Items of Maintenance	Date	Service Personnel	

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