Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

PHILIPS

Massage Gun


Sports Massage Gun

PPM7323


Quick Start Guide


1. It is recommended to charge the device for 4 hours before \square rst use. Do not turn it on when charging.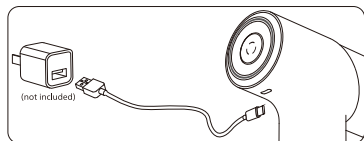


2. When the battery indicator becomes green and remains on, it indicates that the battery is fully charged. When the battery indicator \square ashes red (the battery remaining capacity is lower than 10%), please charge it in time.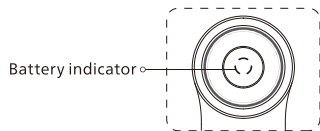


3. The device will power off automatically after continuous use for 10 minutes. Press and hold the operation button to restart the device for use again.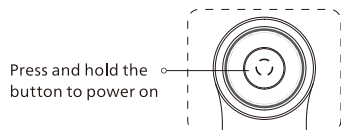


4. During the installation of massage head, insert the appropriate massage head into the front round hole of main unit and press it \square rmly to the end.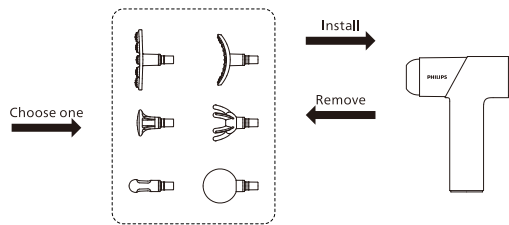

5. Press and hold the operation button to turn on the device. The device will enter Level 1 by default. Press the button to change the levels at a range of Level 1 / Level 2 / Level 3 / Level 4 / Level 5 / low-speed waveform level / high-speed waveform level.

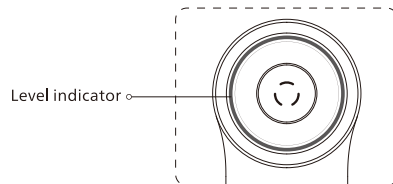6. The massage head is suspended after being blocked. This is to protect the device against abnormal operation and make it switch from operation mode to shutdown mode. Press and hold the button to restart the device.

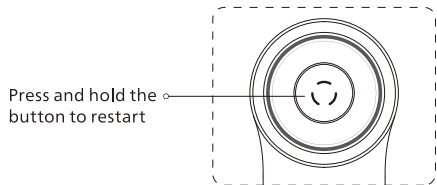! Please choose the appropriate massage strength based on your own situation. Do not massage a single part for more than 10 minutes during each use.

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