Visit us

[www.philips.com](http://www.philips.com)

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

**PHILIPS**

Neck and Shoulder  
Massager

Neck Massager

PPM3306

**Quick Start Guide**

1. Wrap the massager around your neck (or other parts that need to be massaged), and use the pull strap to adjust the tightness.

2. Press and hold the power function button for 2 seconds to start the device (the buzzer rings once). The default option is the Vibration Mode 1 (white light). In this mode, the vibration intensity is low (green light), and the warm compress is working in low gear (green light).

Press and hold the button for 2 seconds to turn on

3. Press the power function button to switch the vibration modes in cycle. Mode 1 (the indicator light turns white and the buzzer rings once); Mode 2 (the indicator light turns ice blue and the buzzer rings twice); Mode 3 (the indicator light turns dark blue and the buzzer rings three times)

Press the button to switch the modes

4. Press the warm compress button to switch the levels for warm compress in cycle. Warm compress in low level (the indicator light turns green and the buzzer rings once); Warm compress in high level (the indicator light turns orange and the buzzer rings twice); Turn off the warm compress (the indicator light is off and the buzzer keeps ringing)

Press the button to switch the levels for warm compress

5. Press the vibration intensity button to switch the vibration intensities in cycle. Low vibration intensity (the indicator light turns green and the buzzer rings once); High vibration intensity (the indicator light turns orange and the buzzer rings twice); Turn off the vibration (the indicator light is off and the buzzer rings for 2 seconds)

Press the button to switch the levels for vibration

6. The product will be automatically shut off after working for 15 minutes. To discontinue halfway, press and hold the power function button to turn it off.

7. When the battery is low, both the vibration and warm compress functions will be turned off automatically, the blue indicator light will flash, and the buzzer will remind you to charge the product. When charging, the dark blue indicator light will be flashing slowly. When the product is fully charged, the dark blue indicator light remains on.

- 1. If you feel uncomfortable during the massage, stop using it immediately.
- 2. When the massager is not in use, please turn it off and let it stop working. In case of machine failure, turn off the power to avoid accidents.

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