Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

PHILIPS

Back and Waist Massager


Waist Massager

PPM4325

Quick Start Guide

1. Push the belt inside out through the clasp of the massager and secure it with a Velcro.

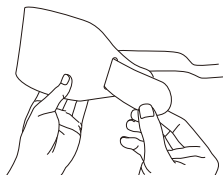
2. Mount the battery compartment (controller) into the slideway of the massager from right to left so that the battery compartment is firmly mounted on the massager.


3. Put the massager on the waist, push the belt through the clasp and secure it with a Velcro.

4. Press and hold the power button for 2 seconds to turn on/off the product, the buzzer keeps beeping for 2 seconds, and the power indicator turns orange. After power-on, the heating temperature by default is 45°C and the level 1 heating indicator is turns orange.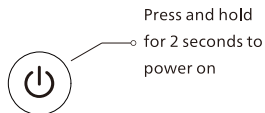

5. Press the temperature adjustment button to switch the heating levels circularly, which is in the order as follows: level 1 (45°C) – level 2 (50°C) – level 3 (55°C) – level 1 (45°C), and the corresponding level indicator turns orange, and the buzzer will give a beep for each operation.

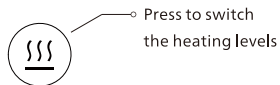6. The massager is set to shut off automatically after 30 minutes of operation, and the buzzer keeps beeping for 2 seconds and then the product enters the standby state.

- 1.The abdominal skin is sensitive, so it is recommended to use the massager with clothes on for fear of scalding if choosing 50℃ and above;
- 2.The fabric area should not be washed. Sterilize the fabric and remove sweat stains with alcohol or wipe it with a clean wet towel.
- 3.When the battery remaining capacity is less than 10%, the power indicator quickly flashes for 2 seconds and then the product will shut off.
- 4.Make sure to power off before charging. Do not use the product while charging.

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