Visit us

[www.philips.com](http://www.philips.com)

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

**PHILIPS**

Back and  
Waist Massager

Massage Back Brace


PPM4361

**Quick Start Guide**


### 1. Wear the Massage Back Strap

Straighten your back, wear the massager on your back, adjust the elastic shoulder strap to fit comfortably on your shoulder and back, and then press and secure the Velcro on the elastic belt. Adjust the tightness of the shoulder strap until you feel comfortable.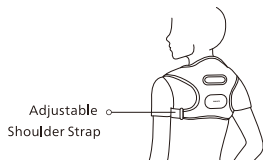


### 2. Select the Working Mode

Slide the working mode selection switch "☺" to switch to the required working mode. "☺" on the left side is the ordinary massage mode, "☺" on the right side is the shoulder and neck care mode.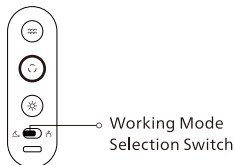


### 3. Ordinary Massage Mode

In the ordinary massage mode, the massage and warm compress level can be set after the massager is powered on. The massager will automatically shut down after operating for 10 minutes each time.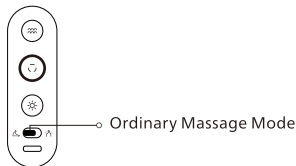


### 4. Shoulder and Neck Care Mode

In the shoulder and neck care mode, arched back reminder is enabled, and a vibration reminder will be given when the user's posture tilt is detected. At the same time, the warm compress level can be set after starting. In this mode, the vibration massage cannot be enabled, and the arched back reminder and warm compress function will continue to work until it automatically shuts down due to low battery or the user shuts it down manually.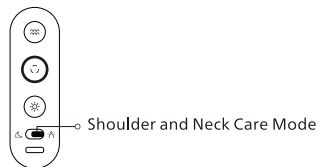


### 5. Power On

Press and hold the "☺" button for 1 second, then the massager is powered on and enter the corresponding working mode according to the state of the working mode selection switch. The button icon and the lamp ring light on. In the massage mode, the default massage area is the shoulders and the cervical vertebrae, and the massage mode is vibration massage level 1 and the warm compress level 1. In the shoulder and neck care mode, when the massager is powered on, only the arched back reminder is enabled by default. The user can manually enable the warm compress function and adjust the warm compress position and level. The vibration massage cannot be enabled.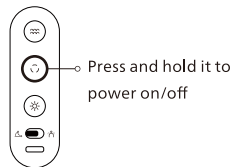

## 6. Select the Massage Area

After power-on, the shoulders and cervical vertebrae mode is enabled by default. "☺" button and the lamp ring turns cyan; press the button to switch to the shoulders mode, and the lamp ring turns blue; press the button again to switch to the cervical vertebrae mode, and the lamp ring turns green; and repeat in this way.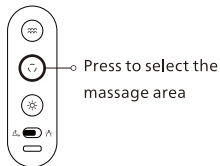

## 7. Select the Vibration Massage Level


In the massage mode, the default vibration massage intensity is in level 1 after power-on, the "☺" button turns green, press the button to switch to intensity level 2, and the button turns yellow; press the button again to switch to intensity level 3, and the button turns orange; press the button again to turn off the vibration massage, and the button lamp is off. In the shoulder and neck care mode, the vibration massage cannot be enabled, and the button lamp remains off.

## 8. Select the Warm Compress Gear


In the massage mode, the default warm compress is in level 1 after power-on, and "☺" button turns green; press the button to switch to the warm compress level 2, and the button turns yellow; press the button again to switch to the warm compress level 3, and the button turns orange; press the button again to turn off the warm compress, and the button lamp is off.

In the shoulder and neck care mode, the warm compress is turned off by default after power-on. Press the button as per the above steps to adjust the levels of warm compress in turn.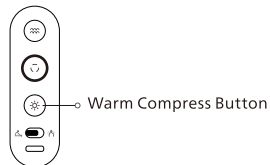

## 9. Low Battery Reminder and Charge

When the battery remaining capacity is less than 5% under the working state, the "☺" button working indicator will flash red to remind the user. Please connect the charging cable to charge the device. To avoid battery damage, do not start and use the device during charging. After the device is fully charged, the lamp ring will turn green and remain on.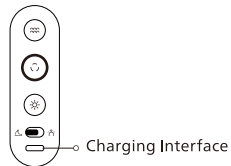

- 1. Use the original charging cable. Try not to use any charging cables of unknown origin.
- 2. The massage back strap is an electronic product, thus please avoid soak cleaning and ironing. After each use, clean the surface with a clean wet cloth and store it after airing.

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