Visit us

[www.philips.com](http://www.philips.com)

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

**PHILIPS**

Leg Massager

Knee Massager

PPM5521


**Quick Start Guide**


1. Wear the massager on the knees, push the binding strap through the fixing buckle, fold it back and pull it to a comfortable state, and then fasten it to the fabric of the massager with the Velcro on the front of the binding strap.


2. Press and hold the power function button for 2 seconds to turn on/off the massager. After startup, the massager will enter the wakeup mode by default, enabling the warm compress at 40°C with a voice prompt "Massage On".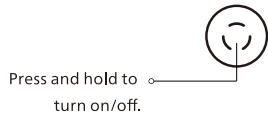

3. Press the mode button to switch the massage mode (Wakeup Mode - Relaxing Mode - Energizing Mode - Powerful Mode - Turn off air pressure) circularly.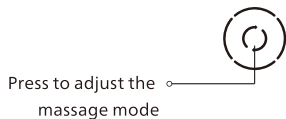

4. Press the temperature button to switch the temperature (40°C, 45°C, 50°C, heating off) circularly, and the display screen in the middle will display the current selected temperature.

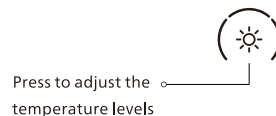5. When charging, the indicator lamp ring flashes blue slowly until the massager is fully charged, then the blue lamp remains on.

6. When the battery remaining capacity is less than 10%, the indicator lamp ring flashes orange for 5 seconds to prompt, and then the massager shuts off.

7. The default timing of the product is 15 minutes. It is set to shut off automatically after 15 minutes of operation and enter the standby state with the voice prompt: "Turn off massage".

- 1. As each person has different sensitivity to temperature, it is recommended to use the massager with clothes on for fear of scalding if choosing 45 °C and above.
- 2. For the initial, it is recommended to select a low-level temperature and then turn the temperature up gradually when the body gets used to it.
- 3. The fabric area should not be washed. Sterilize the fabric and remove sweat stains with alcohol or wipe it with a clean wet towel.

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