Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

PHILIPS

Massage Gun

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
PPM7303

Quick Start Guide


1. It is recommended to charge the device for 4 hours before first use. Do not turn it on when charging.

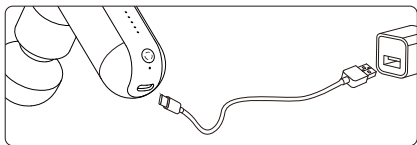
2. When the battery indicator is always green, it indicates that the battery is fully charged. When the indicator flashes red (the battery remaining capacity is lower than 5%), please charge it in time.


3. After continuous use of the device for 10 minutes, it will automatically switch off. Restart the device if you need to use it again.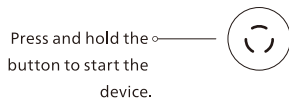


4. When installing the massage head, insert it into the front round hole of the massage gun, and then press it firmly to the end.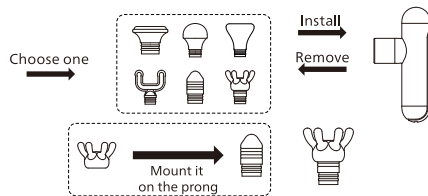

5. Press and hold the operation button to start the device, then the device will enter Level 0 by default, and the level indicator will have 5 lamps on and flash. The massage head does not move when it is in Level 0. If the hot compress massage head is inserted, the hot compress function is enabled. Press the button once to increase one level. Press the button at Level 5 to enter Level 0 again, and repeat in this way.

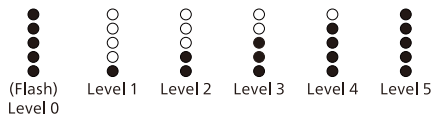6. The massage head is suspended after being blocked. This is to protect the device against abnormal operation and make it switch from operation mode to standby mode. Press the button to restart the device.

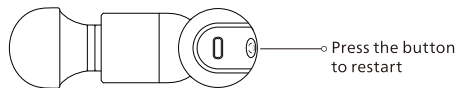! Please choose the massage intensity reasonably according to your own situation. Do not use the device for more than 10 minutes at a time.

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