Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2023 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev A MAR 23

PHILIPS

Back and
Waist Massager


Waist Massager

PPM4721

Quick Start Guide

1. Remove the accessories on the waist and abdomen, and then wear the massager on the waist and adjust it to a comfortable state.

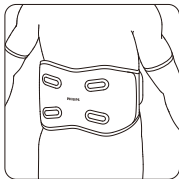2. Press and hold the power function button for 2 seconds to turn on/off. After power-on, the massager will enter the wake-up mode, and the buzzer makes a long sound of "drip ~". The status indicator turns orange, and the middle of screen displays the current temperature.

3. Press the power function button to turn on /off the heating function.

4. Press the mode adjustment button to switch the massage modes (wake-up mode - energizing mode - relaxing mode - soothing mode - massage off) circularly. During the adjustment, the corresponding mode display lamp will turn white.

5. Press the vibration level button to adjust the vibration (level 1- level 2 - level 3) circularly. During the adjustment, the corresponding mode display lamp will turn white, the display screen will display the vibration level (L1-L2-L3), and then automatically switch to temperature display 2 seconds later.

6. Press the temperature "<", ">" buttons to adjust the temperature (40°C, 45°C, 50°C, 55°C) circularly, and the middle of screen will display the current selected temperature.

7. When the device is working normally, press and hold the lock screen button for 2 seconds to lock/unlock the screen.

8. When the battery remaining capacity is lower than 10%, the indicator lamp ring will flash blue for 5 seconds. The buzzer will make a long sound of "drip ~", and the product will be powered off. When charging, the indicator lamp ring flashes blue slowly until the massager is fully charged, then the blue lamp remains on.

9. The massager will be automatically powered off after operation for 30 minutes, and the buzzer will make a long sound of "drip ~" to enter the power-off state.

- ❗ 1. As each person has different sensitivity to the temperature, it is recommended to use the massager with clothes on for fear of scalding if choosing 45°C and above;
- 2. It is recommended to select the first level of intensity at the beginning of the use of the product. Your waist and abdomen may become red after use, which can be relieved by itself after a period of time.
- 3. The fabric area should not be washed. Sterilize the fabric and remove sweat stains with alcohol or wipe it with a clean wet towel.

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