Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2024 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Guangdong Deerma Technology Company Limited and Guangdong Deerma Technology Company Limited is the warrantor in relation to this product.

Rev B APR 24

PHILIPS

Massage Gun

Sports Massage Gun

PPM7323

User Manual

Contents

1 Readme

Symbol Description

Applicable Users

Precautions for Use

Precautions for Maintenance

2 Product Features

3 Components

4 How to Use

Preparations before Use

Installation

Operation

Charging

Indicator Description

5 Maintenance

Cleaning

Disposal

6 Troubleshooting

7 Technical Parameters

8 Service Guide for Product Warranty

Warranty and Service

Warranty Card

9 Packing List

1 Readme

Please read this manual carefully and keep it properly before using the product. The manual is available at www.philips.com

Symbol Description

To avoid personal injury or property damage, be sure to observe the contents below.

- The following symbols are used to illustrate what you should observe.

Indicates prohibition

Indicates the actions that need to be subject to instructions

Applicable Users

The following users should use the device with caution:

- Users with skin sensory disorders
- Users who are implanted with metal, silicon or plastic materials in body due to a fracture or plastic surgery
- Users with sequelae due to traffic accidents, etc.
- Users who are vulnerable to electromagnetic failures due to cardiac pacemakers or other implanted medical electronic devices in their bodies
- Users who are deemed to have lost their sense of temperature
- Users with malignant tumors

- Users with heart disease
- Users who are pregnant or shortly after giving birth
- Users with perceptual impairment caused by diabetes
- Users who are injured but not healed
- Users with muscle damage
- Users who need peace and those in poor physical condition
- Users with osteoporosis, severe fractures, sprains, muscle pain, and acute pain (headache)
- Users with fever symptoms
- Users with symptoms such as acute inflammation, fatigue, aversion, and changes in blood pressure
- Other users who are treated in medical institutions

Precautions for Use

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

Children should be supervised to ensure that they do not play with the appliance.

If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.

This appliance contains batteries that are only replaceable by skilled persons.

- ❗ Must be only supplied at safety extra low voltage corresponding to the marking on the appliance.
- ❗ This product is set for 10-minute timing each time. Do not use it for more than 20 minutes each time. After use, stop the device for 30 minutes to ensure the extension of service life.
- ❗ Do not wear excessive clothes during use.
- ❗ Do not overuse the massage function.
- ❗ Stop the device immediately if you feel back pain, dizziness, vomiting, palpitation and other abnormal reactions during use.
- ❗ Keep away from liquid or heat source, and keep the charging port free of dust. Do not use the product during charging.
- ❗ After charging, disconnect the power supply in time.
- ❗ Stop the device immediately if it cannot work properly or you feel abnormal.
- ❗ Before each use, carefully check whether the product can be used normally according to the operation instructions in this manual.
- ⊘ Do not use it while sleeping.
- ⊘ Do not use it under a quilt or blanket.
- ⊘ Do not lie or sit on the product.
- ⊘ Do not bring serious impact and huge load to the product.
- ⊘ Do not use the product in the bathroom or shower.
- ⊘ It is strictly prohibited to disassemble, modify or repair the product by yourself.
- ⊘ Do not use the product under the following circumstances:
 - When you are severely tired
 - Within one hour before and after meals
 - When you are drunk


Precautions for Maintenance

- ❗ Be sure to turn off the device when cleaning.
- ❗ Properly handle the packaging of this product and keep it away from infants.
- ❗ Be sure to confirm whether the switch is normal in advance before it is used again if it has not been used for a long time.
- ❗ Do not use volatile oil, thinner, alcohol, strong alkaline detergent, bleach and other cleaning agents for maintenance.
- ❗ Frequently check the device for any sign of wear or damage. If there is such sign, or if the device has been misused, contact the supplier for maintenance before reuse.
- ⊘ This product contains lithium batteries. Do not disassemble, modify, immerse the product (controller) in water or throw it into fire, otherwise the product may be damaged.
- ⊘ Keep out of reach of children and infants. In addition, do not allow them to touch or operate the product. Otherwise, burns, electric shock, or personal injury may occur.
- ⊘ Do not damage, modify, bend or pull the power cord during charging. In addition, do not place heavy objects or squeeze the power cord.
- ⊘ This product contains a lithium battery. Keep it away from ignition sources as it may cause an explosion.

2 Product Features

This product is available for relaxing the tense muscles by massaging the soft tissues of the body with high frequency. A variety of massage heads can be combined to massage muscles all over the body.


3 Components

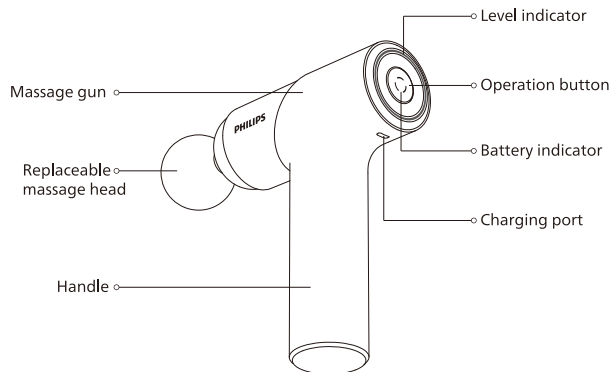Pointed
massage head

Flat
massage head

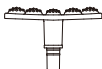Finger pressing
massage head

Spherical
massage head

Claw-shaped
massage head

Large-area multi-point
massage head


4 How to Use

Preparations before Use

1. Check whether the device and accessories are complete.
2. Remove the pendants and accessories when using, so as to avoid affecting the massage area.
3. Install the massage head in place.

Installation


According to the massage demand, insert the appropriate massage head into the front round hole of main unit and press it firmly to the end. As shown in the following figure, it indicates that it is installed in place.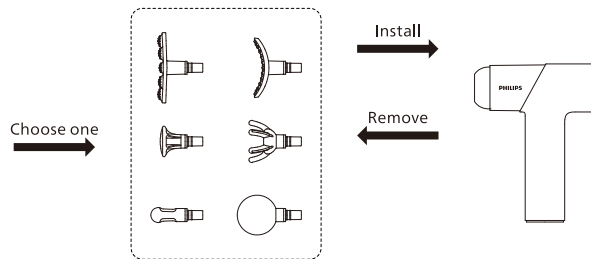

Operation



1. Press and hold the operation button to turn on the device. The device will enter Level 1 by default, and the level indicator will light up.
2. Press the operation button to change the levels at a range of Level 1 / Level 2 / Level 3 / Level 4 / Level 5 / low-speed waveform level / high-speed waveform level.
3. Press and hold the operation button to turn off the device at any level, and the level indicator and battery indicator will light off.
4. The device will power off automatically after continuous use for 10 minutes. Press and hold the operation button to restart the device for use again.

❗ The massage head is suspended after being blocked. This is to protect the device against abnormal operation and make it switch from operation mode to shutdown mode. Press and hold the button to restart the device.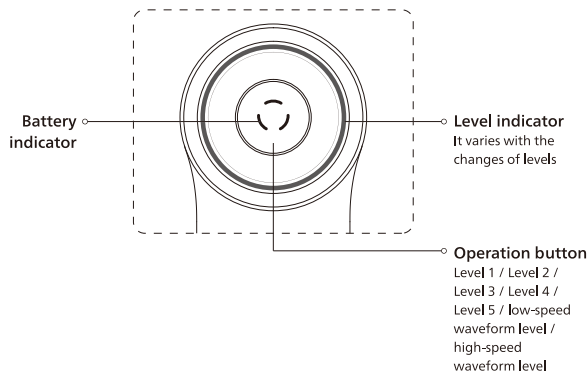

Charging

1. It is prohibited to charge in an upright position, so as to prevent injury to the human body when the product falls.
2. Before the first use, charge the device for 4 hours until it is fully charged.
3. When charging, connect the charging cable provided to the charging port and power socket of the device.
4. The battery indicator shows the battery capacity through the color of light, indicating that it is charging.
5. When the battery indicator flashes red, it indicates that the battery capacity is less than 10%. Please charge it in time.
6. The battery can be charged at any time and at any battery capacity. This product cannot be turned on or used during charging.

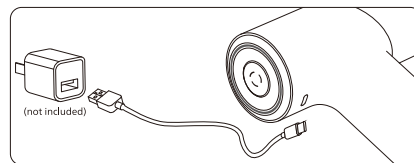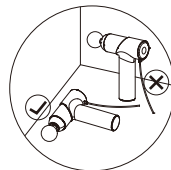Indicator Description

Press and hold the operation button to power on. The device will enter Level 1 by default, and the level indicator varies with the change of levels. The level indicator status is described as follows:

Level Indicator	Status	Description
Cyan lamp lights on in 1 section	Remain ON	Level 1
Cyan lamp lights on in 2 sections	Remain ON	Level 2
Cyan lamp lights on in 3 sections	Remain ON	Level 3
Cyan lamp lights on in 4 sections	Remain ON	Level 4
Cyan lamp lights on in 5 sections	Remain ON	Level 5
Low-speed revolving lantern	Revolving lantern	Low-speed waveform level
High-speed revolving lantern	Revolving lantern	High-speed waveform level

After power-on, the power indicator will light up, indicating the current battery capacity through the colors:

Battery Capacity Indicator	Status	Description
Green	Remain ON	Battery capacity >50%
Yellow	Remain ON	Battery capacity 10-50%
Red	Flash	Battery capacity <10%

When the device is being charged, the power indicator displays the current battery capacity in different colors:

Charging Indicator	Status	Description
Red	Flash slowly	Battery capacity 0-10%
Yellow	Flash slowly	Battery capacity 11-50%
Green	Flash slowly	Battery capacity 51-99%
Green	Remain ON	Battery capacity 100%

5 Maintenance

Cleaning

1. Clean the surface of the product with a slightly damp towel, and then dry it with a soft cloth.
2. Put the device in the appropriate portable bag for storing or traveling.
3. Store the device in a dry and well-ventilated place without corrosive gas.
4. Clean and pack the device if it is not used for a long time. It is recommended to charge it every 1 month.
5. This product is suitable for transportation by car, train, ship and airplane. During transportation, the product shall be fixed against violent vibration, collision, rolling and falling, and shall be placed upward and be handled with care and rain proof.
6. To protect the environment, do not directly dispose of the entire device. Please abide by the discarding method for each part.

 During maintenance, the power cord must be unplugged, but do not unplug it with wet hands.

Disposal

The lithium-ion battery used in this product is a valuable resource that can be recycled. Take out the lithium-ion battery when it is used up and discarded, and then hand it over to a professional institution that can recycle the rechargeable batteries for disposal.

Battery removal:

Pry the bottom of main unit, and then remove the battery from the device. (Users cannot replace the battery by themselves)

- The battery must be removed from the device before disposal;
- The device must be powered off before removing the battery;
- The battery should be disposed of safely.

6 Troubleshooting

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center in your region.

Fault Description	Solution
Massage head slips out or falls off	Check whether the massage head is installed in place
Start-up failure	Check whether the battery capacity is sufficient (charge for more than 4 hours if not used for a long time)
Charging failure	1. Check whether the voltage and current of the charging head are in line with the product requirements 2. Check whether the charging connector and power supply are plugged in place
Sudden stop after continuous operation for 10 minutes	This is a setting for protection of long-term operation. It is normal to switch from operation mode to shutdown mode. Press and hold the button to restart.

The massage head is suspended after being blocked	This is a setting for protection against abnormal operation. It is normal to switch from operation mode to shutdown mode. Press and hold the button to restart.
Reasonable use of the product can effectively prolong its service life. It is recommended that: 1. Do not throw or drop the device during use. 2. Do not use the device continuously for more than 20 minutes. Keep away from humid environment when using or storing. 3. Please charge the device every 1 month if it is not used for a long time.	

If the product has been left unused for a long time, be sure to confirm that its functions is normal before using it again.		
Regular check	Check for the following faults	In the event of the above faults, stop using the device immediately and cut off the power supply to prevent accidents. Please send it to our maintenance center.
	• Whether the charging cable or plug is hot • Whether the charging cable is deeply scratched or deformed, and has a burnt smell • Whether there is abnormal sound or vibration during work • Whether the product can work properly after switch on • Whether there are other abnormalities or faults	

Notice:

If the fault cannot be eliminated or the product still cannot operate properly after the remedy, contact the after-sales service personnel, and do not disassemble or attempt to repair by yourself.

7 Technical Parameters

Product Name	Philips Massage Gun
Model	PPM7323
Power Supply	Rechargeable Lithium Battery
Operating Voltage	7.4V ---
Operating Power	30W
Charging Voltage	5V ---
Charging Current	2A
Battery Capacity	1500mAh
Product Dimension	157x62x204mm
Net Weight	Approx. 0.7kg
Levels	7 levels
Interface Type	USB Type-C
Charging Duration	2-3 Hours

8 Service Guide for Product Warranty

Warranty and Service

If you need information or have any questions, please visit www.philips.com/-support or contact consumer service center in your country/region. If there is no consumer service center in your country/region, please contact your local distributor.

Within two years from the date of purchase, you will receive free warranty service for any damage caused by manufacturing process problems, or any damage to components under normal use conditions confirmed by our maintenance service personnel.

Warranty service excludes frequently replaced consumables, auxiliary equipment, shipping cost and door-to-door service. Please show proof of purchase to service personnel when servicing.

Malaysia

Consumer Care: +60-128868274

Email: philips.massage@realx.tmy.net

Singapore

Consumer Care: +65-89420233

Email: philips.massage@realx.tsg.net

Philippines

Consumer Care: +63-9604111314

Email: Philips.massager.ph@orohome.com

Thailand

Consumer Care: +66-20051888

Email: Philips_massager@orohome.co.th

Indonesia
Consumer Care: +021-39715012 / +021-39715013
Email: Philips_Massager.ID@orrohomeindonesia.com

Vietnam
Consumer Care: +84-909800829
Email: Philips.massager.VN@orrohome.vn

Warranty Card

Product Model		User Name	
Sales Agency		Phone Number	
Date of Purchase		Serial Number	
Special Technical Service Center		User Address	
Items of Maintenance	Date	Service Personnel	

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