Visit us

www.philips.com

There may be certain differences between the content of this manual and your product. Please refer to the actual product. We reserve the right to update the product at any time without prior notice.

© 2024 AquaShield

All rights reserved

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license.

This product has been manufactured by and is sold under the responsibility of Hong Kong AquaShield Health Technology Company Limited and Hong Kong AquaShield Health Technology Company Limited is the warrantor in relation to this product.

Rev A JUL 24
Made in china

PHILIPS

Neck and Shoulder
Massager

Neck Massager

PPM3502

User Manual

Contents

1 Instructions for User

Symbol Description

People Suitable to Use

Precautions for Use

Precautions in Operating Site

Precautions for Maintenance

2 Product Features

3 Product Overview

4 How to Use

Operation

Charging

Description of Power Indicator

5 Maintenance

Cleaning

Discarding

6 Technical Parameters

7 Troubleshooting

8 Service Guideline for Warranty

Service and Support

Warranty

Warranty Card

9 Packing List

1 Instructions for User

Please read this manual carefully before using the product and keep this manual properly.

Symbol Description

To avoid personal injury or property damage, be sure to observe the following.

- The following symbols respectively illustrate the items that you must observe

 Indicates that it is prohibited

 Indicate that you should do as instructed

Inapplicable Group

 The product should be used with caution by the following groups

People under treatment at a medical institution or in the following conditions should consult with a doctor or physical therapist before using the product.

- People under medical treatment who feel physically abnormal
- People who are taking medication
- People with abnormal blood pressure
- People with infectious diseases

- People with skin perception disorders or skin abnormalities
- People who are taking medication
- People with abnormal blood pressure
- People who cannot freely express their wills
- People with perception disorder due to high peripheral circulation disorder such as diabetes
- The surface of the device may become hot, which must be paid attention to when used by people who are not sensitive to heat

People with following situations are prohibited from using the product

People who are prohibited from exercising or stretching activities as advised by doctor, such as thromboembolism, severe aneurysm, acute venous aneurysm, various skin inflammations, and various skin infections, because they can easily cause deterioration.

- Those who cannot express their will or cannot operate the device by themselves are prohibited from using the product.
- Children under the age of 6 are prohibited from using this product. Adolescents aged 6-14 should use it under the guidance of parent or adult guardian.
- If you experience abnormal body reactions such as back pain, dizziness, vomiting, throbbing, etc. during use, please stop the product immediately.
- This product is set for 15-minute timing at a time. Do not use it for no more than 30 minutes each time. After each use, stop the device for 30 minutes to ensure the extension of its service life.
- People with following situations are prohibited from using the product
 - When you are severely tired
 - Within an hour before and after meals
 - When you are drunk

Precautions for Use

- ❗ Before using, make sure that the product is in good condition. If any damage is found, regardless of size, stop using it immediately.
- ❗ Clean the charger regularly. Dust on the plug can easily accumulate and lead to a fire due to poor insulation caused by moisture.
- ❗ Make sure the switch is working properly before using the product.
- ❗ Stop using it immediately if it doesn't work or you feel abnormal.
- ❗ Unplug the charger when the power goes out.
- ❗ This appliance is not suitable for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.
- ❗ If the power cord is damaged, it must be replaced with a special cord or assembly available from the manufacturer or its service agent.
- ❗ The battery contained in this appliance should be replaced by qualified personnel only.
- ❗ The appliance should only be powered by a safety extra-low voltage as marked on the appliance.
- ❗ Be sure to turn off the power when not in use.
- ❗ The surface of the device may become hot, which must be paid attention to when used by people who are not sensitive to heat
- ⊘ Do not place load on the charger to prevent damage.
- ⊘ Do not stand on the product or use the product as a toy to avoid accidents. NEVER disassemble, modify or repair the product by yourself. Do not open the zipper by yourself.
- ⊘ Do not use this product when your body is wet.
- ⊘ Do not remove the outer cover for use, as it may cause accidents.

- ⊘ Do not forcefully pull the product or stab it with a knife or sharp object.

Precautions in Operating Site

- ⊘ Do not use the product near the stove, or inside the electric quilt, or on heating equipment such as electric blankets, as it can easily cause a fire or malfunction.
- ⊘ Do not cover the product with towels or cloth or other objects for use.
- ⊘ Do not use the product near bathtubs, showers, washbasins, or other high humidity utensils that hold water.

Precautions for Maintenance


- ❗ During maintenance, be sure to turn off the power and unplug the charger.
- ❗ When not in use for a long time, be sure to turn off the power and unplug the charger.
- ⊘ Do not drop the product, or hit it with strong force.
- ⊘ Do not intentionally apply excessive force on the product.
- ⊘ Do not spray water on the product.
- ⊘ Do not use the product in areas with a room temperature above 35°C or in direct sunlight.
- ⊘ Do not use the product in bathrooms or places with high humidity.
- ❗ If the product has not been used for a long time, be sure to confirm in advance whether the switches and other units work properly before using it again.
- ⊘ Do not clean the product with volatile oil, diluent, alcohol, strong alkaline cleaner or bleaching agent for cleaning during the maintenance.
- ❗ Simple shockproof facilities are provided for the product so that it can be transported by air, rail, road and ship. Avoid rain and snow, inversions and collisions during transport.

- ❗ Please keep the product out of reach of children.
- ❗ When not in use for a long time, store the product in a safe, dry, ventilated and cool place after fully charging.
- ❗ When not in use for a long time, fully charge the product every other month.

2 Product Features

This product can relax the neck by simulating the hand kneading, finger pressing and heating. Multiple massage mode and kneading intensity are available. It is suitable for daily massage and relaxation of different groups.

3 Product Overview

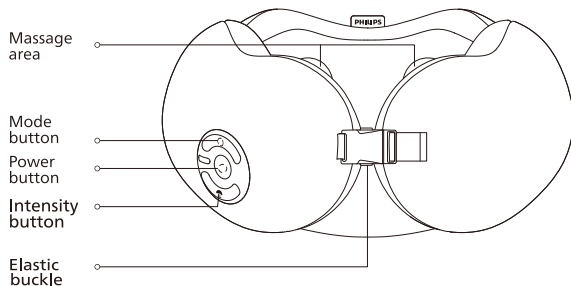4 How to Use

Preparations for Use

1. Check that the product and accessories are complete.
2. Remove the pendants and decorations before use to avoid affecting the massage.
3. Wrap the massager around your neck, and then lock the buckle to keep it fully attached to your body.

Operation

Power button

1. Press and hold the power button for 1.5 seconds to turn on the device, with a voice prompt of "Massage On". The device will work in "soothing mode" by default, with a voice prompt of "Soothing Mode", which includes forward kneading, low intensity, and low temperature.
2. Press it to change temperature in an order of [Low Temperature] - [High Temperature] - [Heating OFF], with the corresponding voice prompt. The initial grade of each mode shall be subject to the specific mode.
3. Double click the power button with in 1.5 seconds to turn off/on voice prompt, with the corresponding voice prompt.
4. It will automatically shut down after 15 minutes of massage. If reuse, press and hold the power button for 1.5 seconds to restart massage.

Mode button

Press it to switch modes in an order of:

[Soothing Mode] - [Vitality Mode] - [Wake-Up Mode] - [Neck Caring Mode] - [Kneading Mode], with the corresponding voice prompt.

Intensity button

Press it to change the intensity in an order of [Low Intensity] - [Medium Intensity] - [High Intensity], with the corresponding voice prompt. The initial intensity of each mode shall be subject to the specific mode.

Charging

- 1. Connect the charging cable to the charging port at the control panel.
- 2. Plug the charger into a 220V-50Hz outlet.
- 3. Use a 5V-/2A adapter for charging.
- 4. When the power is less than 10%, the power button indicator will flash, with a voice prompt that "low battery level, please charge it now". The device will automatically shut down after 1 minute, and can only be restarted after charging.
- 5. Do not use the product during charging.

Description of Power Indicator

During working, the indicator displays different colors to show the current power.

Power indicator	Status	Description
Green light	Remain on	Low temperature
Orange light	Remain on	High temperature
Blue light	Remain on	Heating OFF
Green light / Orange light / Blue light	Flashing	Current power <10%

During charging, the indicator displays different colors to show the current power.

Power indicator	Status	Description
Red light	Blinking	Current power <10%
Orange light	Blinking	Current power 11%-50%
Green light	Blinking	Current power 51%-99%
Green light	Remain on	Current power 100%

5 Maintenance

Cleaning

- 1. Remove the dust on the product cover by a vacuum cleaner, rather than patting it.
- 2. Remove the surface dirt with a clean cloth.
- 3. Use the wet dishcloth with neutral detergent and then gently wipe off the stubborn stains.
- 4. Do not let the water into the product. Massage components and the control panel are prohibited from getting wet.
- 5. Do not expose the product in direct sunlight or moisture. If not used for a long time, remove the dirt, and then cover it with a cloth to prevent dust contamination.
- 6. Be sure to shut down the device for maintenance.

- ⚠ Do not remove the product cover when cleaning.
- ⚠ Keep the product out of reach of children.
- ⚠ Do not store the product in direct sunlight, high temperature, or humid places.
- ⚠ Keep the product in a dry and ventilated place.
- ⛔ DO NOT disassemble, modify or repair the product by yourself, as it may cause malfunctions.

Discarding

The lithium-ion battery used in this product is a valuable resource that can be recycled. Please remove the lithium-ion battery after use, and send it to a professional agency that can recycle the rechargeable battery for disposal.

- 1. Do not replace the battery during normal use.
- 2. Be sure to remove the battery from the product before discarding, with the steps as follows:
 - (1) Remove the product cover.
 - (2) Remove the battery box assembly and open the bottom cover of the control box.
 - (3) Remove the battery by disconnecting the battery connection wires inside.
- 3. Be sure to shut down the product before removing the battery.
- 4. Dispose of the battery in a safe manner.

6 Technical Parameters

Product Name	Philips Neck Massager
Product Model	PPM3502
Operating Voltage	7.4V 
Operating Power	10W
Charging Voltage	5V 
Charging Current	2A
Battery Capacity	2500mAh
Charging Time	Approximately 4 hours
Power Interface	Type-C
Product Size	247x257x126mm
Net Weight	约0.92kg
Date of Production	See product compliance certificate

7 Troubleshooting

This chapter lists the most common problems to the product. If the problems you encounter are not included in the following tables, please call our service hotline or contact the authorized service center at the region where you are located.

Fault Description	Cause	Solution
Failure to work	Battery power is low if not use for long term	After recharging, press and hold the power button again for 1.5 seconds to boot.
	The power button is not turned on	Press and hold the power button for 1.5 seconds to boot
	This product has a timer function, which automatically shuts down after a 15-minute cycle ends.	To continue using, press and hold the power button again for 1.5 seconds.
Sudden stop during the operation	The product is low in power	After recharging, press and hold the power button again for 1.5 seconds to boot.
A sound from the product (vibration of the massage head)	The sound is caused by the mechanical structure of the product	It is normal.
Failure to charge or no charging light display	It is not well connected	Check that the USB cable, main unit and adapter are connected properly.

If the product has been left unused for a long time, be sure to confirm that its function is normal before reuse		
	Check for the following faults	
Regular check	<ul style="list-style-type: none"> Whether the charging cable is hot. Whether the charging cable is deeply scratched or deformed, and has a burnt smell. Whether there is abnormal sound or vibration during work . Whether the product can work properly after powering on. Whether there are other abnormalities or faults. 	In the event of the above faults, stop using the device immediately and cut off the power supply to prevent accidents. Please send the defective product to our maintenance center.

Note:.

If the fault cannot be eliminated or the product still cannot operate satisfactorily, contact the after-sales. Do not disassemble or attempt to repair it by yourself.

8 Service Guideline for Warranty

Service and Support

This Product is a well-designed household appliance made of high-quality components, which should perform well under normal use and maintenance. Within two years after the date of purchase, in case of any damage caused by the manufacturing process or components, Philips China Maintenance Service Center will provide the in-store maintenance service free of charge for the users who hold the warranty card, and replace the parts or the entire product as appropriate.

During or after the warranty period, the maintenance service is available in any country where Philips officially sells the product.

If the product is not sold by Philips in the local market, Philips also provides maintenance service. However, the maintenance duration may be longer due to the lack of required parts. Please contact your local Philips franchiser or Philips service department.

If you need to consult related information, or in case of difficulties, please contact the local Philips Maintenance Service Center, or the Philips Service Hong Kong Importer: Century Field International Ltd.

Address: Room 505, 5/F, Nam Wo Hong Building, 148 Wing Lok Street
Sheung Wan, Hong Kong

Hotline: 852-3188-4987

Service Email: philips.message.hk@centuryfield.com

Warranty

- Within two years after the date of purchase, you will receive free warranty service for any damage caused by the manufacturing process or components under normal use confirmed by the personnel of our authorized maintenance center.
 - This copy shall be properly kept by users.
 - This card and the original invoice shall be presented for servicing.
 - This card will be invalid if altered.
 - This card is valid only with the original invoice.
 - Free services will not be available under the following circumstances
 1. The product is artificially damaged due to improper use, such as improper access to power supply, use of improper accessories, improper installation, failure to follow the instructions, misuse or negligence; the product is damaged by transportation and other accidents; maintenance and modification not approved by our company; other damages caused by force majeure (such as natural disasters, abnormal voltage, etc.).
 2. The product is damaged by use outside the general household (such as industrial and commercial use).
 3. The product is aging and worn caused by normal use, which does not affect the normal use of the product.
- This card is valid only within the territory of Hong Kong.
- The warranty card serves as a guarantee for free maintenance within the prescribed period and conditions, without restricting the legal rights of consumers.

Warranty Card

Product Model		User Name	
Sales Agency		Phone Number	
Date of Purchase		Serial Number	
Special Technical Service Center		User Address	
Item of Maintenance	Date	Service Personnel	

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