General description

The Philips Avent storage cups are part of a reusable storage system specifically designed to collect and store breast milk and feed your baby. The Avent storage cups have twist-on lids to provide leakproof transport and storage.

Read all instructions carefully before use. Keep the user manual and product packaging for future reference. An electronic copy of these instructions can be found at www.philips.com/support.



Important safety information

Read this important information carefully before you use the appliance and save it for future reference.

For your child's safety and health

- Continuous and prolonged sucking of fluids will cause tooth decay.
 - Always check food temperature before feeding.
 - Before each use, inspect the product. Throw away at the first sign of damage or weakness.
 - Keep components not in use out of the reach of children.
 - Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
 - Always use this product with adult supervision.
- It is not recommended to use a microwave to warm up baby food or drinks. Microwaves may destroy valuable nutrients in food/drinks and may produce localized high temperatures. If you decide to use the microwave, take extra care to stir heated food/drinks to ensure even heat distribution and check the temperature before serving. Only place the container in the microwave, without the lid. To avoid tooth decay, drinks other than milk and water, such as fruit juices and flavoured sugary drinks are not recommended. If they are used, they should be well diluted and only used for limited periods, not constantly sipped. Do not allow children to play with small parts or walk/run while using cups. Do not bring the storage cup parts in contact with or place on surfaces with abrasive or antibacterial cleaning agents. Excessive concentrations of cleaning agents may eventually cause plastic parts to crack. Should this occur, replace the cracked part immediately. This product may break if dropped. If you use a hot liquid to heat the food, check the temperature carefully.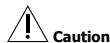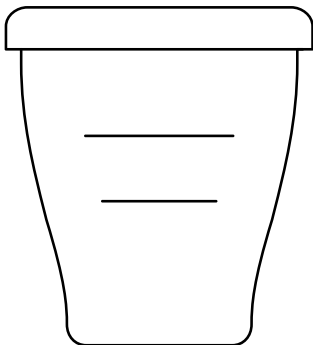

- Never refreeze thawed breast milk or add fresh breast milk to already frozen breast milk.
- You can store breast milk in the storage cup in the refrigerator for up to 4 days or in the freezer for up to 6 months.
- When you take breast milk out of the refrigerator, use it within 4 hours.
- Do not fill the cup beyond the 180 ml/6 oz level, as breast milk expands when frozen.

For food storage cups SCF721 only:

- The spoon is suitable for children aged 6 months and older.
- It is not recommended to put the spoon in a microwave.
- Do not use the spoon in combination with oily or very acidic food.
- Do not fill the cup beyond the 240 ml/8 oz level, as breast milk expands when frozen.

Before first use

Disassemble all parts, clean and then sterilize the parts by placing in boiling water for 5 minutes or sterilize by using a Philips Avent sterilizer. This is to ensure hygiene.

Assembly

Assemble the storage cup following the steps shown in the images. If you decide to use the storage cup with a feeding teat, make sure that the feeding teat is fully pulled through the screw ring as shown in the image.

Heating breast milk

To ensure a gentle and even warming of your milk, place it in warm water or use a Philips bottle warmer. Always remove lids first if using bottle warmers or microwaves.

Defrosting breast milk

To defrost breast milk overnight, place the storage cup in the refrigerator. To defrost breast milk prior to feeding, place the storage cup in warm water.

Note: Always discard any breast milk that is left over at the end of a feed.

Storage of milk and food

Expressed breast milk or baby food can be stored in cleaned, sterilized and closed Philips Avent storage cups in the refrigerator for up to 4 days or in the freezer for up to 6 months. Write the expression date on the storage cup with a pencil or marker. Never refreeze breast milk or add fresh breast milk to already frozen milk. Expressed breast milk can be stored outside the fridge for up to 4 hours.

Cleaning and storage

Clean all parts before each use to ensure hygiene. After each use, disassemble all parts, wash in warm, soapy water, remove any food residues and rinse thoroughly. Then sterilize using a Philips Avent sterilizer or boil for 5 minutes. Philips Avent storage cups are suitable for all methods of sterilization. Make sure that you wash your hands thoroughly and that the surfaces are clean before contact with sterilized components. During sterilization with boiling water, prevent the parts from touching the sides and bottom of the pot. This can cause irreversible product deformation, defect or damage that Philips cannot be held liable for. Food colorings may discolor parts. This product is dishwasher safe. Clean them on the top rack of the dishwasher. Do not put any other objects on top of the storage cups. Do not place in a heated oven. Do not bring the cup parts in contact with or place on surfaces with abrasive or antibacterial cleaning agents. When not in use, store in a clean and dry place.

Compatibility

With the adapter for the cup, the Philips Avent storage cup is compatible with Philips Avent breast pumps, screw rings and feeding teats of the Natural and Anti-colic ranges.

Ordering accessories

To buy accessories or spare parts, visit www.philips.com/support or go to your Philips dealer. You can also contact the Philips Consumer Care Center in your country.

Support

If you need information or support, please visit www.philips.com/support.

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Trademarks are the property of Koninklijke Philips N.V.
All rights reserved.