PHILIPS

sonicare

- EN** Patient Trial Program
Disinfection and handling procedure
- DE** Patiententestprogramm
Desinfektions- und Handhabungsverfahren
- DK** Program med patientforsøg
Desinfektions- og håndteringsprocedure
- ES** Programa de prueba en
Procedimiento de desinfección y manipulación
- FI** Potilaiden kokeiluohjelma
Desinfiointi- ja käsittelyohjeet
- FR** Programme d'essais pour les
Procédure de désinfection et de manipulation
- IT** Programma di prova nei pazienti
Procedura di disinfezione e trattamento
- NL** Proefprogramma patiënten
Procedure voor desinfectie en gebruik
- NO** Testprogram for pasienter
Desinfeksjons- og behandlingsprosedyre
- PT** Programa de testes para pacientes
Procedimento de desinfeção e manuseamento
- SV** Testprogram för patient
Desinfektions- och hanteringsförfarande

— |

| —

— |

| —

— |

| —

— |

| —


English 5
Dansk 10
Deutsch 15
Español 20
Français 25
Italiano 30
Nederlands 35
Norsk 40
Português 45
Suomi 50
Svenska 55

Introduction

The Philips Sonicare Patient Trial Program is an opportunity for patients to use the Sonicare Power Toothbrush, especially if they haven't tried it before. Please follow the tooth brushing instructions precisely and use appropriate infection control techniques during this procedure when handling a clean or used toothbrush. Save these instructions for future reference.


Preparation

- 1 Put on gloves.
- 2 Take the brush handle off the charger and inspect the handle for any damage.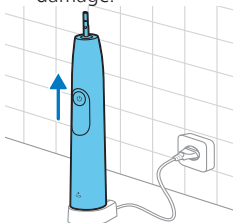

- 3 Hold the handle with buttons facing you and fit one of the protective sleeves from the kit through the small opening on the sleeve.


Note: Do not pierce or damage the sleeve.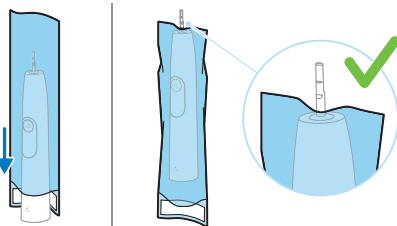

- 4 Pull the excess end of the sleeve back up to the handle and peel away the tape to expose adhesive, then fold it across the bottom of the handle and seal the sleeve closed.


Note: If the sleeve is impaired in any way, discard it and place a new sleeve on the handle

6 English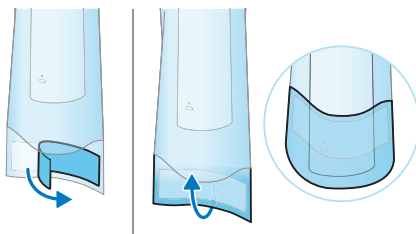

- 5 Unpack a new brush head and press the brush head onto the handle in such a way that the bristles will face the front of the handle. Firmly press the brush head down onto the metal shaft until it stops. The brush head will fit over the top of the sleeve.

Note: It is normal to see a slight gap between the brush head and the handle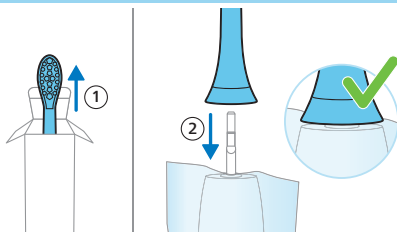


- 6 Ask the patient to wash his hands and to brush teeth for 2 minutes following the separate brushing instructions.

Reminder: Patient should brush over a sink.


Cleaning and Disinfecting handle after each use

- 1 Hold handle upside-down and remove protective sleeve. Invert the dirty side within the sleeve so only the protected side is touched. If brush head is still attached, remove used brush head within the sleeve. Discard used sleeve and used brush head.

Note: Do not put toothbrush handle in any type of autoclave.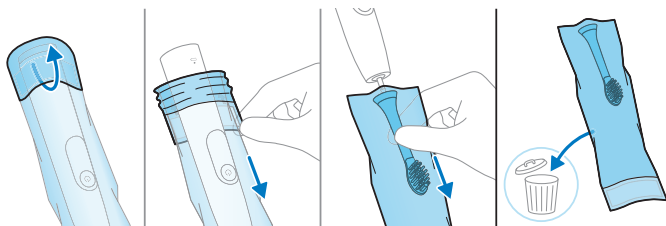


- 2 Pick up the used handle using one piece of CaviWipes™.
- a Wrap a wipe around the used handle and wipe handle and metal shaft up and down for at least 15 seconds, ensuring all surfaces are clean. Discard the wipe.
 - b Use a new wipe to repeat the same procedure for 15 seconds, assuring the surfaces are clean.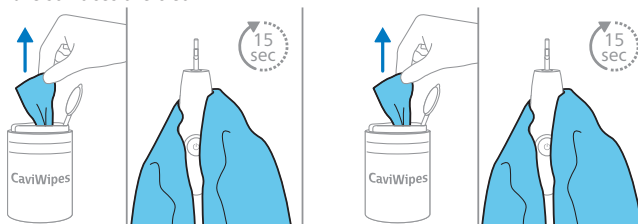

- 3 Wear clean gloves and place the handle in 1 gallon plastic zipping bag. Ensure that the handle is lying flat.

Note: Take care the metal tip does not pierce the plastic bag.


Fill the plastic bag with CaviCide™ disinfectant and squeeze out excess air so the handle is fully immersed in solution; carefully seal the bag. A cold sterile container may be used instead of a zipping bag.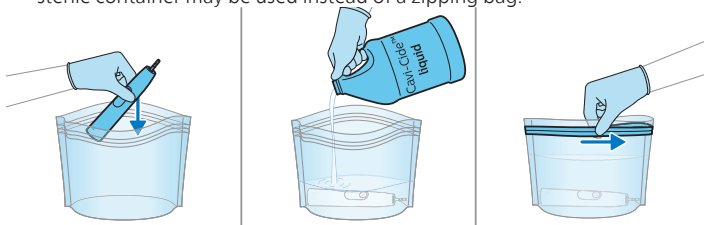

- 4 Soak the handle for 3 minutes (use a timer).

Do not soak handle for an extended period of time as this might cause damage to the product.


- Put on gloves. Turn on the water faucet, open the bag over a sink, and empty the liquid with water running from the faucet; discard bag. Rinse the handle with distilled water. Wipe the handle with a 70% isopropyl alcohol wipe, to aid in drying the handle.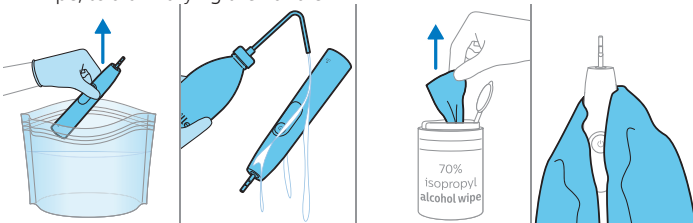

- Allow handle to air-dry; then charge the handle on the charger for at least 15 minutes before next Professional Trial. Discard used gloves.

Note: Charge handle overnight at the end of each day. Store and use at room temperature.

Note: If unit is damaged or non-functioning, do not modify or repair the product. Contact your Sonicare® Representative for replacement.

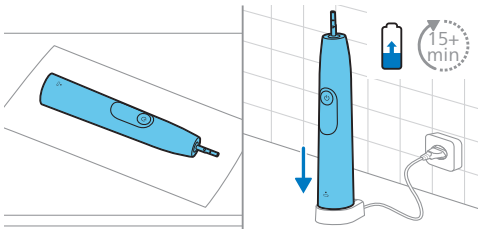- After the handle is disinfected, record the number of uses on the Handle Cleaning and Disinfection Log. Use this log to track the of number of times the handle was disinfected and used. Do not exceed 240 brushings.

Note: The starter pack includes brush heads and sleeves for 60 trials. Per starter kit, you can re-order up to 3 refill packs for both brush heads and sleeves, to drive up to 240 trials per handle.

1	☒	2	☒	3	☒
13	☒	14	☒	15	☒
25	☒	26	☒	27	☒
37	☒	38	☐	39	☐
49	☐	50	☐	51	☐
61	☐	62	☐	63	☐
73	☐	74	☐	75	☐
85	☐	86	☐	87	☐

The Test Drive handle should not be used more than 240 times. Please contact your Sonicare® Representative for a replacement.

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©2024 Koninklijke Philips N.V. (KPNV). All rights reserved. Philips and the Philips shield are trademarks of KPNV. Sonicare and the Sonicare logo are trademarks of Philips Oral Healthcare, LLC and/or KPNV.