PHILIPS

AVENT

Collection cups

Flacons de
recueil

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
Hands-free


Mains libres




Indossabile

SCF481/11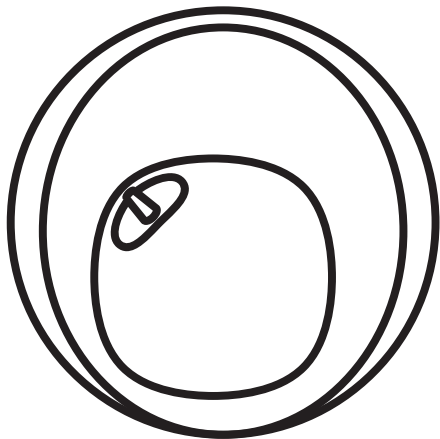

www.philips.com/support

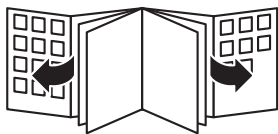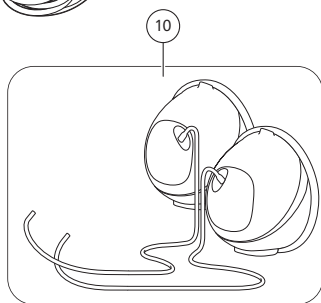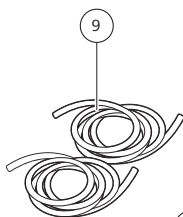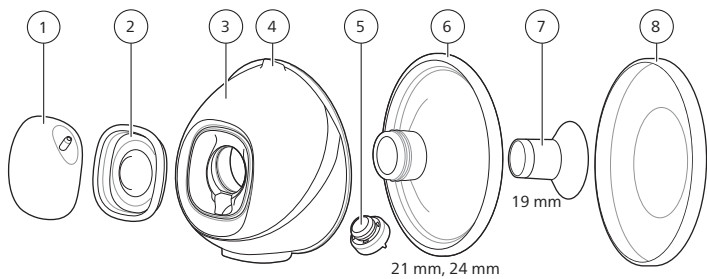English	6
Deutsch	15
Español	25
Français	34
Italiano	44
Nederlands	53
Polski	63
Português	73
Ελληνικά	83

Introduction

Welcome to Philips Avent! To learn more about this product or the Philips Avent product range, scan the QR code on the user manual or visit us at www.philips.com/support.

General description (Fig. 1)

i Note: The numbers mentioned below refer to the numbers in the figure on the front foldout page of this user manual.

- | | |
|----------------------------|--------------------------------------|
| 1 Front cap (2 pcs) | 6 Breast shield (21 mm, 24 mm) |
| 2 Membrane (2 pcs) | (2 pcs each) |
| 3 Cup (2 pcs, 200 ml each) | 7 Insert (19 mm) (2 pcs) |
| 4 Pouring spout | 8 Breast shield (dust) cover (2 pcs) |
| 5 Valve (2 pcs) | 9 Silicone tube (2 pcs) |
| | 10 Collection cups (assembled) |

The Philips Avent hands-free collection cups are intended to be used with the Philips Avent Hands-free Electric Breast Pump, electric breast pump Premium, Premium Plus, or Advanced. The device is intended for a single user.

Important safety information

Read this user manual carefully before you use the product and save it for future reference.

This user manual can also be found online at: www.philips.com/support.

⚠ Warning

Warnings to avoid choking, strangulation and injury:

- Children and pets should be supervised to ensure that they do not play with the packaging materials or accessories.
- Make sure to use the silicone tubes carefully to avoid strangulation.

Warnings to avoid contamination and to ensure hygiene:

- For hygienic reasons, the collection cups are only intended for repeated use by a single user.
- Clean and disinfect all parts that come into contact with the breast and breast milk before first use and after each use. If you use inserts and/or breast shields of different sizes during the pumping session, make sure to clean and disinfect all such parts.
- Store and transport the collection cup using the breast shield (dust) cover.

Warnings to avoid breast and nipple problems including pain:

- Do not use the collection cup if the membrane appears to be damaged or broken. See 'Ordering accessories and spare parts' for information on how to obtain replacement parts.
- Only use accessories and parts recommended by Philips Avent.

Warning to avoid cross-contamination:

- The collection cups are personal care items intended for use by a single user only and should not be shared between users.

⚠ Caution**Caution to prevent damage to and malfunctioning of the breast pump:**

- Keep all parts away from heated surfaces to avoid overheating and deformation of these parts.


Any serious incident that has occurred in relation to this device should be reported to Philips via **www.philips.com/support** and to the competent authority of the Member State in which the user and/or patient is established.

Before first use

Disassemble, clean and disinfect all parts that come into contact with the breast and breast milk before you use the collection cups for the first time and after each use.

Cleaning and disinfecting

- Disassemble, clean and disinfect the following parts before first use and after every use: membrane, cup, valve, breast shield, insert (if used), breast shield (dust) cover.

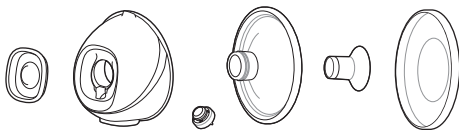- Clean the front cap and silicone tube(s) when needed. Wipe with a clean, damp cloth with water and mild detergent.

Step 1: Disassembling

Completely disassemble all parts that come into contact with the breast and breast milk. Make sure that you:

- Remove the valve from the cup.
- Separate the membrane and front cap.
- Remove the insert (if used) from the breast shield.

Step 2: Cleaning

⚠ Warning: Do not use antibacterial or abrasive cleaning agents to clean the collection cup parts as this may cause damage.

Clean the following parts before first use and after every use: membrane, cup, valve, breast shield, insert (if used), breast shield (dust) cover. You can clean these parts manually or in the dishwasher.

Step 2A: Manual cleaning

Supplies needed:

- Mild dishwashing liquid
- Drinking-quality water
- Soft, clean brush
- Clean tea towel or drying rack
- Clean sink or bowl

⚠ Caution: Be careful when you remove and clean the valve. If it gets damaged, your breast pump does not function properly. To clean the valve, rub it gently between your fingers in warm water with some dishwashing liquid. Do not insert objects into the valve, as this may cause damage.

- 1 Rinse the parts under a running tap with lukewarm water.
- 2 Soak the parts for 5 minutes in warm water with some mild dishwashing liquid.
- 3 Clean the parts with a cleaning brush in warm water with some mild dishwashing liquid.
- 4 Thoroughly rinse the parts under a running tap with drinking-quality water.
- 5 Leave the parts to air-dry on a clean tea towel or drying rack.

Step 2B: Cleaning in the dishwasher

Supplies needed:

- Mild dishwashing liquid or a dishwashing tablet
- Drinking-quality water

ⓘ Note: Food colorings may discolor parts.

- 1 Place all parts on the top rack of the dishwasher. Use a dishwasher basket for the smallest parts.

- 2 Put detergent or a dishwashing tablet in the machine and run a standard program.
- 3 Leave the parts to air-dry on a clean tea towel or drying rack.

Step 3: Disinfecting

Supplies needed:

- Household pot
- Drinking-quality water

⚠ Caution: During disinfection with boiling water, prevent the parts from touching the sides and bottom of the pot. This can cause irreversible product deformation or damage that Philips cannot be held liable for.

Disassemble, clean and disinfect the following parts before first use and after every use: membrane, cup, valve, breast shield, insert (if used), breast shield (dust) cover.

- 1 Fill a household pot with enough water to cover these parts and place all the parts in the pot. Let the water boil for 5 minutes. Make sure the parts do not touch the sides and bottom of the pot.
- 2 Allow the water to cool down.
- 3 Gently remove the parts from the water. Leave the parts to air-dry on a clean tea towel or drying rack.
- 4 Store the dry items safely in a clean, protected area until needed. Ensure that the clean parts have air-dried completely before storing them to help prevent germs and mold from growing.

Using the collection cups

Sizes of the breast shields and inserts

⚠ Warning: Using an incorrectly sized breast shield and insert may cause pain, discomfort, and improper nipple positioning during pumping. Prolonged use may lead to nipple or breast tissue injury, inflammation, or blocked milk ducts. Ensure proper fit before use.

The Philips Avent hands-free collection cup has a soft breast shield that can be used individually or with a flexible silicone insert for a better comfort. There are multiple sizes available.

Measure your nipple to find the most suitable breast shield and, if necessary, the insert for you.

- 1 Use the nipple sizing card that is included in the package or refer to **www.philips.com/support** to measure your nipples.
- 2 Stimulate your nipple prior to measuring your size.

- 3 Place the nipple sizing card around your nipple. Make sure there is a space of 0.5–2 mm between your nipple and the hole in the nipple sizing card.
- 4 Try various holes to see which opening fits your nipple best. If needed, view fitting in a mirror.
- 5 Select the breast shield (and the insert) that matches the hole that fits your nipple best.
- 6 Breast shield sizes 21 mm, 24 mm, and insert size 19 mm are included in the package.
- 7 Breast shield size 28 mm and insert sizes 15 mm, 17 mm, 26 mm can be purchased separately (see 'Ordering accessories and spare parts').
- 8 Start pumping by using the selected size(s) and closely monitor your comfort. After pumping, the nipple size might slightly change. If pumping becomes uncomfortable, measure your nipple size again and switch to another size, if needed.

i Note: You might need different sizes for your left nipple and your right nipple. Nipples may also change during your pumping journey. For example, the nipple size may change just after giving birth, after several months of breastfeeding or pumping, or may be different again after a second pregnancy.

Assembling the collection cups

i Note: The figure numbers mentioned below refer to the numbers of the figures on the back foldout pages of this user manual.

⚠ Warning: Be careful, when you have disinfected the collection cup parts by boiling them, they can be very hot. To prevent burns, only start assembling the collection cups when disinfected parts have cooled down.

⚠ Warning: Wash your hands thoroughly with soap and water before you touch collection cup parts and breasts to prevent contamination. Avoid touching the inside of the cup.

Make sure you have cleaned and disinfected the collection cup parts that come into contact with the breast and breast milk. Gather all parts and place them on a clean surface.

Check the parts for damage before use and replace them if necessary.

i Note: Correct placement of the breast shield, membrane, insert (if used), valve and silicone tube is essential for the breast pump to form a proper vacuum and to reduce the risk of milk leakage.

- 1 Wash your hands thoroughly with soap and water.

- 2 Push the valve into the cup as far as possible (Fig. 2).
- 3 Attach the membrane to the cup (Fig. 3).
- 4 Attach the front cap (Fig. 4) to the membrane and the cup. You can choose one of 4 possible positions of the front cap, to orient the silicone tube as preferred. Choose the most comfortable position for you. Push down the membrane cap until it is securely in place.
- 5 If you use an insert, first place the insert (Fig. 5) into the breast shield fully (Fig. 6).

ⓘ Note: If you use an insert, assemble it to the breast shield prior to attaching the breast shield to the cup, to avoid any gaps.

Make sure that you attach the insert to a breast shield of the corresponding size. The insert sizes 15 mm, 17 mm, 19 mm fit on the breast shield size 21 mm. The insert size 26 mm fits on the breast shield size 28 mm. Breast shield size 24 mm can only be used without an insert.

- 6 Attach the breast shield (including the insert, if any) to the cup (Fig. 7). Push the inner part of the breast shield into the cup as far as possible. Make sure that the rim of the breast shield covers the sides of the cup fully (Fig. 8).
- 7 Connect the silicone tube to the front cap (Fig. 9).
- 8 Please refer to the user manual of your Philips Avent electric breast pump for the next assembly steps, for example how to connect the tube(s) to the motor unit.

ⓘ Note: Place the breast shield (dust) cover over the breast shield to keep the collection cup clean when you take it with you and while you are preparing for expressing (Fig. 10).

Pumping instructions

- 1 Wash your hands thoroughly with soap and water. Clean your breast and nipple prior to placing the collection cup.
- 2 Place the assembled collection cup into your bra, on your breast, with the pouring spout (opening) positioned at the top (Fig. 11).
- 3 Make sure that the nipple is in the center of the breast shield.
- 4 Please refer to the user manual of your Philips Avent electric breast pump for the next steps of pumping instructions.

ⓘ Note: The collection cups can be worn in your regular nursing bra. If you experience discomfort or pain, try switching to a more stretchy bra.


After use

⚠ Warning: Refrigerate or freeze expressed breast milk immediately or keep it at a temperature of 16-29 °C for a maximum of 4 hours before you feed your baby. Never refreeze thawed breast milk to avoid a decrease in milk quality. Never add fresh breast milk to frozen breast milk to avoid a decrease in milk quality and unintended thawing of the frozen milk.

- 1 After completing the pumping session, lift the collection cup out of your bra. Place the collection cup on a clean and flat surface (Fig. 12). Be careful not to spill any milk from the pouring spout.
- 2 To store the breast milk: pour the expressed breast milk from the collection cup into a separate milk storage container (e.g., a bottle or a breast milk storage bag) (Fig. 13). Do not store expressed breast milk in the collection cup.
- 3 Disassemble, clean and disinfect all parts that have been in contact with the breast and breast milk (see 'Cleaning and disinfecting').

Compatibility

The Philips Avent hands-free collection cup is compatible with the following motor unit of Philips Avent hands-free electric breast pump (SCF531, SCF532, SCF543, SCF547), electric breast pump Advanced (SCF391, SCF393, SCF395, SCF397), Plus (SCF391, SCF393, SCF373), Premium (SCF392, SCF394, SCF396, SCF398), Premium Plus (SCF392, SCF394), Philips Avent Breast Feeding Starter Set Advanced (SCD340), Philips Avent Breast Feeding Starter Set Premium (SCD330), or Philips Avent Breast Pump Starter Set Hands-Free (SCD553):

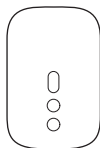Do not combine it with motor units of other Philips Avent breast pumps or different brands.

Ordering accessories and spare parts

To buy accessories for convenience, breast shields and inserts of smaller or larger size, or spare parts, visit **www.philips.com/support** or go to your Philips dealer. You can also contact the Philips Consumer Care Center in your country.

Recycling

The parts are made of plastics that are not harmful to the environment when disposed of as household waste. Recycle or dispose of according to local regulations.

Support

For all product support such as frequently asked questions, please visit www.philips.com/support.

Storage

Keep the collection cups out of direct sunlight as prolonged exposure may cause discoloration. Store the collection cups and their accessories in a safe, clean and dry place.

Technical specifications

Materials — Cup, front cap, breast shield (dust) cover: polypropylene. Breast shield, membrane, valve, insert: liquid silicone rubber. Silicone tube: silicone.

Explanation of symbols

The warning signs and symbols are essential to ensure that you use this product safely and correctly and to protect you and others from injury.

This symbol indicates:

- to follow the instructions for use (Fig. 14).
- important information such as warnings and cautions (Fig. 15).
- usage tips, additional information or a note (Fig. 16).
- the manufacturer (Fig. 17).
- the manufacturing date (Fig. 18).
- compliance with the general safety and performance requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices (Fig. 19).
- manufacturer's catalog number of the collection cup (Fig. 20).
- the batch number of the medical device (Fig. 21).
- the Global Trade Identification Number and the Lot Code of the collection cup (Fig. 22).
- the device is a medical device (Fig. 23).
- the Forest Stewardship Council (Fig. 24).
- that the material used in the product is safe for food contact (Fig. 25).
- the UDI carrier, including the AIDC and human readable information (Fig. 26).

- that the material used for the packaging is corrugated cardboard/fiberboard (Fig. 27).
- that the material used for the packaging is non-corrugated cardboard/fiberboard (Fig. 28).
- name and address of the authorized representative's registered place of business in Switzerland (Fig. 29).
- the entity importing the medical device into the local country (Fig. 30).
- that the packaging to be deposited in the blue container (Fig. 31).

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Trademarks are the property of Koninklijke Philips N.V. / Les marques commerciales sont la propriété de Koninklijke Philips N.V. / I marchi commerciali e depositati sono di proprietà di Koninklijke Philips N.V.
All rights reserved. / Tous droits réservés. / Tutti i diritti riservati.
Philips Consumer Lifestyle B.V.