

Experience the total sleep solution with our Dream Family

Philips DreamStation CPAP machines make it simple to start, customise and continue your therapy.

Philips DreamMapper App helps you track and share your sleep therapy progress.



Talk to your healthcare professional and discover the difference the Philips Dream Family can make for you.



1. Neuzeret PC, Morin L (2017) Impact of different nasal masks on CPAP therapy for obstructive sleep apnea: a randomized comparative trial. Clin Respir J 11(6):990–998.
 2. Data analysis of Aug 2017 Patient Preference trial where n = 85 and prescribed masks include Resmed Airfit F10 and F20, Resironics Amara View and F&P Simplus.
 3. DreamWear Under the Nose Nasal: Data analysis at 30 days of use of 2015 Philips sponsored patient preference trial (n=98). Prescribed masks include ResMed Swift FX, ResMed Mirage FX, Philips Wisp, and ResMed P10. Data analysis at 10 days of use of 2019 Philips sponsored patient preference trial (n=81). Prescribed masks include ResMed Airfit N10, N20, Mirage Fx, Philips Wisp, Pico, ComfortGel Blue, F&P Eson & Eson 2.
 4. DreamWear Full Face: Data analysis of Aug 2017 Patient Preference trial where (n=85) and prescribed masks include ResMed Airfit F10 and F20, Resironics Amara View and F&P Simplus.
 5. Analysis after 30 day of use during Jul/Aug 2020. Pillows preference trial with (n=127). Prescribed masks include ResMed Airfit P10, P30i, Swift Fx, F&P Brevida, Philips DreamWear Gel Pillows, Nuance / Pro.
 6. The design of the mask is such that it does not contact the nasal bridge.
 7. Design of the mask prevents the cushion from touching the bridge of the nose.
 8. Data analysis at 10 days of use of 2014 Philips sponsored patient preference trial (n=35).
- * Masks with magnets may be the right choice for many, but not all. Patients should discuss the use of magnets with their care provider.

ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE.

© 2025 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.
Trademarks are the property of Koninklijke Philips N.V. or their respective owners.

www.philips.com.au/sleep


Version: ANZ07/2025
P00000535-00-00 v1

Philips Sleep and Respiratory Care
65 Epping Road
North Ryde, NSW, 2113
Australia
Telephone: 1800 830 517 (toll-free)

PHILIPS

Set-up for a better night's sleep

A quick guide to help you choose a sleep apnoea mask

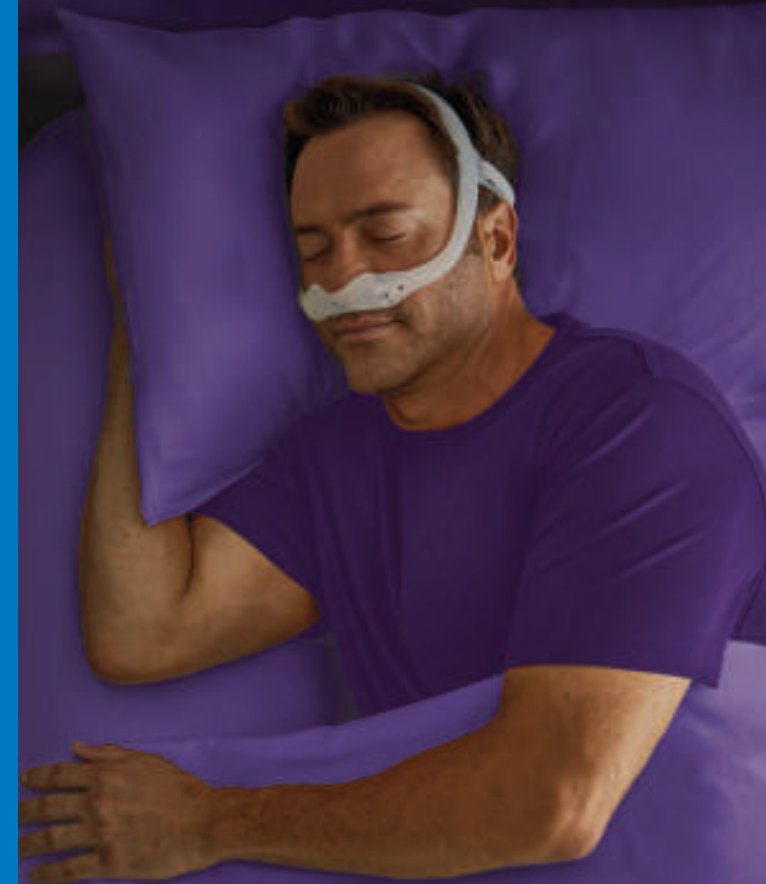



Match your needs with the right mask

When it comes to sleep apnoea, the fit of a mask can make or break a good night's sleep. Choosing a suitable CPAP Mask is an important part of CPAP therapy acceptance and adherence.¹









Philips is committed to helping you easily find the right mask, so you can sleep better. Because that's what matters most.

How to use

Review the following characteristics with your consultant and then discuss the corresponding mask guidance. We recommend talking through the options, such as the different tube connections and mask types, so you are aware of the range of styles available.

Scan this QR code to access more information about Philips CPAP masks and related resources.



Do you...	CPAP mask type	Tube connection	Philips mask guidance
 Is overall comfort most important to you?	Nasal or Pillows	Any	DreamWear – Nasal or Silicone Pillows Cushions ^{3,4,5}
 Do you have nasal congestion?	Full Face	Any	DreamWear Full Face Amara View
 Do you have facial hair?	Nasal or Pillows	Any	DreamWear – Nasal or Silicone Pillows Cushions
 Do you toss and turn at night or sleep on your side?	Any	Top	DreamWear – Nasal, Full Face or Silicone Pillows Cushions ^{2,3,4,5}
 Do you sleep on your stomach?	Nasal or Pillows	Top	DreamWear – Nasal or Silicone Pillows Cushions ^{3,4,5}
 Do you breathe through your mouth while sleeping?	Full Face	Any	DreamWear Full Face Amara View
 Do you want an open field of vision?	Nasal, Pillows or Total	Any	DreamWear – Nasal or Silicone Pillows Cushions ^{3,4,5} FitLife Total Full Face
 Do you wear glasses in bed while watching TV, reading, etc.?	Various	Any	DreamWear – Nasal, Full Face or Silicone Pillows Cushions Amara View Pico

Get to know your Philips mask options



DreamWear – Nasal or Silicone Pillows Cushions

- Designed for your comfort^{3,4,5}
- The DreamWear mask with Nasal Cushion is our most minimal design – cushion rests under your nose, not on it.⁶
- The innovative Silicone Pillows design causes less redness, discomfort and irritation on the face, nose and nostrils.⁵



DreamWear – Full Face

- Designed to prevent red marks and discomfort on the nose bridge.⁵
- Magnetic clips for quick assembly and release.*



Pico – Nasal

- Minimal contact nasal mask that is lightweight and small.
- Pico offers simplicity, comfort and seal with the freedom to watch TV or read while wearing it.⁸



Amara View – Full Face

- Innovative tube in front design offers an open field of vision and helps prevent red marks, discomfort or irritation on the nose bridge.⁷
- One frame for three cushion sizes (S,M,L).



FitLife – Total Full Face

- Unrestricted view.
- Large surface area.

For more information on masks and mask parts, please contact your consultant.