PHILIPS


AVENT



Single/Double
Breast Pump Kit

Kit tire-lait
simple/double

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SCF335, SCF337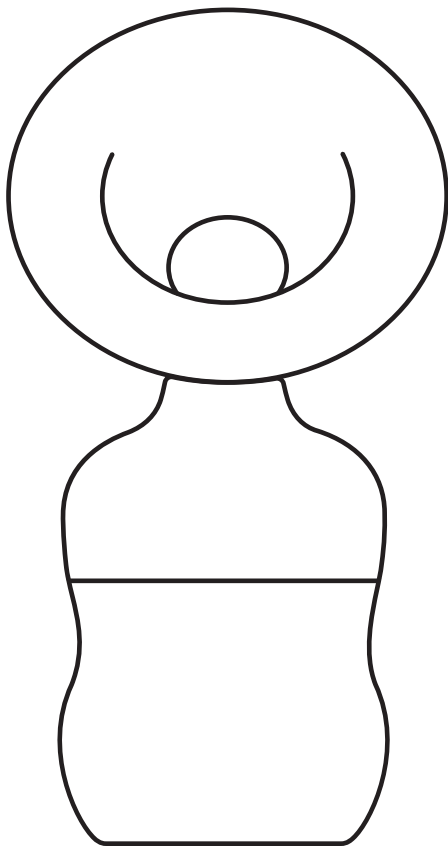

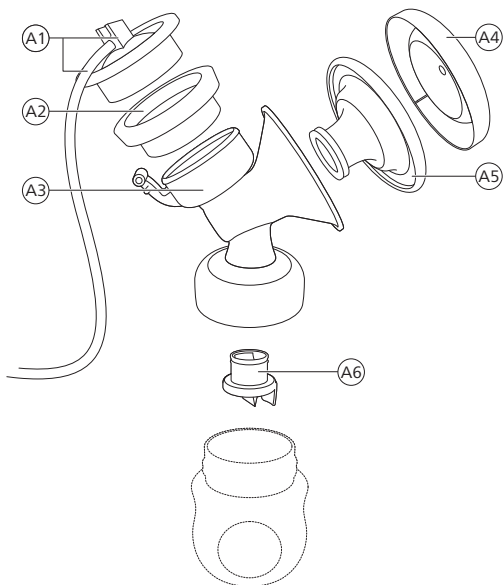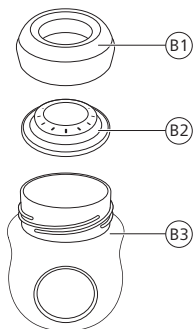
A**B**

English	6
Dansk	12
Deutsch	18
Español	25
Français	32
Italiano	39
Nederlands	46
Norsk	53
Português	59
Suomi	66
Svenska	72
Ελληνικά	78
עברית	85

Introduction

Welcome to Philips Avent! To learn more about this product or the Philips Avent product range, scan the QR code on the user manual or visit us at **www.philips.com/support**. On this website, you will also find the full user manual, videos and frequently asked questions.

The Philips Avent single/double breast pump kit can only be used in combination with the following motor unit of Philips Avent breast pump Advanced (SCF391, SCF393, SCF395, SCF397), Plus (SCF391, SCF393, SCF373), Premium (SCF392, SCF394, SCF396, SCF398), Premium Plus (SCF392, SCF394), Philips Avent Breast Feeding Starter Set Advanced (SCD340), or Philips Avent Breast Feeding Starter Set Premium (SCD330):

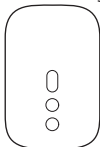Do not combine it with motor units of other Philips Avent breast pumps or different brands.

General description (Fig. 1)

i Note: The numbers mentioned below refer to the numbers in the figure on the front foldout page of this user manual.

i Note: The double breast pump kit comes with two pieces of the items shown in Figures A and B.

Product description (Fig. A)

- A1 Silicone tube and cap
- A2 Silicone diaphragm
- A3 Pump body
- A4 Cover
- A5 Cushion
- A6 White valve

Philips Avent Natural bottle (Fig. B)

- B1 Screw ring
- B2 Sealing disc
- B3 Bottle

Intended use

The Philips Avent single/double breast pump kit is intended to express and collect milk from the breast of a lactating woman. The device is intended for a single user.

Important safety information

Read this user manual carefully before you use the breast pump kit and save it for future reference.

The full user manual can be found online: www.philips.com/support.

Warning

Warnings to avoid contamination and to ensure hygiene:

- For hygienic reasons, the breast pump is only intended for repeated use by a single user.
- Clean and disinfect all parts before first use and after each use.

Warnings to avoid breast and nipple problems including pain:

- Do not use the breast pump if the silicone diaphragm appears to be damaged or broken. See 'Ordering accessories and spare parts' for information on how to obtain replacement parts.
- Only use accessories and parts recommended by Philips Avent.
- Never use the breast pump while you are sleepy or drowsy to avoid lack of attention during use.

Before first use

Disassemble, clean and disinfect all parts that come into contact with the breast and breast milk before you use the breast pump kit for the first time and after every use.

Cleaning and disinfecting

i Note: The figure numbers mentioned below refer to the numbers of the figures on the back foldout page of this user manual.

- Clean and disinfect the parts that come into contact with the breast and breast milk (Fig. 2) before first use and after every use.
- Clean the parts that do not come into contact with the breast and breast milk when needed (Fig. 3). Wipe the parts with a clean, damp cloth with water and mild detergent.

Step 1: Disassembling

Disassemble the breast pump kit (Fig. A) and the bottle (Fig. B) completely. Also remove the white valve (Fig. A6) from the pump body (Fig. A3).

Step 2: Cleaning

The parts that come into contact with the breast and breast milk can be cleaned manually or in the dishwasher.

⚠ Warning: Do not use antibacterial or abrasive cleaning agents to clean the breast pump kit parts as this may cause damage.

⚠ Caution: Be careful when you remove and clean the white valve. If it gets damaged, your breast pump does not function properly. To remove the white valve, gently pull at the ribbed tab on the side of the valve. To clean the white valve, rub it gently between your fingers in warm water with some dishwashing liquid. Do not insert objects into the white valve, as this may cause damage.

Step 2A: Manual cleaning

- 1 Rinse all parts under a running tap with lukewarm water.
- 2 Soak all parts for 5 minutes in warm water with some mild dishwashing liquid.
- 3 Clean all parts with a cleaning brush in warm water with some mild dishwashing liquid.
- 4 Thoroughly rinse all parts under a running tap with cold clear water.
- 5 Leave all parts to air-dry on a clean tea towel or drying rack.

Step 2B: Cleaning in the dishwasher

ⓘ Note: Food colorings may discolor parts.

- 1 Place all parts on the top rack of the dishwasher.
- 2 Put detergent or a dishwashing tablet in the machine and run a standard program.
- 3 Leave all parts to air-dry on a clean tea towel or drying rack.

Step 3: Disinfecting

⚠ Caution: During disinfection with boiling water, prevent the bottle or other parts from touching the sides of the pot. This can cause irreversible product deformation or damage that Philips cannot be held liable for.

- 1 Fill a household pot with enough water to cover these parts and place all the parts in the pot. Let the water boil for 5 minutes. Make sure the parts do not touch the sides of the pot.
- 2 Allow the water to cool down.
- 3 Gently remove the parts from the water. Place the parts on a clean surface, tea towel or drying rack to air-dry.
- 4 Store the dry items safely in a clean, protected area until needed. Ensure that the clean parts have air-dried completely before storing them to help prevent germs and mold from growing.

Using the breast pump kit

Cushion size

The Philips Avent single/double breast pump kit has a soft, active cushion. There is only one cushion size. It gently stimulates your nipple to trigger milk flow. The cushion is made of flexible silicone, fitting nipple sizes up to 30 mm.

Assembling the breast pump kit

Make sure you have cleaned and disinfected the parts of the breast pump kit that come into contact with the breast and breast milk.

⚠ Warning: Be careful, when you have disinfected the breast pump parts by boiling them, they can be very hot. To prevent burns, only start assembling the breast pump when disinfected parts have cooled down.

⚠ Warning: Wash your hands thoroughly with soap and water before you touch breast pump parts and breasts to prevent contamination. Avoid touching the inside of containers or lids.

Check the breast pump kit parts for wear or damage before use and replace them if necessary.

❗ Note: Correct placement of the white valve, cushion, silicone diaphragm and silicone tube is essential for the breast pump to form a proper vacuum.

- 1 Insert the white valve into the pump body from underneath. Push the white valve in as far as possible (Fig. 4).
- 2 Screw the pump body clockwise onto the bottle until it is securely fixed (Fig. 5).
- 3 Place the cushion into the pump body (Fig. 6) and make sure that the rim covers the pump body. Push in the inner part of the cushion into the funnel against the line indicated with an arrow (Fig. 7).

- 4 Place the silicone diaphragm into the pump body (Fig. 8). Make sure that the silicone diaphragm fits securely around the rim by pressing down with your thumbs (Fig. 9).
- 5 Attach the silicone tube and cap onto the silicone diaphragm. Push down the cap until it is securely in place (Fig. 10).
- 6 Please refer to the user manual of your Philips Avent electric breast pump for the next assembly steps, for example how to connect the tube(s) to the motor unit.

Pumping instructions

- 1 Wash your hands thoroughly with soap and water and make sure that your breast is clean.
- 2 Place the breast pump kit onto your breast. Make sure that you center your nipple (Fig. 11).
- 3 Please refer to the user manual of your Philips Avent electric breast pump for the next steps of pumping instructions.


After use

- 1 Disassemble, clean and disinfect all parts that have been in contact with the breast and breast milk (see 'Cleaning and disinfecting').
- 2 Please refer to the user manual of your Philips Avent electric breast pump for instructions on how to store the expressed breast milk.

Compatibility

The Philips Avent single/double breast pump kit is compatible with all baby bottles in the Philips Avent Natural range and Philips Avent storage cups with adapter. We do not recommend using the Philips Avent Natural glass bottles with the breast pump kit.

The Philips Avent single/double breast pump kit can only be used in combination with the following motor unit of Philips Avent breast pump Advanced (SCF391, SCF393, SCF395, SCF397), Plus (SCF391, SCF393, SCF373), Premium (SCF392, SCF394, SCF396, SCF398), Premium Plus (SCF392, SCF394), Philips Avent Breast Feeding Starter Set Advanced (SCD340), or Philips Avent Breast Feeding Starter Set Premium (SCD330):

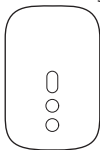Do not combine it with motor units of other Philips Avent breast pumps or different brands.

Ordering accessories and spare parts

To buy accessories or spare parts, visit www.philips.com/support or go to your Philips dealer. You can also contact the Philips Consumer Care Center in your country.

Support

For all product support such as frequently asked questions, please visit www.philips.com/support.

Storage conditions

Keep the breast pump kit out of direct sunlight as prolonged exposure may cause discoloration. Store the breast pump kit and its accessories in a safe, clean and dry place.

Technical information

Operating conditions: temperature: 5 °C to 40 °C, relative humidity: 15 to 90 % (non-condensing).

Storage and transportation conditions: temperature -25 °C to 70 °C, relative humidity up to 90 % (non-condensing).

Operating and storage pressure: 700-1060 hPa (<3000 m altitude).

Materials: polypropylene, silicone.

Explanation of symbols

The warning signs and symbols are essential to ensure that you use this product safely and correctly and to protect you and others from injury.

This symbol indicates:

- important information such as warnings and cautions (Fig. 13).
- that the product or packaging needs to be sorted or brought to a recycling point (Fig. 14).

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Trademarks are the property of Koninklijke Philips N.V. / Les marques commerciales sont la propriété de Koninklijke Philips N.V. / I marchi commerciali e depositati sono di proprietà di Koninklijke Philips N.V.
All rights reserved. / Tous droits réservés. / Tutti i diritti riservati.
Philips Consumer Lifestyle B.V.